


600mm BARIATRIC KATIE CHAIR

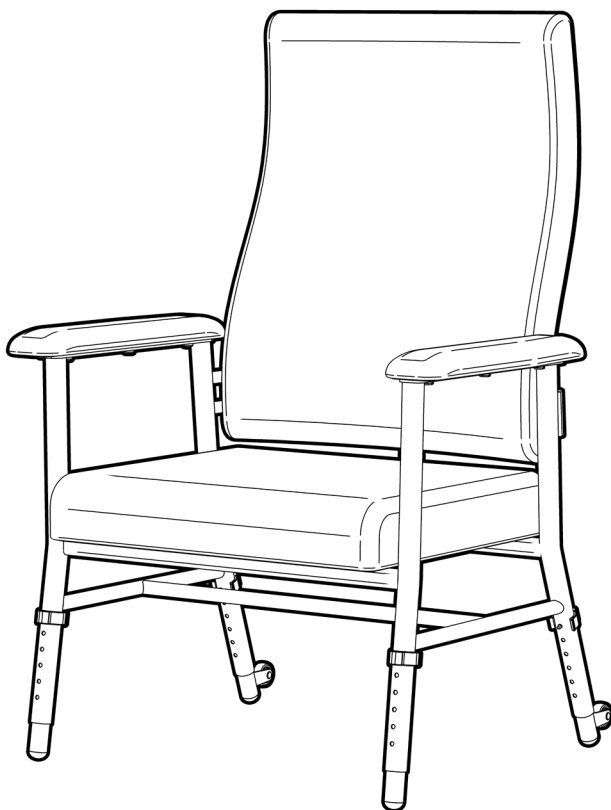


RG632

Edition 1



USER MANUAL

Please read and keep these instructions

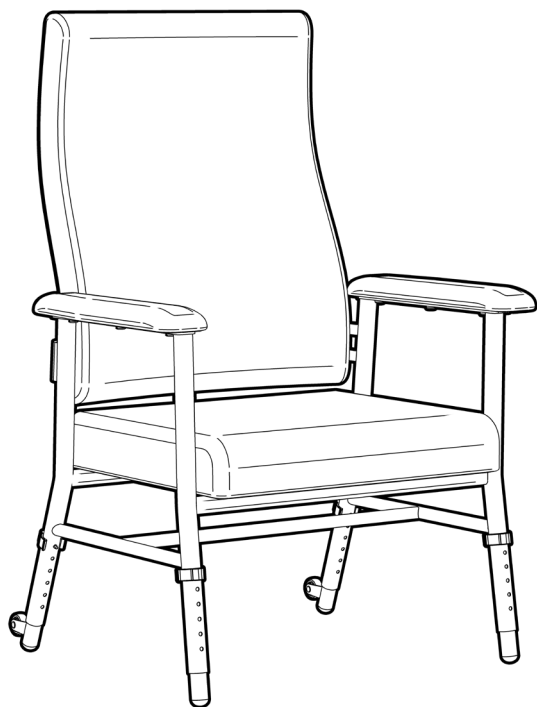

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Bariatric Katie Chair user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**




REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **SET UP** - UNPACK and INSPECT SUPPLIED PARTS
4. **SET UP** - FIT BACKREST
5. **SET UP** - ADJUST SEAT HEIGHT
6. **SET UP** - ADJUST SEAT HEIGHT (continued)
7. **SET UP** - ADJUST SEAT DEPTH
8. **USAGE GUIDELINES**
9. **PRODUCT CARE** - INSPECTIONS / CLEANING
10. **WARRANTY**
11. **KATIE CHAIR OPTIONS**
12. **REDGUM BRAND RANGE**

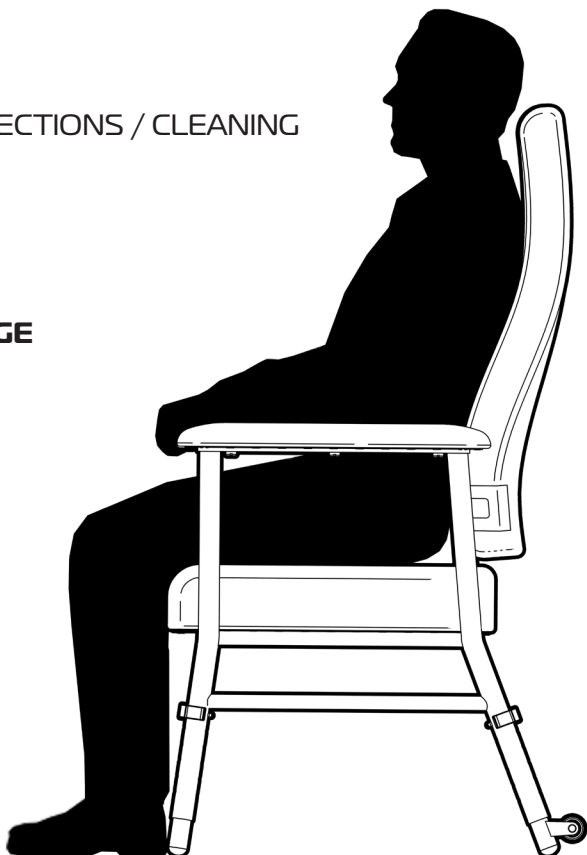

INTRODUCTION

Thank you for purchasing this REDGUM Brand product.

If you have any issues or questions about the Bariatric Katie Chair please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180


E: info@redgumbrand.com.au

A: 1 Business Way, Malaga, Western Australia 6090

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use the Bariatric Katie Chair for its intended purpose which is as an INDOOR seating solution that can adjust to suit the users individual requirements.

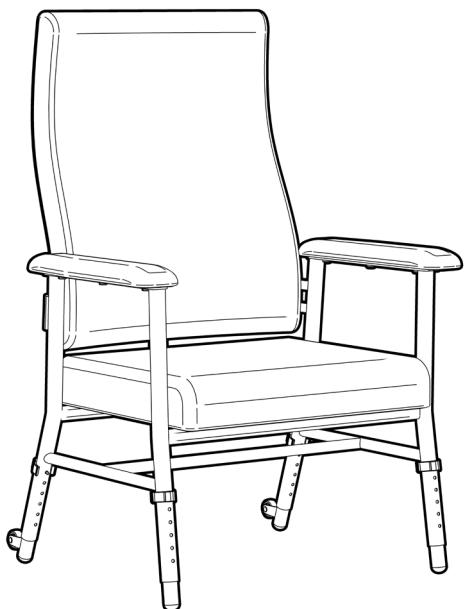

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 350kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.


1.

ASK FOR THE BRAND

PRODUCT OVERVIEW

FEATURES

- 350kg weight load capacity
- 600mm between armrests
- Adjustable seat height
- Adjustable seat depth
- Contoured padded backrest
- Comfortable padded seat
- Large padded armrests
- Chair mobility wheels on rear legs
- Reinforced metal framework
- Vinyl covered upholstery

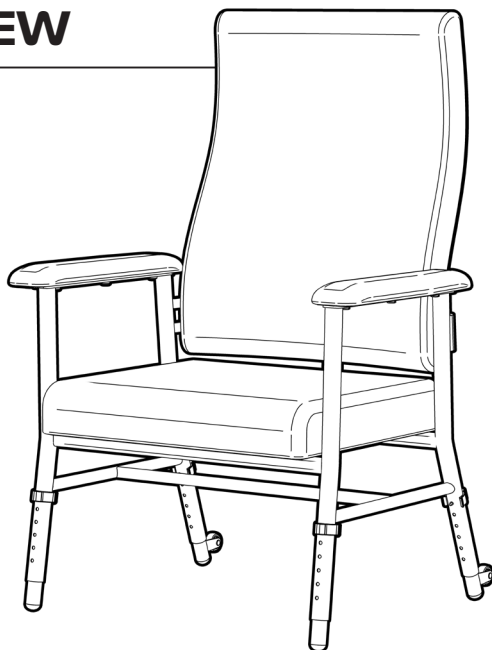


SPECIFICATIONS (NOTE: All dimensions shown are nominal)

- Overall Width: **790mm**
- Overall Depth: **645mm** (at lowest setting)
- Overall Height: **1160mm** (at lowest setting)
- Seat Width: **600mm** (between armrests)
- Seat Depth: **450 – 490mm** (front of seat to middle of backrest)
- Seat Height: **450 - 575mm**: 6 settings (floor to top of seat)
- Armrest Height: **220mm** (top of seat to top of armrest)
- Backrest Width: **600mm** (at widest point)
- Backrest Height: **700mm** (seat to top of backrest)
- Backrest Angle: **98°**
- Mobility Wheel Diameter: **Ø40mm**
- Product Weight: **00kg**
- Maximum User Weight: **350kg**


Please contact REDGUM Brand if you would like any further information on the Bariatric Katie Chair.

SET UP UNPACK and INSPECT SUPPLIED PARTS


Ensure you have been supplied with the following parts:

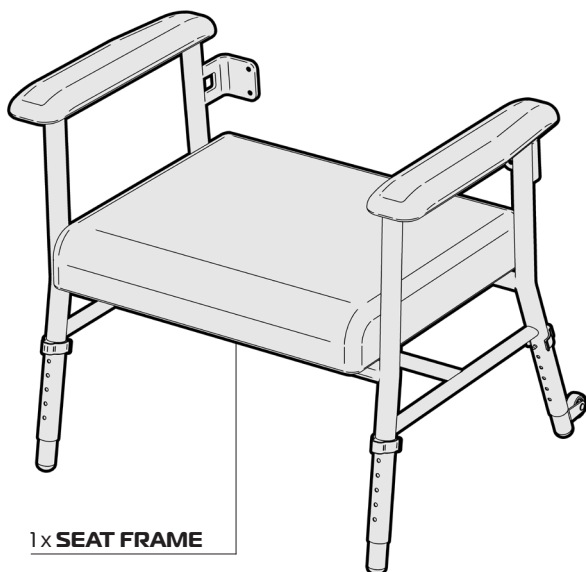


1 x **SEAT FRAME**

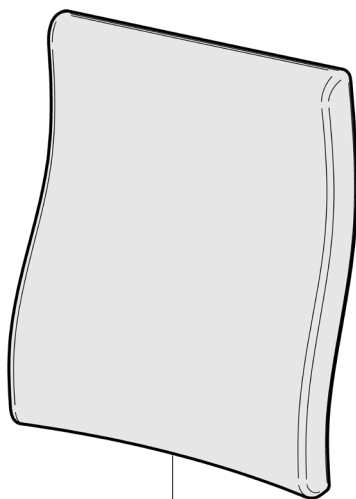

1 x **BACKREST**

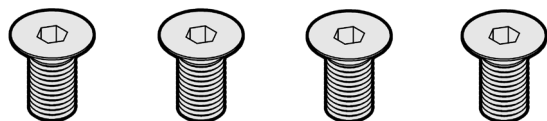

4 x **FIXING**

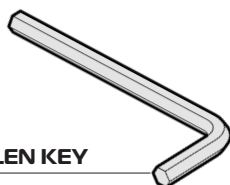

1 x **ALLEN KEY**

IMPORTANT NOTE

Inspect the product prior to assembling. If you see any damage or suspect a fault **DO NOT USE THE PRODUCT** and contact your supplier for support.

3.

ASK FOR THE BRAND




SET UP FIT BACKREST

Please follow these steps to fit the backrest onto the seat frame:

1. Align the backrest with the seat frame as shown.

2. Align the holes in the backrest with the holes in the mount brackets on the seat frame.

3. Using the fixings and allen key supplied fix backrest in place

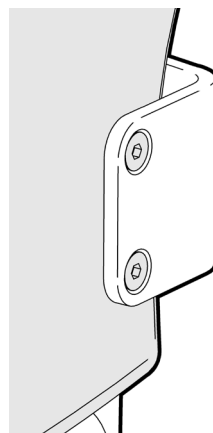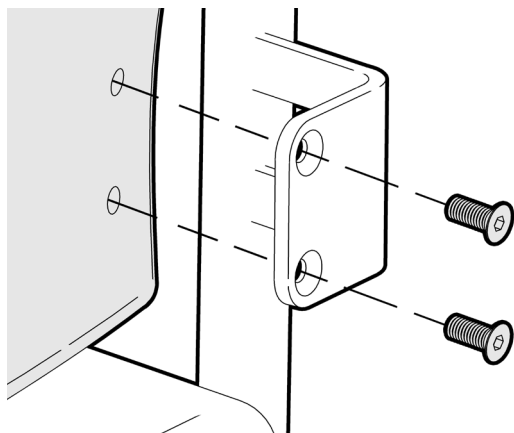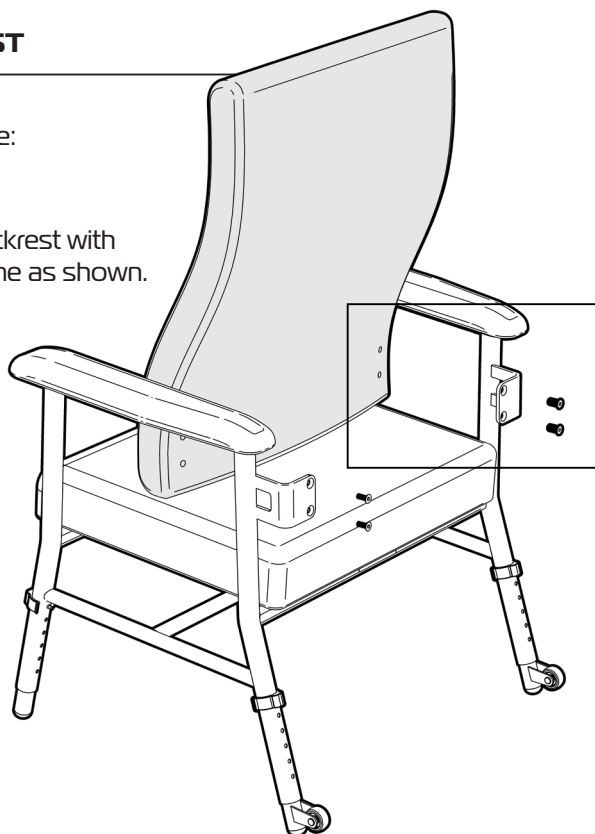

IMPORTANT NOTE

Ensure all four fixings are sufficiently tightened prior to use.

4.

SET UP ADJUST SEAT HEIGHT

The Bariatric Katie Chair has an adjustable height seat that can be changed to suit the user.

The legs on the chair should be adjusted so the top of the seat is inline with the back of the users knee.

Measure from the floor to the bend in the knee and set the leg height to the closest hole setting:

Seat Height

(Floor to top of seat)

Hole 1: 450mm


Hole 2: 475mm

Hole 3: 500mm

Hole 4: 525mm

Hole 5: 550mm

Hole 6: 575mm

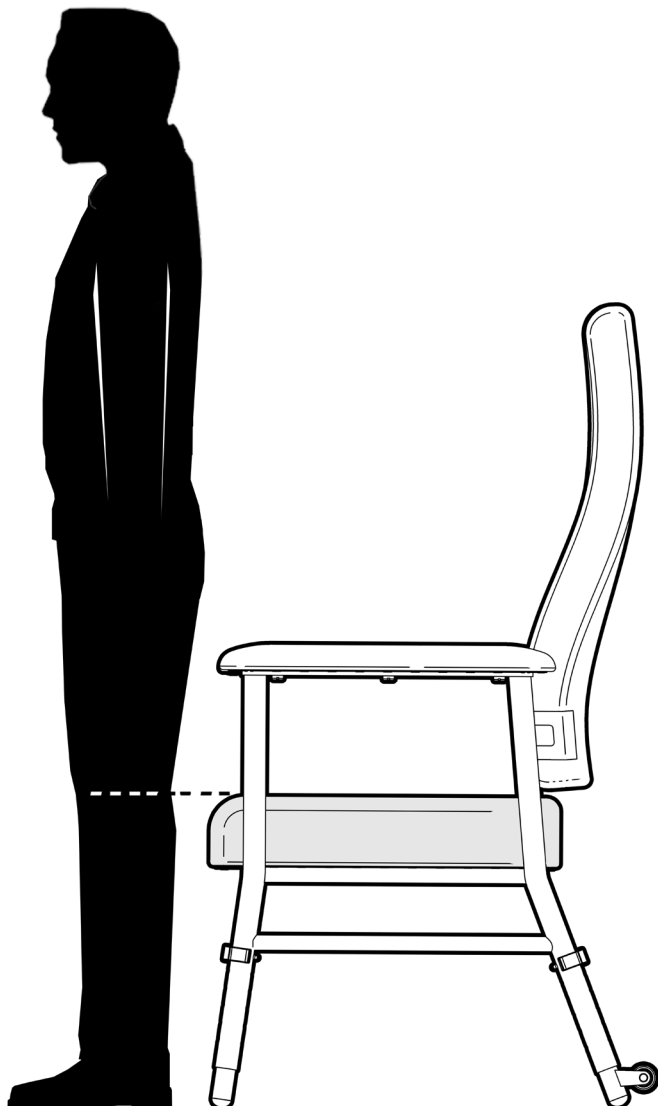



5.

ASK FOR THE BRAND

SET UP ADJUST SEAT HEIGHT

Please follow these steps to adjust the legs to the required height:

1. Carefully position the chair so the legs facing up.

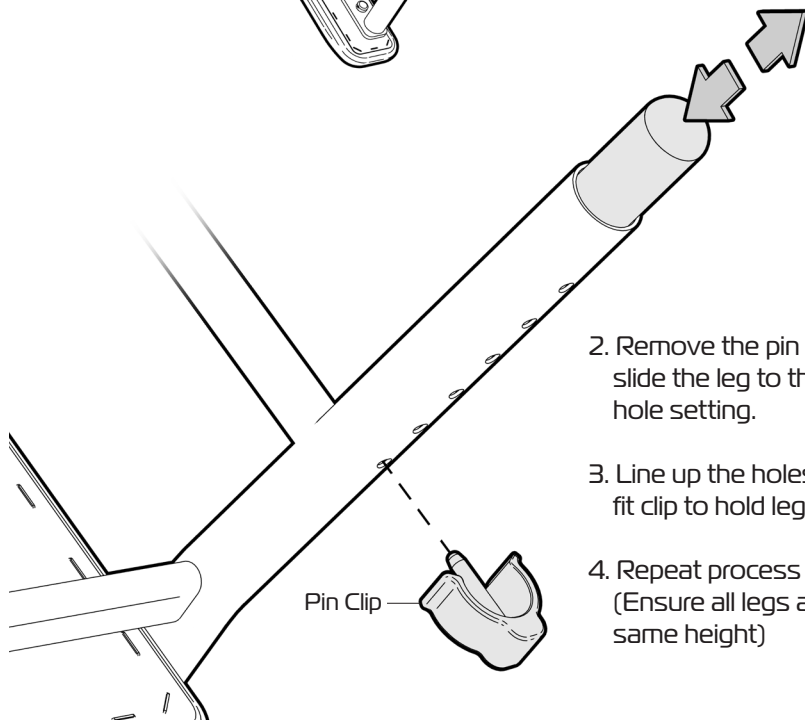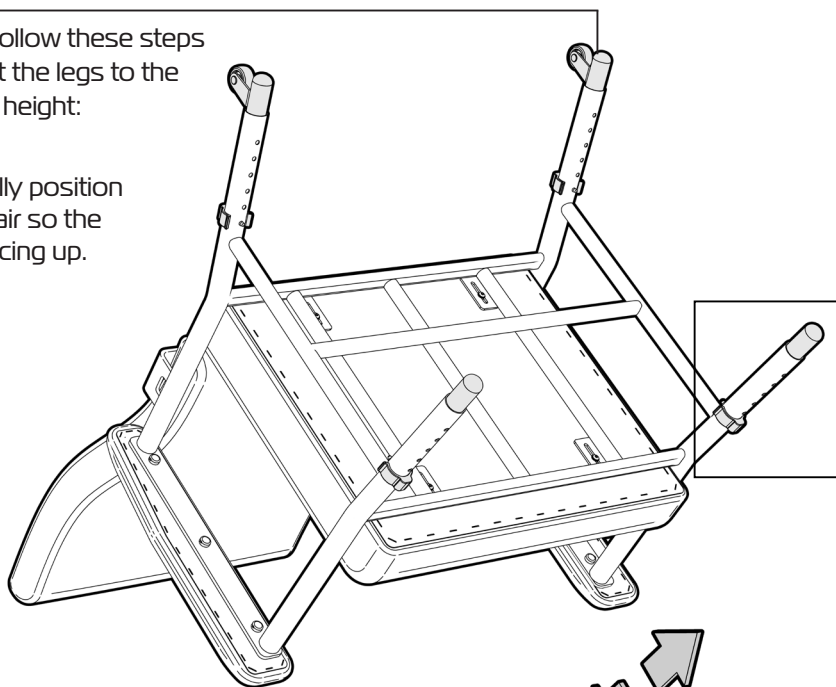


2. Remove the pin clip and slide the leg to the required hole setting.
3. Line up the holes and fit clip to hold leg in place.
4. Repeat process on all four legs (Ensure all legs are set to the same height)

6.


SET UP ADJUST SEAT DEPTH

The Bariatric Katie Chair has adjustable seat depth that can be changed to suit the user.

Adjust the chair so the user can lean comfortably against the backrest and have the back of the knee past the front of the seat.

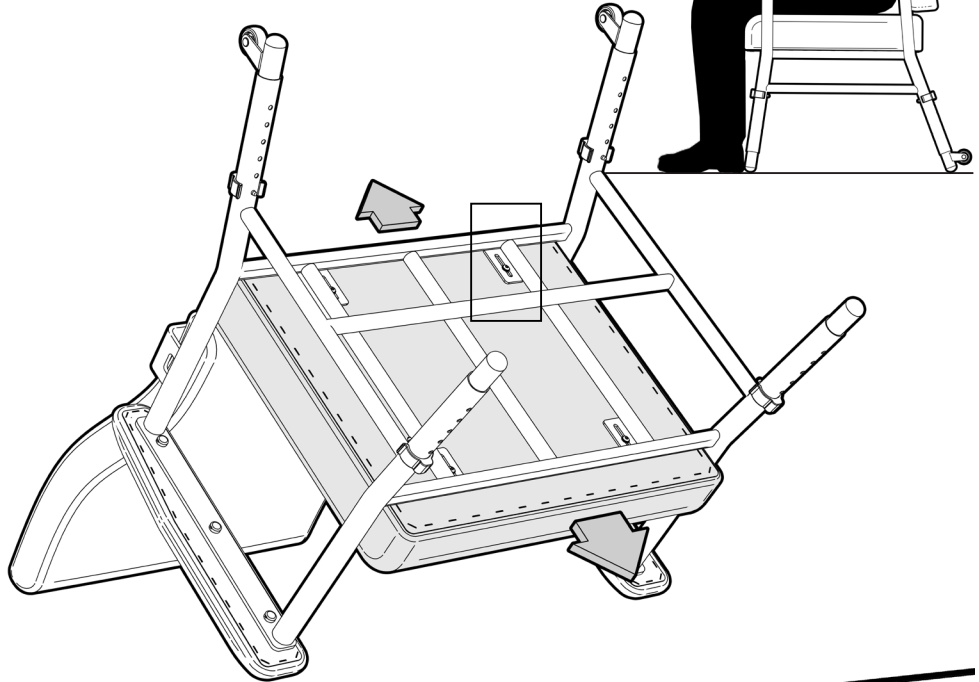

1. Carefully position the chair so the legs facing up.
2. Using the allen key provided - loosen all four fixings that hold the padded seat on the frame.
3. Slide the seat to the required position.
4. Tighten all four fixings to hold the seat in position.

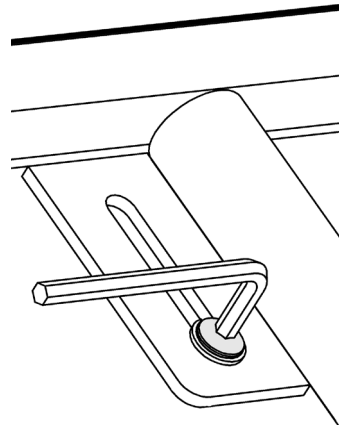

7.

ASK FOR THE BRAND

USAGE GUIDELINES


Please follow these guidelines to get the optimal benefit when using the Bariatric Katie Chair.

Failure to follow these guidelines may result in damage to the chair and put the user at risk.


350kg Maximum User Weight.

Always adjust the chair to the required height setting to suit the user.

Ensure **ALL FOUR LEGS** remain on ground at all times when chair is occupied.

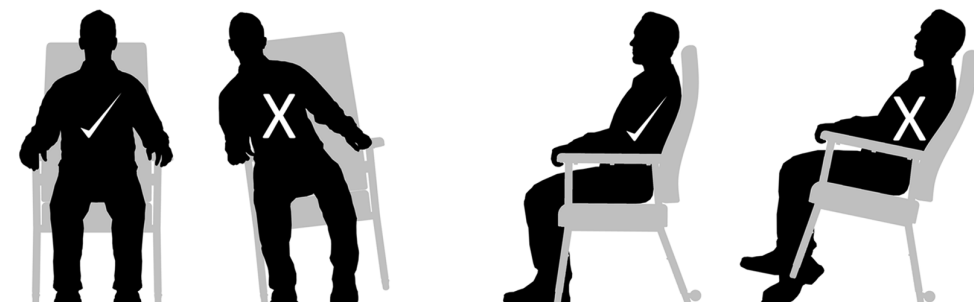

Do not lean too far to the side or backward causing legs to leave the ground.

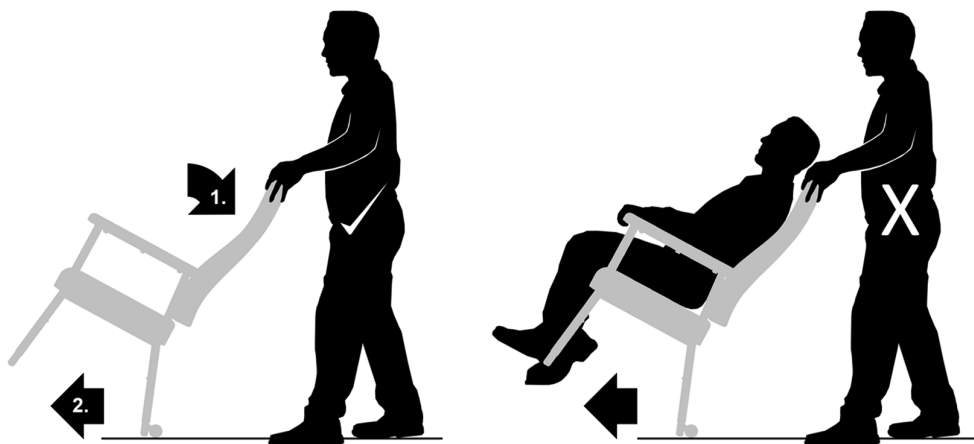

To move the chair, rotate back until the mobility wheels engage with the ground.
NEVER move the chair when occupied.

PRODUCT CARE INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the Bariatric Katie Chair periodically for signs of damage or wear prior to use. Please check all aspects of the chair and pay particular attention to the following areas:

Metalwork – Ensure the frame is not damaged or distorted – no visible cracks.

Legs - Ensure all four legs are not damaged or distorted.

Fasteners – Check all bolts are done up tight / leg press buttons are engaged

Upholstery – Ensure all coverings are not ripped or torn.

Overall check that no components are loose or worn.



Withdraw the Bariatric Katie Chair immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING

Clean the Bariatric Katie Chair using disinfectant wipes.

We do not recommend cleaning the product with soap and water.

Do not use any abrasive cleaners or abrasive cleaning pads as they could damage the toilet surround.

If you have any questions about the cleaning of the product please contact your supplier or REDGUM brand direct.



WARRANTY

REDGUM Brand - RG632: 600mm Bariatric Katie Chair is covered by a 12 month warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if the product needs to be returned to the place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser.
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.


Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the Bariatric Katie Chair in any way or fit any parts or accessories that are not specified as an option. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the Bariatric Katie Chair please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **info@redgumbrand.com.au**

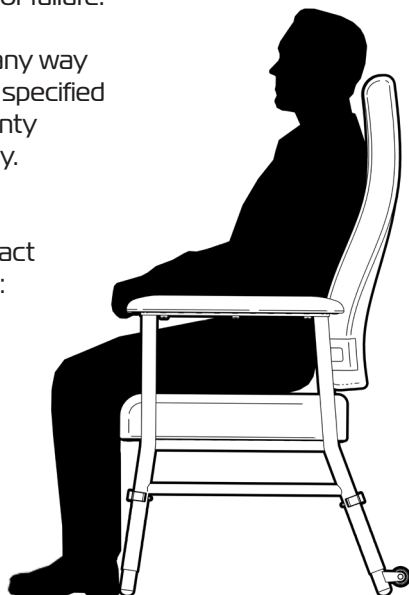


KATIE CHAIR OPTIONS

REDGUM Brand offer a range of options to accompany the Bariatric Katie Chair:

KATIE LEG REST

- Can be used as a legrest or stool
- Adjustable height and angle
- Padded upholstery to match Bariatric Katie Chair

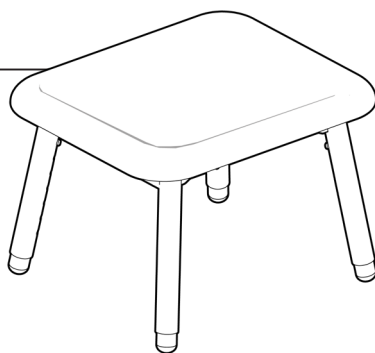

Available in:

BLUE - RG63LRBL

GREY - RG63LRGR

IVORY - RG63LRIV

SAGE GREEN - RG63LRSG

KATIE CHAIR RUBBER LEG TIPS

RG63RLT


- Can be fitted onto Katie Chair or Leg Rests to improve grip on hard flooring
- Durable non slip grey rubber leg tip
- Ribbed grip pattern on underside
- Easy to fit and remove



KATIE SIDE FOLDING TABLE

RG63SFT

- Handy side table
- Can be fitted to left or right hand side
- Folds down when not required

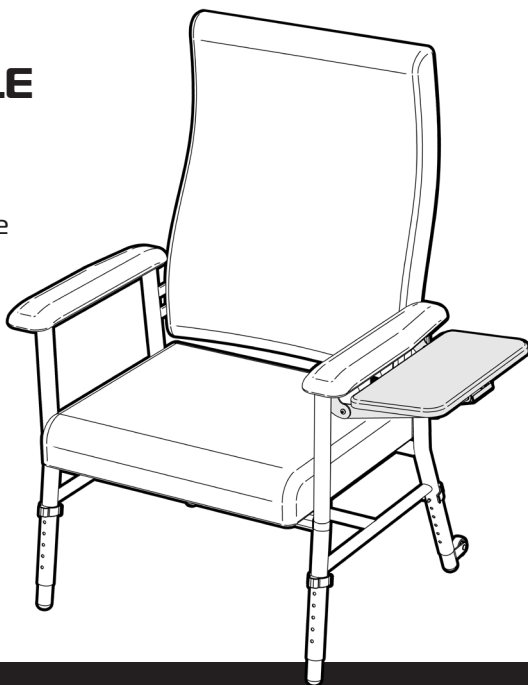


REDGUM BRAND RANGE

REDGUM Brand offer a comprehensive range of products including:

SEAT WALKERS
WALKING FRAMES
WALKING STICKS
QUAD CANES
CRUTCHES
REACHERS
WHEELCHAIRS
SHOWER CHAIRS
SHOWER STOOLS
TOILET SEAT RAISERS
MOBILE SHOWER COMMODOES
GRAB RAILS
DAY CHAIRS
LIFT CHAIRS
CUSHIONS
ELECTRIC BEDS
BED RAILS
SELF HELP POLES
RAMPS

**Primo
Seat Walker**
RG4212ADJ

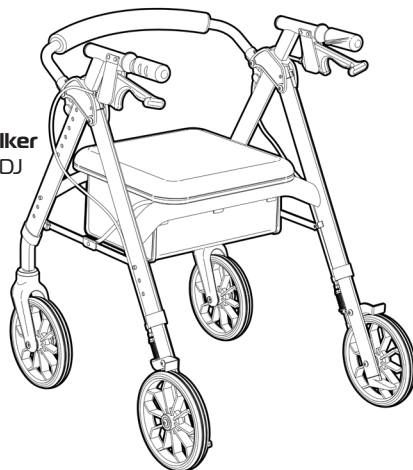

**Space Saver
Shower Chair**
RG555H



REDGUM Brand stock a large range of spare parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:

T: +61 8 9248 4180 E: info@redgumbrand.com.au

DISTRIBUTOR:



**Explore our wide
range of products!**