


REDGUMbrand
ASSISTED LIVING PRODUCTS

KATIEplus⁺

Day Chair Range

RG631: **KATIE PLUS** HIGH BACK
RG636: **KATIE PLUS** LOW BACK

Edition 1

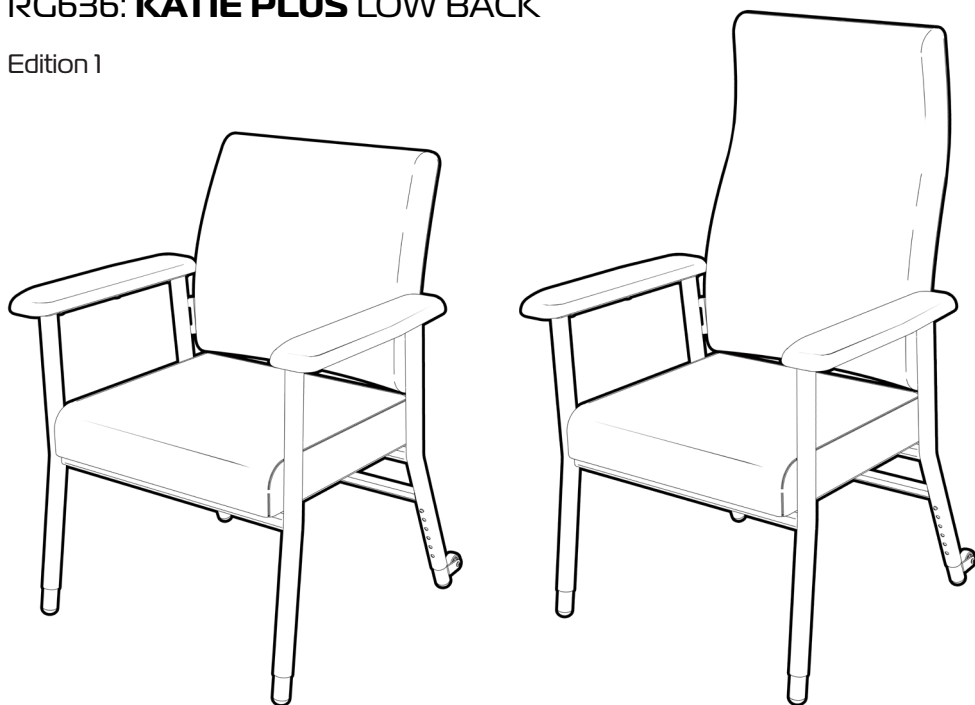


USER MANUAL

Please read and keep these instructions

ASK FOR THE BRAND

redgumbrand.com.au

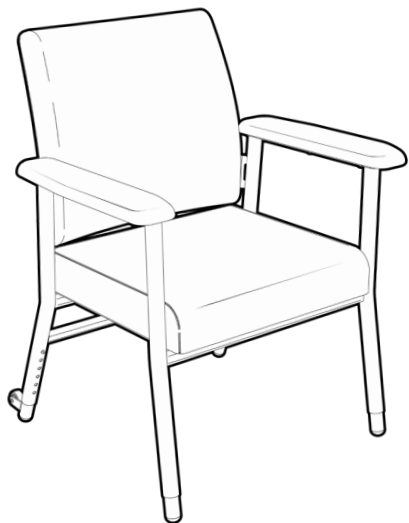

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Katie Plus Day Chair Range user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**




REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **SET UP** - UNPACK and INSPECT SUPPLIED PARTS
4. **SET UP** - FIT BACKREST
5. **SET UP** - FIT BACKREST (continued)
6. **SET UP** - ADJUST SEAT HEIGHT
7. **SET UP** - ADJUST SEAT DEPTH
8. **USAGE GUIDELINES**
9. **PRODUCT CARE** - INSPECTIONS / CLEANING
10. **WARRANTY**
11. **KATIE CHAIR OPTIONS**
12. **REDGUM BRAND RANGE**

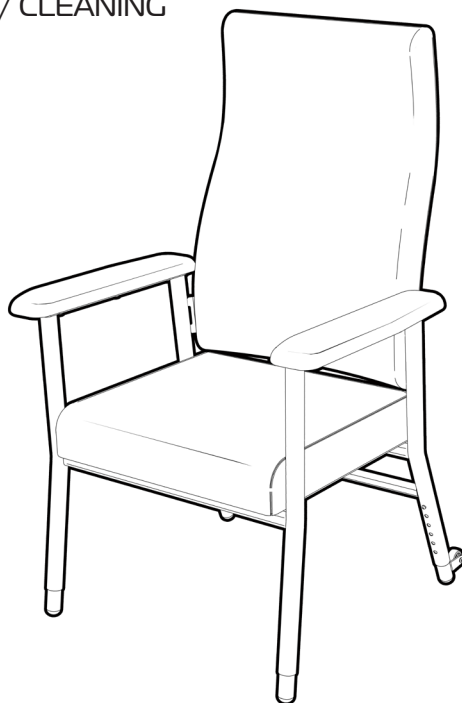

INTRODUCTION

Thank you for purchasing this REDGUM Brand product.

If you have any issues or questions about the Katie Plus Chair please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**


E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use the Katie Plus Chair for its intended purpose which is as an INDOOR seating solution that can adjust to suit the users individual requirements.

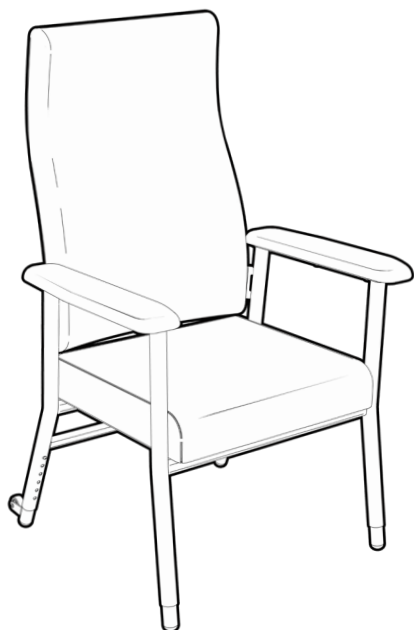

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 200kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

1.


ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

540mm between armrests
Adjustable seat height
Adjustable seat depth
Contoured padded backrest
Comfortable padded seat
Large padded armrests
Chair mobility wheels on rear legs
Backrest can be folded down for transit /storage
Strong lightweight construction
Epoxy powder coated framework
Vinyl covered upholstery

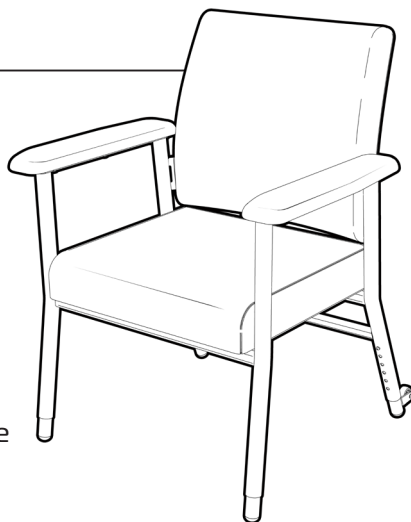

Katie Plus Chairs High / Low Back are available in the following colours:

High Back BLUE: RG631BL

Low Back BLUE: RG636BL

High Back GREY: RG631GR

Low Back GREY: RG636GR


SPECIFICATIONS (NOTE: All dimensions shown are nominal)

Overall Width: **715mm** (at lowest setting)
Overall Depth: **650mm** (at lowest setting)
Overall Height: **1175mm** (high back) **960mm** (low back) (at lowest setting)
Seat Width: **540mm** (between armrests)
Seat Depth: **435 – 475mm** (front of seat to middle of backrest)
Seat Height: **450 – 575mm**: 6 settings (floor to top of seat)
Armrest Height: **220mm** (top of seat to top of armrest)
Backrest Width: **480mm** (at widest point)
Backrest Height: **425mm** (low back) - **700mm** (high back) (seat to top of backrest)
Backrest Angle: **98°**
Mobility Wheel Diameter: **Ø40mm**
Product Weight: **13kg** (low back) - **15kg** (high back)
Maximum User Weight: **200kg**


Please contact REDGUM Brand if you would like any further information on the Katie Plus Chair range.

SET UP UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

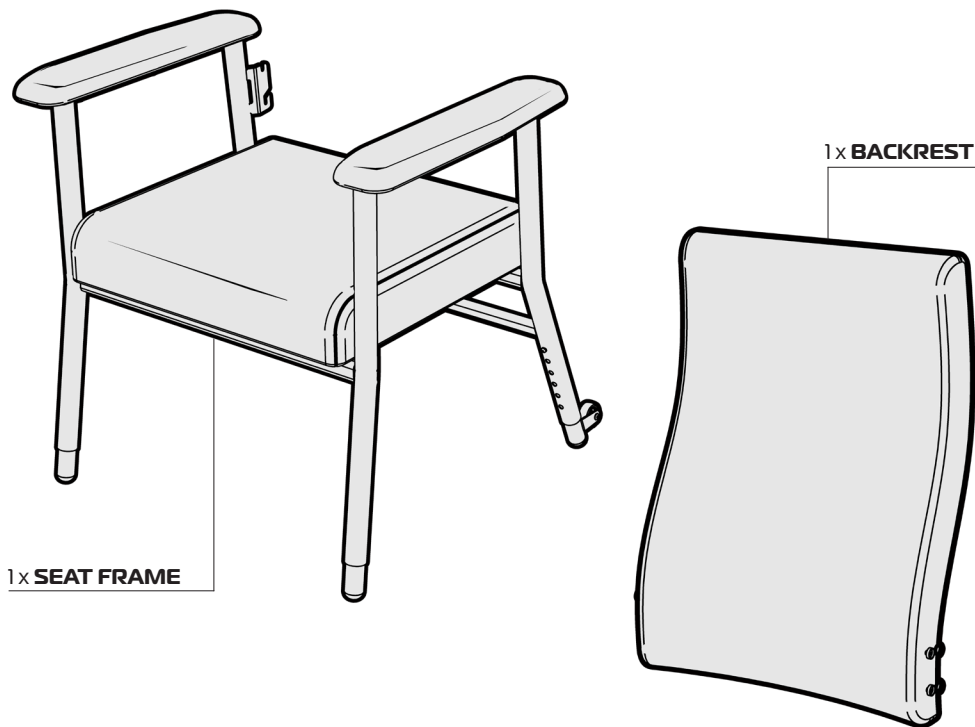

TOOLS (not shown to scale)

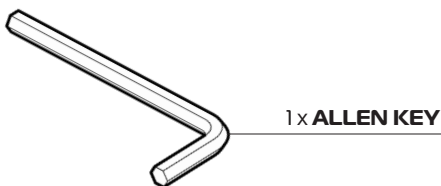

IMPORTANT NOTE

Inspect the product prior to assembling. If you see any damage or suspect a fault **DO NOT USE THE PRODUCT** and contact your supplier for support.


3.

ASK FOR THE BRAND




redgumbrand.com.au

SET UP FIT BACKREST

Please follow these steps to fit the backrest onto the seat frame correctly:

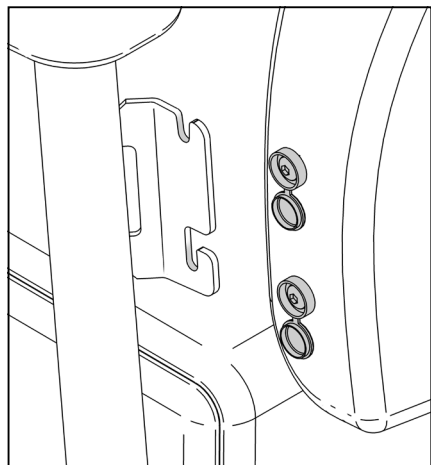

1. Align the backrest with the seat frame as shown.

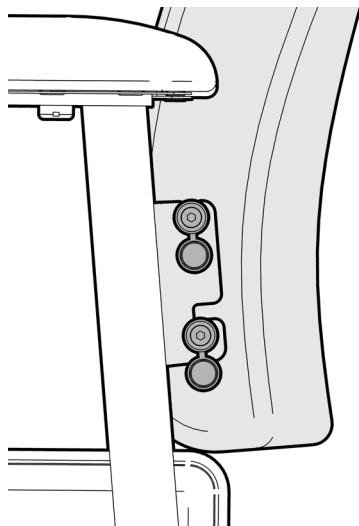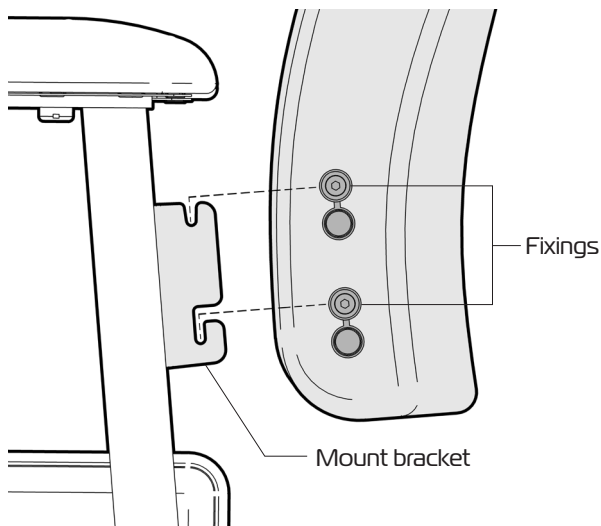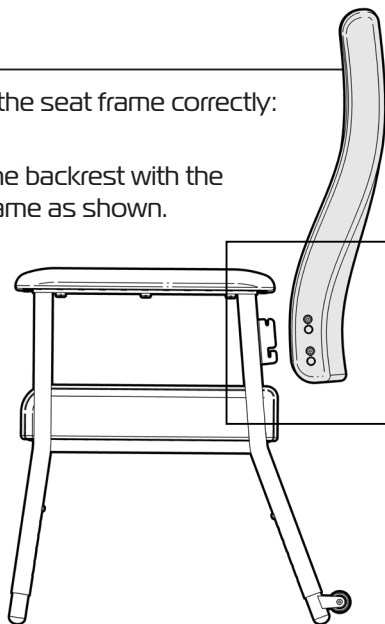


2. Align the fixings on the backrest with the mount brackets on the seat frame.
With the cover caps on the outside of the mount brackets - carefully locate the fixings onto the slots on the mount brackets on both sides.


Please refer to next page for more instruction on fitting backrest....

4.

SET UP FIT BACKREST (CONTINUED)

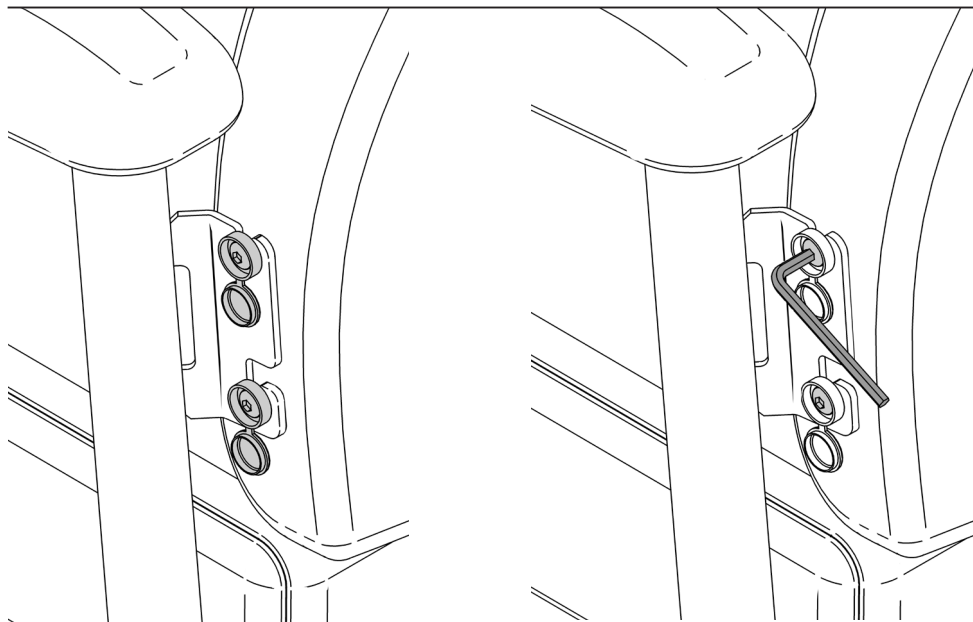

3. Using the allen key provided - tighten all four fixings.

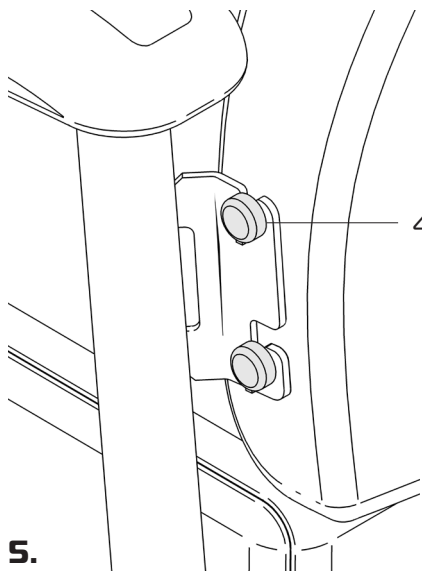

4. Once all four fixings are sufficiently tightened close the cover caps.

5.

ASK FOR THE BRAND



redgumbrand.com.au

SET UP ADJUST SEAT HEIGHT


Please follow these instructions to adjust the seat to the correct height to suit the user.

Adjust the Katie Plus Chair so the top of the seat is inline with the back of the knee.

1. Carefully lay the Katie Plus Chair on its side to access the adjustable legs.

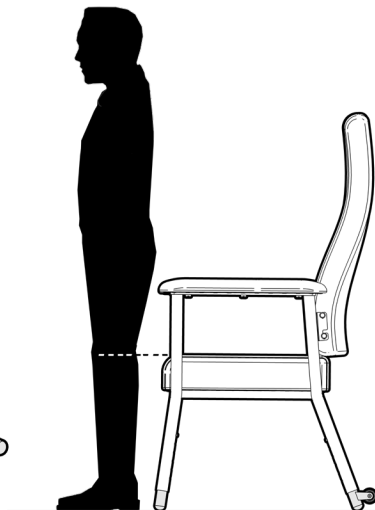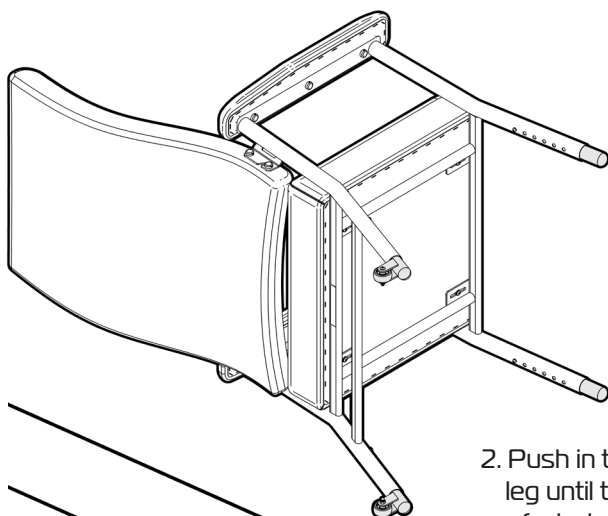

2. Push in the press button and slide the leg until the press button pops back out of a hole at the required height setting.

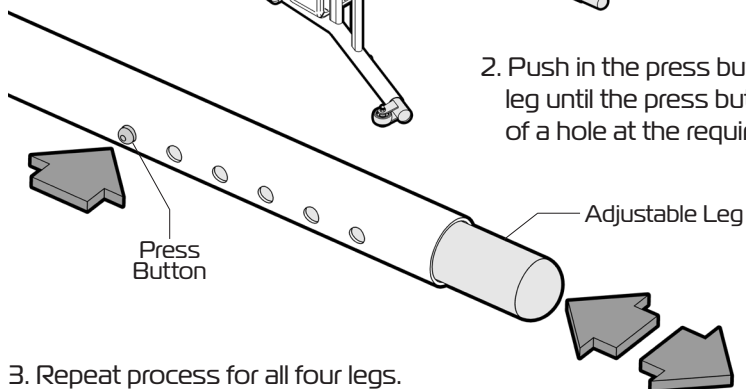

3. Repeat process for all four legs.

4. Carefully place the Katie Chair back on its legs and check to see if the seat is at the required height setting.



Adjust legs are at the same height setting if a flat seat is required.
If an reclining seat is required set both the rear legs lower.

6.

SET UP ADJUST SEAT DEPTH

Please follow these instructions to adjust the seat to the correct depth to suit the user.

Adjust the Katie Plus Chair so the user can lean comfortably against the backrest and have the back of the knee past the front of the seat.

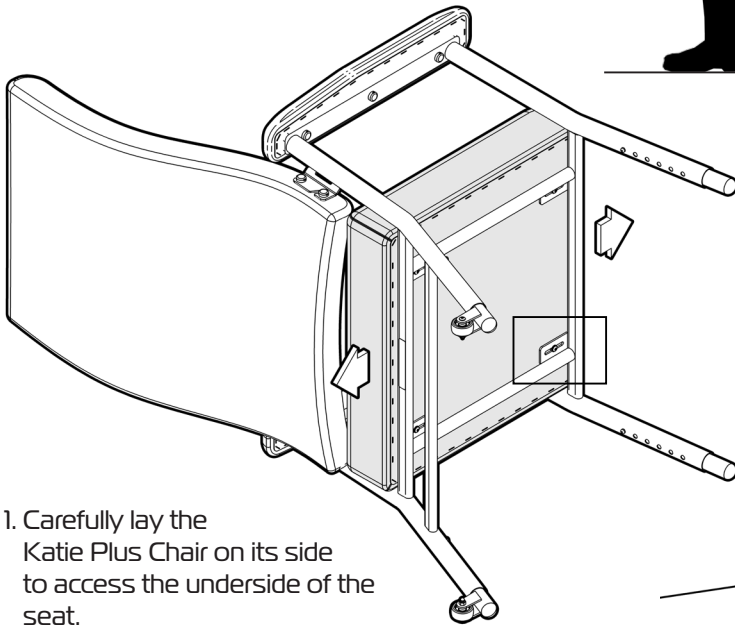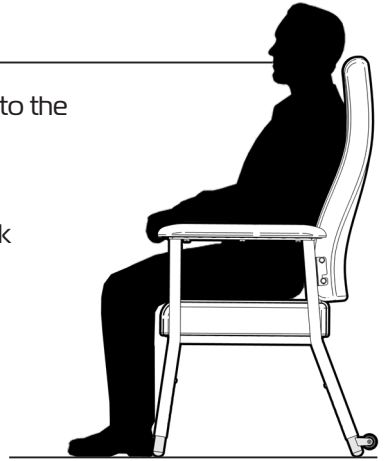

1. Carefully lay the Katie Plus Chair on its side to access the underside of the seat.


2. Using the allen key provide undo all four fixings that hold the padded seat on the frame.

3. Slide the seat to the required position.

4. Tighten all four fixing to hold seat in position.

5. Carefully place the Katie Plus Chair back on its legs and check to see if the seat is at the required depth setting.

7.

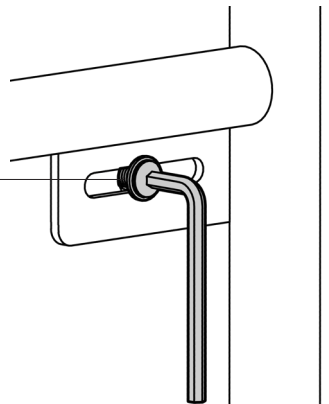


USAGE GUIDELINES

Please follow these guidelines to get the optimal benefit when using the Katie Plus Chair.


Failure to follow these guidelines may result in damage to the chair and put the user at risk.

200kg Maximum User Weight.


Always adjust the chair to the required height setting to suit the user (Refer to page 6.)

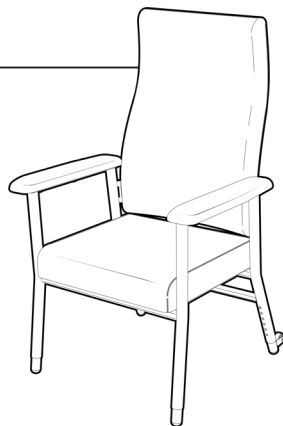

Ensure **ALL FOUR LEGS** remain on ground at all times when chair is occupied.

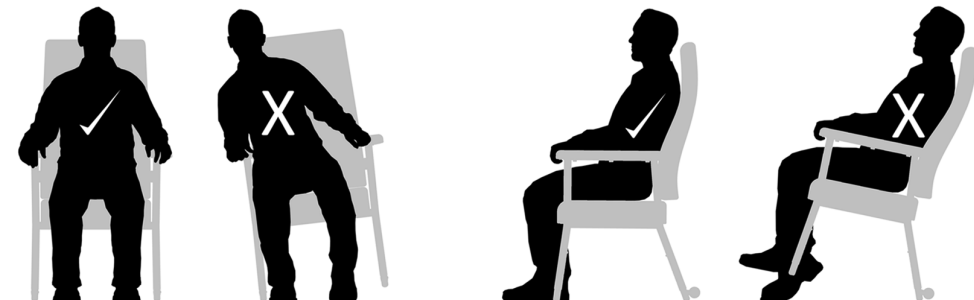

Do not lean too far to the side or backward causing legs to leave the ground.

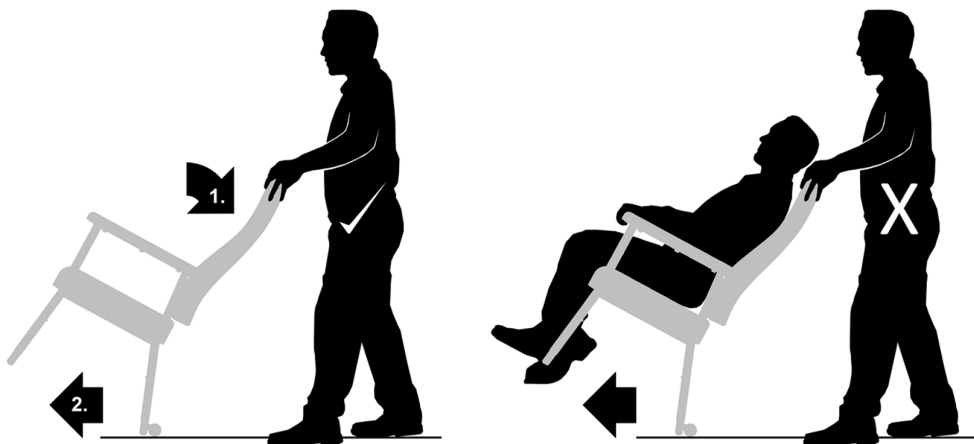

To move the chair, rotate back until the mobility wheels engage with the ground. NEVER move the chair when occupied.

8.

ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT CARE

INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the Katie Plus Chair periodically for signs of damage or wear prior to use. Please check all aspects of the chair paying particular attention to the following areas:

Metalwork – Ensure the frame is not damaged or distorted – no visible cracks.

Legs - Ensure all four legs are not damaged or distorted.

Fasteners – Check all bolts are done up tight / leg press buttons are engaged

Upholstery – Ensure all coverings are not ripped or torn.

Overall check that no components are loose or worn.



Withdraw the Katie Plus Chair immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.


CLEANING

Clean the Katie Plus Chair using disinfectant wipes.

We do not recommend cleaning the product with soap and water.

Do not use any abrasive cleaners or abrasive cleaning pads as they could damage the toilet surround.

If you have any questions about the cleaning of the product please contact your supplier or REDGUM brand direct.

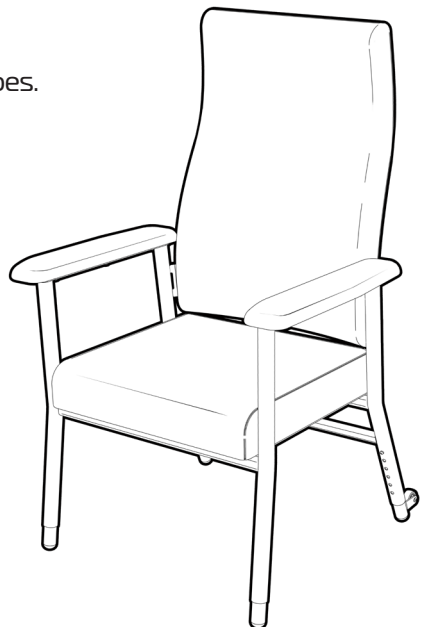

WARRANTY

REDGUM Brand:

Katie Plus High Back Chair (RG631) / Katie Plus Low Back Chair (RG636)
is covered by a 12 month warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if the product needs to be returned to the place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser.
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.


Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the Katie Plus Chair in any way or fit any parts or accessories that are not specified as an option. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the Katie Plus Chair please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

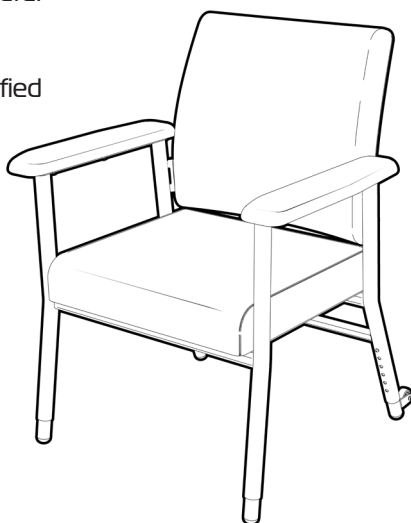


KATIE CHAIR OPTIONS

REDGUM Brand offer a range of options to accompany the Katie Plus Chair:

KATIE LEG REST

- Can be used as a legrest or stool
- Adjustable height and angle
- Padded upholstery to match Katie Plus Chair

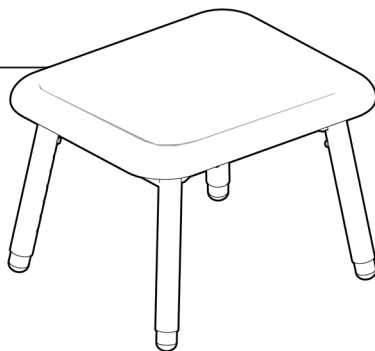

Available in:

BLUE - RG63LRBL

GREY - RG63LRGR

IVORY - RG63LRIV

SAGE GREEN - RG63LRSG

KATIE CHAIR RUBBER LEG TIPS

RG63RLT


- Can be fitted onto Katie Chair or Leg Rests to improve grip on hard flooring
- Durable non slip grey rubber leg tip
- Ribbed grip pattern on underside
- Easy to fit and remove




KATIE SIDE FOLDING TABLE

RG63SFT

- Handy side table
- Can be fitted to left or right hand side
- Folds down when not required

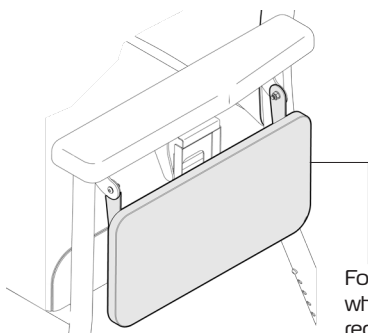

Folds down
when not
required

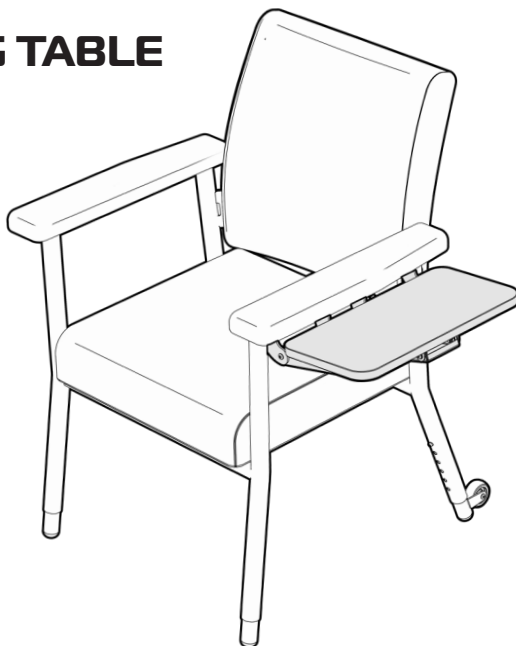


REDGUM BRAND RANGE

REDGUM Brand offer a comprehensive range of products including:

SEAT WALKERS
WALKING FRAMES
WALKING STICKS
QUAD CANES
CRUTCHES
REACHERS
WHEELCHAIRS
SHOWER CHAIRS
SHOWER STOOLS
TOILET SEAT RAISERS
MOBILE SHOWER COMMODOES
GRAB RAILS
DAY CHAIRS
LIFT CHAIRS
CUSHIONS
ELECTRIC BEDS
BED RAILS
SELF HELP POLES
RAMPS

**Primo
Seat Walker**
RG4212ADJ

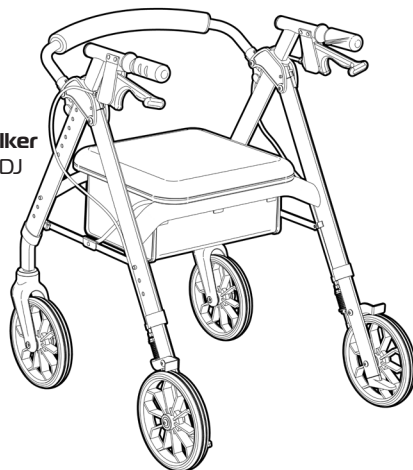

**Space Saver
Shower Chair**
RG555H



REDGUM Brand stock a large range of spare parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:

T: **+61 8 9248 4180** E: **sales@for-de.com.au**

DISTRIBUTOR:

