

REDGUMbrand
ASSISTED LIVING PRODUCTS

AUSTRALIAN
Owned
Operated




JASPER

SELF PROPEL WHEELCHAIR

RG360V

Edition 1

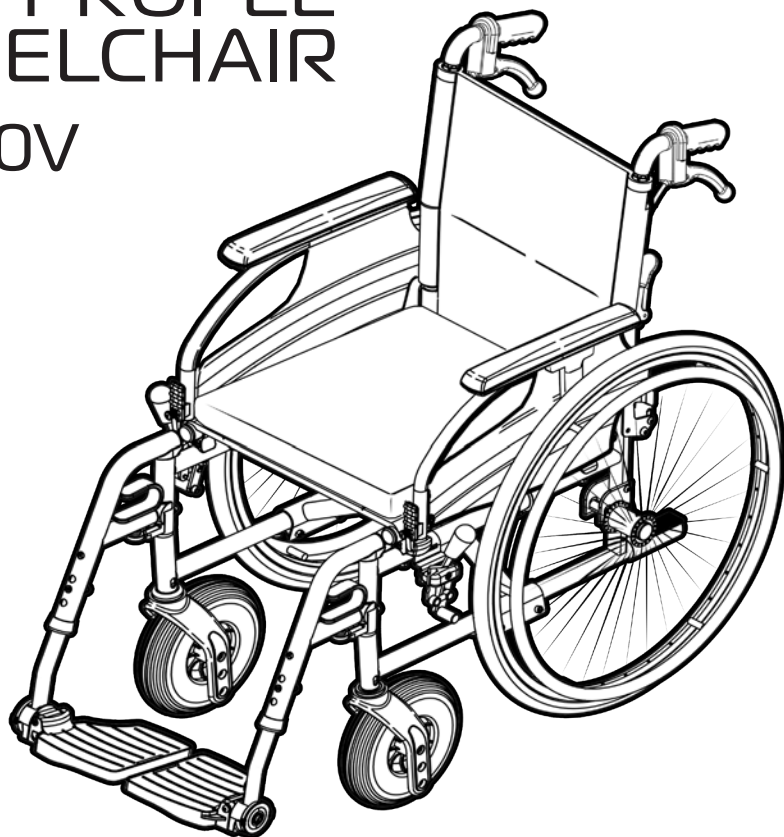


USER MANUAL

Please read and keep these instructions

ASK FOR THE BRAND

redgumbrand.com.au

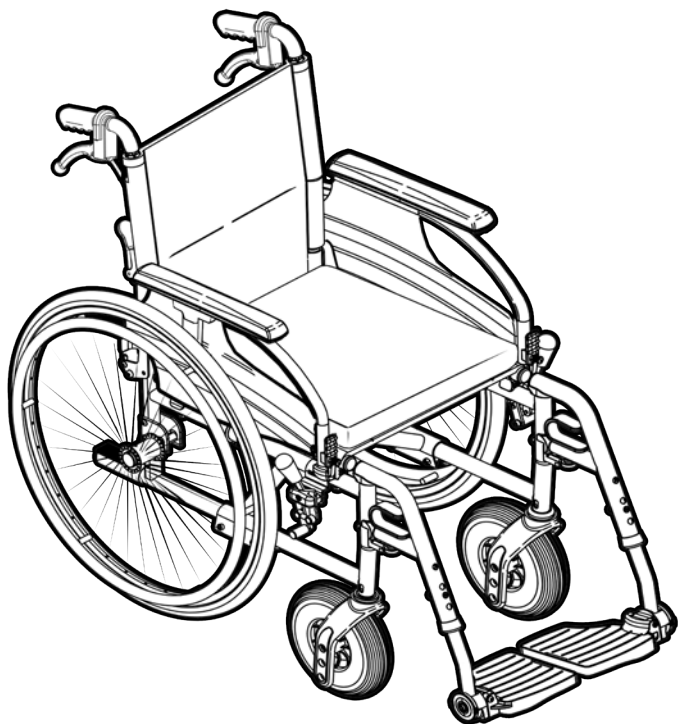

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the JASPER Self Propel Wheelchair user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright REDGUM Brand

FOR-DE
GROUP PTY LTD


REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION**
2. **PRODUCT OVERVIEW**
3. **INSPECT SUPPLIED PARTS**
4. **WHEELCHAIR UNFOLDING / FOLDING**
5. **BACKREST UNFOLDING / FOLDING**
6. **SWING AWAY FOOTRESTS**
7. **ARMRESTS**
8. **HAND BRAKES**
9. **QUICK RELEASE REAR WHEELS**
10. **USAGE GUIDELINES**
11. **PRODUCT CARE**
12. **WARRANTY**

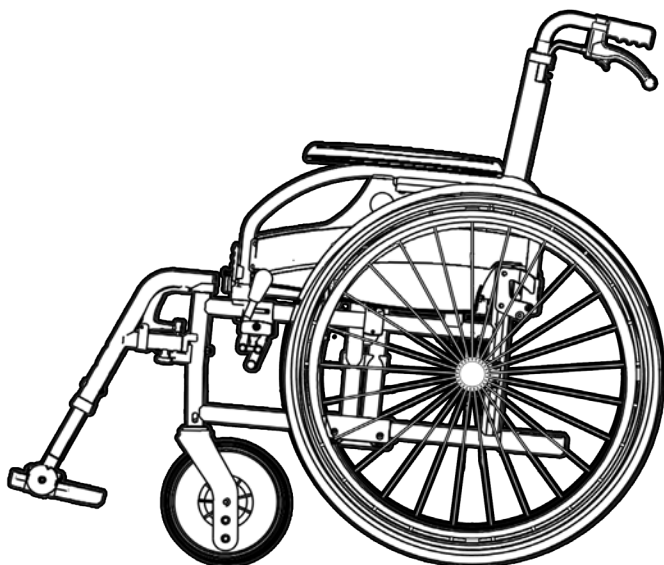

INTRODUCTION

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the JASPER Self Propel Wheelchair please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090


INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your JASPER Self Propel Wheelchair for its intended purpose which is as a short term usage mobile chair for a single user that is operated by the occupant or a attendant / carer. Refer to usage guideline page for information on how to use the wheelchair in a safe manner.

IMPORTANT NOTE

This equipment **MUST** be assembled for use and operated by a competent person.

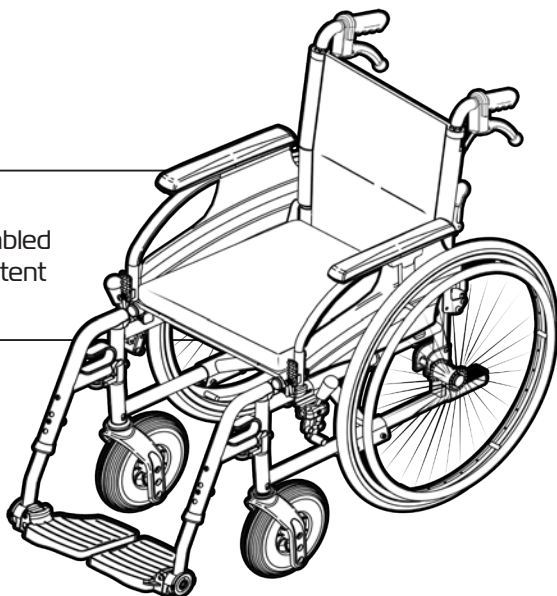

1.


ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

- Lightweight wheelchair (Aluminium frame)
- Folding frame and fold down backrest
- Attendant operated hand brakes on push handles
- Occupant operated hand brakes
- Removable / height and angle adjustable
fold up footplates with heel straps
- Adjustable height / swing up / removable armrests
- Flat free front swivel castors
- Flat free self propelling quick release rear wheels

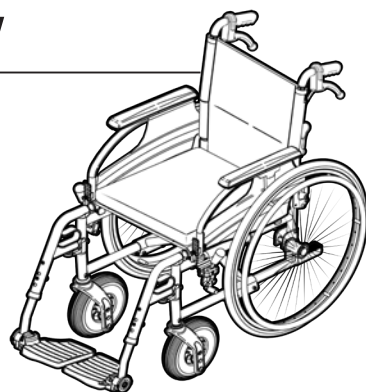

SPECIFICATIONS (All dimensions shown are nominal and are in mm)

	RG360V42	RG360V46	RG360V50	RG360V52
Overall length	1010	1010	1010	1010
Overall width	620	660	700	720
Overall width folded	300	300	300	300
Effective seat width	420	460	500	520
Seat depth	430	430	430	430
Seat height	470	470	530	530
Backrest height	420	420	420	420
Armrest height range	220 - 280	220 - 280	220 - 280	220 - 280
Footrest distance from seat range	380 - 450	380 - 450	380 - 450	380 - 450
Front wheels	Ø200 (8")	Ø200 (8")	Ø200 (8")	Ø200 (8")
Rear Wheels	Ø560 (22")	Ø560 (22")	Ø610 (24")	Ø610 (24")
Product Weight:	17kg	17.5kg	18kg	18.5kg
Max. User Weight:	130kg	130kg	130kg	130kg

Please contact REDGUM Brand if you would like any further information on the JASPER Self Propelling Wheelchair.

INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:


We recommend that you visually inspect all aspects of the wheelchair prior to use.

IMPORTANT NOTE:


If you see any damage or suspect a fault,

DO NOT USE THE PRODUCT

and contact your supplier for support.

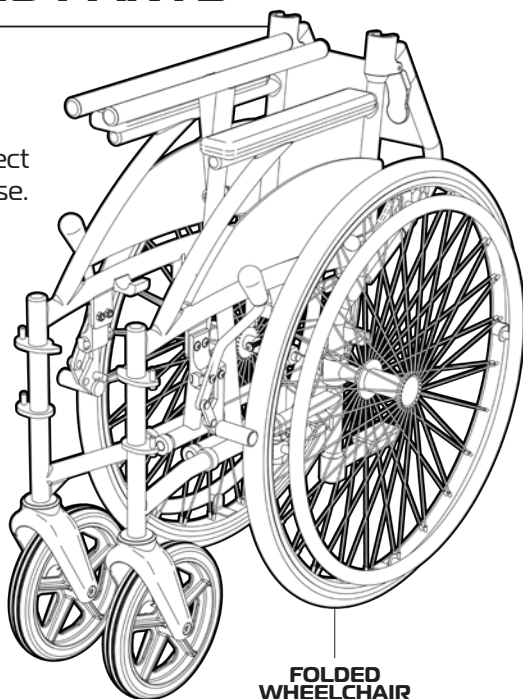


SEAT CUSHION

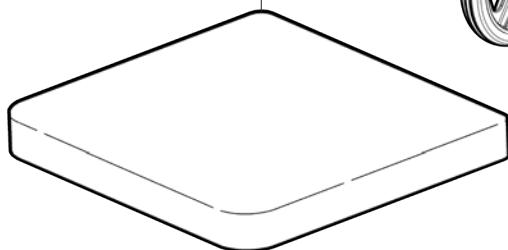


TOOLS SUPPLIED

(Not shown to scale)

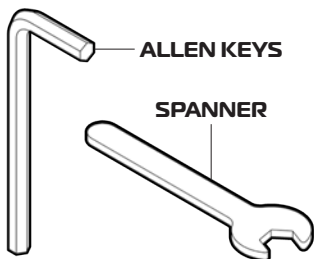

SWING
AWAY
FOOTREST
1 LEFT / 1 RIGHT

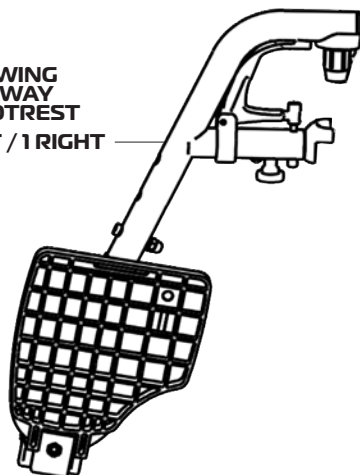

3.

ASK FOR THE BRAND

redgumbrand.com.au

WHEELCHAIR UNFOLDING / FOLDING

The JASPER Self Propel Wheelchair requires unfolding prior to use.

Please follow these steps:


1. Pull **OUT** on both of the armrests at the same time to open out the wheelchair.
2. Push down on the outside edges of seat to ensure wheelchair has been fully unfolded.

Important note:


DO NOT place fingers underneath seat when unfolding wheelchair!

To fold wheelchair:

- Pull **UP** on seat.
- (Ensure footplates are up prior to folding)

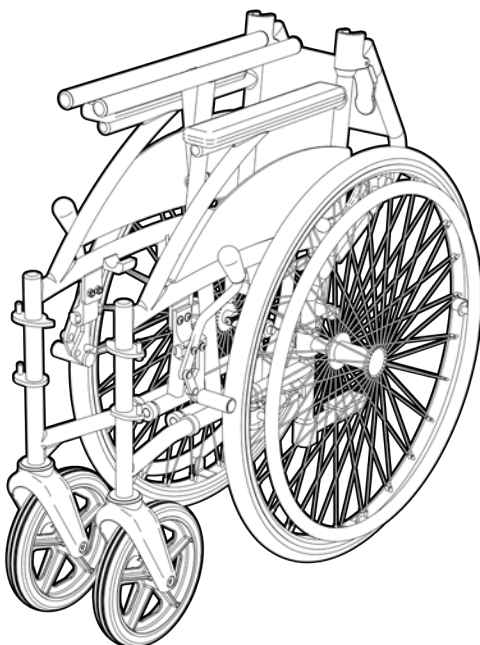


**PULL OUT ON
ARMRESTS**

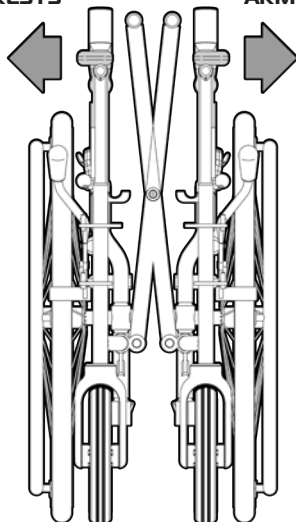

**PULL OUT ON
ARMRESTS**

**PUSH DOWN
ON SIDES OF SEAT**

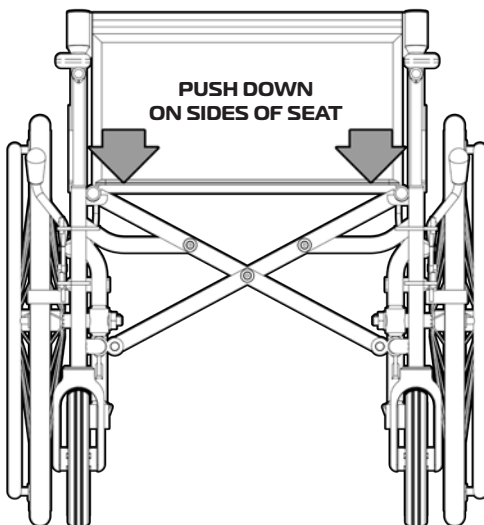


4.

BACKREST UNFOLDING / FOLDING


The JASPER Self Propel Wheelchair comes with the folded backrest and requires unfolding prior to use. Please follow these steps:

1. Place your hand on the armrest.
2. Rotate up each backrest handle until it locks into position.

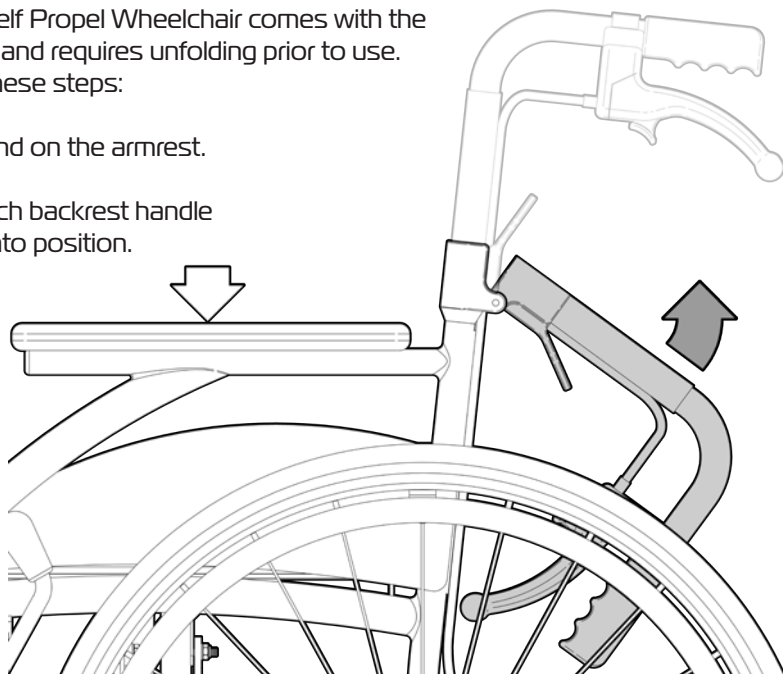

Important note:

Check to ensure both backrest handles have locked into position prior to use. (Loud 'CLICK' should be heard when they have fully located in place)

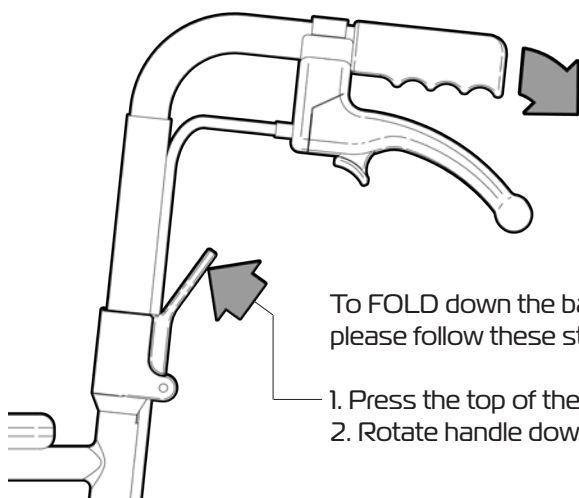

To FOLD down the backrest handles please follow these steps:



1. Press the top of the lever in towards the frame.
2. Rotate handle down.

5.

SWING AWAY FOOTRESTS

The JASPER Self Propel Wheelchair comes with removable swing up footrests. Please follow these steps to fit and remove swing away footrests:

1. Position the swing away at an angle to the wheelchair as shown.
2. Line up the large pin on the end of swing away with the open tube on frame.
3. Gently lower swing away onto frame.
4. Rotate swing away in towards wheelchair until it locks into position.

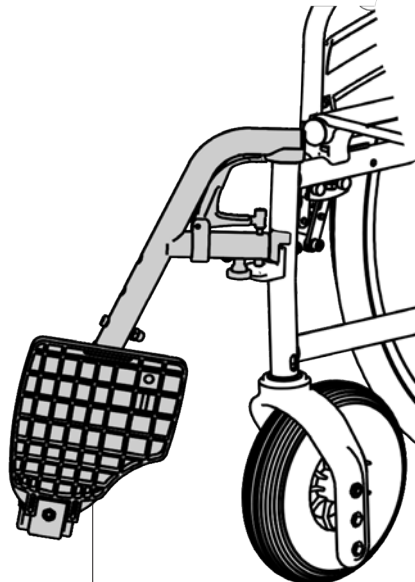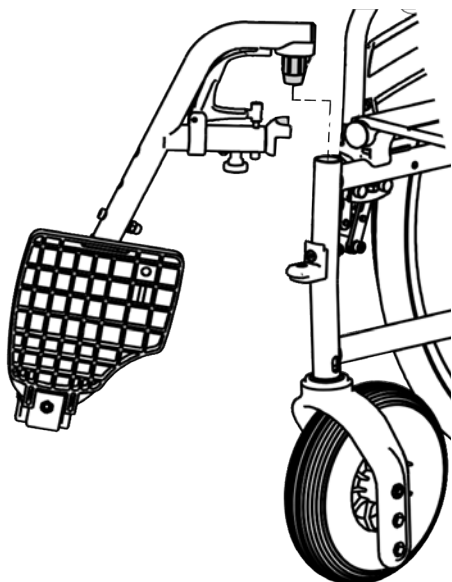


Lower footplate and adjust height and angle as required.

To rotate away or remove the swing away footrest:

Pull up on the release lever and rotate the swing away footrest outwards.

Once the swing away is at an angle away from the wheelchair - pull up to remove it from the wheelchair if required.

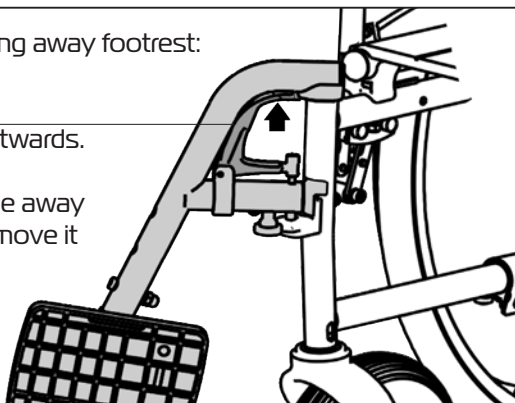

6.


ARMRESTS

The JASPER Self Propel Wheelchair comes with armrests that are:

- Adjustable in height
- Swing up
- Removable


To adjust the height of the armrest:

- Undo knob.
- Pull knob out and move armrest to the required height setting.
- Release knob and move armrest till it locks in place.
- Tighten knob.

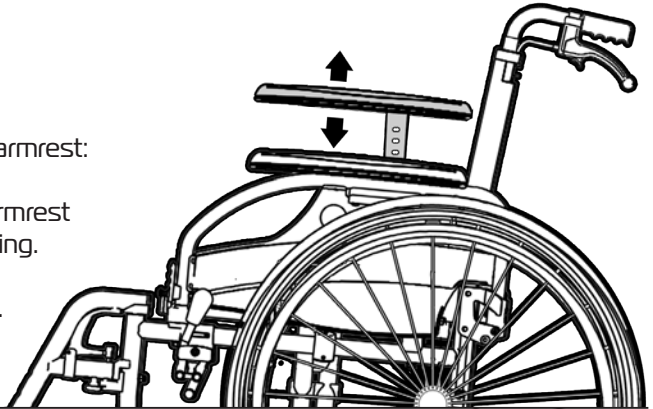

To swing up the armrest:

- Press the lever in at the top.
- Rotate the arm toward the back of the wheelchair

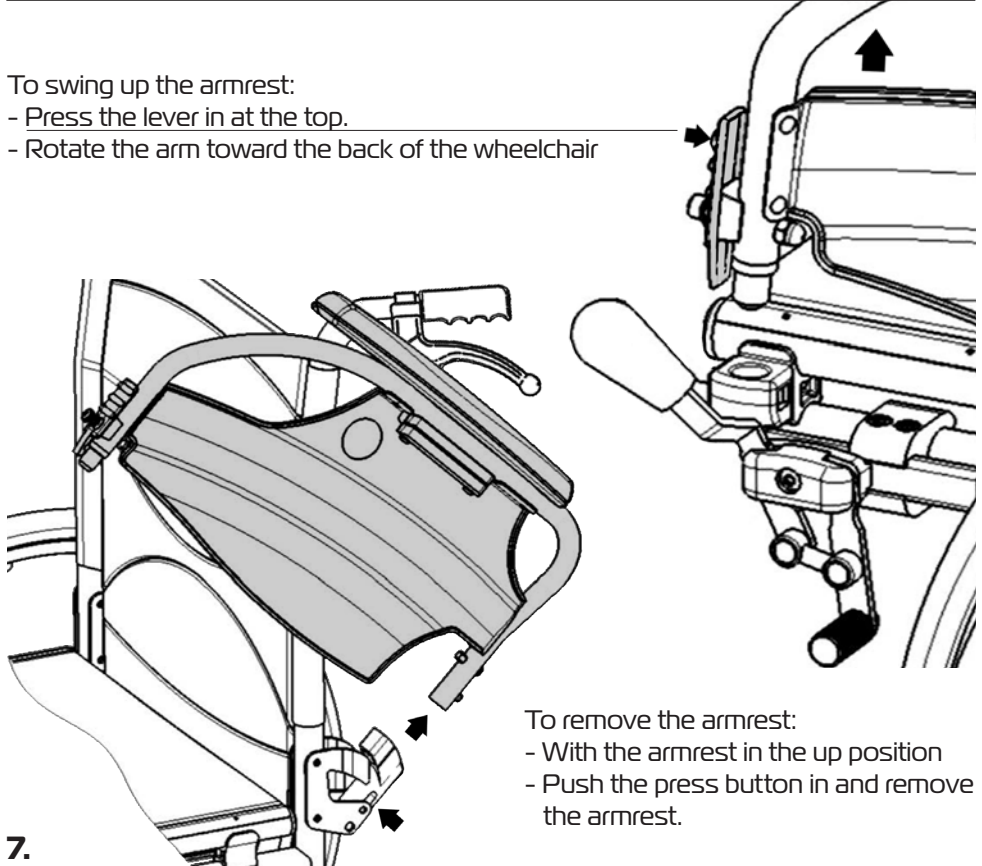

To remove the armrest:

- With the armrest in the up position
- Push the press button in and remove the armrest.

7.

ASK FOR THE BRAND

redgumbrand.com.au


HAND BRAKES

The JASPER Self Propel Wheelchair comes with attendant operated hand brakes located on the push handles.

To **ENGAGE** brake:

Pull up on lever.

Brake will disengage when lever when released.

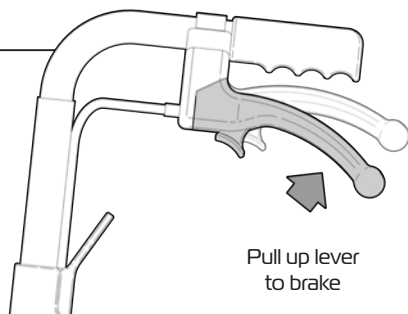

Pull up lever
to brake

To **ENGAGE PARK** brake:


Pull up on lever and then push small lever forward.

Brake will remain engaged when lever is released.

To **DIS-ENGAGE** PARK brake:

Pull up on brake lever.

Park brake will disengage automatically.

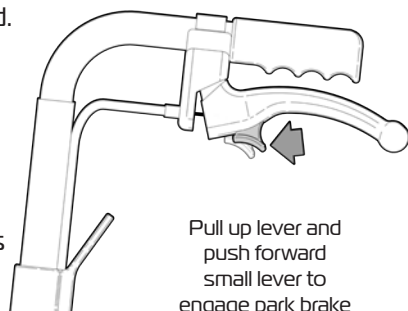

Pull up lever and
push forward
small lever to
engage park brake

The JASPER Self Propel Wheelchair also comes with occupant operated hand brakes located on each side of wheelchair in front of rear wheels.

To **ENGAGE PARK** brake:



Push lever forward.

Brake will remain engaged when lever is released.

To **DIS-ENGAGE** PARK brake:

Pull lever back.

Park brake will disengage when lever when released.

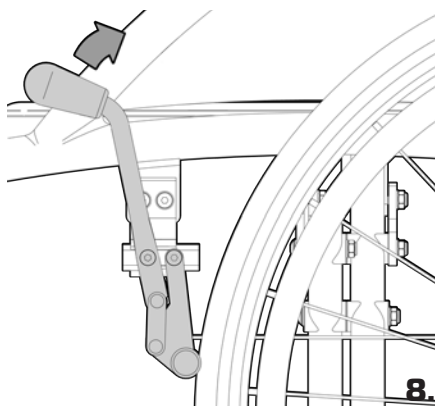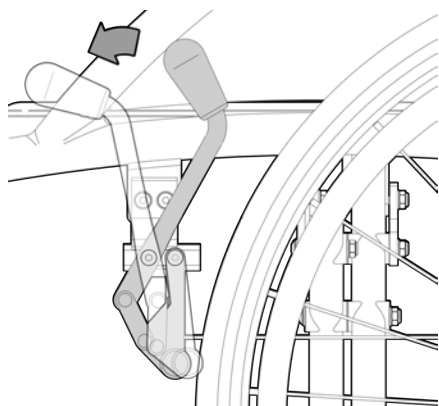

Note:


It is recommended that the park brake is engaged when wheelchair is in use and is stationary.

QUICK RELEASE REAR WHEELS

The JASPER Self Propel Wheelchair comes with quick release self propel rear wheels.

To remove the rear wheels please follow the following steps:

1. Using one hand - slightly tilt the wheelchair so the wheel being removed is off the ground.
2. With the other hand - depress the button in the centre of the wheel and pull the wheel out away from the wheelchair.

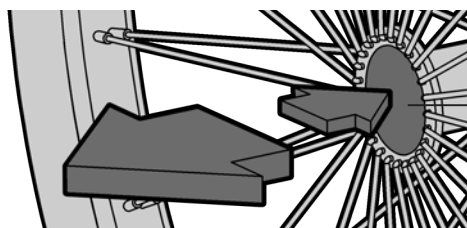

Button

IMPORTANT NOTE:

The wheelchair will be unstable with one wheel removed and will fall towards you.

Repeat process to remove the other wheel.

To refit wheels



Depress button and slide wheel axle into socket on wheelchair frame.

Release button and check to ensure wheel is securely fixed in place prior to using wheelchair.

9.

ASK FOR THE BRAND

redgumbrand.com.au

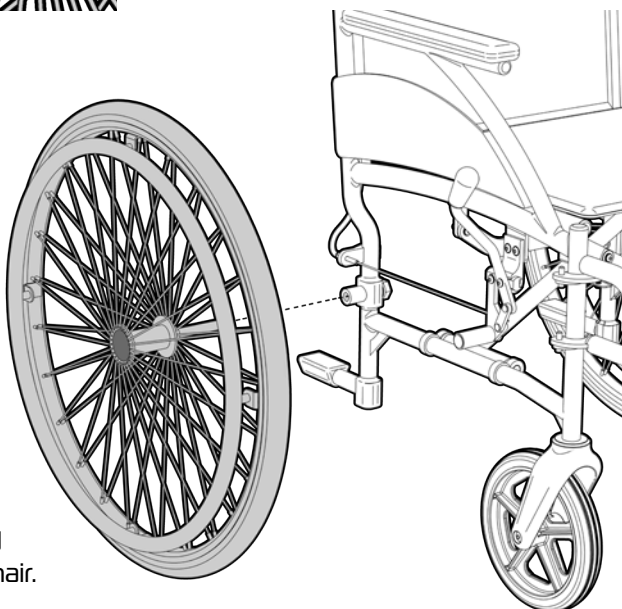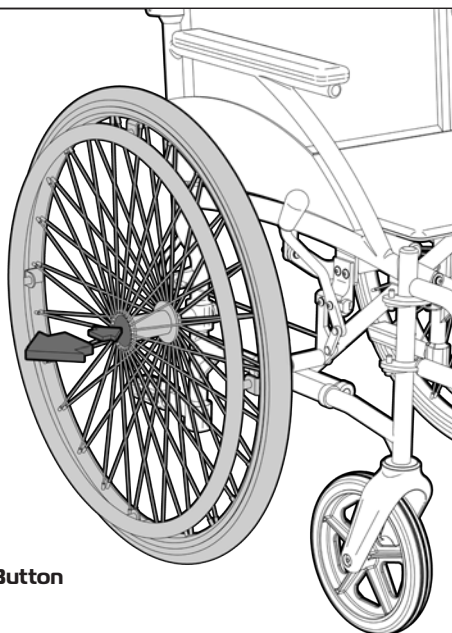


USAGE GUIDELINES

Please follow these guidelines to get the optimal benefit from the JASPER Self Propel Wheelchair. Failure to follow these guidelines may result in damage to the wheelchair and put the user at risk.

The JASPER Self Propel Wheelchair is designed to take 130kg Maximum Load (evenly distributed). Do not overload the wheelchair!


The footplates are not designed to be load bearing. Do not stand on the footplates at any time.

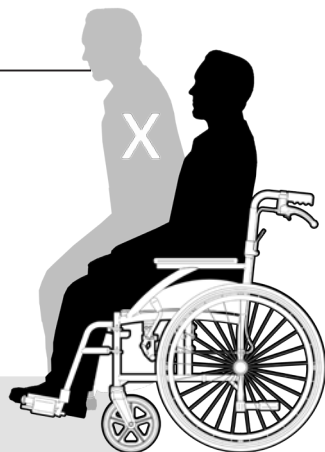

Always engage the park brakes when user is getting on or off the wheelchair.

Go up or down inclines directly. Do not try to cross on an angle as this will cause instability and wheelchair may tip over.


Tipper bars have been incorporated to assist when manoeuvring the wheelchair over a step. The attendant can raise the front wheel of the ground by simultaneously pushing down on the push handles and pushing down on the tipper bars with their foot.

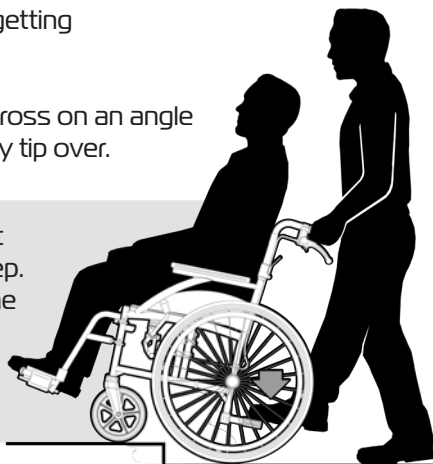

The JASPER Self Propel Wheelchair has been crash tested and APPROVED for use as a seat in a vehicle when using a four point strap tie down system and a 3 point occupant restraint system. Follow the manufacturers fitting instructions of the restraint systems being used.

Ensure the wheelchair and occupant are securely restrained prior to travel.

If you have any issues or questions about restraining the wheelchair please contact your supplier or REDGUM Brand direct.

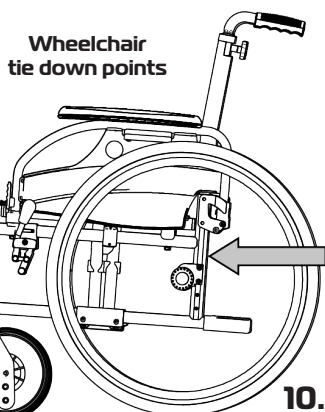

10.

ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT CARE

INSPECTIONS

We suggest that the JASPER Self Propel Wheelchair is visually checked periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the wheelchair immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING

Clean the JASPER Self Propel Wheelchair using disinfectant wipes. We do not recommend cleaning the product (except upholstered parts) with soap and water.


Abrasive cleaners or abrasive cleaning pads could damage the wheelchair and should not be used.

Seat cushion cover and calf strap can be removed and machined washed if required:
Cold water wash - gentle cycle
with mild detergent.

Tumble dry on lowest temperature.

Ensure all parts are dry after cleaning prior to using wheelchair.

If you have any issues or questions about cleaning the wheelchair please contact your supplier or REDGUM Brand direct.

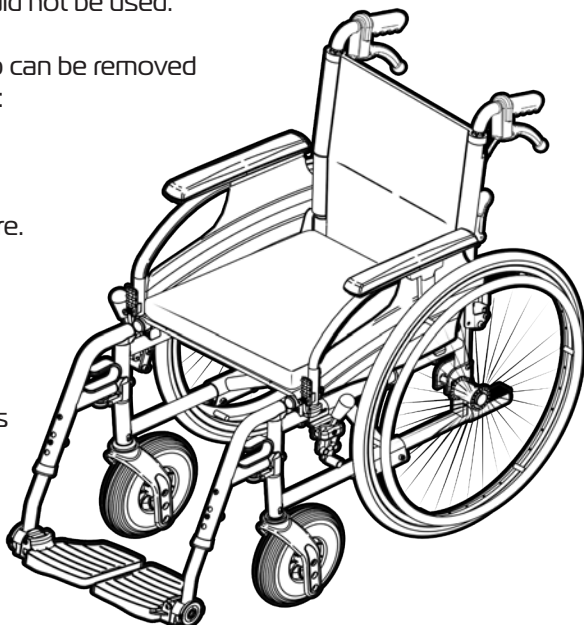

WARRANTY

The REDGUM Brand JASPER Self Propel Wheelchair (RG360V) is covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.


Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the JASPER Self Propel Wheelchair in any way or fit any parts or accessories that are not specified as an option for the product. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the JASPER Self Propel Wheelchair please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

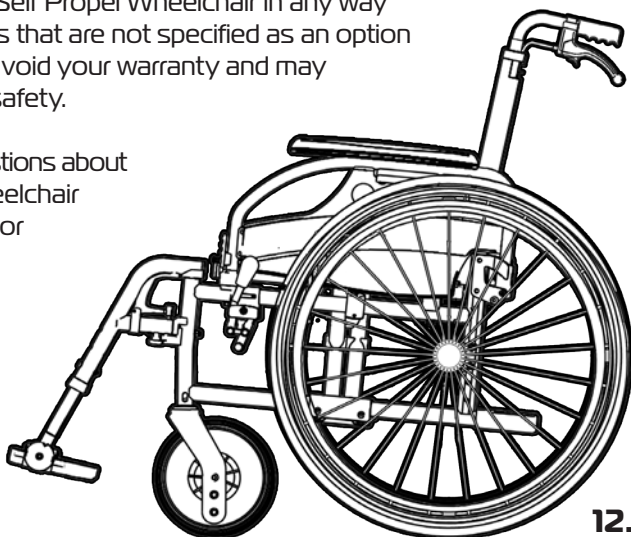

12.



DISTRIBUTOR:

