


**HARD CANOPY** 

SE-HC

Edition 2

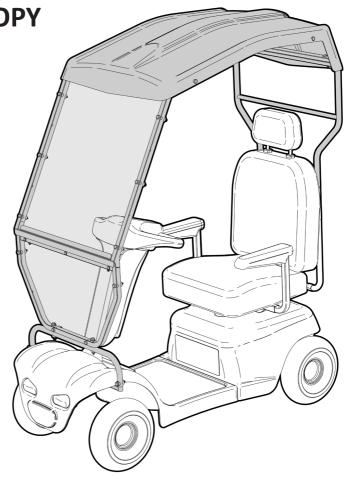

# **USER MANUAL**

Please read and keep these instructions





**SCOOTER XTRAS** owns the copyright to the Hard Canopy user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of Scooter Xtras.

**SCOOTER XTRAS** strives to provide the highest quality products possible. Products are subject to change without notice.

### © Copyright **SCOOTER XTRAS** 2021



### **CONTENTS**

- 1. INTRODUCTION INTENDED USE
- 2. PRODUCT OVERVIEW FEATURES / SPECIFICATIONS
- 3. ASSEMBLY UNPACK and INSPECT SUPPLIED PARTS
- 4. **ASSEMBLY** BAG / WALKER CARRIER LOCATION
- 5. **ASSEMBLY** MOUNT KIT LOCATION
- 6. **ASSEMBLY** MOUNT KIT 1 FIXING
- 7. **ASSEMBLY** CANOPY PRE-ASSEMBLY (UPPER WINDSCREEN)
- 8. **ASSEMBLY** CANOPY PRE-ASSEMBLY (LOWER WINDSCREEN)
- 9. **ASSEMBLY** CANOPY PRE-ASSEMBLY (MOUNT KIT CROSS TUBE)
- 10. **ASSEMBLY** CANOPY PRE-ASSEMBLY (REAR FRAME)
- 11. ASSEMBLY CANOPY ON SCOOTER
- 12. USAGE GUIDELINES DO's + DON'TS
- 13. **PRODUCT CARE** INSPECTIONS / CLEANING
- 14. WARRANTY
- 15. SCOOTER XTRAS HARD CANOPY SIDE COVERS / ACCESSORIES
- **16. NOTES**

### INTRODUCTION

Thank you for purchasing this quality **SCOOTER XTRAS** product. If you have any issues or questions about the **Hard Canopy** please contact your supplier or Scooter Xtras direct:

T: +61 8 9248 4180 F: +61 8 9248 9452 E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

#### **INTENDED USE**

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your **Hard Canopy** for the purpose for which it is intended, which is as a wind and sun shelter solution - it is not designed to be a rain shelter solution as mobility scooters should not be operated in the rain.

#### **IMPORTANT NOTE**

This equipment MUST be assembled by competent people. We recommend that 2 people install the hard canopy onto the mobility scooter.

Tools required (not supplied) for assembly:

- 4mm Allen key
- 10mm Spanner
- Cordless drill with phillips head driver
- Rule and marking pen

Using a mobility scooter fitted with a Hard Canopy in strong winds and adverse weather can seriously compromise your safety and may damage the mobility scooter beyond repair.



## **PRODUCT OVERVIEW**

#### **FEATURES**

- Provides excellent wind + sun protection
- Clear polycarbonate windscreens
- Lightweight metal framework
- Used in conjunction with SE-BC: Bag Carrier
- Easily removable for servicing
- Retro fittable to certain Shoprider mobility scooters

#### **Optional Extras**

- Retro fittable Clear PVC side + rear covers



### **SPECIFICATIONS** (All dimensions shown are nominal)

Overall Dimensions: Length: 1300mm Width: 720mm Height: 1400mm

**Product Weight:** 

8kg

#### **IMPORTANT NOTE:**

Hard Canopy can only be fitted to the following Mobility Scooters:

SHOPRIDER ROCKY 4 (Mount Kit 1 required)

SHOPRIDER ROCKY 6 (Mount Kit 1 required)

SHOPRIDER ALLROUNDER (Mount Kit 2 required)


SHOPRIDER 889SL (Mount Kit 2 required)

**SE-BC:** Bag + Walking Sticker Carrier or **SE-WWC:** Wheelie Walker Carrier is required to fit Hard Canopy to any mobility scooter.

Please contact **SCOOTER XTRAS** if you would like any further information on your **Hard Canopy**.

## **ASSEMBLY**- UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

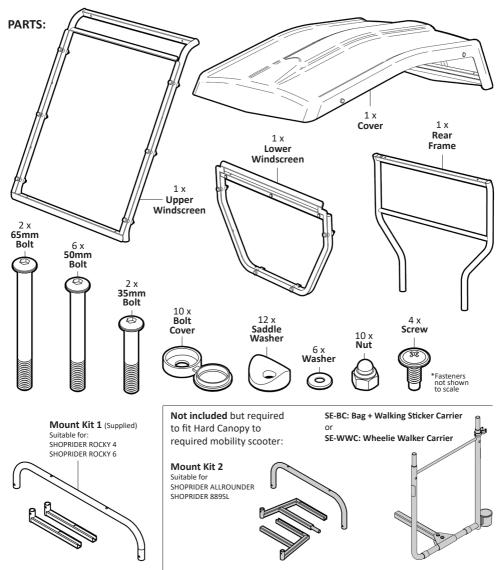


### **IMPORTANT NOTE**

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

## **ASSEMBLY**- BAG / WALKER CARRIER LOCATION

Before the Hard Canopy can be fitted to a mobility scooter...

SE-BC: Bag + Walking Stick Carrier or SE-WWC: Wheelie Walker Carrier must first be fitted. Assemble the carrier as required and then fit to scooter.

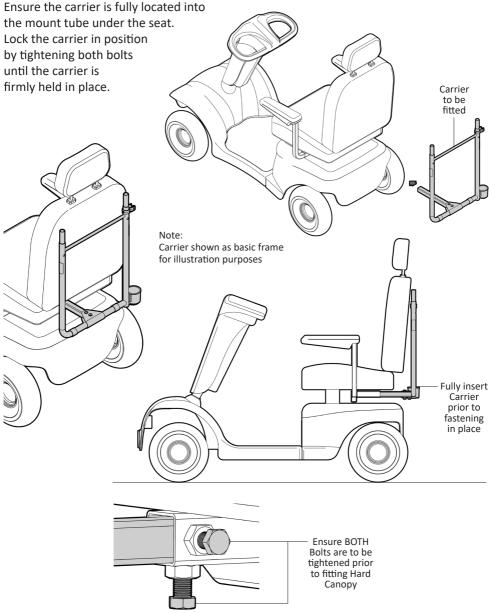


### **ASSEMBLY**- MOUNT KIT LOCATION

Before the Hard Canopy can be fitted to a mobility scooter, a front mount kit needs to be fitted. Please ensure you have been supplied with the correct mount kit for your mobility scooter model:


SHOPRIDER ROCKY 4 (Mount Kit 1 required)
SHOPRIDER ROCKY 6 (Mount Kit 1 required)
SHOPRIDER ALLROUNDER (Mount Kit 2 required)
SHOPRIDER 889SL (Mount Kit 2 required)

Insert the outrigger into the mount tube located above the front tyres of the scooter and underneath the cover. The uprights on ends of mounts to be positioned just off the cover - ready to accept the front mount tube.

#### **Mount Kit 1**

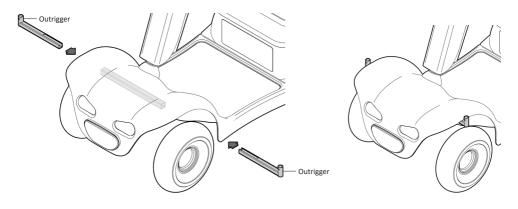

#### **Mount Kit 2**

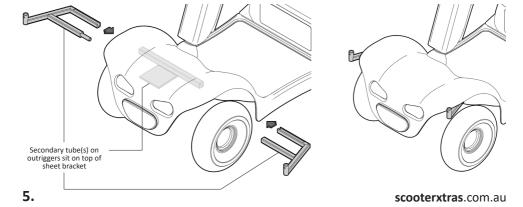


## **ASSEMBLY**- MOUNT KIT 1 FIXING


We recommend that the outriggers on Mount Kit 1 be screwed into place to create a stable mounting platform for the hard canopy. Follow these steps to correctly fit both outriggers to frame underneath the mobility scooter:




- 2. Locate the mount kit cross tube onto the outriggers.
- 3. Using the rule and pen mark screw positions as shown below.
- 4. Using a cordless drill with phillips head driver fix 4 x self tapping screws through frame and outriggers.




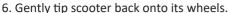

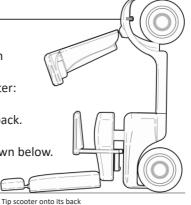


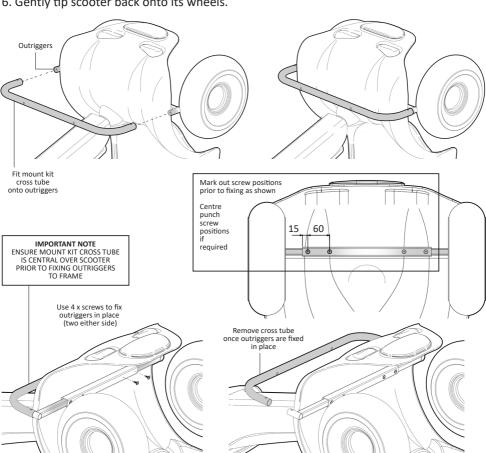


# **ASSEMBLY**- CANOPY PRE ASSEMBLY (UPPER WINDSCREEN)

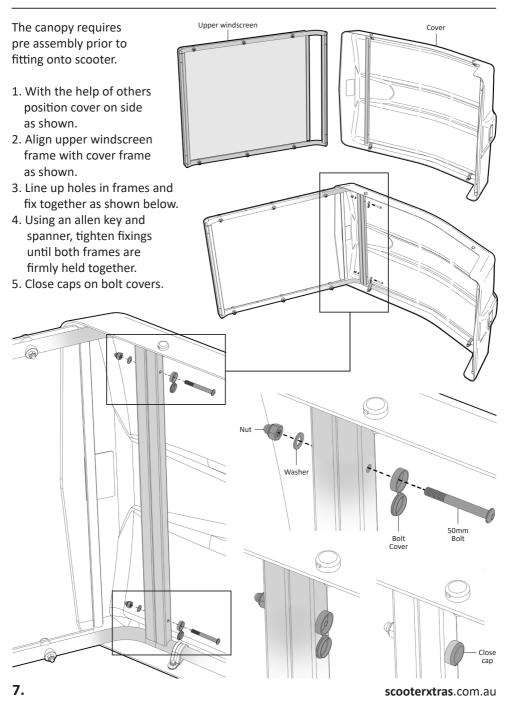


# **ASSEMBLY**- CANOPY PRE ASSEMBLY (LOWER WINDSCREEN)

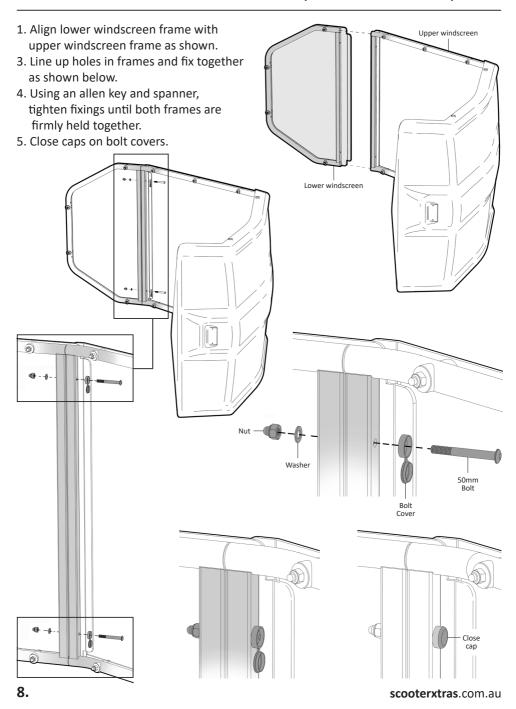


# **ASSEMBLY**- CANOPY PRE ASSEMBLY (MOUNT KIT CROSS TUBE)

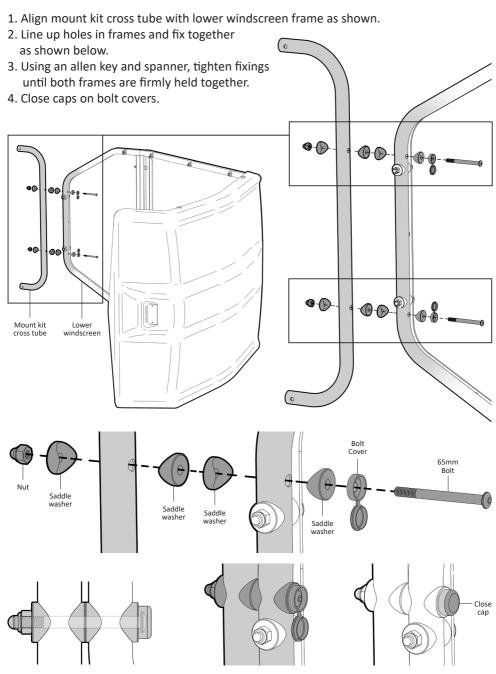

# **ASSEMBLY**- CANOPY PRE ASSEMBLY (REAR FRAME)

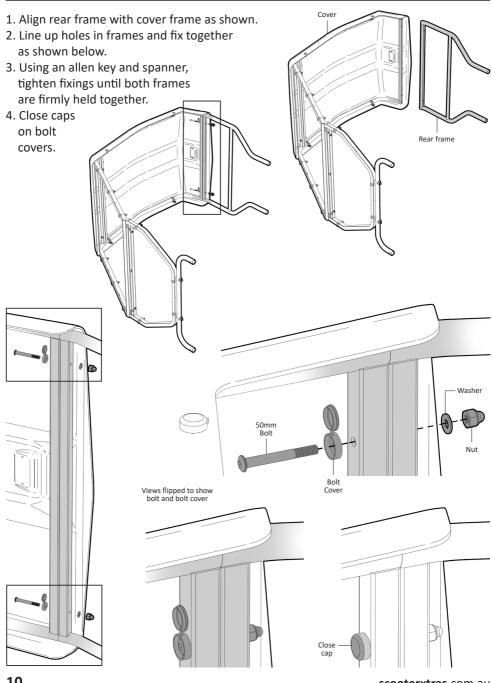

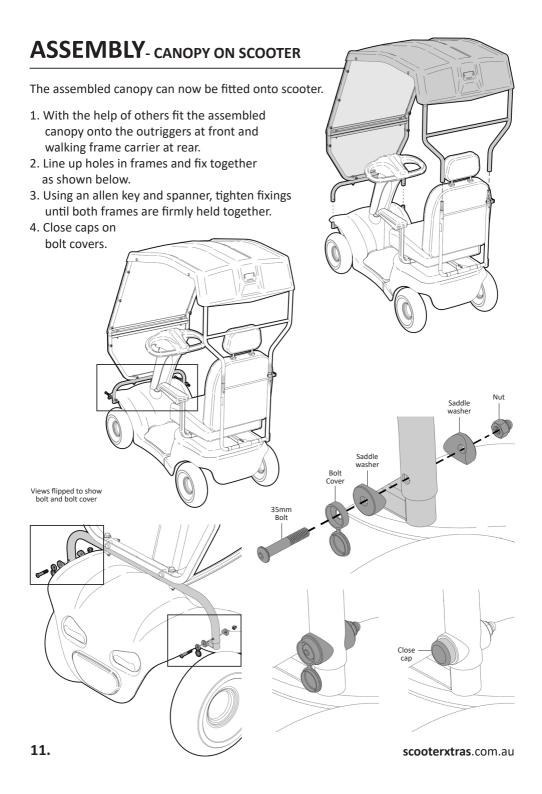

### **USAGE GUIDELINES** - DO's + DON'TS

Please follow these guidelines to get the optimal benefit from your Hard Canopy.

Failure to follow these guidelines may result in damage to the Hard Canopy, mobility scooter and put the user at risk.

Please ensure all the parts of the hard canopy have been fitted as instructed and all fasteners have been tightened sufficiently.

## **IMPORTANT NOTE!**

Using a mobility scooter fitted with a Hard Canopy in STRONG WINDS and adverse weather can seriously compromise your safety and may damage the mobility scooter beyond repair.

Only use your **Hard Canopy** for the purpose for which it is intended, which is as a wind and sun shelter solution.

The Hard Canopy is not designed to be a rain shelter solution as mobility scooters should not be operated in the rain.

The user should not use the Hard Canopy to assist them with getting on and off the mobility scooter. The Hard Canopy has not been designed to be used as a assistive support structure.



## **PRODUCT CARE** - INSPECTIONS / CLEANING

#### **INSPECTIONS**

We suggest that you check your Hard Canopy visually periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the Hard Canopy immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

#### **IMPORTANT NOTE**

Do not fit any parts or accessories to your Hard Canopy that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

#### **CLEANING**

Clean the Hard Canopy using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Hard Canopy and should not be used.



13.

### WARRANTY

Your Scooter Xtras **SE-HC: HARD CANOPY** is covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE**, **MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY**.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

If you have any issues or questions about your Hard Canopy please contact your supplier or SCOOTER XTRAS direct on:

T: +61 8 9248 4180 F: +61 8 9248 9452 E: sales@for-de.com.au A: 1 Business Way, Malaga Western Australia 6090

Please keep this
User Manual
in a safe place
for future reference.



## **SCOOTER XTRAS** - HARD CANOPY SIDE COVERS

Optional clear PVC side and rear covers can be retro fitted to the hard canopy. (Product Code: SE-HCSC)

### **FEATURES**

- Provides excellent shelter from the elements
- Clear PVC panels for good visiblity
- Zip access flaps
- Zip up side windows
- Rear cover (can be rolled up)
- Side Covers are easily removable if required



SCOOTER XTRAS also offer various accessories that can be fitted to the hard canopy.

SE-020: Bottle Holder SE-022: Phone Holder

For more details please contact your supplier or SCOOTER XTRAS.



# **NOTES**

DISTRIBUTOR:





**Proudly Australian Owned and Operated**