


VARI*air*

FLOTATION CUSHION

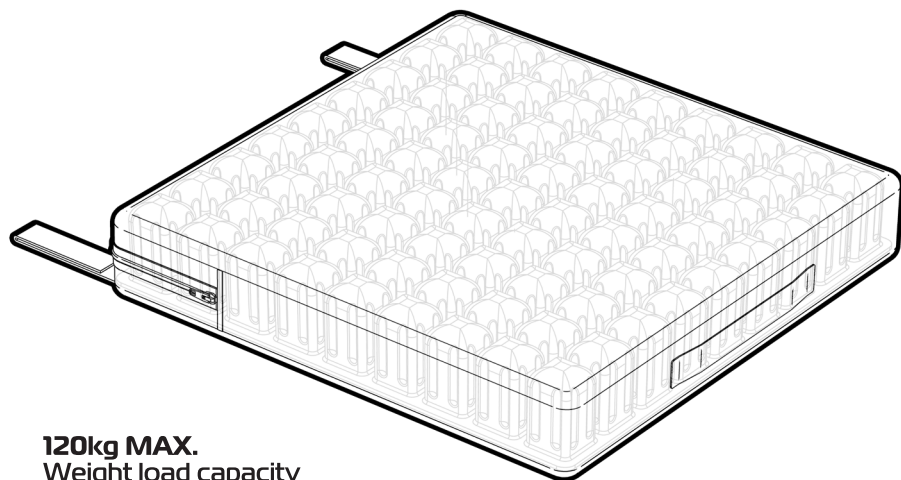

120kg MAX.
Weight load capacity

RG4044: 400mm W x 440mm D x 70mm H (8 x 9 air pockets)


RG4545: 450mm W x 450mm D x 70mm H (8 x 8 air pockets)

RG4848: 480mm W x 480mm D x 70mm H (8 x 8 air pockets)


INSPECT SUPPLIED PARTS

Check the product to ensure that you have been supplied with all the required parts.


Air Bladder
x1

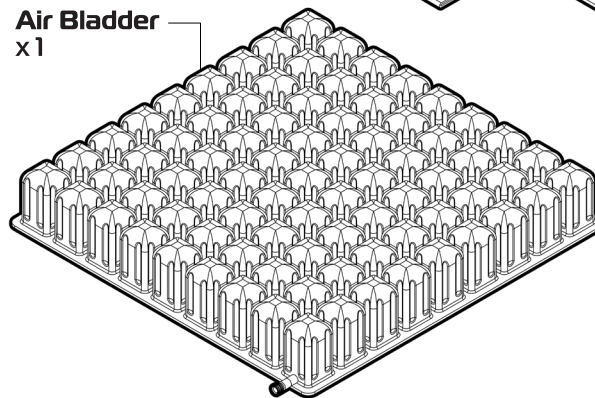

Cushion Cover
x1

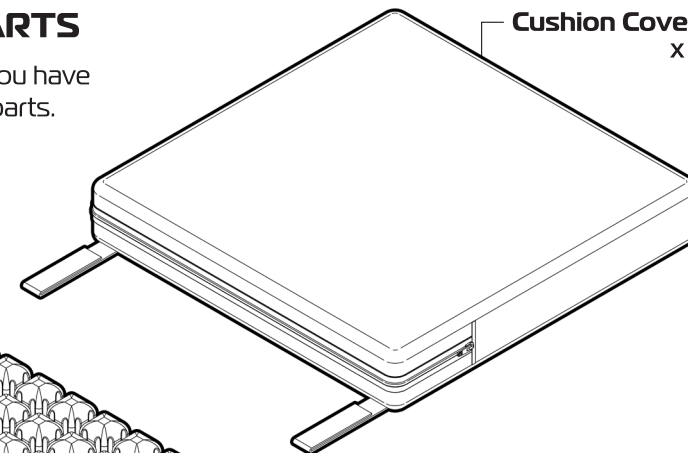


Hand Pump
x1

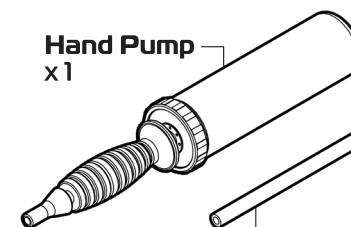

Pump Hose
x1

Repair kit
Glue tube x1
Patch x1

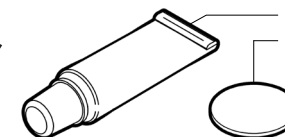

IMPORTANT NOTE:

If any parts are missing or you suspect a fault,
DO NOT USE THE PRODUCT and
contact your supplier for support.

Thank you for purchasing this quality **REDGUM Brand** product. If you have any issues or questions about the **VARI AIR** Flotation Cushion please contact your supplier or **RED-GUM Brand** direct on: T: **+61 8 9248 4180** E: **orders@for-de.com.au** A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the **VARI AIR** Flotation Cushion for the purpose for which it is intended, which is as a pressure relieving seating solution.


Please turn over for more instructions

ASK FOR THE BRAND

redgumbrand.com.au


VARI*air*

FLOTATION CUSHION

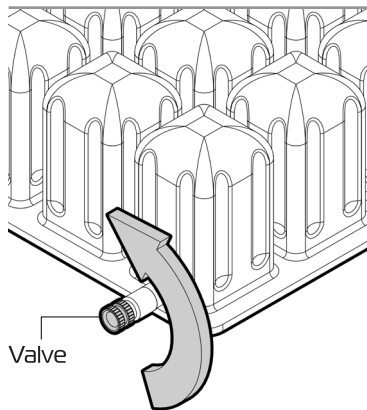


Step 1.
Open the valve by turning the end anti-clockwise.

Step 2.
Fit one end of the pump hose onto the valve and the other end onto the hand pump end as shown.

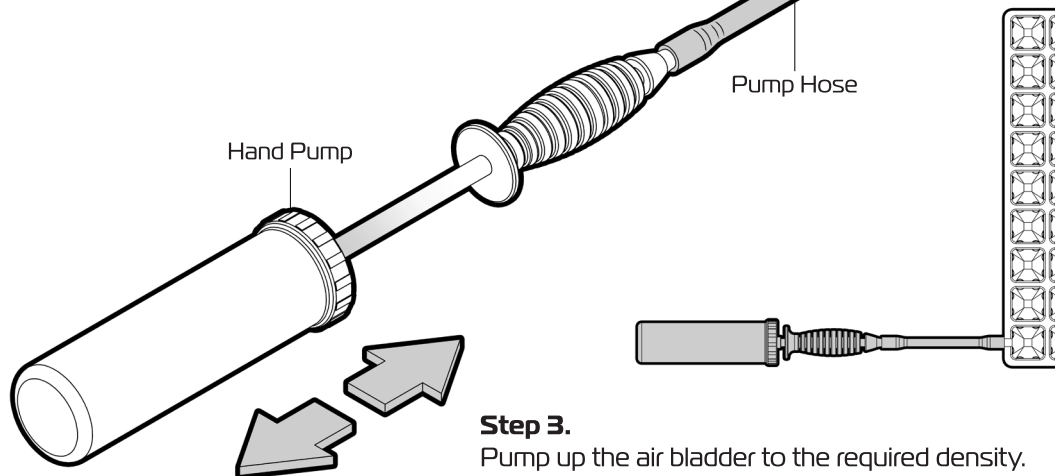

Step 3.
Pump up the air bladder to the required density.

Step 4.
Remove the pump hose. Close the valve by turning the end clockwise to close.

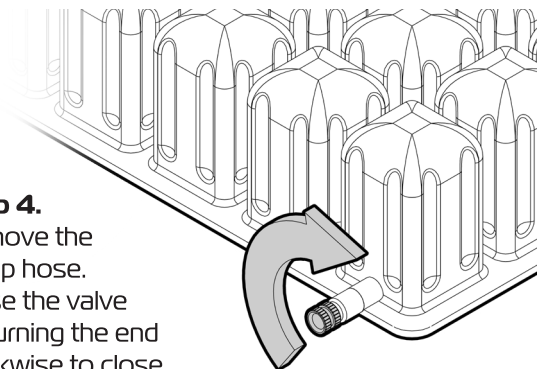

Step 5.
Fit the air bladder inside the cushion cover with the air pockets facing up.

Step 6.
Place the air cushion onto the sitting surface with the carry handle facing forward. Use the velcro straps at the rear of the cushion cover to attach the cushion to seat frame if required. Open and close the value to adjust the cushion so there is approx. 10mm between the body and seat.

INSPECTIONS

We suggest that you visually periodically check the VARI AIR Flotation Cushion for signs of damage or wear prior to use. Remove the cushion immediately from use if you suspect damage or any other fault. Fix any air leaks with the repair kit (patch and glue) supplied if possible. Put the air bladder under water to determine where the air leak is. Rough up the surface around the hole, apply glue and then fit patch over the hole. Allow glue to dry before reinflating air bladder. If in any doubt, please do not issue or use, but immediately contact your supplier for support.

CLEANING

Remove the air bladder from the cushion cover prior to cleaning. Machine wash the cushion cover only and hand wash the air bladder in warm soapy water. Allow to dry prior to use. Abrasive cleaners or abrasive cleaning pads could damage the cushion and should not be used when cleaning the cushion cover or the air bladder.

VARI AIR Flotation Cushion (RG4044 / RG4545 / RG4848) are covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

ASK FOR THE BRAND

redgumbrand.com.au