


REDGUM^{BRAND}
Daily Living and
Lifestyle Products

WALKING TUTOR

RG4310AA

Edition 1

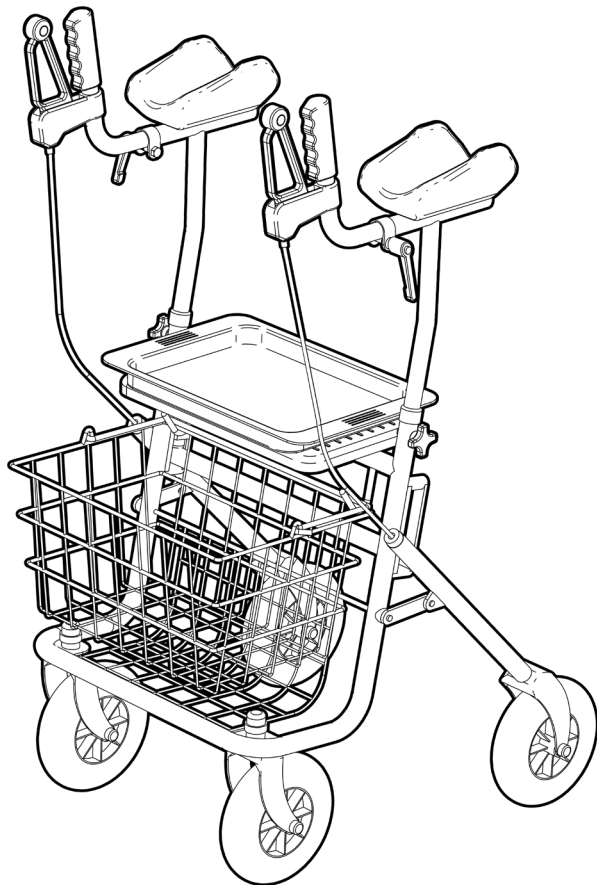


USER MANUAL

Please read and keep these instructions

ASK FOR THE BRAND

redgumbrand.com.au

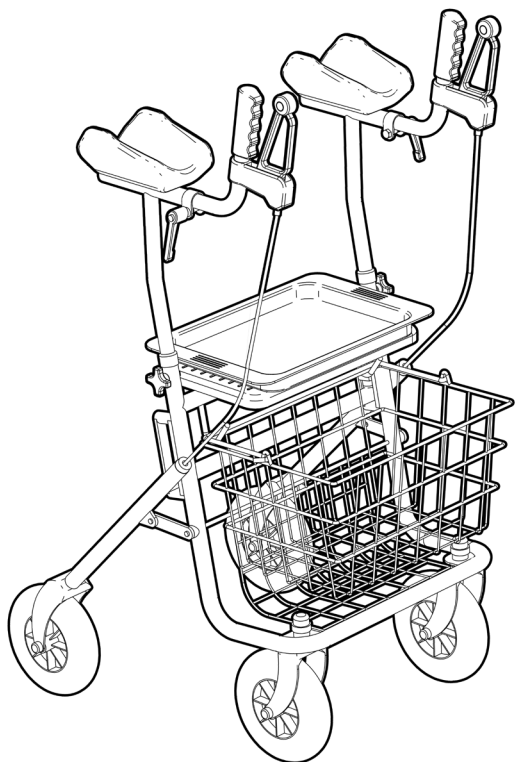

REDGUM^{BRAND}

*Daily Living and
Lifestyle Products*

REDGUM Brand owns the copyright to this walking tutor user manual.
Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible.
Products are subject to change without notice.

© Copyright **REDGUM Brand**

FOR-DE
GROUP PTY LTD


REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - UNFOLD FRAME
5. **ASSEMBLY** - FIT UPRIGHTS
6. **ASSEMBLY** - FIT HANDLES
7. **ASSEMBLY** - FIT TRAY
8. **ASSEMBLY** - FIT BASKET
9. **OPERATION** - BRAKES
10. **USAGE GUIDELINES** - SAFETY PRECAUTIONS
11. **PRODUCT CARE** - INSPECTIONS / CLEANING
12. **WARRANTY**

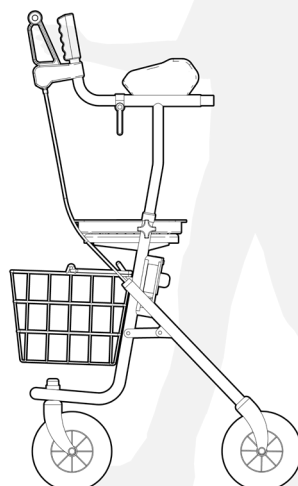

INTRODUCTION

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the walking tutor please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.


Only use the walking tutor for its intended purpose which is as a walking aid that can adjust to suit the users individual requirements.

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 120kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

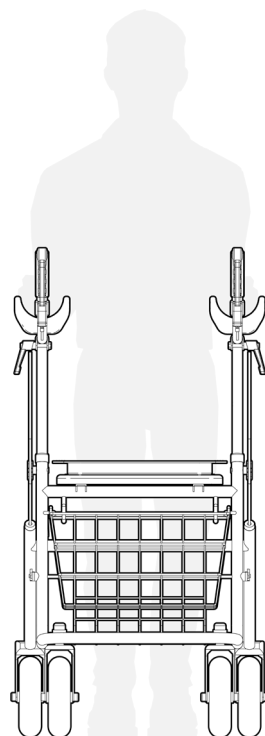

1.

ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

To assist those with limited hand and wrist strength.

Mobile walking frame.

Folding frame for transit and storage.

Height adjustable padded forearm supports.


Depth adjustable handles.

Dual hand and park brakes.

Flat free wheels.

Removable clear tray.

Removable wire basket with carry handle.

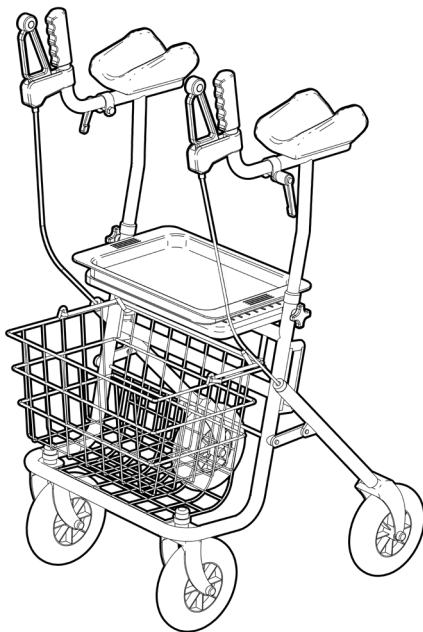

SPECIFICATIONS

(All dimensions shown are nominal)

Overall: **590mm** (W) x **700mm** (D) x **1160mm** (H - at lowest setting)

Overall (Folded with arms removed): **590mm** (W) x **320mm** (D) x **690mm** (H)

Armrest height adjustment: **965 - 1140mm** (From forearm gutter to floor) 8 settings

Handle depth adjustment: **270 - 330mm** (From handle to end of forearm gutter)

Seat: **380mm** (L) x **220mm** (W)

Tray: **320mm** (L) x **235mm** (W) x **28mm** (D)

Basket: **400mm** (L) x **270mm** (W) x **200mm** (D)

Wheel Diameter: **Ø200mm / 8"**

Product Weight:

Walking tutor: **13kg** (excluding basket and tray)

Wire basket: **1kg**


Tray: **0.3kg**

Maximum User Weight: **120kg**


ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:


Note:
Frame and handles are connected by cables

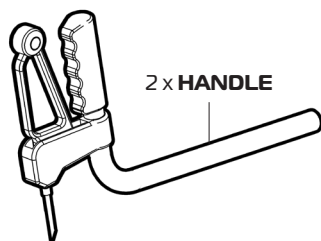


2 x **HANDLE**

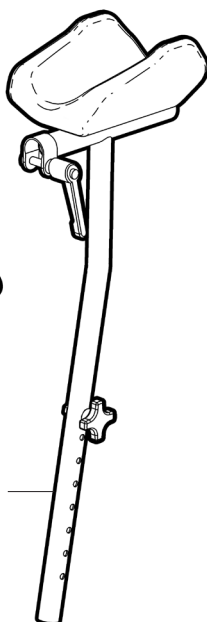

2 x **UPRIGHT**

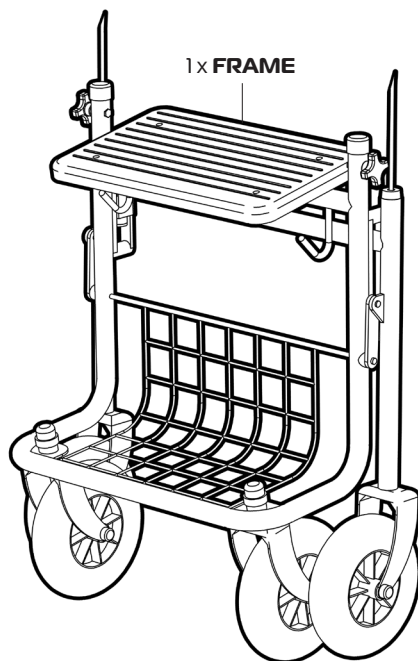


1 x **FRAME**

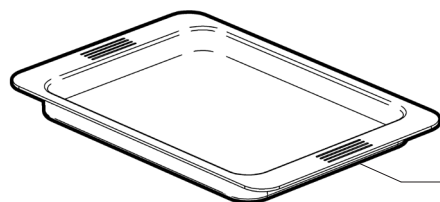

1 x **TRAY**

1 x **BASKET**

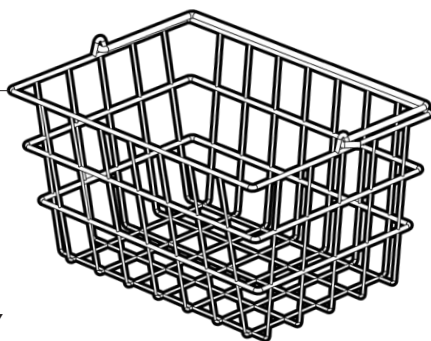

IMPORTANT NOTE

Remove all the protective wrapping and check all the parts. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.


3.

ASK FOR THE BRAND


redgumbrand.com.au

ASSEMBLY - UNFOLD FRAME

Please follow these steps to unfold the frame as required:

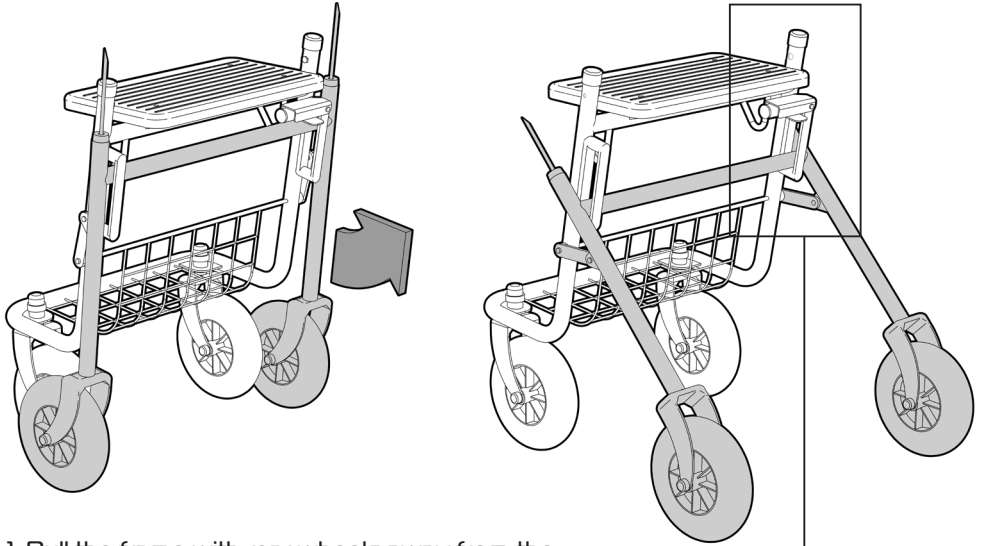

1. Pull the frame with rear wheels away from the main frame of walking tutor.
2. Rotate the locking bracket down to fix rear legs in place.

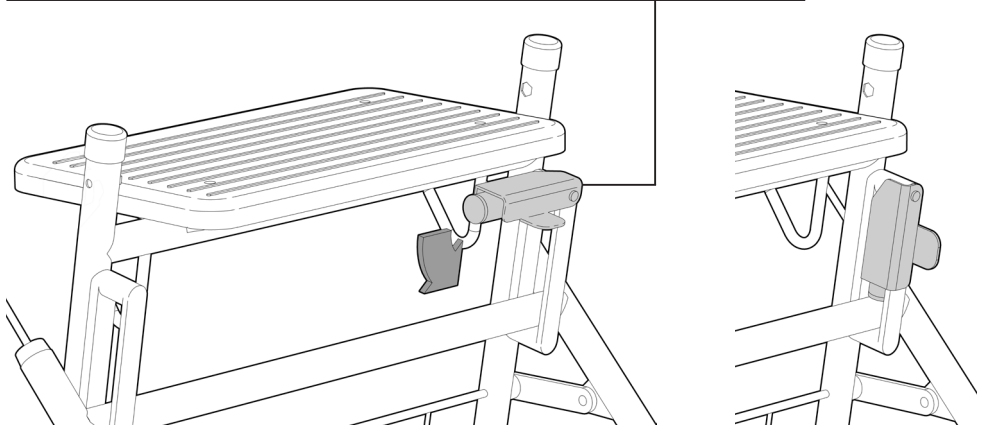


To fold the walker tutor for transit or storage:
Rotate the locking bracket up and push the rear wheel frame in towards the main frame of walking tutor.

ASSEMBLY - FIT UPRIGHTS


Please follow these steps to fit the uprights onto the frame as required:

1. Remove the hand wheel, washer and bolt from the upright tube.
2. Align the upright with the frame as shown.
3. Insert upright into frame.
4. Align required hole in upright with hole in frame.
5. From the inside face - Insert the bolt into the hex hole. Ensure the head of the bolt sits inside hex hole.
6. Slide washer onto bolt and screw hand wheel onto bolt.
7. Repeat process to fit other upright.

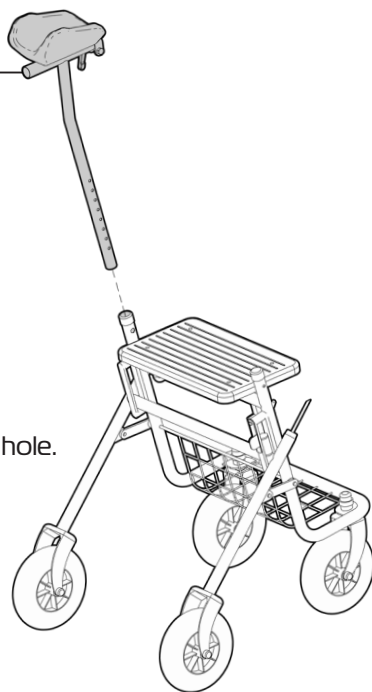


Height settings
(from forearm pad to floor):

- Hole 1: 965mm
- Hole 2: 990mm
- Hole 3: 1015mm
- Hole 4: 1040mm
- Hole 5: 1065mm
- Hole 6: 1090mm
- Hole 7: 1115mm
- Hole 8: 1140mm

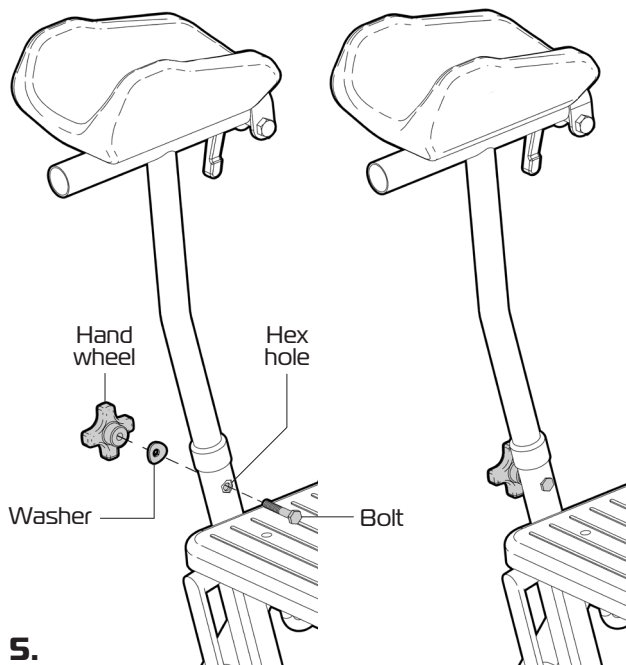

5.

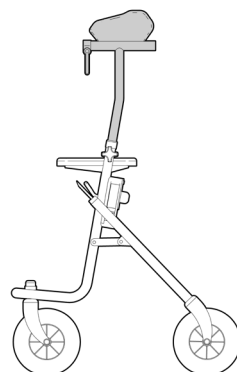


ASK FOR THE BRAND

redgumbrand.com.au

ASSEMBLY - FIT HANDLES

Please follow these steps to fit the handles onto the uprights as required:




1. Line up tube on handle with tube on underside of forearm pad with the hand grip and brake lever facing up.
2. Slide handle in to the required depth.
3. Tighten lever to fix handle in place.

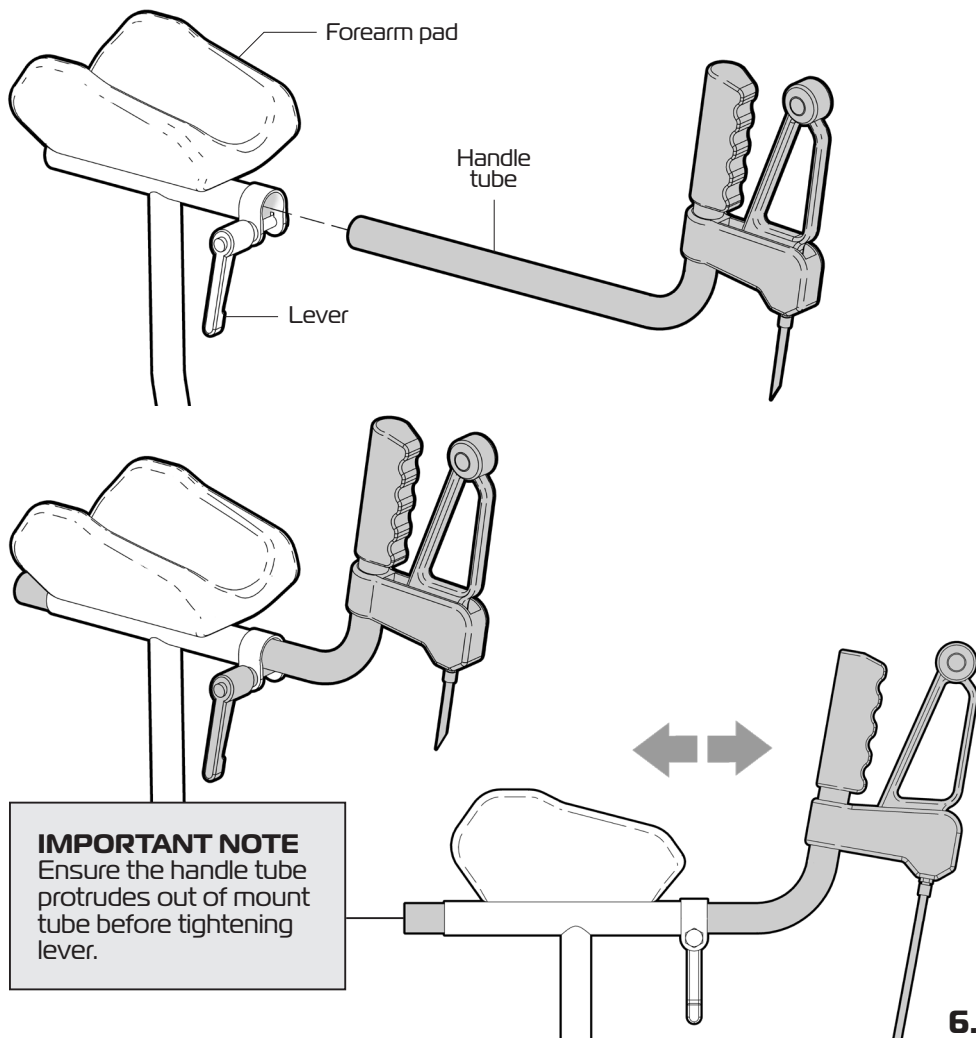

ASSEMBLY - FIT TRAY

Please follow these steps to fit the tray onto the walking tutor as required:

1. Line up pins on underside of tray with holes in top of seat.
2. Firmly push tray down onto seat.

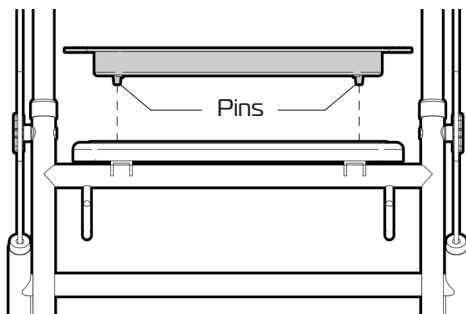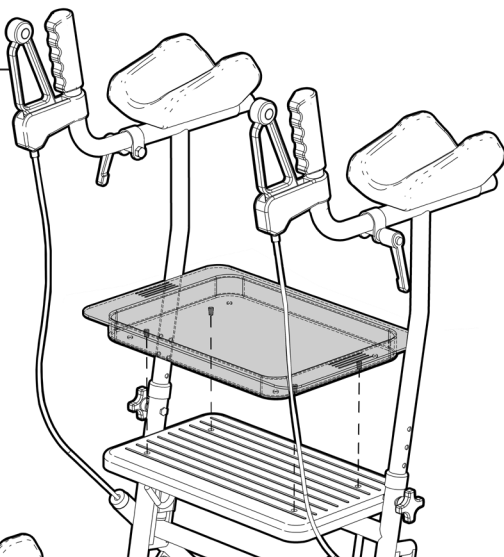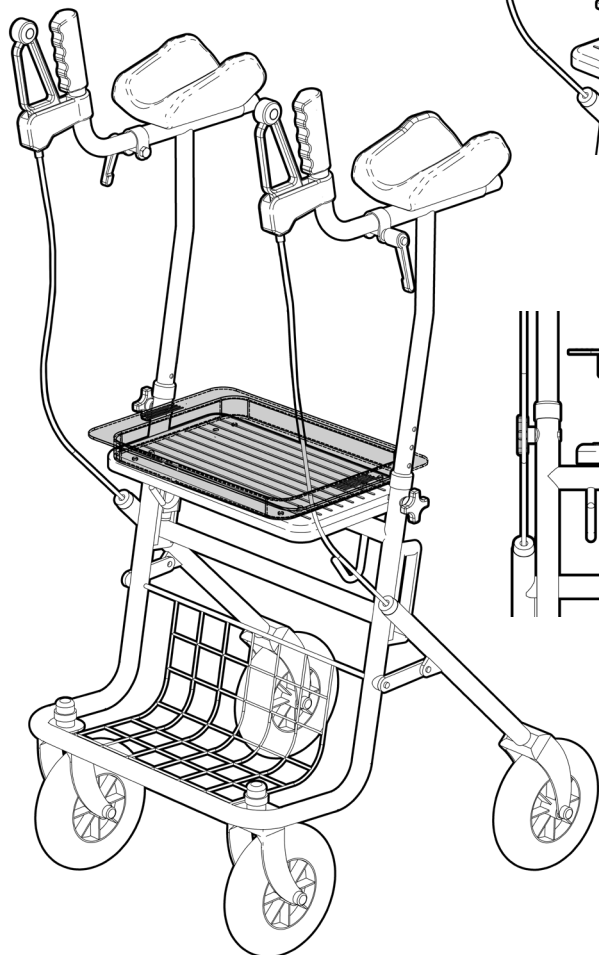

Tray max. load: **2kg**

7.



ASK FOR THE BRAND

redgumbrand.com.au

ASSEMBLY - FIT BASKET



Please follow these steps to fit the basket onto the walking tutor as required:

1. Centrally align the basket with the front of the walking tutor.
2. Hook the top rim of the basket onto the two hooks located on the crossbar underneath the seat.

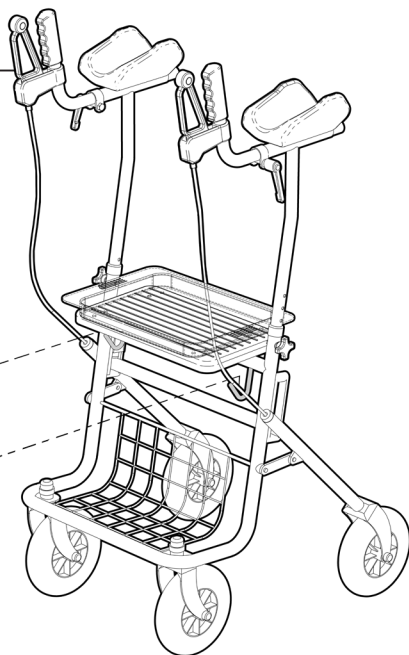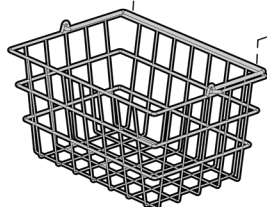

IMPORTANT NOTE

Do not overload basket
Max. load: **5kg**

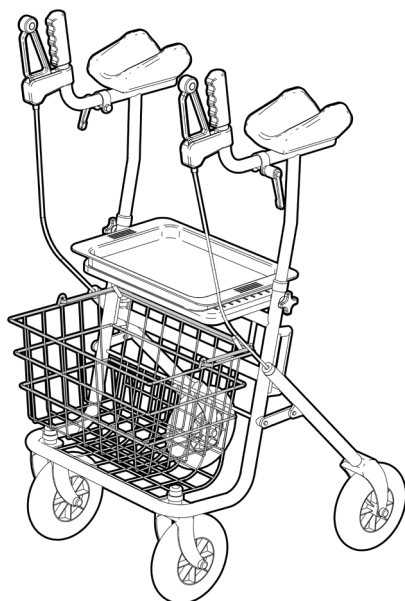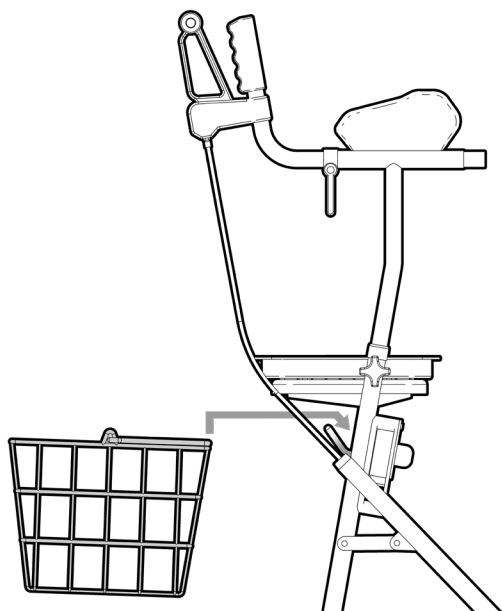


OPERATION - BRAKES

The walking tutor has independent hand brakes on both handles.

IMPORTANT NOTE:


Both hand brakes should be used simultaneously to ensure the walking tutor stops as required and the user is in full control.

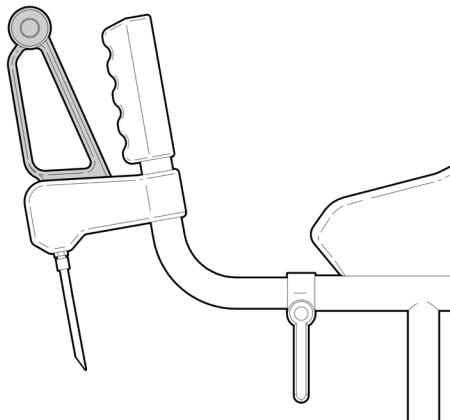

SLOW DOWN AND STOP

To slow down or stop the walking tutor from rolling gently pull back on both hand brakes simultaneously.

Release both hand brakes to allow the walking tutor to move again.

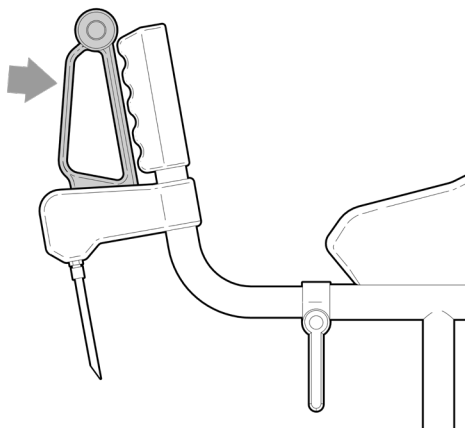


PARK BRAKE

To engage the park brakes on the walking tutor gently push forward on both hand brakes simultaneously until a loud click is heard.

This will stop the walking tutor from rolling.

To release park brakes pull back on both hand brakes simultaneously until a loud click is heard.

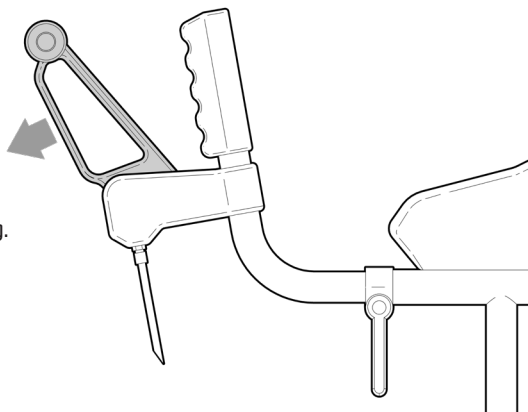

9.

ASK FOR THE BRAND

redgumbrand.com.au

USER GUIDELINES - SAFETY PRECAUTIONS

Please follow these guidelines to get the optimal benefit from walking tutor. Failure to follow these guidelines may result in damage to the product and put the user at risk.

Maximum user weight of walking tutor is **120kg**.

Do not overload the wire basket under seat. (Max. load: 5kg)

Periodically inspect the walking tutor prior to use to ensure it is in good working order. Always check that brakes are working prior to use.

The walking tutor is designed to be used indoors and outdoors.

Take care when using the walking tutor on uneven or wet ground.

Never self propel the walking tutor while seated.
(The walking tutor is not a wheelchair)

Take care when walking tutor is in the folded position as it may fall over if not supported or laid down.

Never hang any objects off the side of the walking tutor as it may tip over.

Ensure all four wheels remain in contact with the ground when the walking tutor is in use. Do not lean too far to one side or lean backwards causing the wheels to leave the ground.

If you have any issues or questions about the walking tutor please contact your supplier or REDGUM Brand direct.



PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the walking tutor periodically for signs of damage or wear prior to use, paying particular attention to the following areas:

Framework – check for damage or distortion.

Wheels - check to ensure walking tutor rolls as required

Brakes - check to ensure both brakes work as required (brake and park brake)

Hand grips / Forearm pads - check for cracks / tears in material

Overall check that no components are loose or worn and walking tutor operates as required.




Withdraw the walking tutor immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING

Clean the walking tutor using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the walking tutor and should not be used.

If you have any questions about the cleaning of the walking tutor please contact your supplier or **REDGUM Brand** direct.

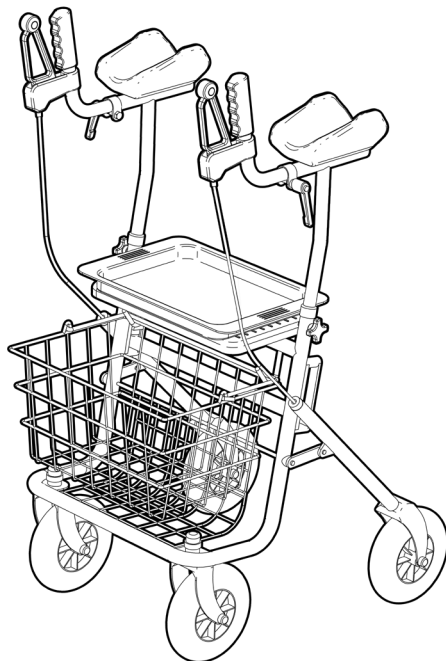

WARRANTY

REDGUM Brand walking tutor is covered by a 12 month warranty from the date of purchase.

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Do not modify the walking tutor in any way or fit any parts or accessories that are not specified as a option for the walking tutor. To do so will void your warranty and may seriously compromise your safety.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

DISTRIBUTOR:

REDGUM **BRAND**
*Daily Living and
Lifestyle Products*



Proudly
Australian Owned
and Operated

ASK FOR THE BRAND

redgumbrand.com.au