




HEAVY DUTY **TRI WALKER**


RG4405

Edition 1

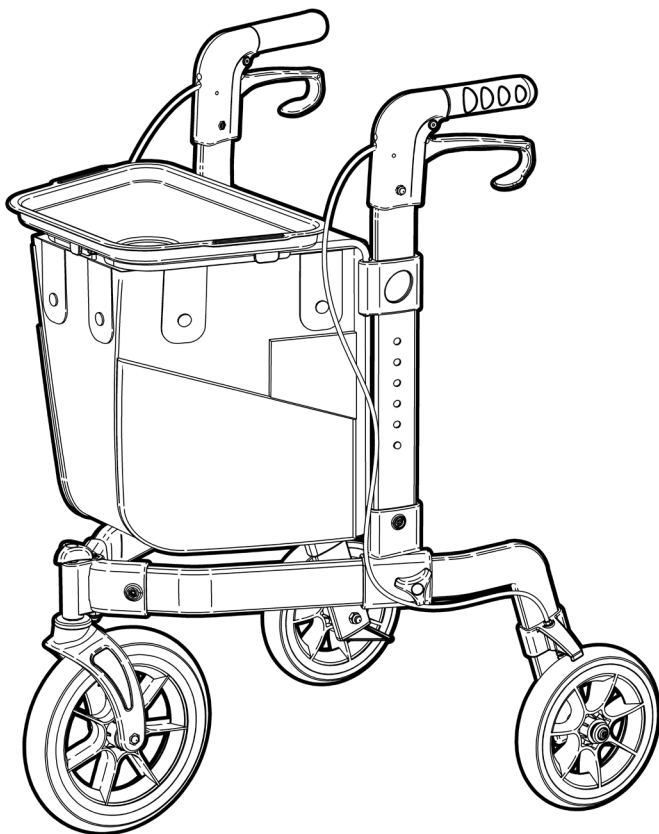

USER MANUAL

Please read and keep these instructions

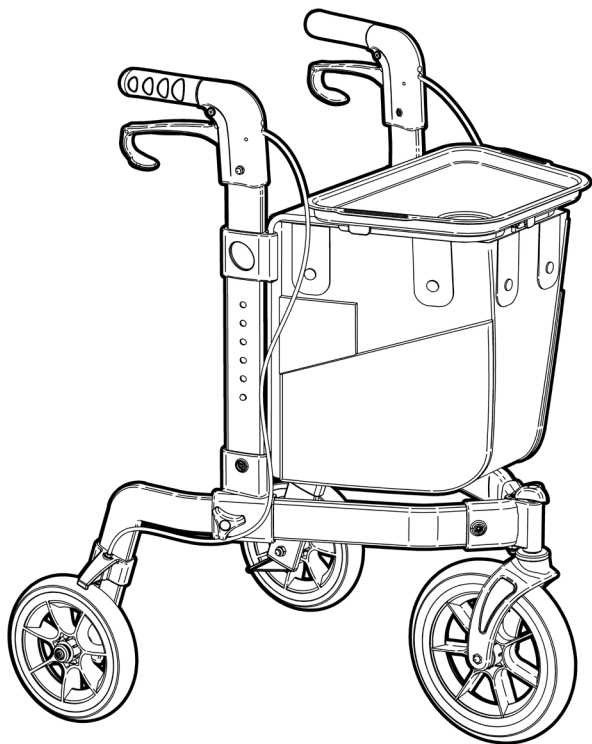

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Heavy Duty Tri Walker user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**




REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrandmobility.com.au

CONTENTS

1. **INTRODUCTION**
2. **PRODUCT OVERVIEW**
3. **UNPACK and INSPECT PARTS**
4. **UNFOLD FRAME**
5. **FIT FRONT WHEEL**
6. **FIT REAR WHEELS**
7. **FIT HANDLES**
8. **ADJUST HANDLES**
9. **FIT TRAY**
10. **BRAKING**
11. **USAGE**
12. **FOLDING WALKER**
13. **PRODUCT CARE**
14. **WARRANTY**
15. **REDGUM BRAND RANGE**
16. **NOTES**

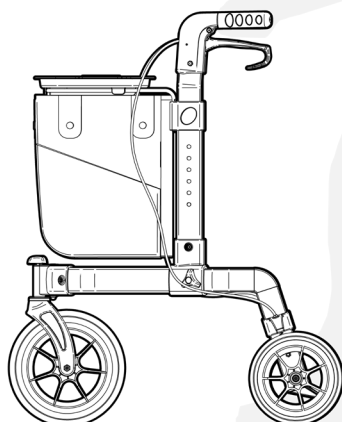

INTRODUCTION

Thank you for purchasing this REDGUM Brand product.

If you have any issues or questions about the Heavy Duty Tri Walker please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your Heavy Duty Tri Walker for its intended purpose which is as a walking aid that can adjust to suit the users individual requirements.


Suitable for indoor and outdoor use.

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 136kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

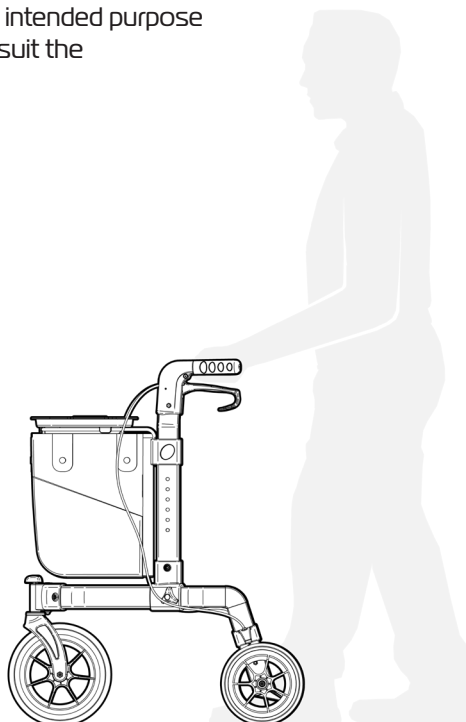

1.


ASK FOR THE BRAND

redgumbrandmobility.com.au

PRODUCT OVERVIEW

FEATURES

- Great walking and stability aid
- Perfect for shopping
- Large storage bag
- Height adjustable handles
- Ergonomic rubber hand grips
- User friendly brake levers
- Removable plastic tray
- Large wheels for excellent mobility
- Folds up for transport and storage

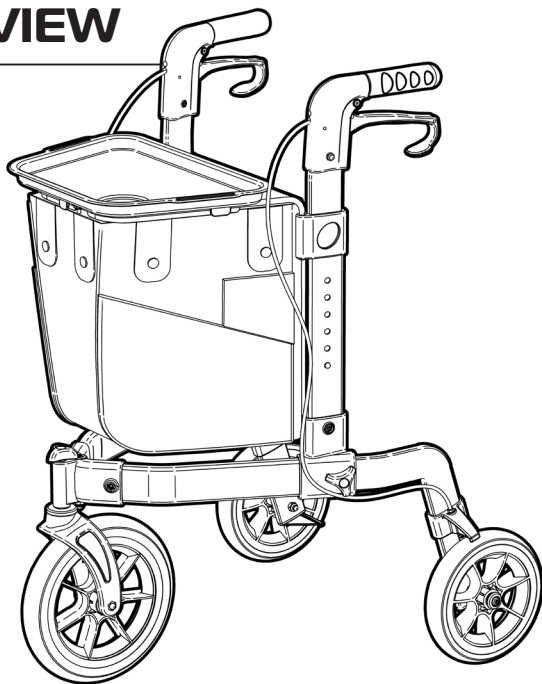

SPECIFICATIONS


(All dimensions shown are nominal)

- Overall: **740mm L x 600mm W x 905mm H** (at lowest setting)
- Overall Folded: **780mm L x 220mm W x 905mm H**
- Handle Height (6 settings - floor to top of handle):
905mm, 930mm, 955mm, 980mm, 1005mm, 1030mm
- Storage Bag: **270mm L x 200mm (front) / 330mm (rear) W x 350mm D**
- Wheel Diameter: **Front: Ø230mm / Rear: Ø200mm**
- Product Weight: **8kg**
- Maximum Weight Capacity: **136kg**
- Maximum Storage Bag Weight Capacity: **10kg**


Please contact **REDGUM Brand** if you would like any further information on the Heavy Duty Tri Walker.

UNPACK and INSPECT PARTS

Ensure you have been supplied with the following parts:

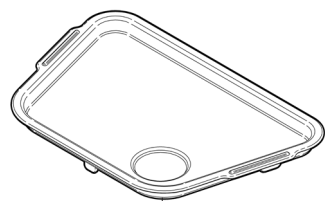


Tray

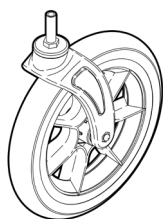


Front Wheel




Large
Tri Wheel
(not to scale)

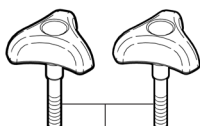


Small
Tri Wheel
(not to scale)

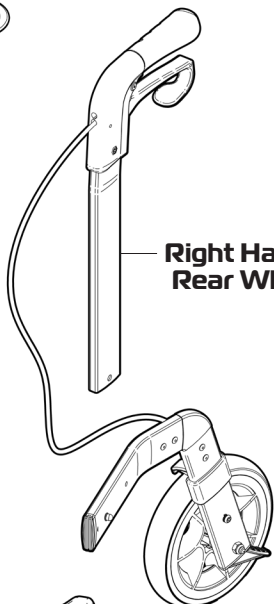

Right Handle
Rear Wheel

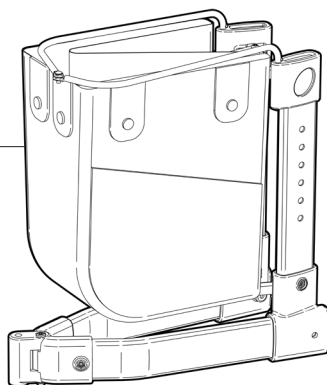

Frame

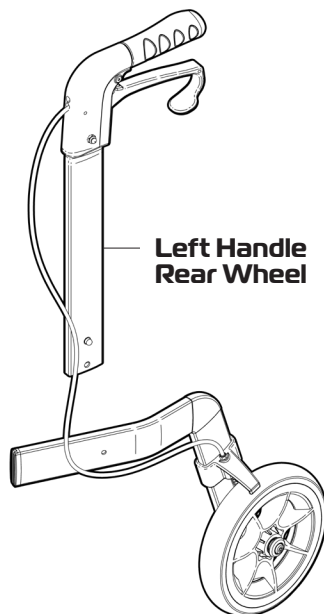

Left Handle
Rear Wheel

Please take care when unpacking the product and removing the protective packing materials. Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE THE PRODUCT** and contact your supplier for support. Please recycle packaging accordingly :)

3.


ASK FOR THE BRAND

redgumbrandmobility.com.au


UNFOLD FRAME

Please follow these steps to unfold the frame:

1. Using both hands pull the frame apart.

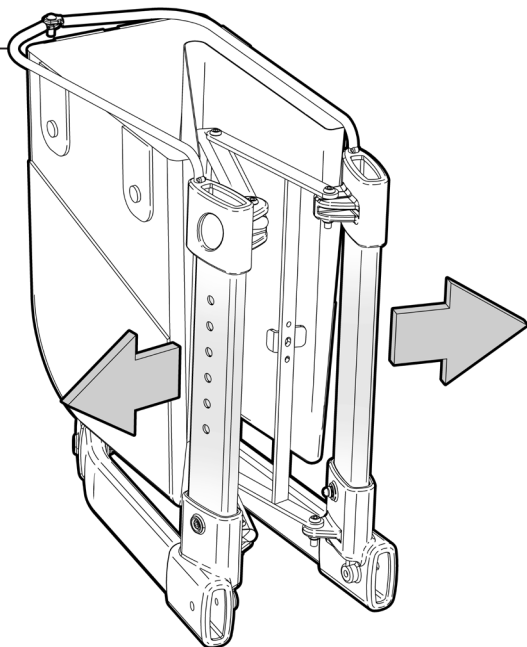

2. Pull the red lever back until a loud "click" is heard which indicates that the frame is now fully opened and locked in position.

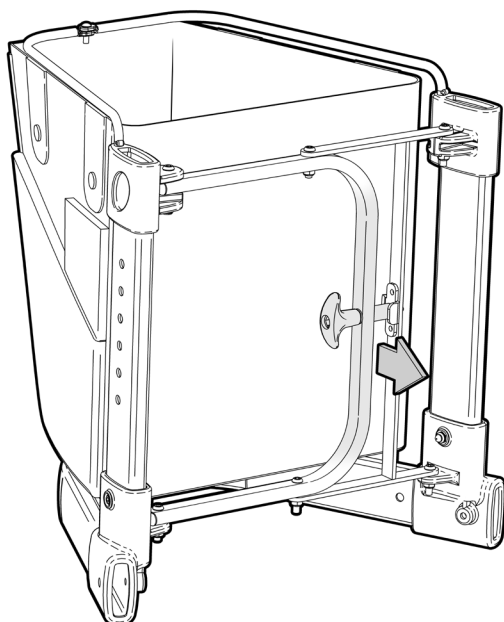



4.

FIT FRONT WHEEL

Please follow these steps to fit the front wheel:

1. Slide the sleeve on the front wheel up into the front housing as shown.

2. Fix the wheel in place using the LARGE tri wheel.

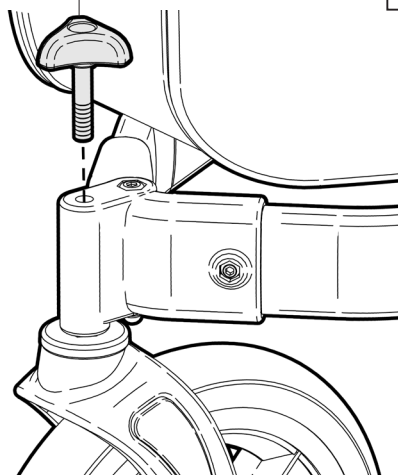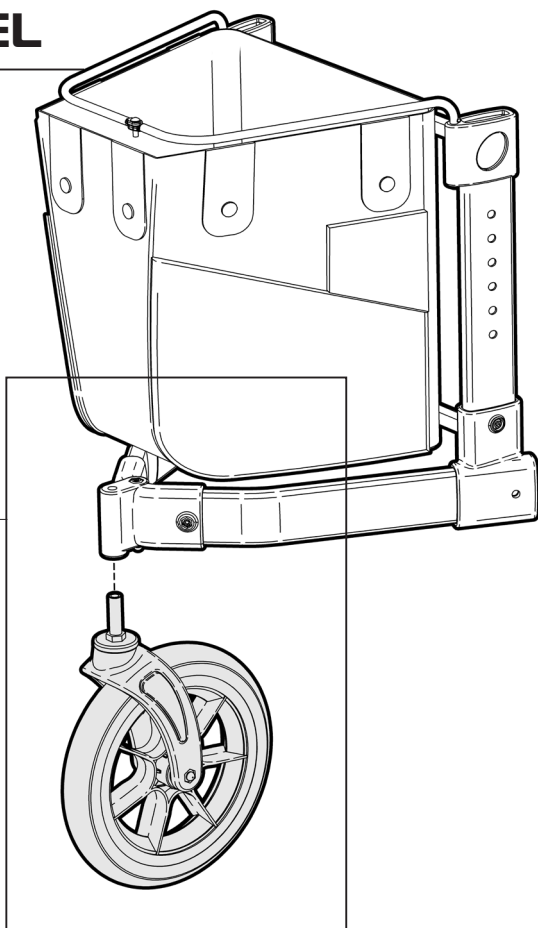

5.


ASK FOR THE BRAND

redgumbrandmobility.com.au


FIT REAR WHEELS

Please follow these steps to fit the rear wheels:


1. Determine the correct rear wheel assembly prior to fitting. Wheel must be on the outside with the brake cable facing up.

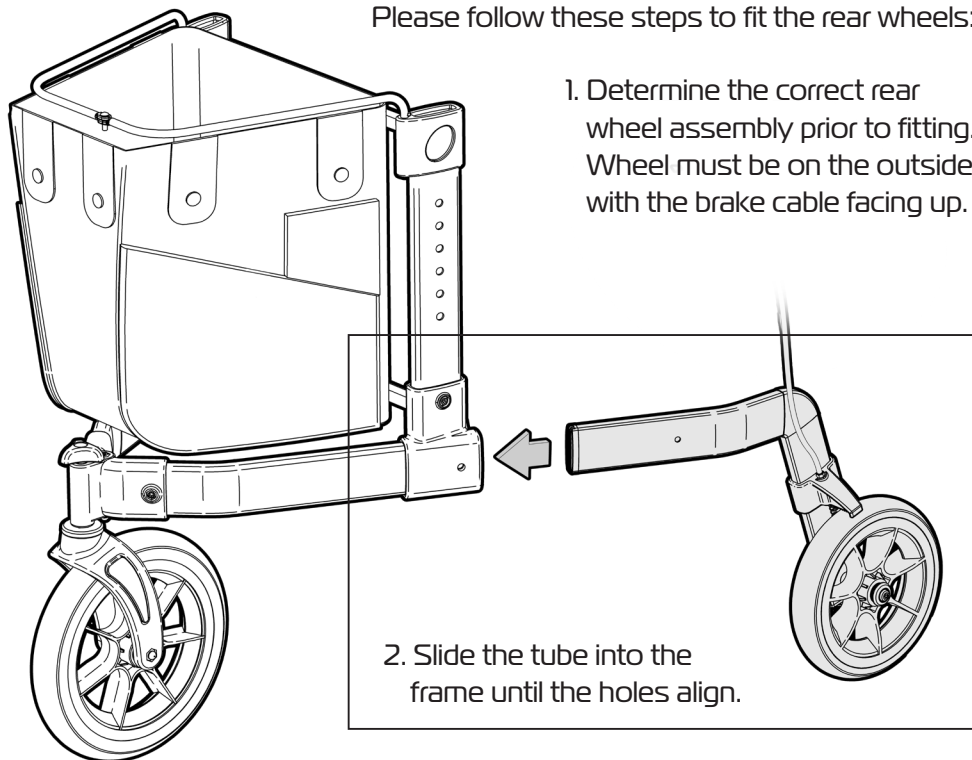

2. Slide the tube into the frame until the holes align.

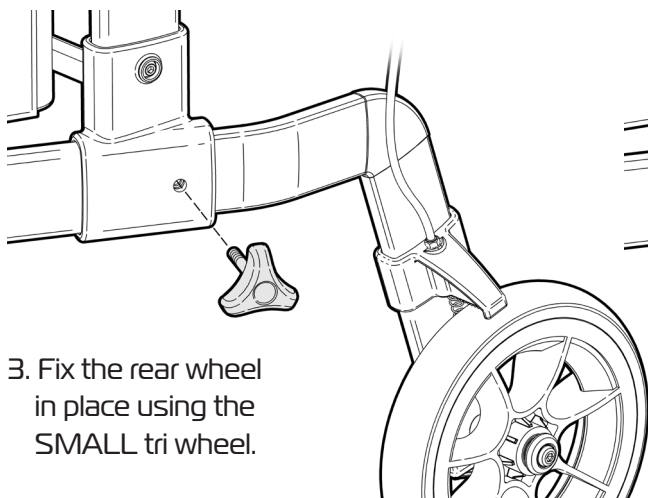

3. Fix the rear wheel in place using the SMALL tri wheel.

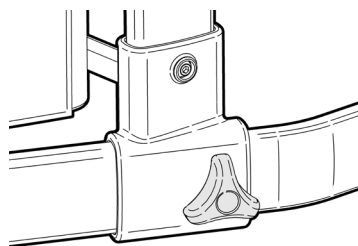



4. Repeat process to fit other rear wheel.

6.


FIT HANDLES

Please follow these steps to fit the handles to the walker:

1. Determine the correct handle assembly prior to fitting. Hand grip must be facing to the rear and the brake cable on the handle is joined to the corresponding rear wheel.
2. Slide the handle tube into the frame as shown.

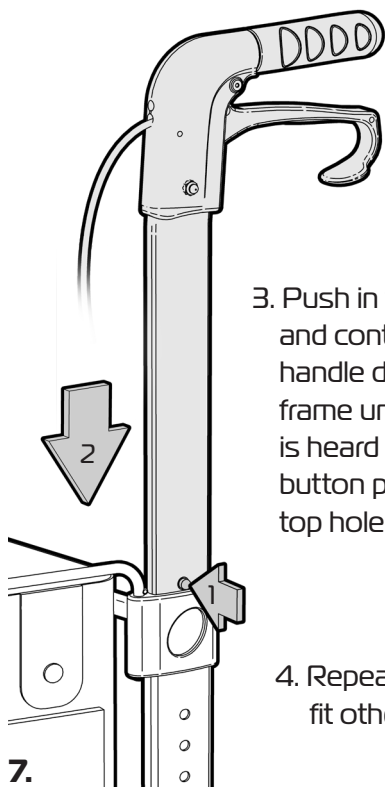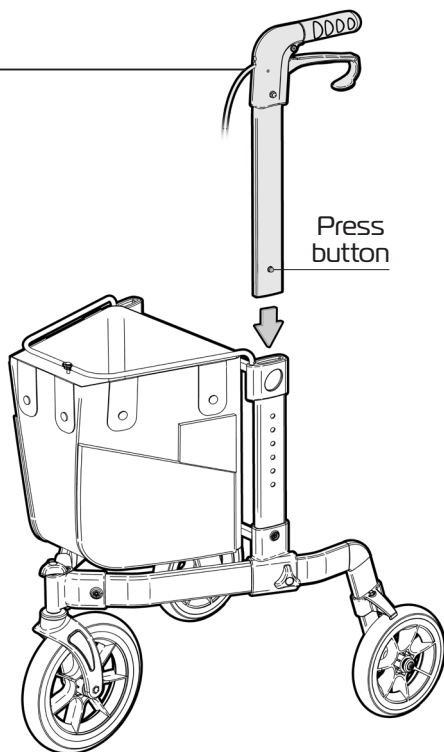

3. Push in the press button and continue to slide the handle down into the frame until a loud "click" is heard and the press button pops out of the top hole in frame.

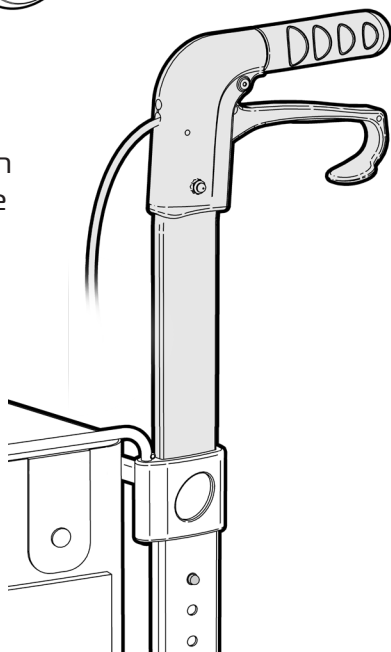

4. Repeat process to fit other handle.



7.

ADJUST HANDLES

Please follow these steps to adjust the handles:

It is very important that the handles are set to the correct height to suit the user.

Stand behind the tri walker with your shoulders relaxed and your arms hanging loosely at your sides. The height of the handles should be adjusted so that they are level with your wrists. This will position the arms at the recommended 20 degree bend in the elbow when using the walker.

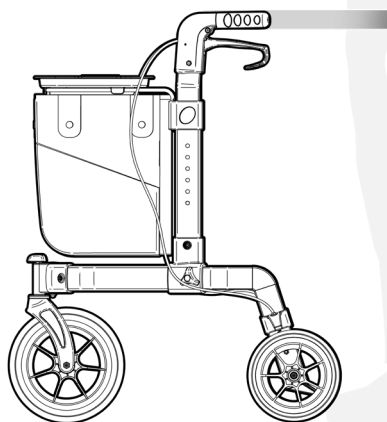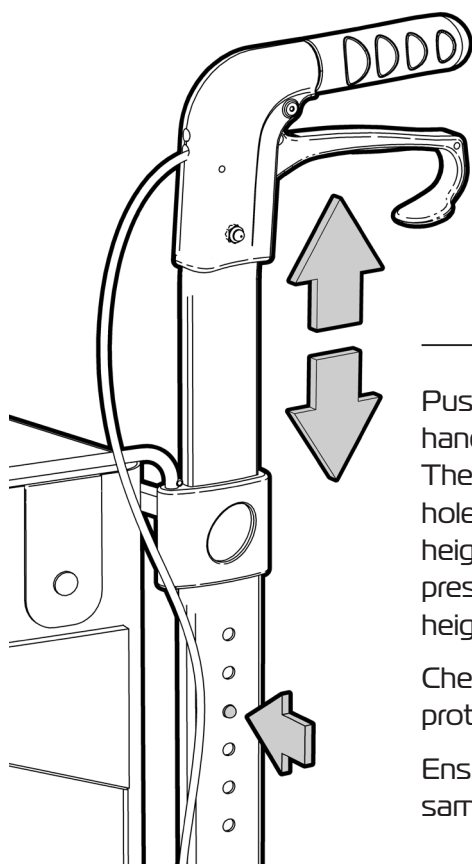

Push in the press button and slide the handle down to the required height setting. The press button will pop out of every hole setting as you adjust the handle height so you will need to push the press button again until you reach the height setting required.


Check to ensure that the press button protrudes out of the frame.

Ensure both handles are set at the same height prior to use.

8.


FIT TRAY

Please follow these steps to fit the tray:

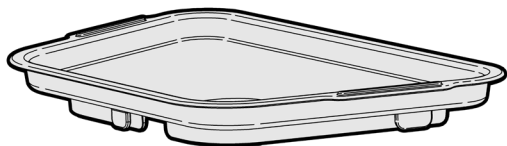


1. Align the tray with the taper facing toward the front of the tri walker.

Rod frame

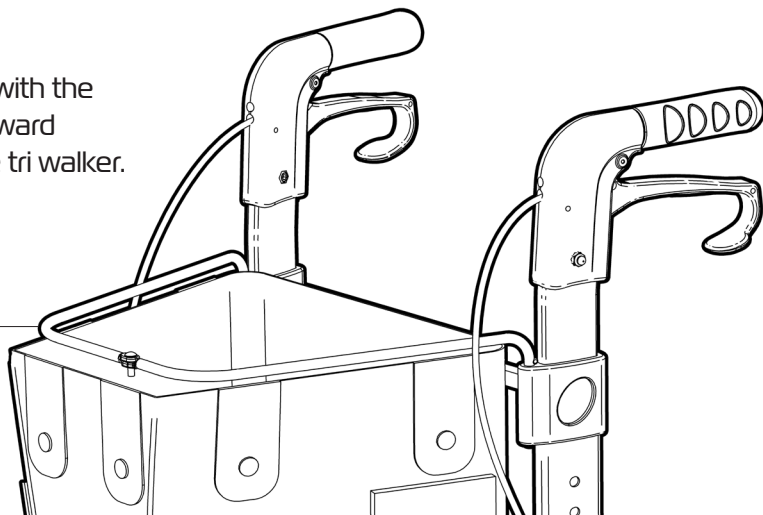

2. Locate the guides on the underside of tray onto the rod frame that holds the bag in place.

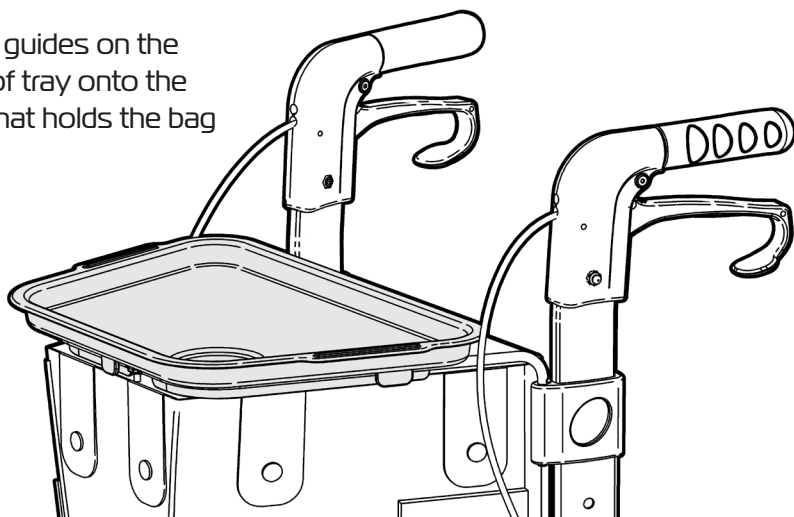

9.


ASK FOR THE BRAND

redgumbrandmobility.com.au

BRAKING

Please follow these guidelines to brake the tri walker in a safe manner:

Both hand brakes should be used simultaneously to ensure the walker stops as required and the user is in full control.

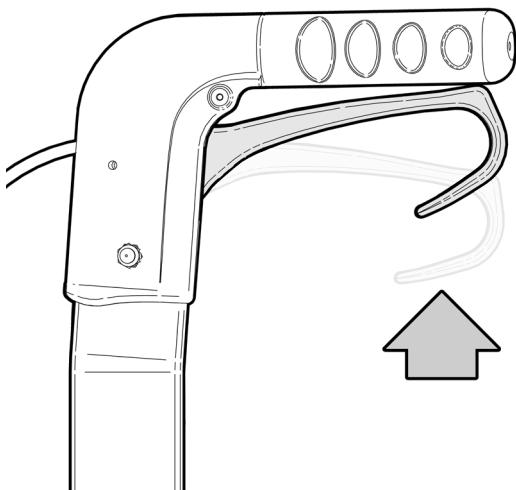


SLOW DOWN AND STOP

To slow down or stop the tri walker from rolling:

Gently pull up on both hand brakes simultaneously.

Release both hand brakes to allow the tri walker to move again.

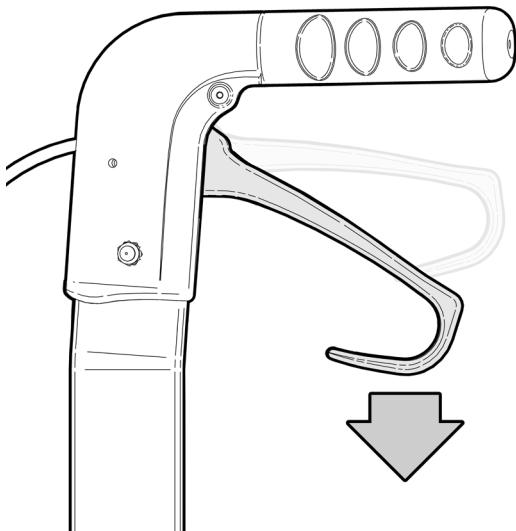

PARK BRAKE

To engage the park brakes on the tri walker:

Gently push down on both hand brakes simultaneously until a loud click is heard.

To release park brakes:

Pull up on both hand brakes simultaneously until a loud click is heard.

USAGE

Please follow these guidelines to use the Heavy Duty Tri Walker in a safe manner. Failure to follow these guidelines may result in damage to the product and put the user at risk.

Do not overload the tri walker **(136kg max. load)**

Do not overload the storage bag **(10kg max. load)**



Pinch points do exist on this product. Please keep hands and any other body parts away from moving parts when folding the walker.

We recommend that the park brakes are only engaged if the walker may roll away.

Go up or down inclines directly. Do not try to cross on an angle as this will cause instability and the walker may tip over.

All three wheels should always remain on the ground unless mounting kerbs.




Push down on the foot peg to lift the front of the walker off the ground to get over kerbs.

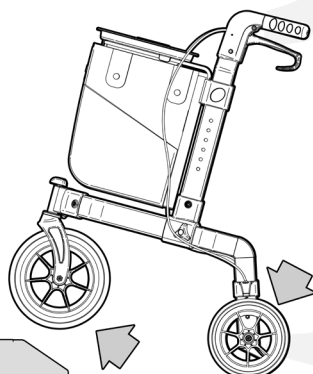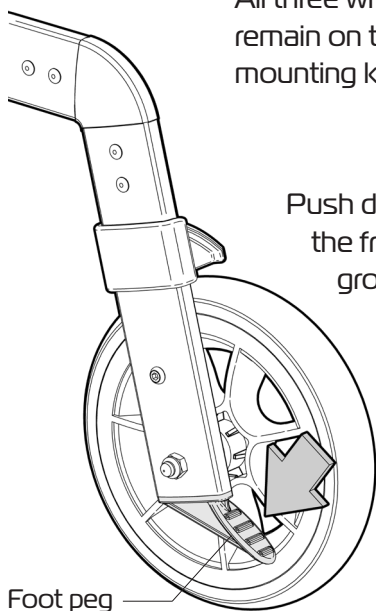

FOLDING WALKER

Please follow these steps to fold the tri walker:

1. Remove the tray.

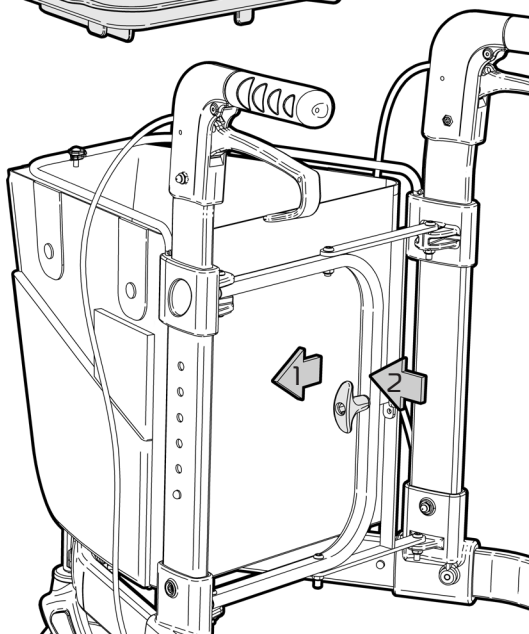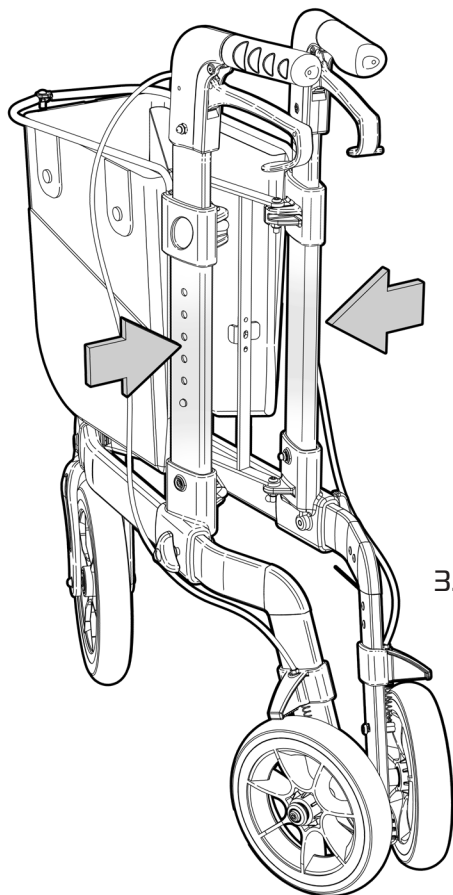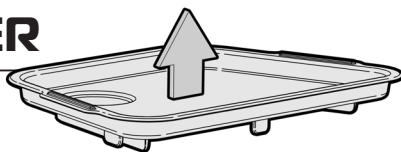

2. Pull up red lever in towards the middle of the walker and push inwards to start folding the frame.

3. Using both hands push the frame together.

Important Note:

Tri Walker can become unstable and fall over when folded up.

Ensure the front swivel wheel is inline with the frame when folded.

PRODUCT CARE

INSPECTIONS

We suggest that you visually check the tri walker periodically for signs of damage or wear prior to use, paying particular attention to the following areas:

Framework: Check for damage or distortion.

Wheels: Check to ensure the tri walker rolls as required.

Brakes: Check to ensure both brakes work as required

Hand grips: Check for cracks / tears in material.

Overall check that no components are loose or worn and that the tri walker operates as required.



Withdraw the tri walker immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for support.

CLEANING

Clean the tri walker using disinfectant wipes. We do not recommend cleaning the product with soap and water.

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Do not use any abrasive cleaners or abrasive cleaning pads as they could potentially damage the tri walker.

If you have any questions about the cleaning of the product please contact your supplier or REDGUM brand direct.



WARRANTY

REDGUM Brand Heavy Duty Tri Walker (RG4405)
is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the tri walker in any way or fit any parts or accessories that are not specified as an option for the product. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about this product please contact your supplier or REDGUM Brand direct on:

T: +61 8 9248 4180 E: orders@for-de.com.au

REDGUM Brand - Product Range

REDGUM Brand offer a comprehensive range of products including:

Seat Walkers

Side Folding Seat Walkers

Walking Frames

Wheelchairs

Walking Sticks + Quads

Crutches

Reachers

Shower / Bathroom products

Toilet products

Cushions: Air / Wedge / Foam

Electric Lift Chairs

Electric Beds / Bed accessories


Aluminium Ramps

Pedal Exercisers



REDGUM Brand stock a large range of spare parts for all REDGUM brand products.

Please contact **REDGUM Brand** for further information.

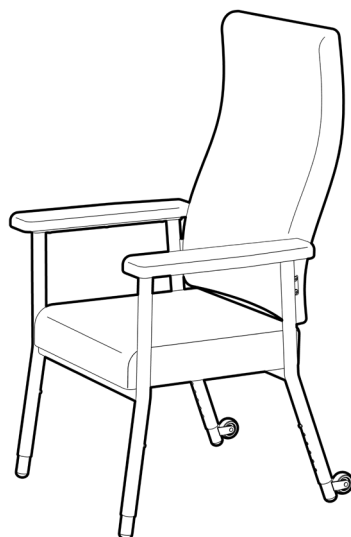

A line drawing of a three-wheeled medical walker. It features a large, rectangular front basket with two side pockets. The walker has three wheels: two large rear wheels and one smaller front wheel. The handlebars are adjustable, with the right handlebar showing a series of holes for height adjustment. A small, curved handle is visible on the left side of the frame. The drawing is a simple black and white line art.





DISTRIBUTOR:



**Explore our wide
range of products!**