


**REDGUM brand**  
ASSISTED LIVING PRODUCTS



# ULTRA

## LIGHTWEIGHT SEAT WALKER

RG4410

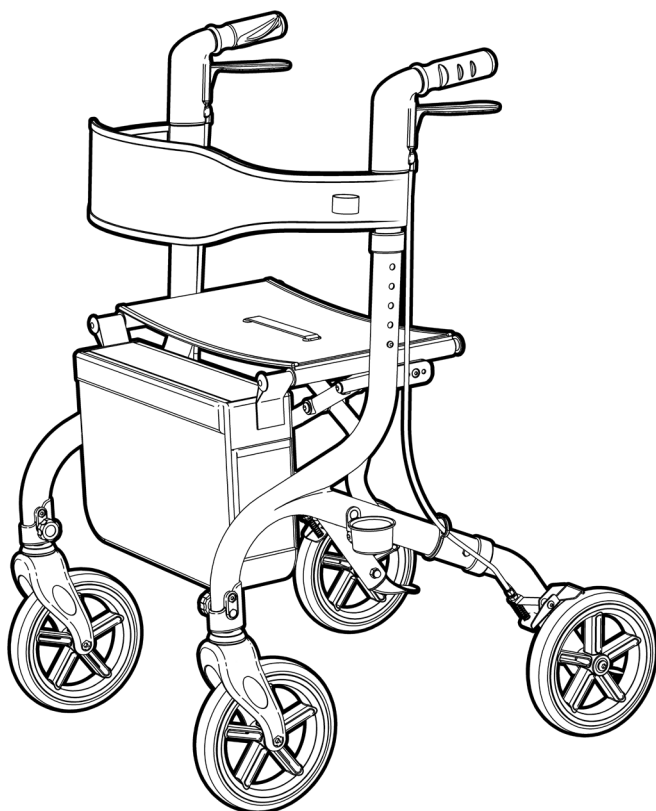


## USER MANUAL

Please read and keep these instructions

ASK FOR THE BRAND

[redgumbrandmobility.com.au](http://redgumbrandmobility.com.au)

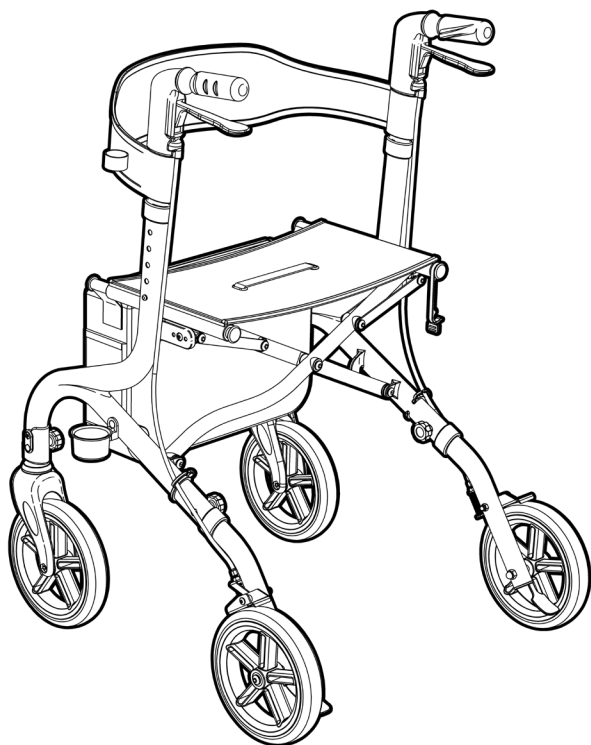

# REDGUM brand

## ASSISTED LIVING PRODUCTS

**REDGUM Brand** owns the copyright to the ULTRA Lightweight Seat Walker user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

**REDGUM Brand** strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**

**FOR-DE**  
GROUP PTY LTD

**REDGUM Brand** is part of **FOR-DE Group Pty Ltd**


*ASK FOR THE BRAND*

**redgumbrand**mobility.com.au

# CONTENTS

---

1. **INTRODUCTION**
2. **PRODUCT OVERVIEW**
3. **UNPACK and INSPECT PARTS**
4. **UNFOLD FRAME**
5. **FIT FRONT WHEELS**
6. **FIT REAR WHEELS**
7. **FIT HANDLES**
8. **ADJUST HANDLES**
9. **FIT BAG**
10. **FIT WALKING STICK CUP**
11. **USAGE**
12. **BRAKING**
13. **FOLDING WALKER**
14. **PRODUCT CARE**
15. **WARRANTY**
16. **REDGUM Brand - Product Range**

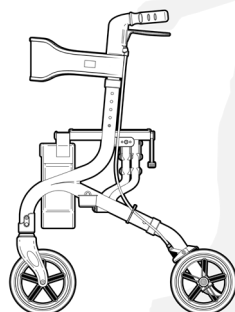

# INTRODUCTION

---

Thank you for purchasing this REDGUM Brand product.

If you have any issues or questions about the ULTRA Lightweight Seat Walker please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

## INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use the ULTRA Lightweight Seat Walker for its intended purpose which is as a walking and sitting aid that can adjust to suit the users individual requirements.


Suitable for indoor and outdoor use.

### IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

#### **Maximum User Weight: 136kg**

Exceeding the weight limit could cause damage to the product and could put the user at risk.

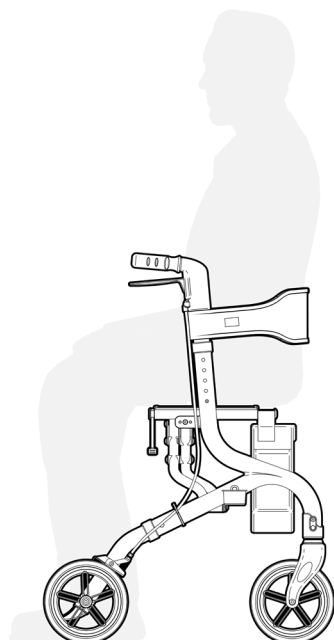

1.


*ASK FOR THE BRAND*

**redgumbrand**mobility.com.au

# PRODUCT OVERVIEW

## FEATURES

- Great walking and stability aid
- **SUPER LIGHTWEIGHT: 4.8kg**  
(Front wheels / Bag removed)
- Side folding seat walker
- Height adjustable handles
- Ergonomic rubber hand grips
- Hand brakes with park brake
- Mesh seating
- Removable storage bag
- Walking stick holder
- Large wheels for excellent mobility
- Folds up for transport and storage

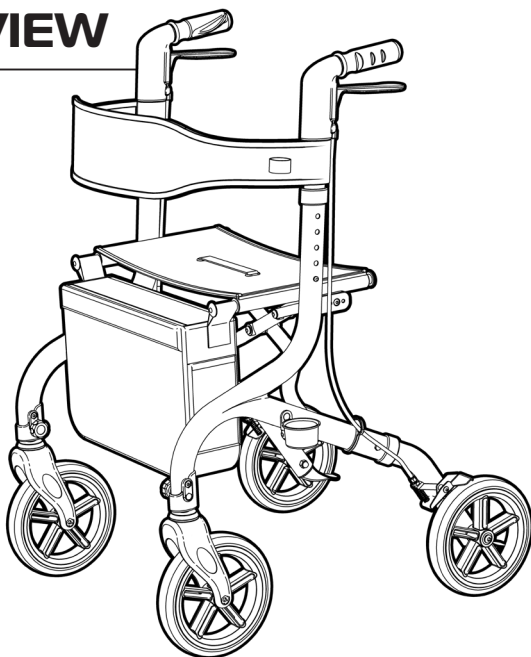


## SPECIFICATIONS (All dimensions shown are nominal)

- Overall: **680mm L x 640mm W x 890mm H** (at lowest setting)
- Overall Folded: **680mm L x 310mm W x 890mm H** (at lowest setting)
- Handle Height: **890 - 990mm (5 settings)**
- Seat: **480mm W x 245mm D x 550mm H** (floor to seat)
- Wheel Diameter: **Ø200mm / 8"**
- Storage Bag: **390mm W x 140mm D x 260mm H**
- Product Weight: **6kg total / 4.8kg** (front wheels / bag removed)
- Maximum User Weight: **136kg**
- Maximum Carry Bag Loading Weight: **5kg**

Please contact **REDGUM Brand** if you would like any further information on the ULTRA Lightweight Seat Walker.

# UNPACK and INSPECT PARTS

Ensure you have been supplied with the following parts:

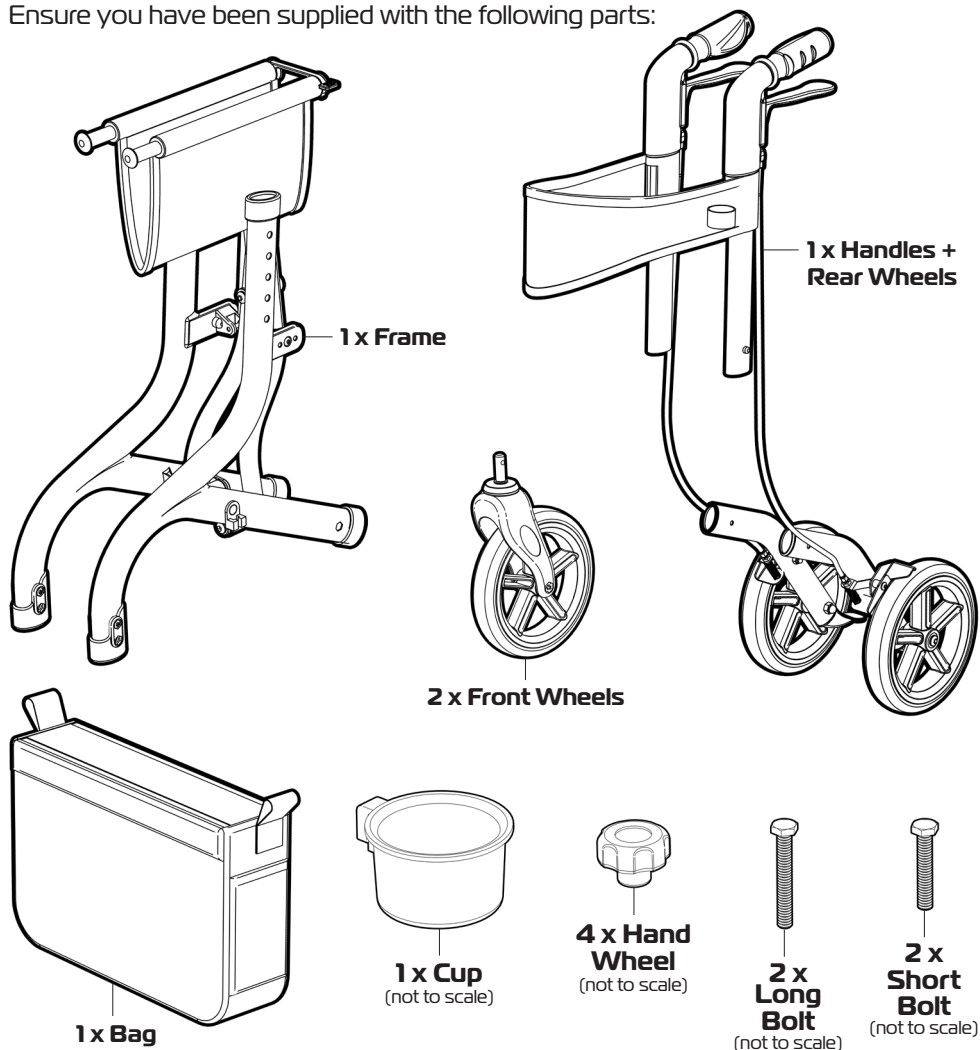

Please take care when unpacking the product and removing the protective packing materials. Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE THE PRODUCT** and contact your supplier for support. Please recycle packaging accordingly : )


3.

ASK FOR THE BRAND


redgumbrandmobility.com.au

# UNFOLD FRAME


Please follow these steps to unfold the frame:

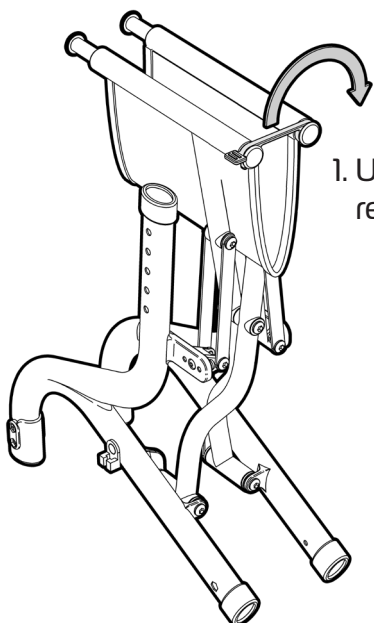

1. Undo the retaining clip.

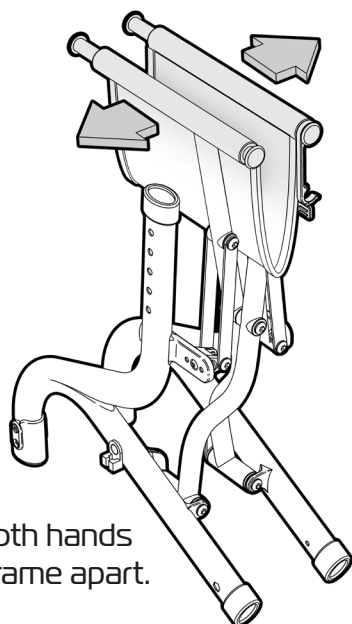

2. Using both hands pull the frame apart.

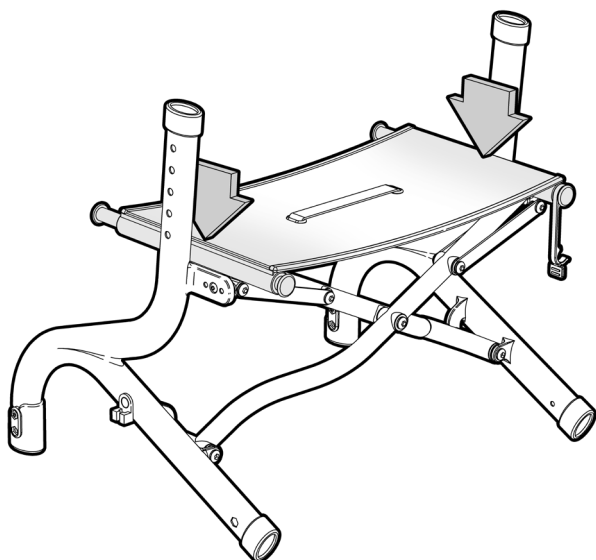


3. Push down on both sides of the seat until a loud "click" is heard which indicates that the seat is now locked in position.

4.

# FIT FRONT WHEELS


Please follow these steps to fit the front wheels:

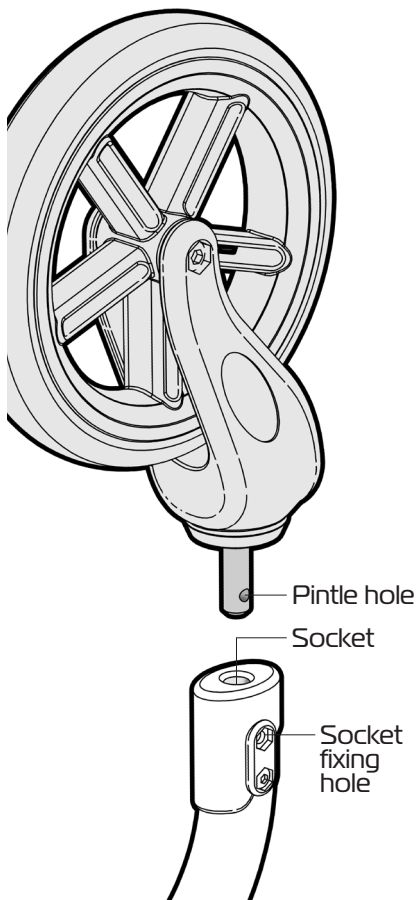

2. Line up the pintle hole on the wheel with the socket fixing hole on the walker frame.


Slide pintle into socket.

5.

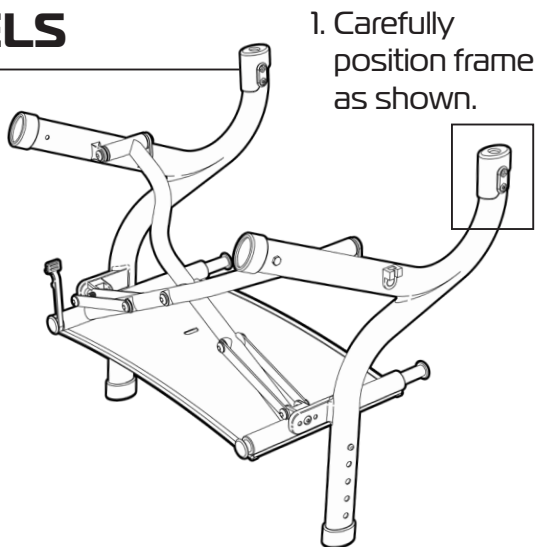


1. Carefully position frame as shown.

3. Insert LONG bolt into hole. Align hex head of bolt with hex recess.

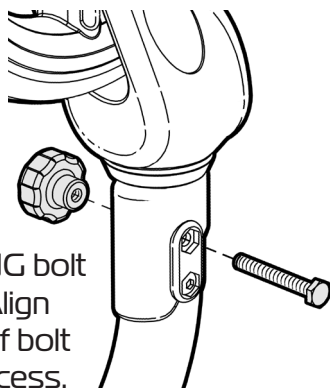


4. Wind hand wheel onto bolt to fix wheel in place.

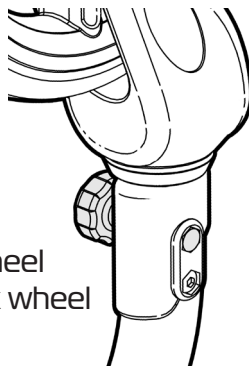




# FIT REAR WHEELS

Please follow these steps to fit the rear wheels:

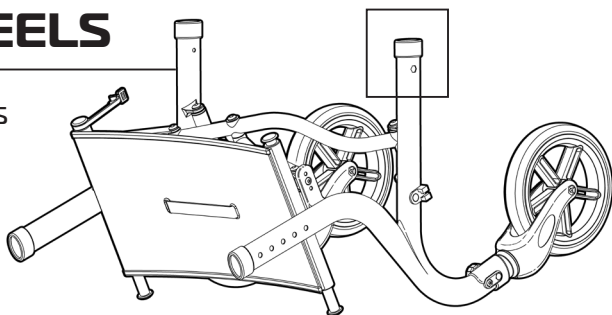


1. Carefully position frame as shown.

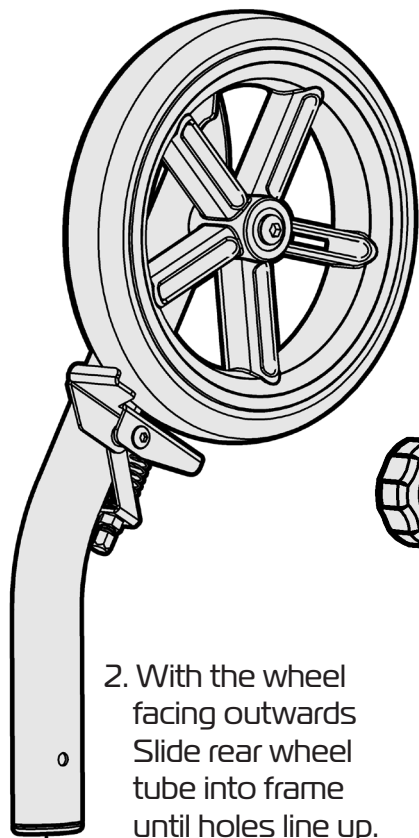

2. With the wheel facing outwards Slide rear wheel tube into frame until holes line up.



3. Insert SHORT bolt into hole. Align hex head of bolt with hex hole in tube.

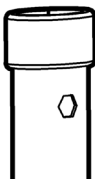



4. Wind hand wheel onto bolt to fix wheel assembly in place.

6.


# FIT HANDLES

Please follow these steps to fit the handles:

1. Carefully position the frame onto the wheels.
2. Determine the correct handle assembly prior to fitting.  
Hand grip must be facing to the rear and the brake cable is inline with the rear wheel.
3. Slide the handle tube into the frame as shown.

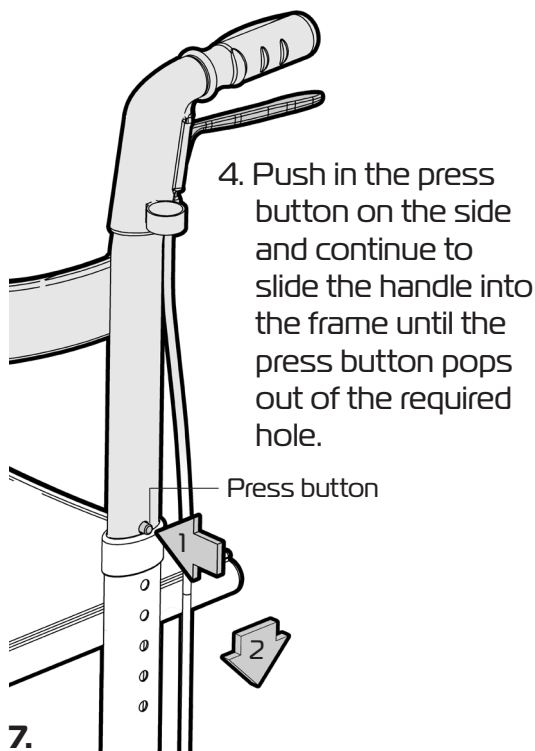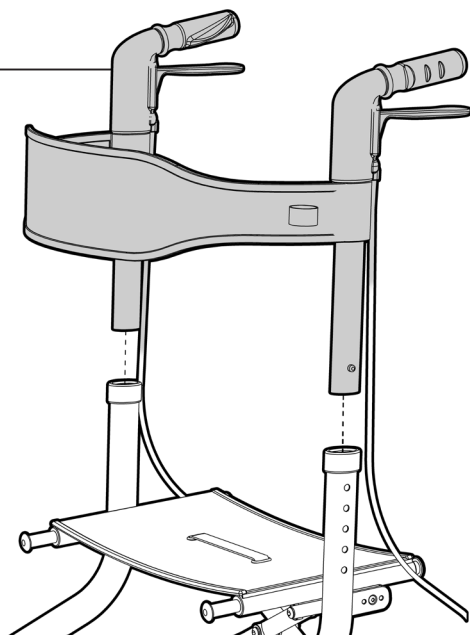

4. Push in the press button on the side and continue to slide the handle into the frame until the press button pops out of the required hole.

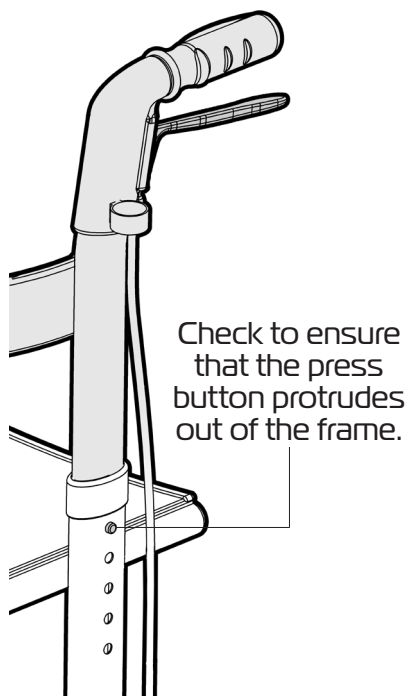



Check to ensure that the press button protrudes out of the frame.

# ADJUST HANDLES

Please follow these steps to adjust the handles:

It is very important that the handles are set to the correct height to suit the user.

Stand behind the seat walker with your shoulders relaxed and your arms hanging loosely at your sides. The height of the handles should be adjusted so that they are level with your wrists. This will position the arms at the recommended 20 degree bend in the elbow when using a seat walker.

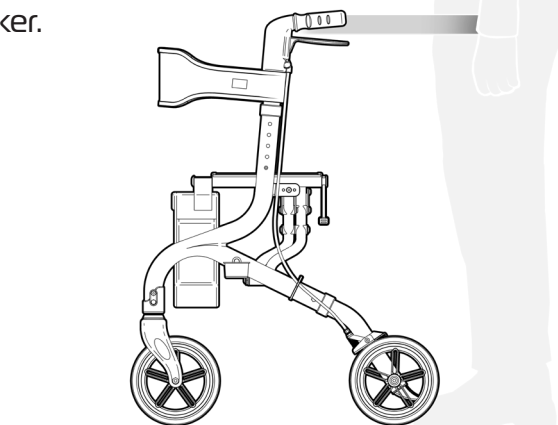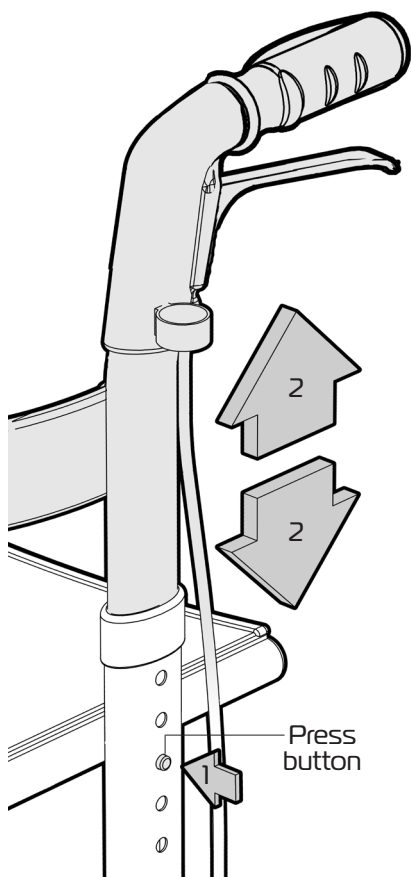

1. Push in the press button on the side and slide the handle as required. The press button will pop out of every hole setting as you adjust the handle height so you will need to push the press button again until you reach the height setting required.


Check to ensure that the press button protrudes out of the frame.

Ensure both handles are set at the same height prior to use.

8.


# FIT BAG

Please follow these steps to fit the storage bag:

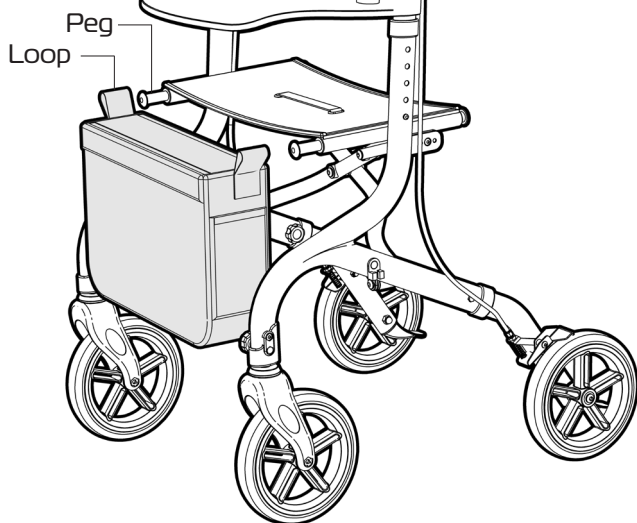

1. Align the bag so the high visibility strip is facing toward the front of the walker.

2. Slide the loops on the sides of the bag onto the pegs on the ends of the seat tubes.

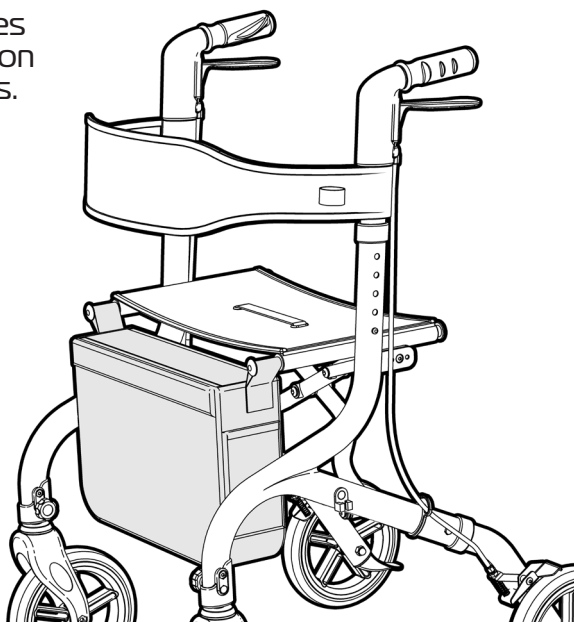

9.




ASK FOR THE BRAND

[redgumbrandmobility.com.au](http://redgumbrandmobility.com.au)

# FIT WALKING STICK CUP

Please follow these steps to fit the walking stick cup:

1. Locate the walking stick cup onto the mount bracket located on the side of the seat walker frame.

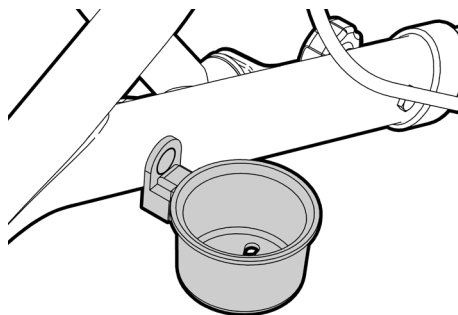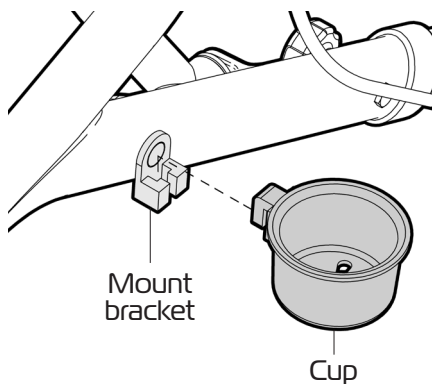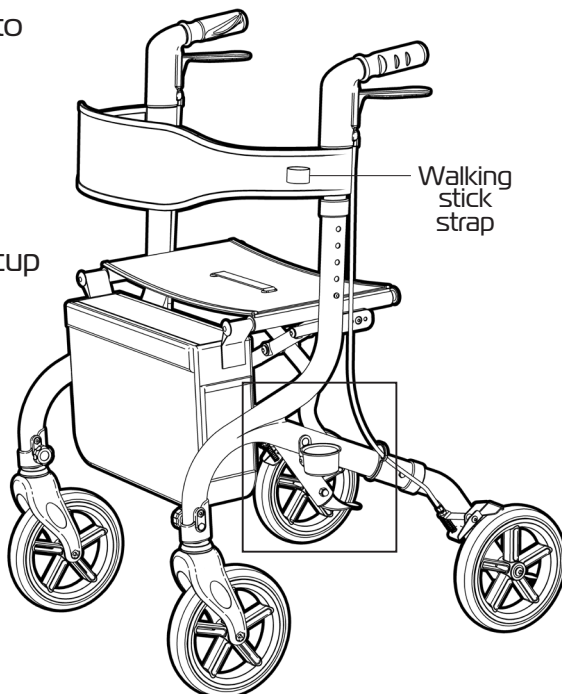

2. Once located on the mount bracket rotate the cup to be inline with the walking stick strap.

# USAGE

---

Please follow these guidelines to operate the ULTRA Lightweight Seat Walker in a safe manner. Failure to follow these guidelines may result in damage to the product and put the user at risk.

Do not overload the seat walker (**136kg max. load**)


Do not overload the bag (**5kg max. load**)

Pinch points do exist on this product. Please keep hands and any other body parts away from moving parts when folding the seat walker.

We recommend that the park brakes are engaged when the user is getting on or off the seat walker.


Go up or down inclines directly. Do not try to cross on an angle as this will cause instability and the seat walker may tip over.

All four wheels should always remain on the ground unless mounting kerbs.

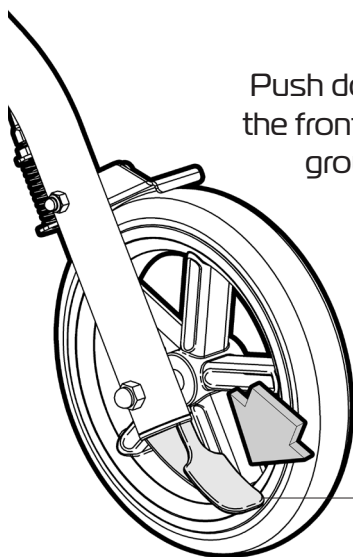

Push down on the foot peg to lift the front of the seat walker off the ground to get over kerbs.

Foot  
peg

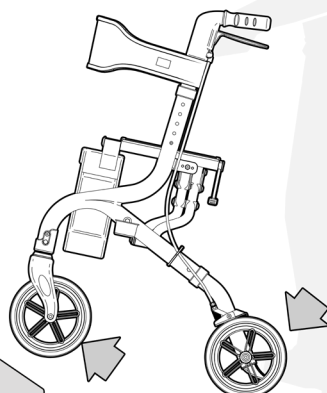


# BRAKING

---

Please follow these guidelines to brake the seat walker in a safe manner:

Both hand brakes should be used simultaneously to ensure the seat walker stops as required and the user is in full control.

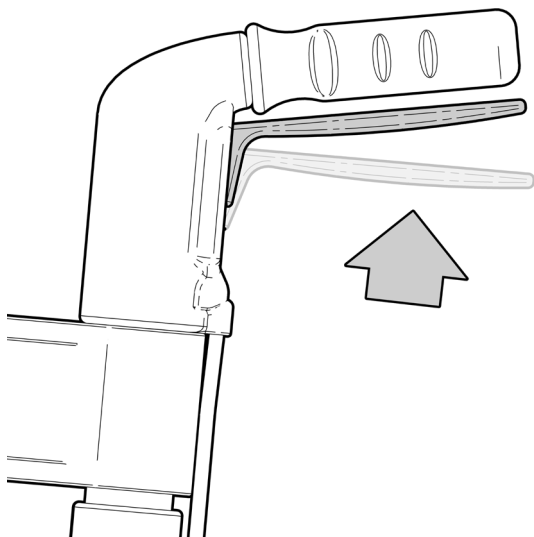


## **SLOW DOWN AND STOP**

To slow down or stop the seat walker from rolling:

Gently pull up on both hand brakes simultaneously.

Release both hand brakes to allow the seat walker to move again.

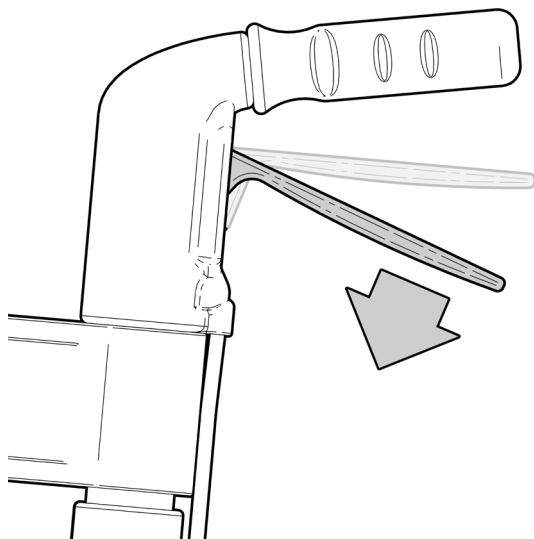

## **PARK BRAKE**

To engage the park brakes on the seat walker:

Gently push down on both hand brakes simultaneously until a loud click is heard.


To release park brakes:

Pull up on both hand brakes simultaneously until a loud click is heard.


# FOLDING WALKER

Please follow these steps to fold the seat walker:

1. Pull up on the handle in the middle of the seat to fold up the seat walker.

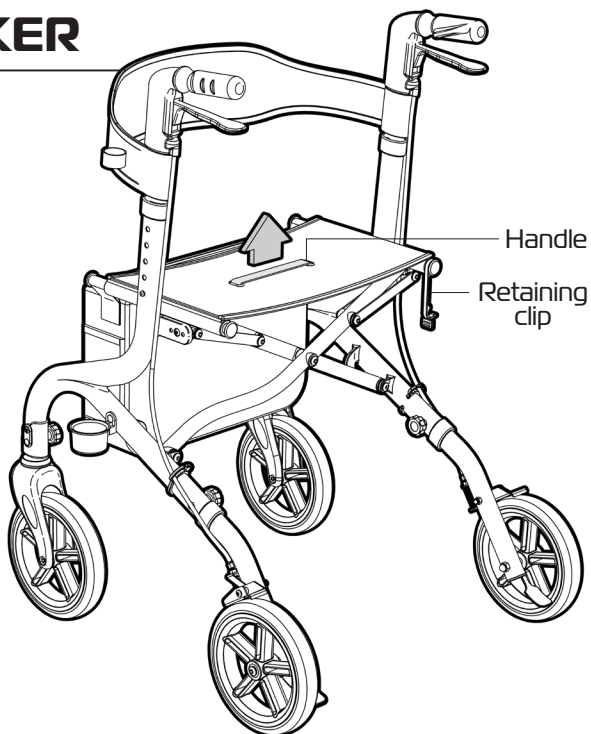

2. Push the handles together and rotate the retaining clip over to hold the walker closed.

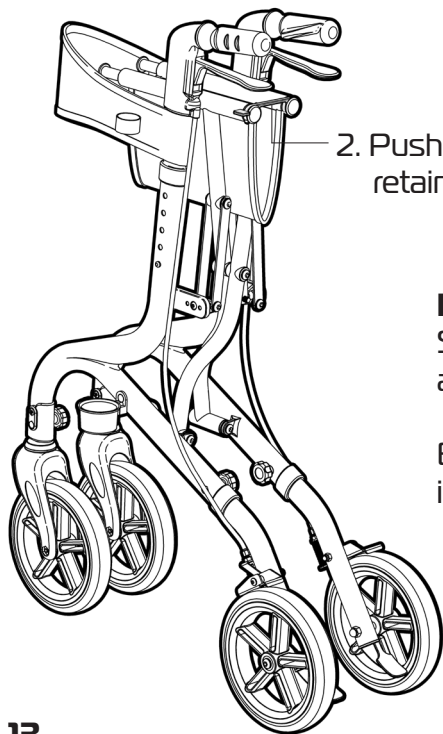

## Important Note:

Seat walker can become unstable and fall over when folded up.

Ensure the front swivel wheels are inline with the seat walker when folded.



# PRODUCT CARE

---

## INSPECTIONS

We suggest that you visually check the ULTRA Lightweight Seat Walker periodically for signs of damage or wear prior to use, paying particular attention to the following areas:

**Framework:** Check for damage or distortion.

**Wheels:** Check to ensure the seat walker rolls as required.

**Brakes:** Check to ensure both brakes work as required

**Hand grips:** Check for cracks / tears in material.

Overall check that no components are loose or worn and that the seat walker operates as required.



**Withdraw the seat walker immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for support.**

## CLEANING

Clean the ULTRA Lightweight Seat Walker using disinfectant wipes. We do not recommend cleaning the product with soap and water.

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Do not use any abrasive cleaners or abrasive cleaning pads as they could potentially damage the seat walker.

If you have any questions about the cleaning of the ULTRA Lightweight Seat Walker please contact your supplier or REDGUM brand direct.

# WARRANTY

---

The REDGUM Brand ULTRA Lightweight Seat Walker (RG4410) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the ULTRA Lightweight Seat Walker in any way or fit any parts or accessories that are not specified as an option for the product. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the ULTRA Lightweight Seat Walker please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**      E: **orders@for-de.com.au**

**15.**

*ASK FOR THE BRAND*

**redgumbrand**mobility.com.au

# REDGUM Brand - Product Range

---

REDGUM Brand offer a comprehensive range of products including:

**Seat Walkers**

**Side Folding Seat Walkers**

**Walking Frames**

**Wheelchairs**

**Walking Sticks + Quads**

**Crutches**

**Reachers**

**Shower / Bathroom products**

**Toilet products**


**Cushions: Air / Wedge / Foam**

**Electric Lift Chairs**

**Electric Beds / Bed accessories**

**Aluminium Ramps**

**Pedal Exercisers**

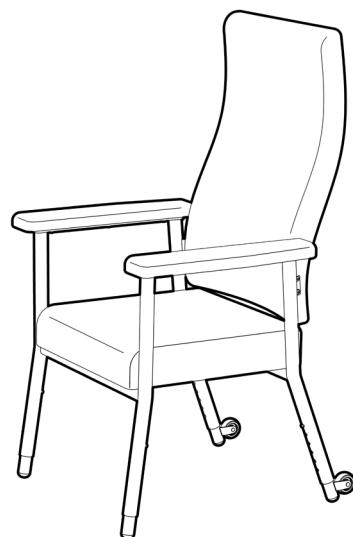

REDGUM Brand stock a large range of spare parts for all REDGUM brand products.

Please contact **REDGUM Brand** for further information.

**DISTRIBUTOR:**



Explore our entire  
range of products!