

# ENDURO SUPER LITE CARBON FIBRE SEAT WALKER

Standard  
RG4414

Low Seat  
RG4414LS



## USER MANUAL

---

Please read and keep these instructions



# **REDGUM** BRAND

*Daily Living and  
Lifestyle Products*

**REDGUM Brand** owns the copyright to the ENDURO Super Lite Carbon Fibre Seat Walker user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

**REDGUM Brand** strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**



**REDGUM Brand** is part of **FOR-DE Group Pty Ltd**


ASK FOR THE BRAND

[redgumbrand.com.au](http://redgumbrand.com.au)

# CONTENTS

---

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **USAGE GUIDELINES** - UNPACK and INSPECT SUPPLIED PARTS
4. **USAGE GUIDELINES** - HANDLE ADJUSTMENT
5. **USAGE GUIDELINES** - UNFOLD WALKER
6. **USAGE GUIDELINES** - UNFOLD WALKER (continued)
7. **USAGE GUIDELINES** - BRAKING
8. **USAGE GUIDELINES** - USING THE SEAT WALKER
9. **USAGE GUIDELINES** - SAFETY PRECAUTIONS
10. **PRODUCT CARE** - INSPECTIONS / CLEANING
11. **WARRANTY**
12. **REDGUM PRODUCT RANGE**

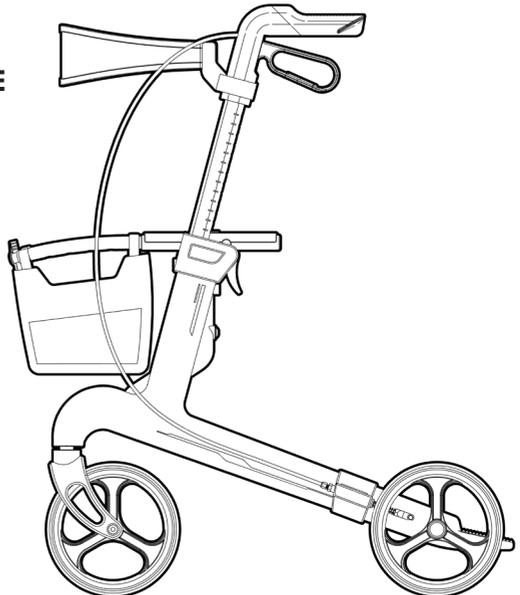

# INTRODUCTION

---

Thank you for purchasing this REDGUM Brand product.

If you have any issues or questions about the ENDURO Super Lite Carbon Fibre Seat Walker please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

## INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your ENDURO Super Lite Carbon Fibre Seat Walker for its intended purpose which is as a walking and sitting aid that can adjust to suit the users individual requirements.

Suitable for indoor and outdoor use.

### IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

#### **Maximum User Weight: 130kg**

Exceeding the weight limit could cause damage to the product and could put the user at risk.



1.


*ASK FOR THE BRAND*

**redgumbrand.com.au**

# PRODUCT OVERVIEW

---

- Carbon Fibre frame
- Height adjustable handles
- Ergonomic handgrips
- User friendly loop brake handles
- Brake + park brakes
- Mesh seating
- Padded Backrest
- Large wheels for excellent mobility
- Removable bag with carry handles
- Folds up easily for transit or storage
- Carry handle

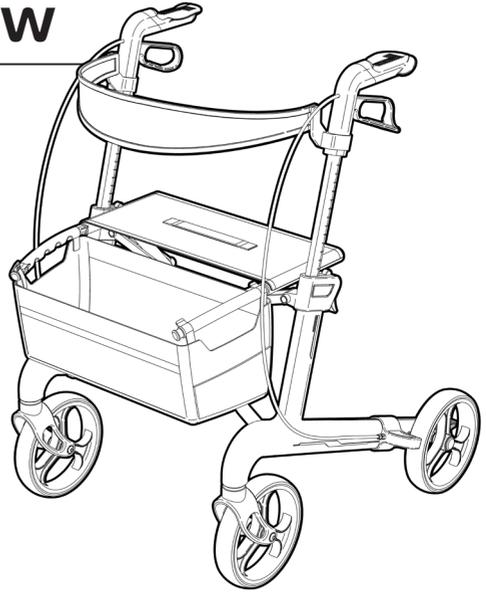

## SPECIFICATIONS (All dimensions shown are nominal)

Overall: **700mm L x 605mm W x 910mm H** (at lowest setting)

Overall Folded: **700mm L x 255mm W x 725mm H**

Handle Height: **725 - 910mm (13 settings)**

Seat: Standard - **450mm W x 195mm D x 580mm H** (floor to seat)  
Low Seat - **450mm W x 195mm D x 540mm H** (floor to seat)

Wheel Diameter: **Ø200mm / 8"**

Carry Bag: **350mm L x 230mm W x 190mm D**

Product Weight: **5.9kg (without bag)**

Maximum User Weight: **130kg**


Maximum Bag Loading Weight: **5kg**

Please contact **REDGUM Brand** if you would like any further information on the ENDURO Super Lite Carbon Fibre Seat Walker.

# ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

---

Ensure you have been supplied with the following parts:

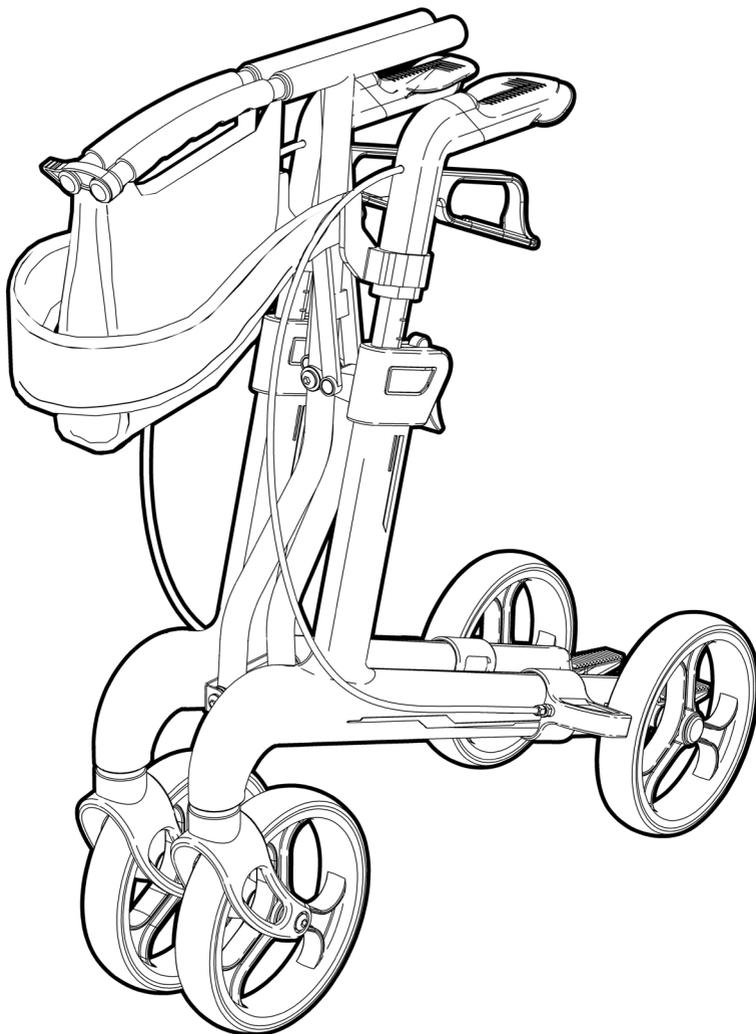

## IMPORTANT NOTE

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

3.

ASK FOR THE BRAND

[redgumbrand.com.au](http://redgumbrand.com.au)

# ASSEMBLY - HANDLE ADJUSTMENT




It is very important that the handles on the ENDURO Super Lite Carbon Fibre Seat Walker are set to the correct height to suit the user.

Stand behind the seat walker with your shoulders relaxed and your arms hanging loosely at your sides.

The height of the handles should be adjusted so that they are level with your wrists.

This will position the arms at the recommended 20 degree bend in the elbow when using a seat walker.

To adjust the handle heights on the ENDURO Super Lite Carbon Fibre Seat Walker please follow these steps:

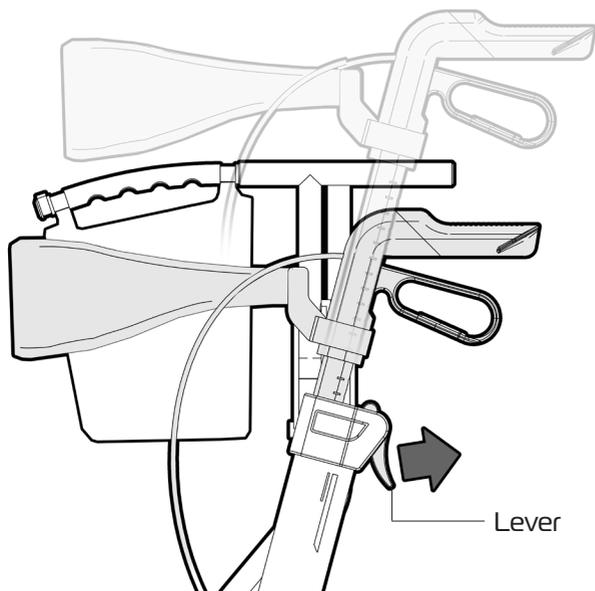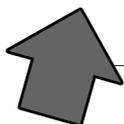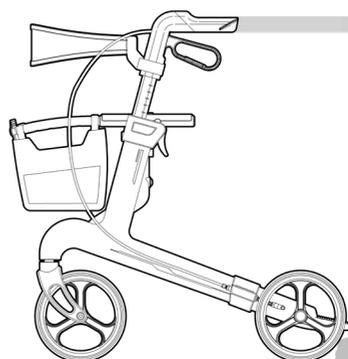

1. Pull and hold the bottom of the lever out.
2. Move the handle to the required height. (You will hear a clicking noise as it passes each height setting point)
3. Release the lever and move the handle until it locates into the nearest height setting hole.


Check to ensure that both handles are securely set at the correct height prior to use.

**4.**



# ASSEMBLY - UNFOLD WALKER

The ENDURO Super Lite Carbon Fibre Seat Walker has a clip that holds the walker together when folded.

The holding clip **MUST** be unclipped prior to unfolding the seat walker.

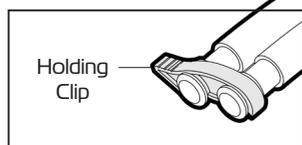

**Note:**  
Carry bag  
and  
backrest strap  
are not shown  
for clarity

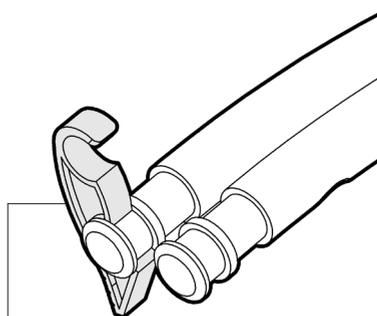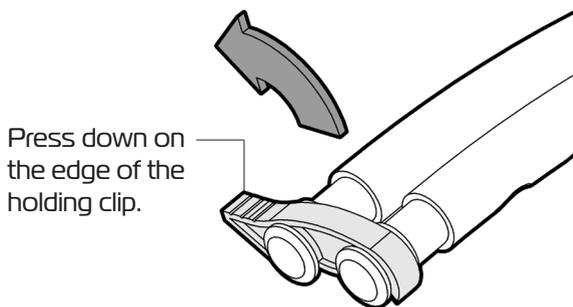

Rotate the Holding Clip away so the seat walker frame can be unfolded.

5.

ASK FOR THE BRAND

redgumbrand.com.au


# ASSEMBLY - UNFOLD WALKER

To unfold the ENDURO Super Lite Carbon Fibre Seat Walker please follow these steps:

1. Pull out on both of the handles at the same time to open out the seat walker.
2. Push down firmly on the outside edges of seat until two loud clicks are heard.
3. Check to ensure seat walker is fully unfolded and locked in position prior to use.


To fold seat walker:

- Remove all items from the carry bag.
- Pull up on the handle in the middle of the seat.
- Hold the frame together using the holding clip.

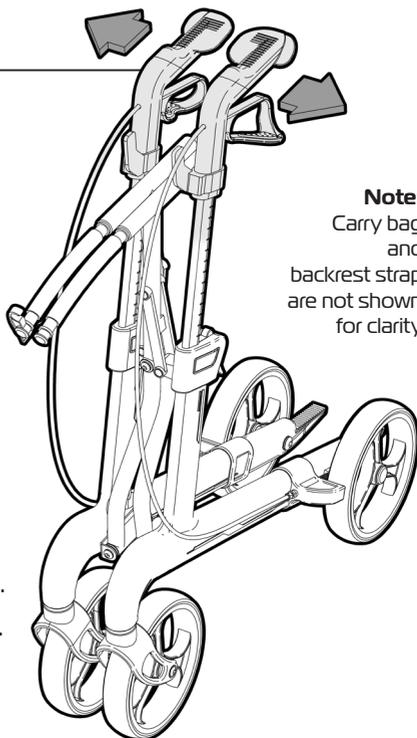


**Note:**  
Carry bag  
and  
backrest strap  
are not shown  
for clarity

**PULL OUT  
ON HANDLES**

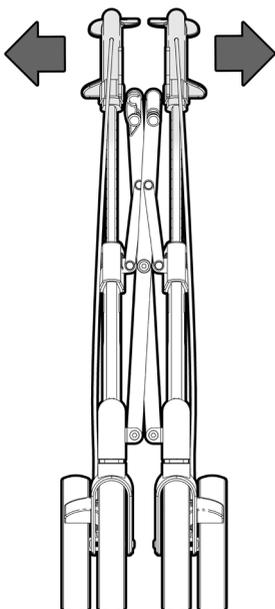

**PUSH DOWN  
ON SIDES  
OF SEAT**

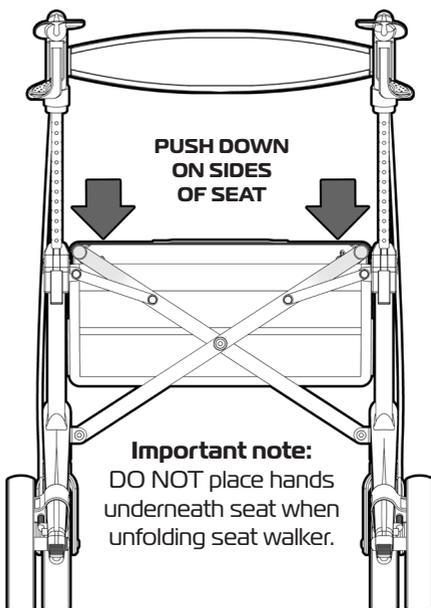

**Important note:**  
DO NOT place hands  
underneath seat when  
unfolding seat walker.

# USAGE GUIDELINES - BRAKING

The ENDURO Super Lite Carbon Fibre Seat Walker has independant hand brakes on both handles.

## IMPORTANT NOTE:


Both hand brakes should be used simultaneously to ensure the seat walker stops as required and the user is in full control.

## SLOW DOWN AND STOP

To slow down or stop the seat walker from rolling:

Gently pull up on both hand brakes simultaneously.

Release both hand brakes to allow the seat walker to move again.

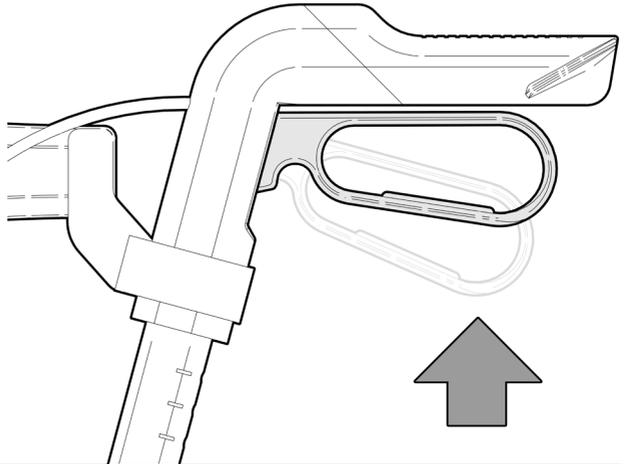

## PARK BRAKE


To engage the park brakes on the seat walker:

Gently push down on both hand brakes simultaneously until a loud click is heard.

This will stop the seat walker from rolling.

To release park brakes:

Pull up on both hand brakes simultaneously until a loud click is heard.

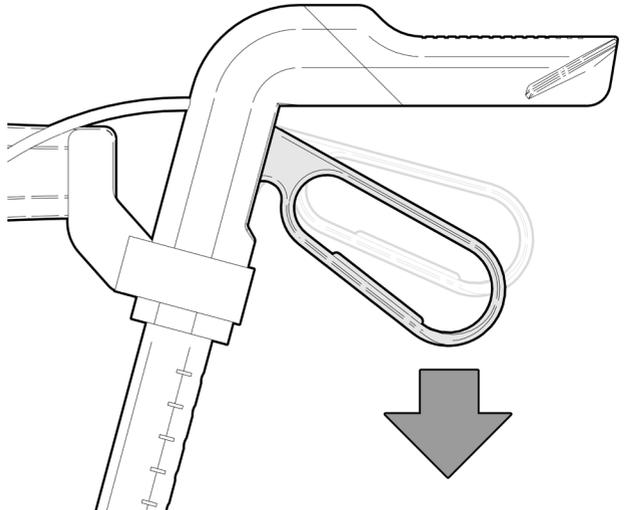

7.

ASK FOR THE BRAND

redgumbrand.com.au


# USAGE GUIDELINES - USING THE SEAT WALKER

Use the seat walker as a walking aid.


Use the seat walker as a sitting aid.

Adjust both handles to suit the user (refer to page 4.)

Store items in the carry bag

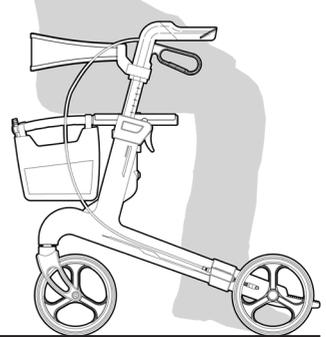

Push down on the rear of side frame to lift the front of the seat walker off the ground to get over kerbs.

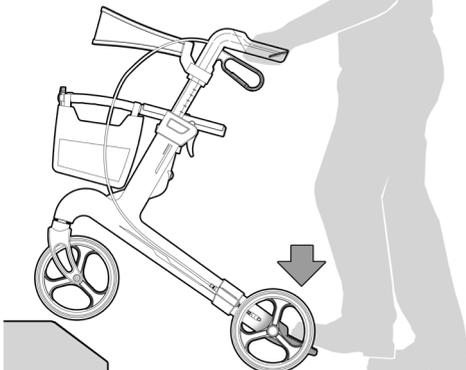

Fold up the seat walker and use the carry handles to lift walker.



8.

# USER GUIDELINES - SAFETY PRECAUTIONS

---

Please follow these guidelines to get the optimal benefit from the ENDURO Super Lite Carbon Fibre Seat Walker. Failure to follow these guidelines may result in damage to the product and put the user at risk.

The ENDURO Super Lite Carbon Fibre Seat Walker is designed to take a maximum Load of 130kg.

- Do not overload the seat walker.
- Do not overload the bag.  
(5kg maximum loading)

Pinch points do exist on this product. Please keep hands and any other body parts away from moving parts when folding the seat walker.

We recommend that the park brakes are engaged when the user is getting on or off the seat walker.

Please take extra care when taking the seat walker offroad.

Go up or down inclines directly. Do not try to cross on an angle as this will cause instability and the seat walker may tip over.

All four wheels should always remain on the ground when possible.

The seat walker is not intended to be self propelled while seated.

## IMPORTANT NOTE

Do not fit any parts or accessories to the ENDURO Super Lite Carbon Fibre Seat Walker that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the ENDURO Super Lite Carbon Fibre Seat Walker please contact your supplier or REDGUM Brand direct.



# PRODUCT CARE - INSPECTIONS / CLEANING

## INSPECTIONS

We suggest that you visually check the ENDURO Super Lite Carbon Fibre Seat Walker periodically for signs of damage or wear prior to use, paying particular attention to the following areas:

**Framework:** Check for damage or distortion.

**Wheels:** Check to ensure the seat walker rolls as required.

**Brakes:** Check to ensure both brakes work as required (brake and park brake).

**Hand grips:** Check for cracks / tears in material.

Overall check that no components are loose or worn and that the seat walker operates as required.



**Withdraw the seat walker immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for service support.**

## CLEANING

Clean the ENDURO Super Lite Carbon Fibre Seat Walker using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the seat walker and should not be used.

If you have any questions about the cleaning of the seat walker please contact your supplier or **REDGUM Brand** direct.



# WARRANTY

---

The REDGUM Brand:

- ENDURO Super Lite Carbon Fibre Seat Walker (Standard: RG4414)
  - ENDURO Super Lite Carbon Fibre Seat Walker (Low Seat: RG4414LS)
- are covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the seat walker in any way or fit any parts or accessories that are not specified as an option for the product. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the seat walker please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga  
Western Australia 6090**

**11.**



*ASK FOR THE BRAND*

**redgumbrand.com.au**

# REDGUM Brand - Product Range

---

REDGUM Brand offer a comprehensive range of products including:

**Seat Walkers**

**Side Folding Seat Walkers**

**Walking Frames**

**Wheelchairs**

**Walking Sticks + Quads**

**Crutches**

**Reachers**

**Shower / Bathroom products**

**Toilet products**

**Cushions: Air / Foam**


**Electric Lift Chairs**

**Electric Beds**

**Aluminium Ramps**

**Pedal Exercisers**

**Personal Alarms**

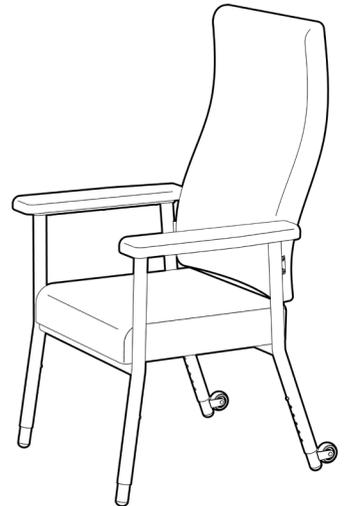

REDGUM Brand stock a large range of Spare Parts for all REDGUM brand products.

Please contact **REDGUM Brand** for further information.

**DISTRIBUTOR:**

**REDGUM** BRAND  
*Daily Living and  
Lifestyle Products*



**Proudly  
Australian Owned  
and Operated**

**ASK FOR THE BRAND**

**redgumbrand.com.au**