

REDGUMbrand
ASSISTED LIVING PRODUCTS

FOLDING TOILET SEAT RAISER

RG518

Edition 1




R REGISTERED
DESIGN

USER MANUAL

Please read and keep these instructions

ASK FOR THE BRAND

redgumbrand.com.au

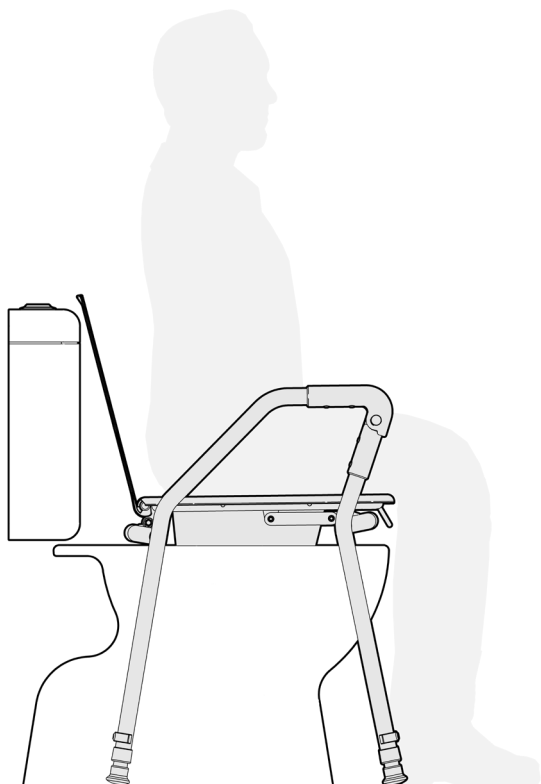

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Folding Toilet Seat Raiser user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**



REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION:** INTENDED USE
2. **PRODUCT OVERVIEW:** FEATURES / SPECIFICATIONS
3. **SET UP:** UNFOLD TOILET SEAT RAISER
4. **SET UP:** ADJUST HEIGHT
5. **USAGE:** SPLASHGUARD
6. **USAGE GUIDELINES**
7. **INSPECTIONS / CLEANING**
8. **WARRANTY**



INTRODUCTION

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the Folding Toilet Seat Raiser please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180** E: **orders@for-de.com.au**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use.

The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use the Folding Toilet Seat Raiser for its intended purpose which is as a adjustable height seating solution to assist with sitting and standing during toileting.



IMPORTANT NOTE

This equipment **MUST** be set up by a competent person.

Maximum User Weight: 130kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

1.

ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

- Elevate the toilet seat and make it easier to transfer on and off the toilet
- Plastic flip up toilet seat and removable splashguard
- Flip up cover flap
- Height adjustable legs
- Non slip leg tips
- Easy to set up (No tools required)
- Easy to position and remove
- Can be folded with splashguard in place for storage or transit
- Can be used as a commode when fitted with optional bowl and lid (RGSB)

SPECIFICATIONS (All dimensions shown are nominal)

Overall unfolded (at lowest height setting)

555mm W x 490mm D x 625mm H

Width between handles: **490mm**

Seat depth: **400mm**

Seat heights (Floor to top of seat flap)

5 settings

Hole 1: **460mm**

Hole 2: **485mm**

Hole 3: **510mm**

Hole 4: **535mm**

Hole 5: **560mm**

Overall folded (at lowest height setting)

555mm W x 290mm D x 750mm H

Product Weight: **4.7kg**

Maximum User Weight: **130kg**



Please contact **REDGUM Brand** if you would like any further information on the Folding Toilet Seat Raiser.




SET UP: UNFOLD TOILET SEAT RAISER

Please follow these instructions to unfold the Toilet Seat Raiser:

1. Pull down on the handle to open out the frame.

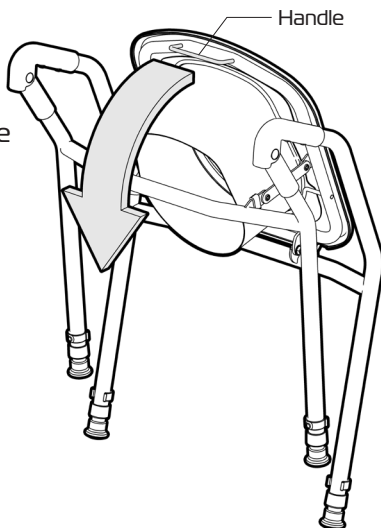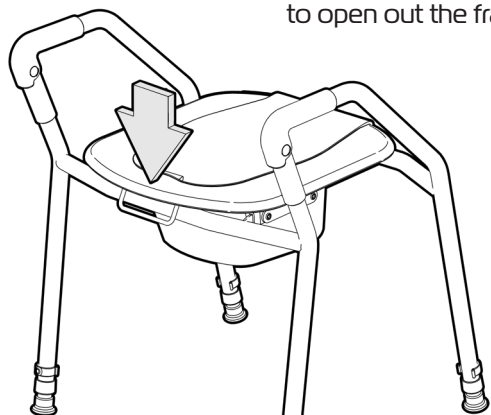

2. Push down on front of seat to ensure frame is fully open.

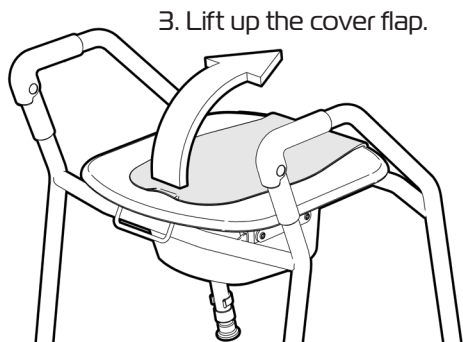

3. Lift up the cover flap.



4. Carefully place the Folding Toilet Seat Raiser centrally over the toilet. Ensure the splashguard is sitting inside the toilet bowl prior to use.

3.

ASK FOR THE BRAND

redgumbrand.com.au

SET UP: ADJUST HEIGHT

Please follow these instructions to adjust the height of the Toilet Seat Raiser to suit the user.

Adjust the legs so the top of the seat flap is inline with the back of the user knee.

Seat Heights
(Floor to top of seat)


Hole 1: **460mm**

Hole 2: **485mm**

Hole 3: **510mm**

Hole 4: **535mm**

Hole 5: **560mm**

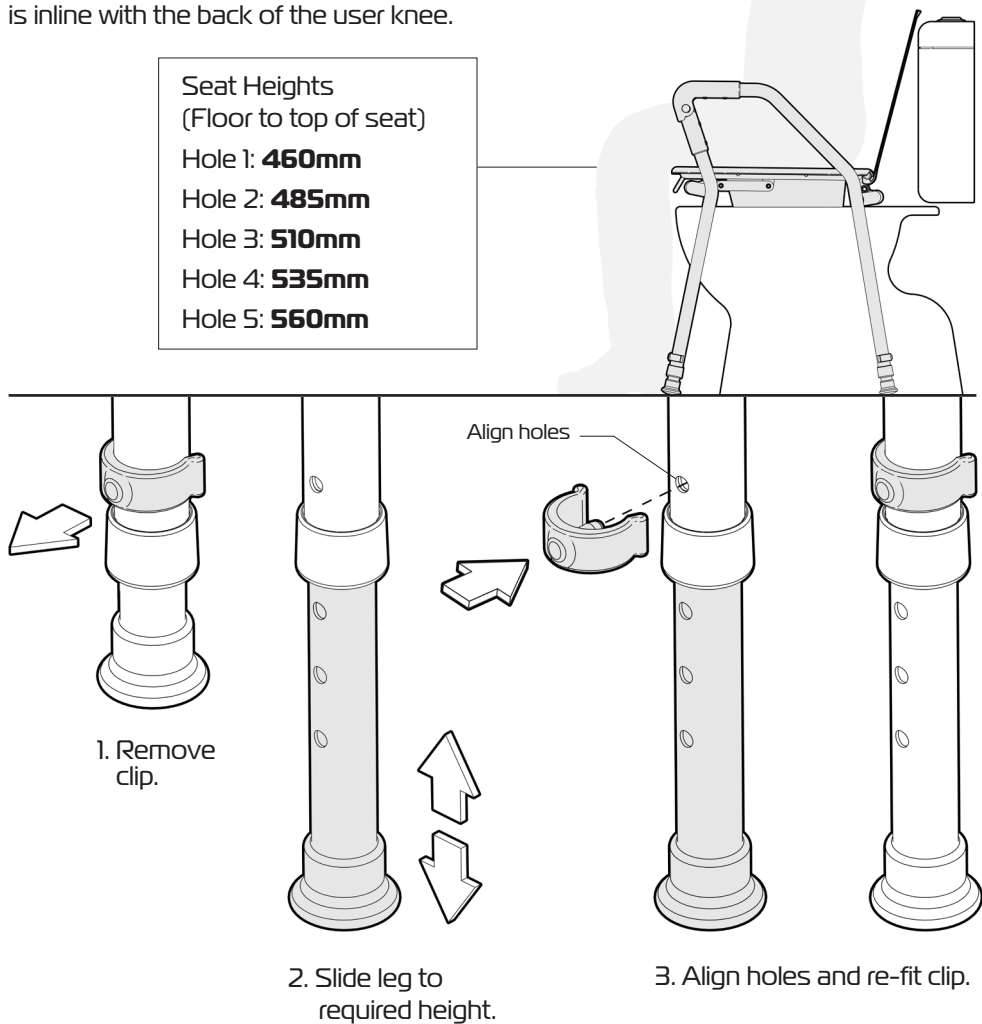

IMPORTANT NOTE


Ensure all legs are set at the same height prior to use.

4.

USAGE SPLASHGUARD

Please follow these instructions to access and remove the splashguard on the Folding Toilet Seat Raiser:

1. Gently rotate up the seat flap until it stays up on its own.


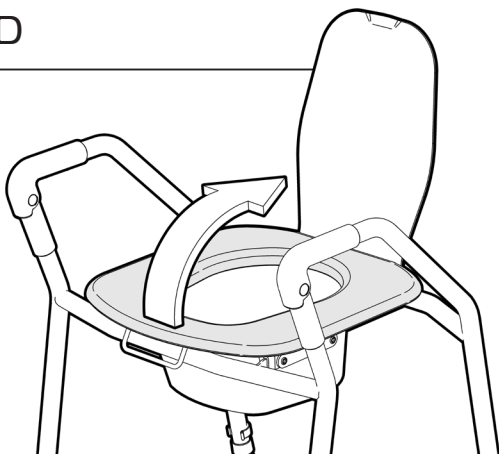


2. Remove the splashguard from the frame.

Finger cut outs are on either side of the splashguard for improved access.



The Folding Toilet Seat Raiser can be used as a commode by replacing the splashguard with a bowl and lid (RG5B)

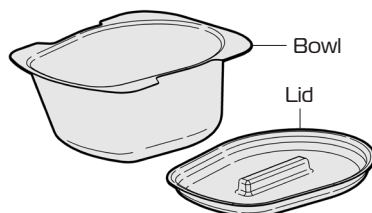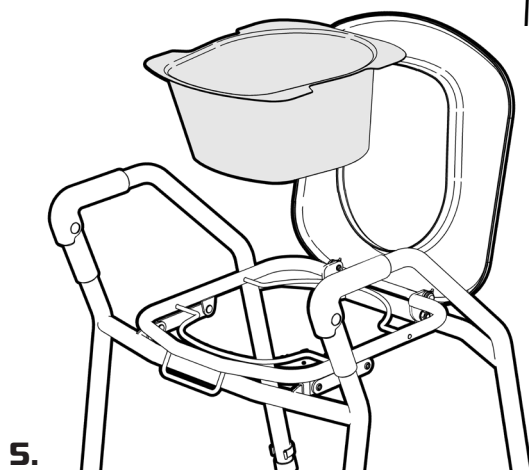

5.

ASK FOR THE BRAND

redgumbrand.com.au

USAGE GUIDELINES

Please follow these guidelines to get the optimal benefit from the Folding Toilet Seat Raiser.

Failure to follow these guidelines may result in damage to the toilet surround and put the user at risk.

Maximum Supportive Load: **130kg**


Ensure that the Folding Toilet Seat Raiser has been set up at the correct height to suit the users requirements.

Top of handles should be within reach when standing in front of the Folding Toilet Seat Raiser.



Always have both hands on the handles and ensure your body weight is central over the Folding Toilet Seat Raiser when getting on and off.

Ensure ALL FOUR LEGS remain on the ground at all times when the Folding Toilet Seat Raiser is being used.

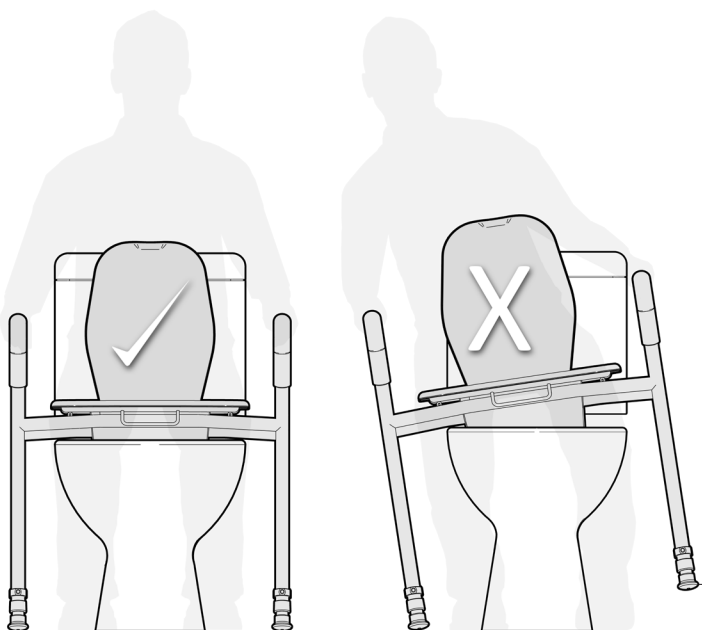

IMPORTANT NOTE

Do not lean too far to the side causing the legs to leave the ground.

6.

INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the Folding Toilet Seat Raiser periodically for signs of damage or wear prior to use. Overall check the following:

- Frame has no distortion.
- Plastic components are not damaged or cracked.
- Clips locate on frame as required and remain clipped around tube.



Withdraw the Dual Grab Folding Toilet Surround immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING

Clean the Folding Toilet Seat Raiser using disinfectant wipes.

We do not recommend cleaning the product with soap and water.

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Lift up the seat flap and remove the splashguard to clean.

Do not use any abrasive cleaners or abrasive cleaning pads as they could potentially damage the Folding Toilet Seat Raiser.

If you have any questions about the cleaning of the Folding Toilet Seat Raiser please contact your supplier or REDGUM brand direct.



WARRANTY

REDGUM Brand Folding Toilet Seat Raiser (RG518)
is covered by a 12 month warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if the product needs to be returned to the place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser.
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the Folding Toilet Seat Raiser in any way or fit any parts or accessories that are not specified as an option. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the Folding Toilet Seat Raiser please contact your supplier or REDGUM Brand direct on:

T: +61 8 9248 4180

E: orders@for-de.com.au



DISTRIBUTOR:

