


REDGUM brand
ASSISTED LIVING PRODUCTS



BARIATRIC TOILET SEAT RAISER

RG520


Edition 1

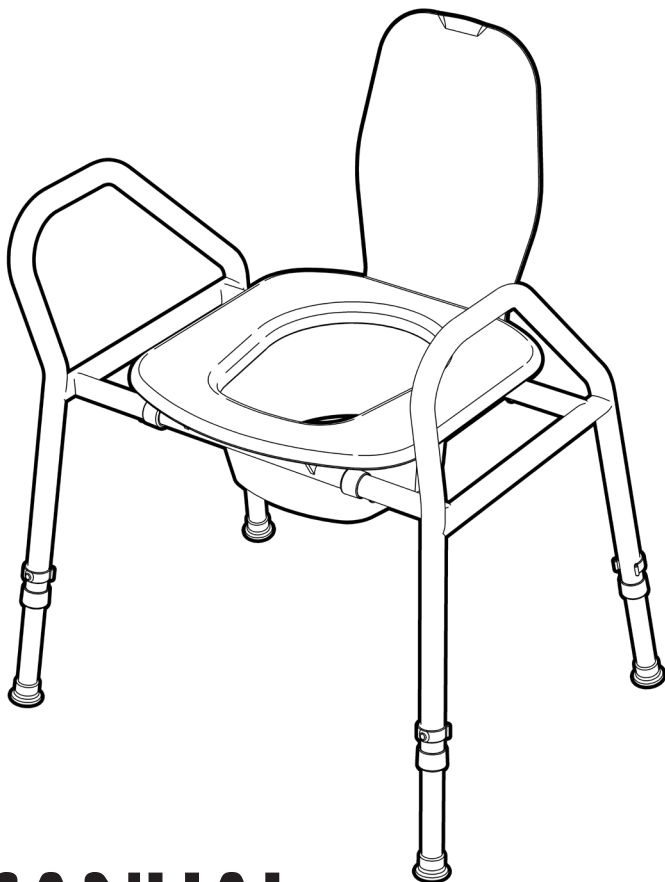

USER MANUAL

Please read and keep these instructions

redgumbrandmobility.com.au

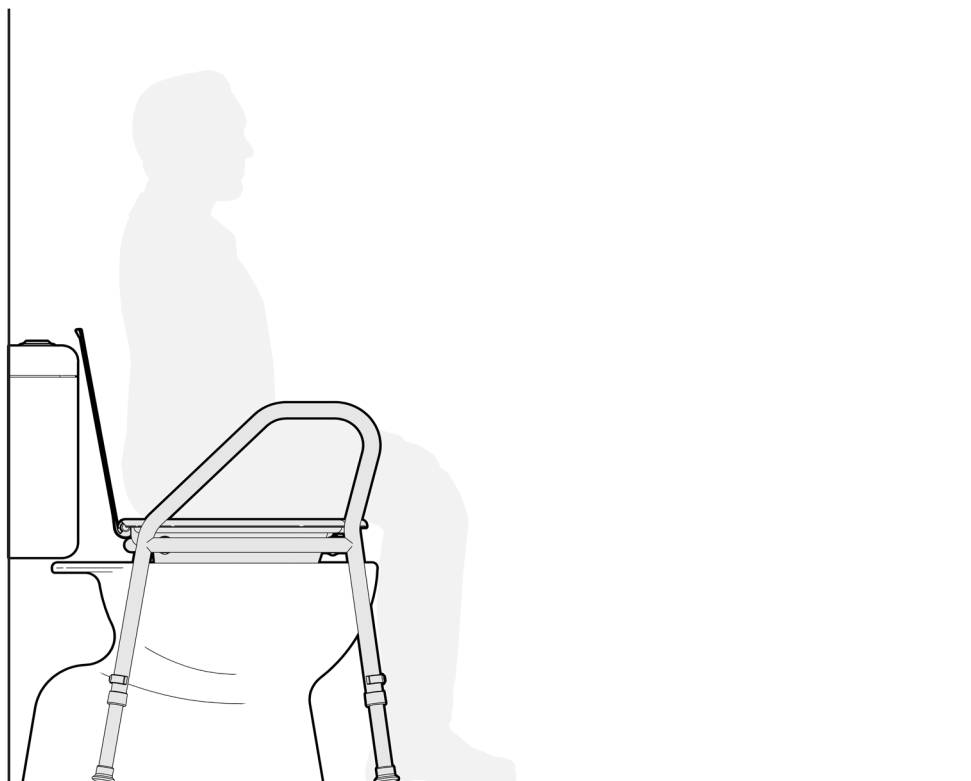

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Bariatric Toilet Seat Raiser user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**



REDGUM Brand is part of **FOR-DE Group Pty Ltd**

CONTENTS

1. **INTRODUCTION**
2. **PRODUCT OVERVIEW:** FEATURES / SPECIFICATIONS
3. **ASSEMBLY:** UNPACK / INSPECT PARTS
4. **ASSEMBLY:** FIT LEGS
5. **ASSEMBLY:** FIT LEGS continued
6. **ASSEMBLY:** FIT SIDE FRAMES
7. **ASSEMBLY:** FIT SIDE FRAMES continued
8. **ASSEMBLY:** FIT SIDE FRAMES continued
9. **USAGE:** GUIDELINES
10. **USAGE:** SPLASHGUARD ACCESS
11. **INSPECTIONS / CLEANING**
12. **WARRANTY**

INTRODUCTION

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the Bariatric Toilet Seat Raiser please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180** E: **info@redgumbrand.com.au**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use.

The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.


Only use the Bariatric Toilet Seat Raiser for its intended purpose which is as a adjustable height seating solution for toileting.

IMPORTANT NOTE

This equipment **MUST** be set up by a competent person.

Maximum User Weight: 160kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

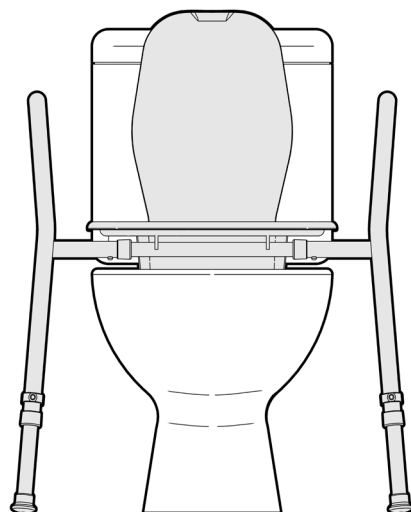

1.

ASK FOR THE BRAND

PRODUCT OVERVIEW

FEATURES

- Assist user when toileting by elevating the toilet seat height
- Armrests to help getting on and off the toilet
- Height adjustable seat
- Plastic flip up toilet seat flap and lid
- Removable splashguard
- Non slip leg tips
- Flat pack design
- Tool free assembly
- Can be converted to a commode
RGSB: Bowl and lid (sold separately)



SPECIFICATIONS (All dimensions shown are nominal)

Overall (at lowest height setting)

625mm W x 495mm D x 650mm H

Width between handles: **500mm**

Seat depth: **400mm**

Seat heights (Floor to top of seat flap)

5 settings

Hole 1: **445mm**

Hole 2: **470mm**

Hole 3: **495mm**


Hole 4: **520mm**

Hole 5: **545mm**

Product Weight: **5kg**

Maximum User Weight: **160kg**

Please contact **REDGUM Brand** if you would like any further information on the Bariatric Toilet Seat Raiser.

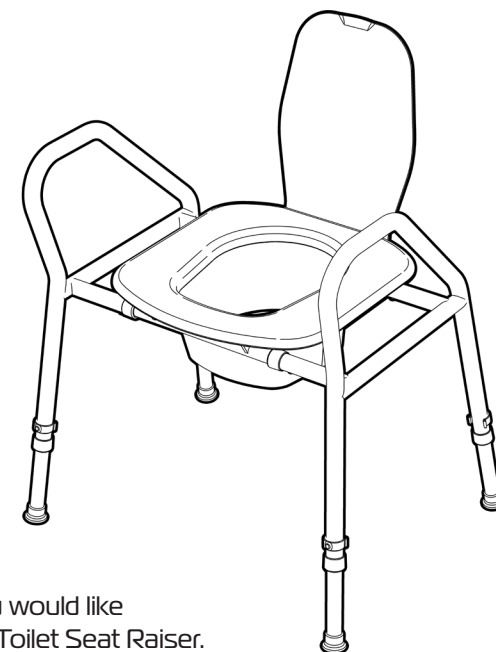


2.

redgumbrandmobility.com.au

ASSEMBLY UNPACK AND INSPECT PARTS

Check to ensure that the following parts have been supplied prior to assembly:

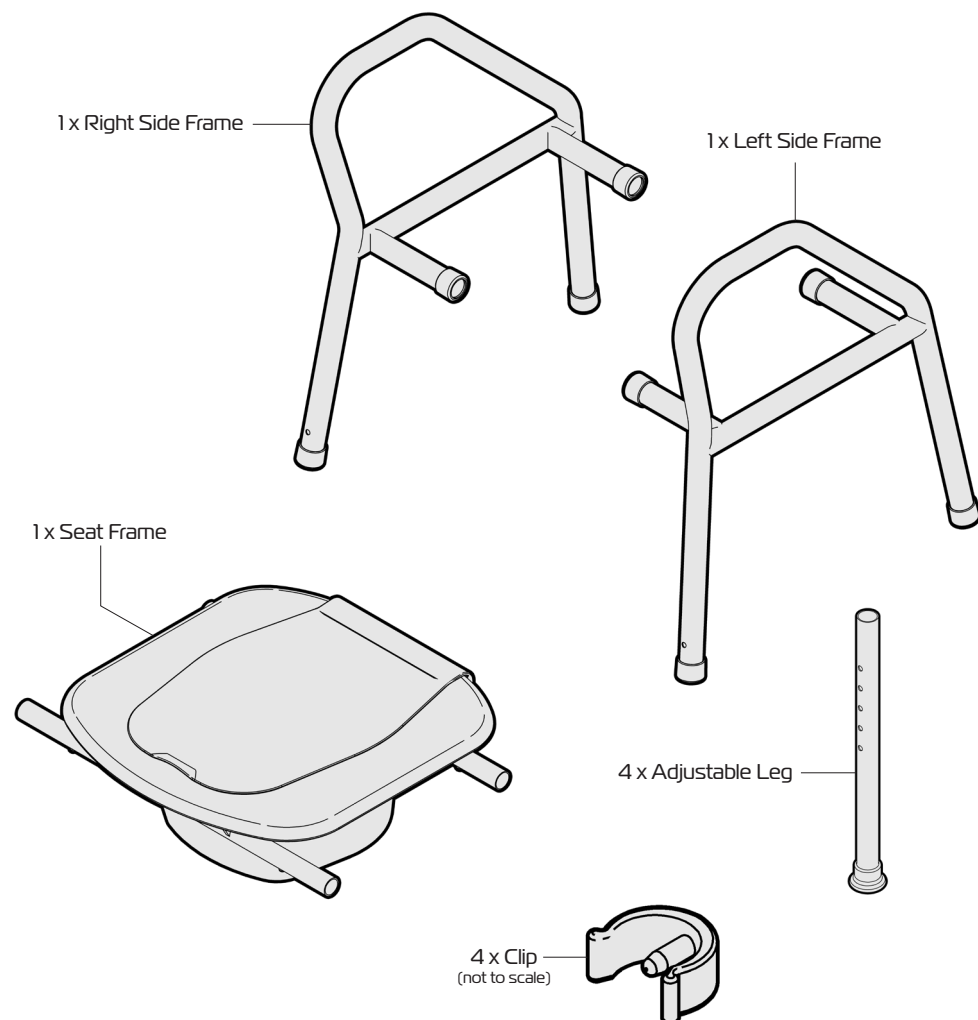

IMPORTANT NOTE

Check the parts for any visible damage. If you see any damage or suspect a fault, **DO NOT USE THE PRODUCT** and contact your supplier for support.

3.

ASK FOR THE BRAND

ASSEMBLY FIT LEGS

The Bariatric Toilet Seat Raisher has height adjustable legs that can be set to the correct height to suit the user.

All four legs should be set so the top of the seat flap is inline with the back of the users knee. Measure from the floor to the back of the users knee and adjust each leg to the closest hole setting shown below.

Seat Height Setting
(Floor to top of seat flap)


Hole 1: **445mm**

Hole 2: **470mm**

Hole 3: **495mm**


Hole 4: **520mm**

Hole 5: **545mm**

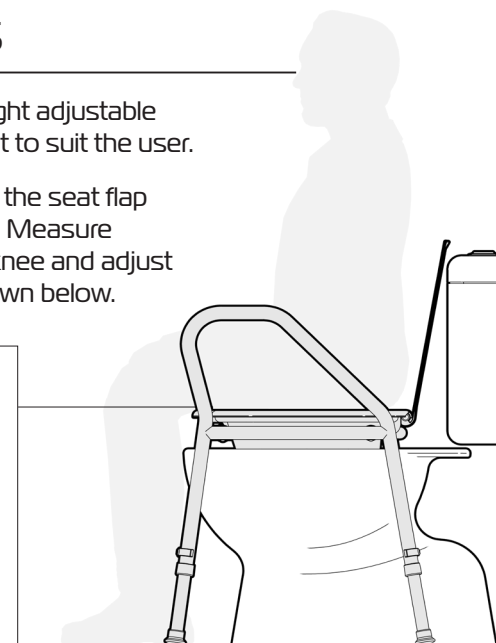

Please follow these instructions to fit the legs to the side frames .

Refer to next page for further instruction....

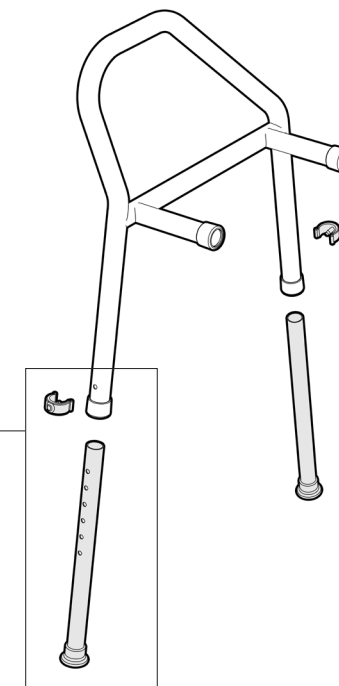

4.

redgumbrandmobility.com.au

ASSEMBLY FIT LEGS continued

1. Line up the holes in leg with hole in side frame.

Determine required height setting:

Hole 5 (545mm)
Hole 4 (520mm)
Hole 3 (495mm)
Hole 2 (470mm)
Hole 1 (445mm)

2. Slide the leg in until the required hole lines up with the hole in the side frame.
At the same time - Insert clip pin into hole and push clip in until it locates around tube.

Repeat process until all 4 legs are fitted.

Ensure each leg is set to the same height.

5.



ASK FOR THE BRAND

ASSEMBLY FIT SIDE FRAMES

Please follow these instructions to fit the side frames to the seat frame.

Determine the correct side frame before fitting to the seat frame.

Armrest on side frame must be angling towards the FRONT of the Toilet Seat Raiser.

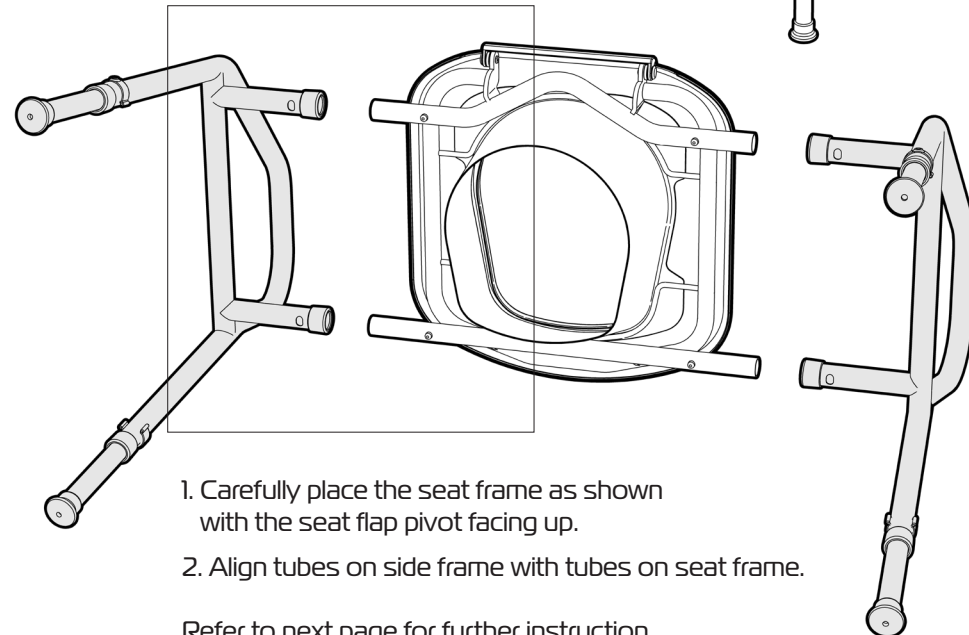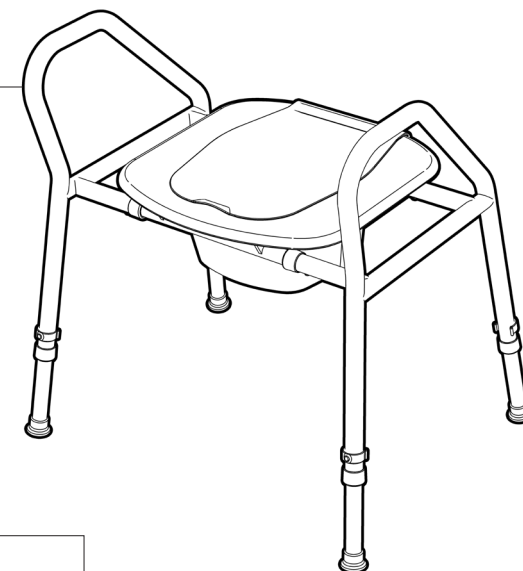

1. Carefully place the seat frame as shown with the seat flap pivot facing up.


2. Align tubes on side frame with tubes on seat frame.

Refer to next page for further instruction....

6.

redgumbrandmobility.com.au

ASSEMBLY FIT SIDE FRAMES continued

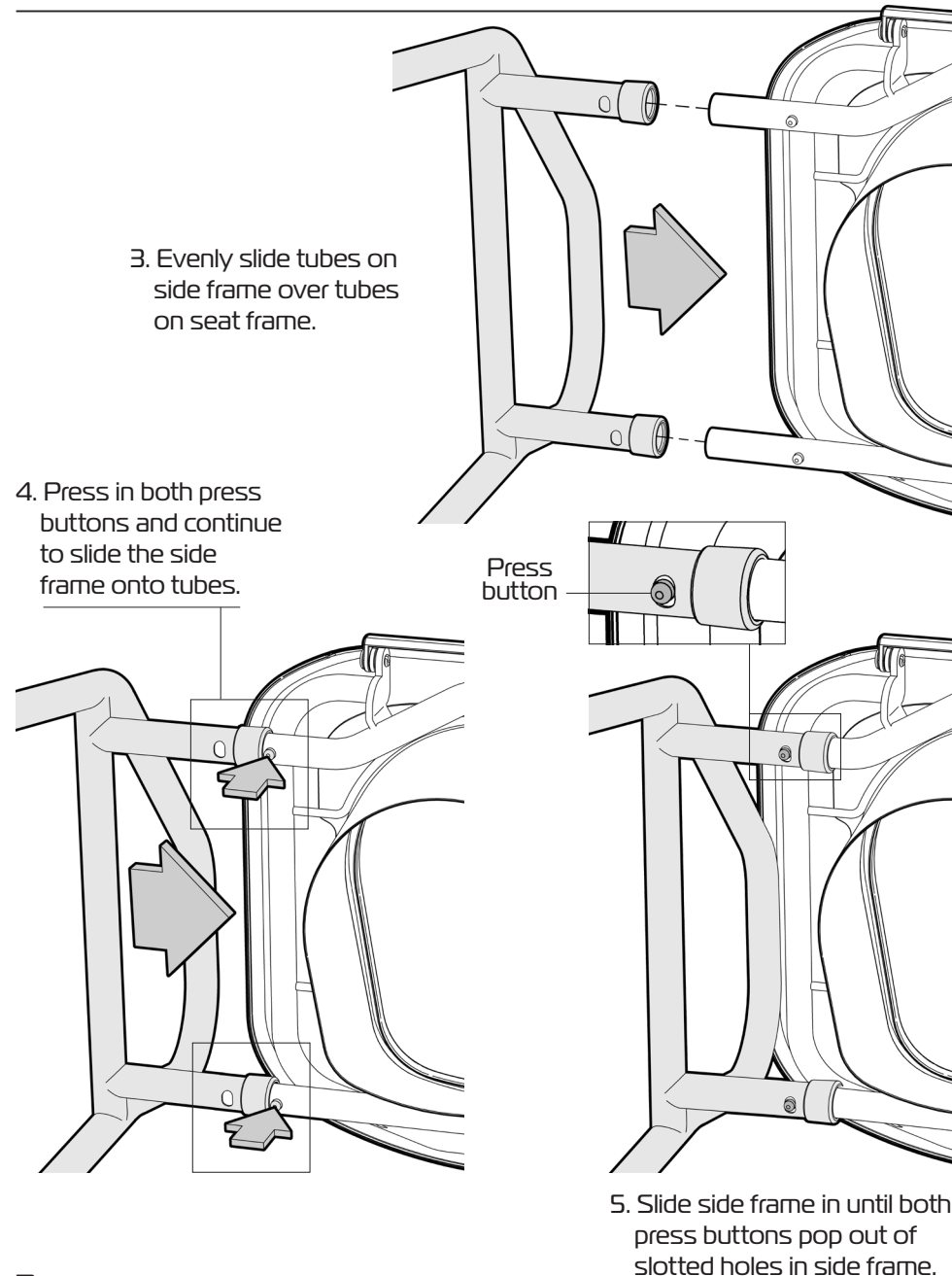


7.

ASK FOR THE BRAND

ASSEMBLY FIT SIDE FRAMES continued

Follow the same procedure to fit the other side frame to the seat frame.

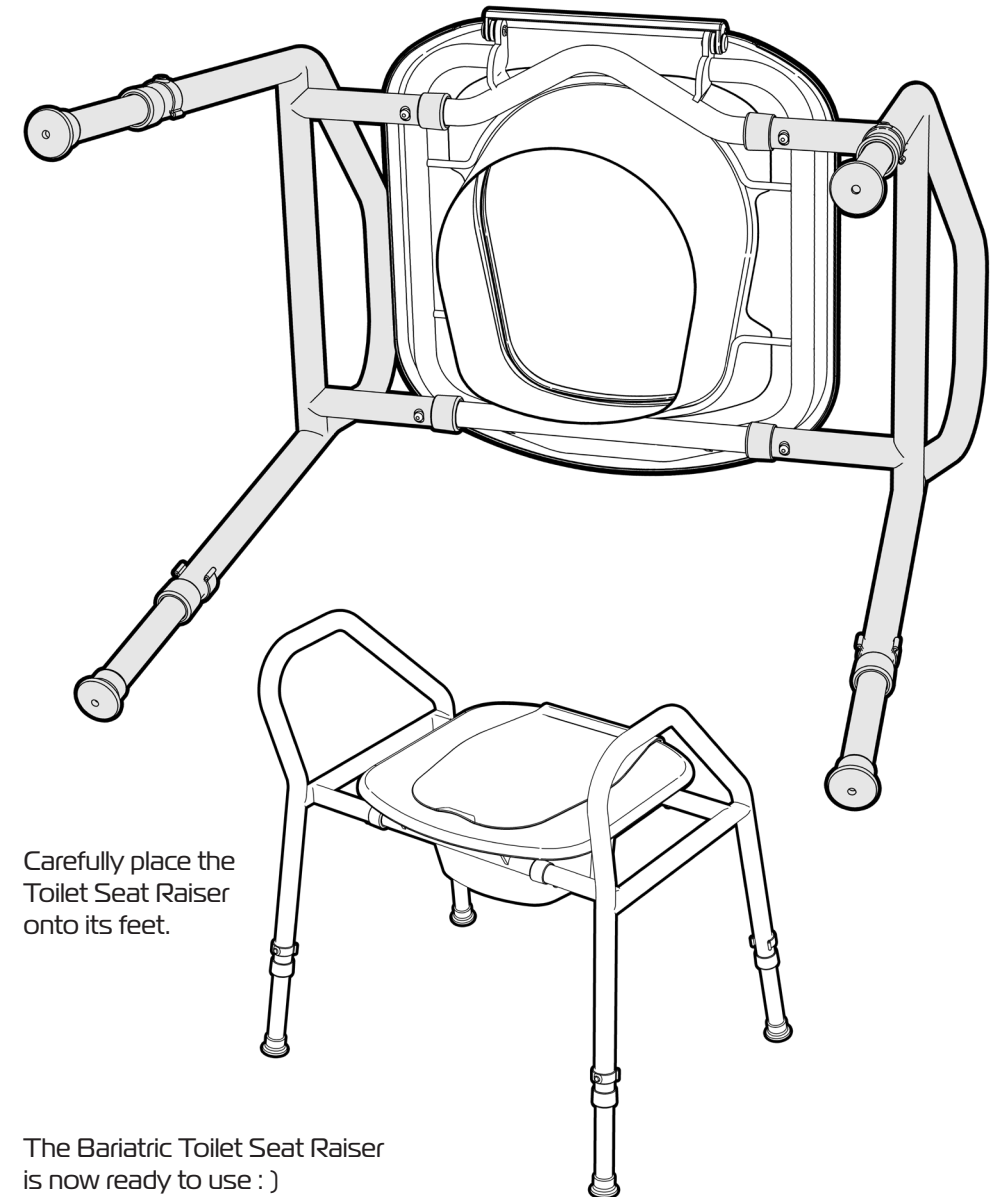

8.

redgumbrandmobility.com.au

USAGE GUIDELINES

Please follow these guidelines to get the optimal benefit from the Toilet Seat Raiser.

Failure to follow these guidelines may result in damage to the product and put the user at risk.


Maximum user weight: **160kg**

Ensure that the Toilet Seat Raiser has been set up at the correct height to suit the user.
(Refer to page 4.)

Top of handles should be within reach when standing in front of the Toilet Seat Raiser.


Always have both hands on the armrests and ensure your body weight is central over the Toilet Seat Raiser when getting on and off.

Ensure ALL FOUR LEGS remain on the ground at all times when the Toilet Seat Raiser is being used.

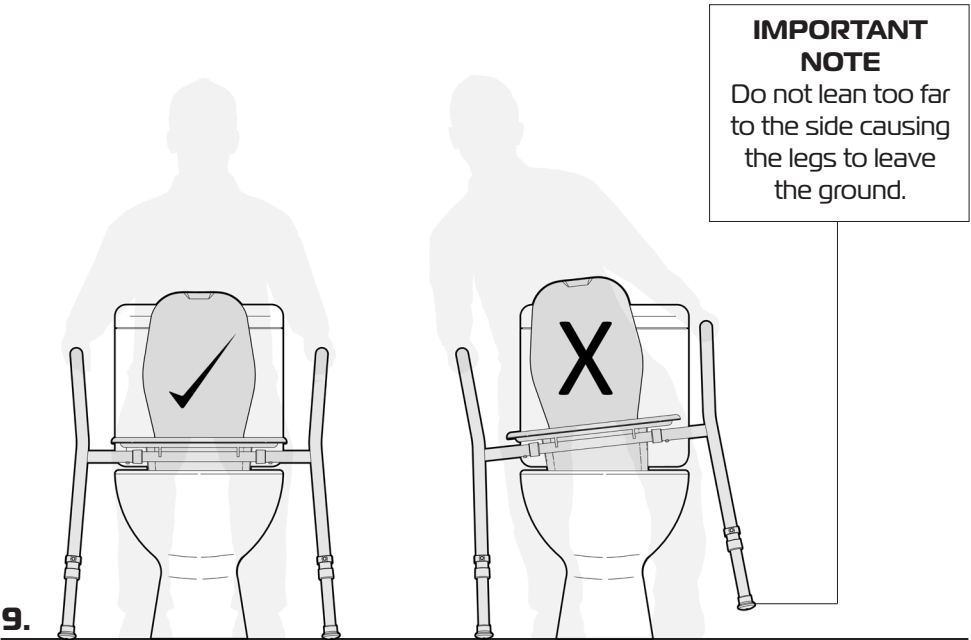

USAGE SPLASHGUARD ACCESS

Please follow these instructions to access the splashguard on the Toilet Seat Raiser:

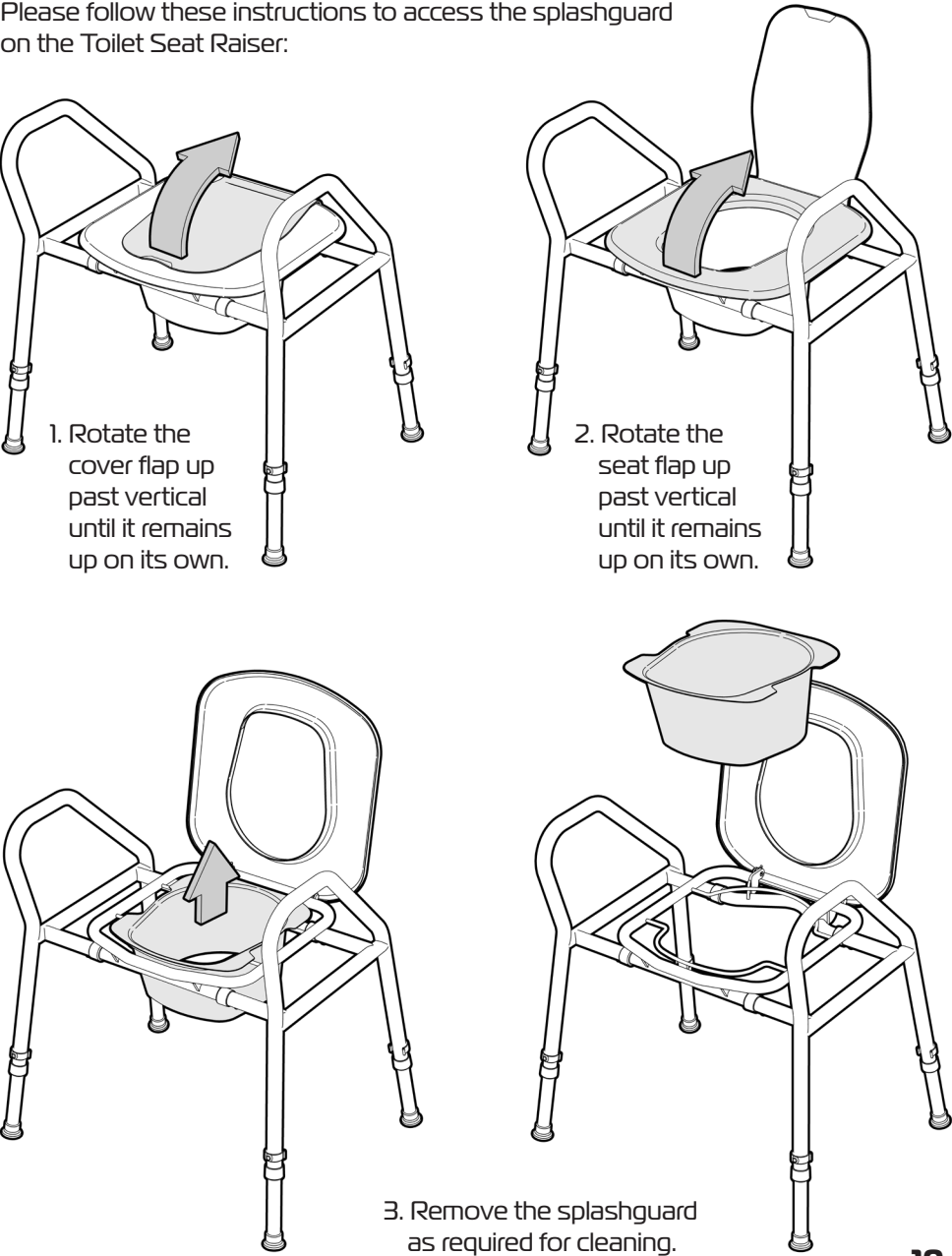

INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the Bariatric Toilet Seat Raiser periodically for signs of damage or wear prior to use. Overall check the following:

- Frame has no distortion.
- Plastic parts are not damaged or cracked.



Withdraw the Bariatric Toilet Seat Raiser immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING


Clean the Bariatric Toilet Seat Raiser using disinfectant wipes. We do not recommend cleaning the product with soap and water.

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Clean the splashguard each time after use if required.

Do not use any abrasive cleaners or abrasive cleaning pads as they could potentially damage the Bariatric Toilet Seat Raiser.

If you have any questions about the cleaning of the Bariatric Toilet Seat Raiser please contact your supplier or REDGUM brand direct.

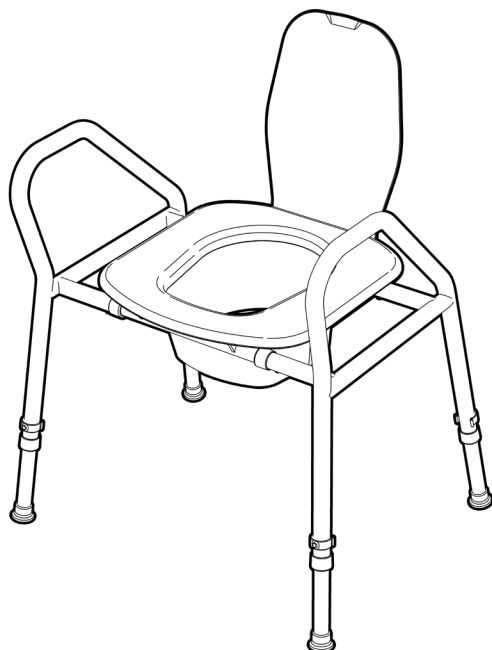

11.

ASK FOR THE BRAND

WARRANTY

REDGUM Brand Bariatric Toilet Seat Raiser (RG520) is covered by a 12 month warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if the product needs to be returned to the place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser.
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the Bariatric Toilet Seat Raiser in any way or fit any parts or accessories that are not specified as an option. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the Bariatric Toilet Seat Raiser please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **info@redgumbrand.com.au**

12.

redgumbrandmobility.com.au

REDGUMbrand
ASSISTED LIVING PRODUCTS



DISTRIBUTOR:



Explore the
REDGUM Brand
range of products



ASK FOR THE BRAND