



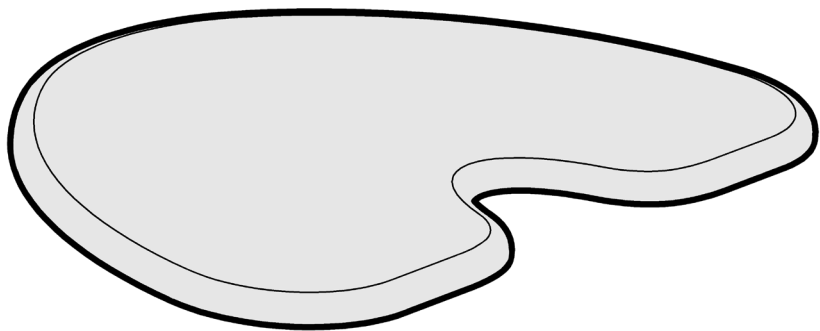

The ComfiGEL Space Saver Corner Shower Stool Gel Pad is designed to fit onto the RG553: Space Saver Corner Shower Stool

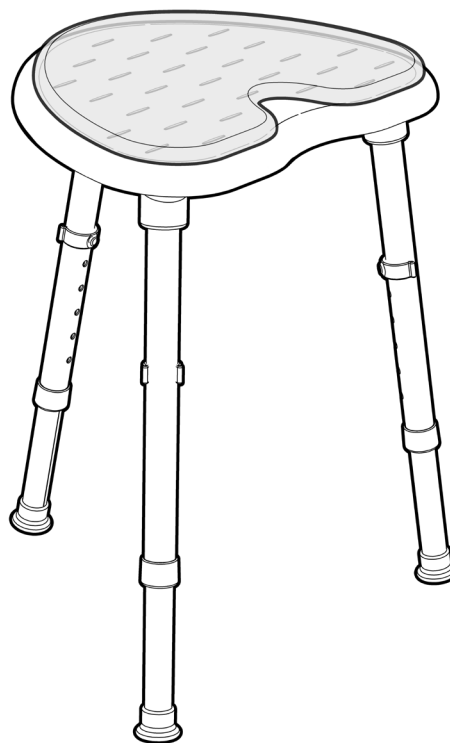

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the Gel Pad please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180** E: **orders@for-de.com.au** A: **1 Business Way, Malaga, Western Australia 6090**

FEATURES

- 10mm Gel Pad to relieve pressure when sitting.
- Washable sticking base.
- Easy to apply and remove.
- Easy to clean.

SPECIFICATIONS

- Width: **365mm**
- Depth: **265mm**
- Height: **10mm**
- Product Weight: **1kg**
- Maximum User Weight: **130kg**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use your Space Saver Corner Shower Stool Gel Pad for the purpose for which it is intended, which is as a pressure relieving seating solution.


INSPECT SUPPLIED PRODUCT

If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.

Please turn over for more instructions

INSTALLATION


1.

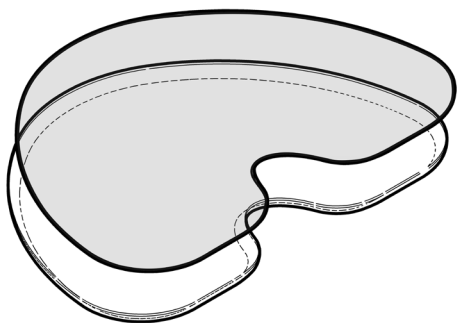

- Turn the Gel Pad upside down and remove the clear plastic backing sheet.


Do not discard the backing sheet in case it needs to be reapplied to the Gel Pad if not being used on the Corner Shower Stool.

2.

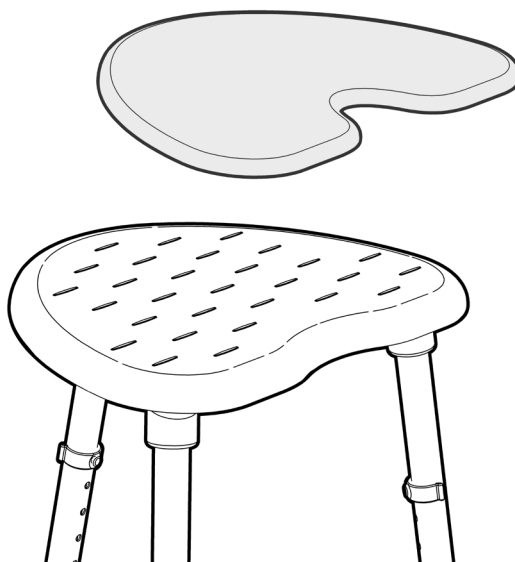

- Turn the Gel Pad over so the sticky side is facing down.
- Align the gel pad in the required position over Corner Shower Stool with the cut out facing toward the front.

3.

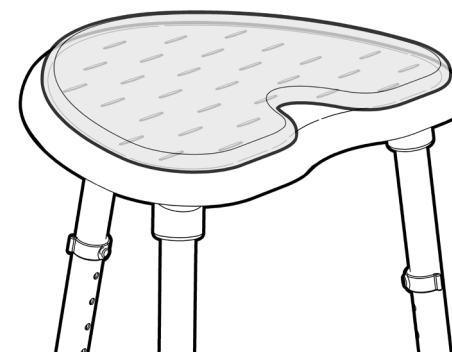

- Place the Gel Pad onto the Corner Shower Stool and push down on the entire surface to fully stick in position.

**If the gel pad is no longer required for use:
Carefully peel the Gel Pad from the Corner Shower Stool and carefully reapply the plastic backing.**

INSPECTIONS

We suggest that you visually periodically check the Space Saver Corner Shower Stool Gel Pad for signs of damage or wear prior to use. Remove the Gel Pad immediately from use if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for support.

CLEANING

Carefully peel the Space Saver Corner Shower Stool Gel Pad from the Shower Stool, wash under warm water and allow to dry completely before refitting to the Shower Stool. Abrasive cleaners or abrasive cleaning pads could damage the Gel Pad and should not be used.

WARRANTY

ComfiGEL Corner Shower Stool Gel Pad (RG53GP) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.