

Ideal to assist with transfers between:

- Beds
- Wheelchairs
- Chairs
- Shower Chairs
- Mobile Shower Commodes

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the Transfer Board please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

E: orders@for-de.com.au

A: 1 Business Way, Malaga, Western Australia 6090

FEATURES

- Assists with lateral sideways movement of a user in a sitting position from one level surface to another.
- Minimises the risk of strain and injury for both user and the carer when transferring.
- Can be used to span a gap of up to 300mm.
- Slope on top edge and smooth rounded edges assist when sliding user onto the Transfer Board.
- Carry handle cut out for transport and storage when using the optional hanging hook.
- Suitable for use in wet areas.

SPECIFICATIONS

- Overall: **800mm (L) x 210mm (W) x 12mm (H)**
- Product Weight: **1.7kg**
- Max. Load: **150kg**

OPTIONS

Transfer Board hanging hook (RG55TRAH)
(Allows for transfer board to be hung on wall)

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Transfer board to assist with lateral sideways movement of a user in a sitting position from one level surface to another. Not to be used for any other purpose.

INSPECT SUPPLIED PARTS If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.



WARNING - IMPORTANT NOTE

Before transferring ensure that the Transfer Board is correctly positioned prior to use.
Always consult your manual handling professional for advise.

Please turn over for more instructions

ASK FOR THE BRAND

redgumbrandmobility.com.au


USER GUIDELINES (Bed and Chair shown as an example)

1.



- Position the surfaces being transferred between as close as possible to each other.
- We recommend that the gap is as small as possible where appropriate.
We DO NOT recommend to have a gap greater than 200mm if possible.
- Max. gap is 300mm with minimum of 55mm on each surface.


2.

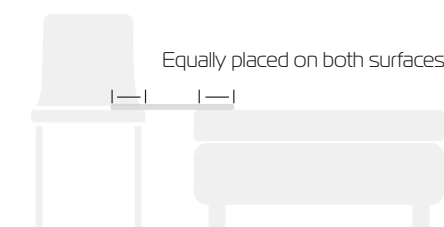

- Carefully place the Transfer Board onto both surfaces.


IMPORTANT NOTE
Surfaces MUST be level with each other and flat!

3.

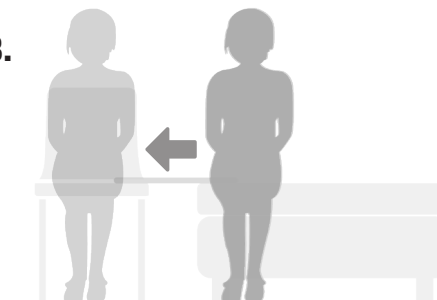

- Always maintain downward force when sliding along the Transfer Board to minimise any movement.
- Keep all body parts on the top of the Transfer Board when being used.
- Potential pinch points exist under the Transfer Board when under load.
- Always remember to remove the Transfer Board when not in use.

4.

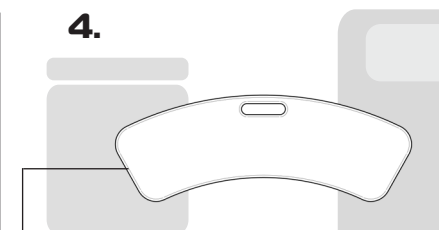

- Ensure the Transfer Board is always positioned centrally onto surfaces being transferred across to minimise potential to fall off.

INSPECTIONS

We recommend that the Transfer Board is visually checked periodically for signs of damage or wear prior to use. Withdrawn the Transfer Board from use immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for support.

CLEANING

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining. Clean the Transfer Board using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners or abrasive cleaning pads could damage the transfer board and should not be used. If you have any questions about the

WARRANTY

Transfer Board (RG55TRA) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

OPTION:

TRANSFER BOARD HOOK

RG55TRAH

