



Thank you for purchasing this **REDGUM Brand** product. If you have any issues or questions about the **PLUO SELF PROPEL WHEELS** please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180 E: orders@for-de.com.au

A: 1 Business Way, Malaga, Western Australia 6090

FEATURES

- Convert the PLUO attendant propelled Mobile Shower Commode to a self propelled version.
- Quick release wheels with flat free tyres
- Push rims
- Easy to retro fit

SPECIFICATIONS


- Wheel: **Ø610mm (24")**
- Product Weight: **2.6kg per wheel**
- Maximum User Weight: **150kg**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the PLUO SELF PROPEL WHEELS to propel the Pluo Mobile Shower commode if the user is capable to do so.

INSPECT SUPPLIED PARTS

If you see any damage or suspect a fault DO NOT USE THE PRODUCT and contact your supplier for support.

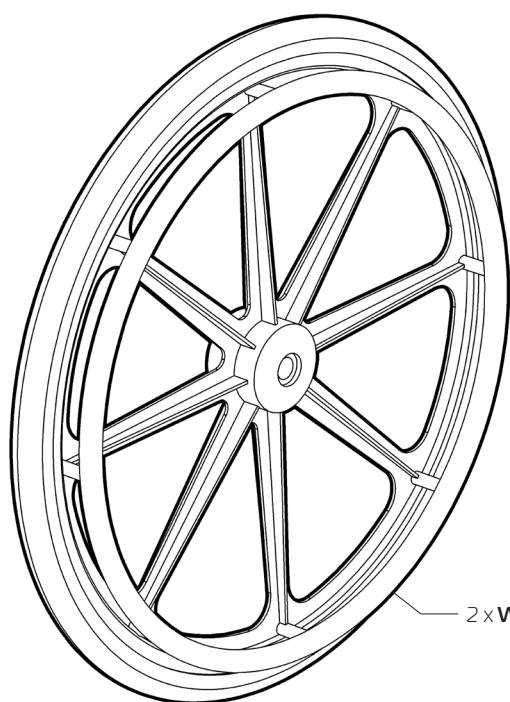

2 x WHEEL

1 x LEFT
HAND BRAKE

1 x RIGHT
HAND BRAKE

2 x LEFT
MOUNT BRACKET

2 x RIGHT
MOUNT BRACKET

2 x AXLE HOUSING

2 x BOLT

2 x AXLE

2 x SPACER

2 x AXLE
HOUSING NUT

2 x NUT

Tools required (not supplied):

- 1 x 5mm Allen Key
- 1 x 6mm Allen Key
- 2 x Adjustable spanners



Please turn over for more instructions

IMPORTANT NOTE: This equipment MUST be assembled for use by a competent person.

RG570WK

ASSEMBLY INSTRUCTIONS


The two rear castors must be removed prior to fitting the Self Propel Wheels to the Pluo Mobile Shower Commode.

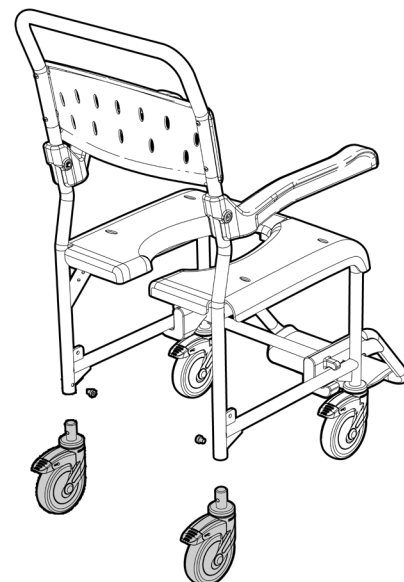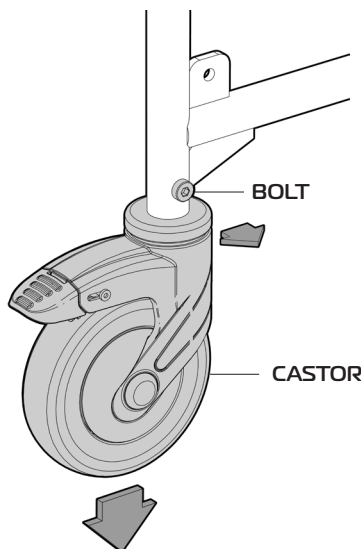

Using a 5mm Allen key - remove the bolt above rear castor and slide castor off frame.

Important Note:



Store castors and bolts in a safe place incase they need to be refitted.

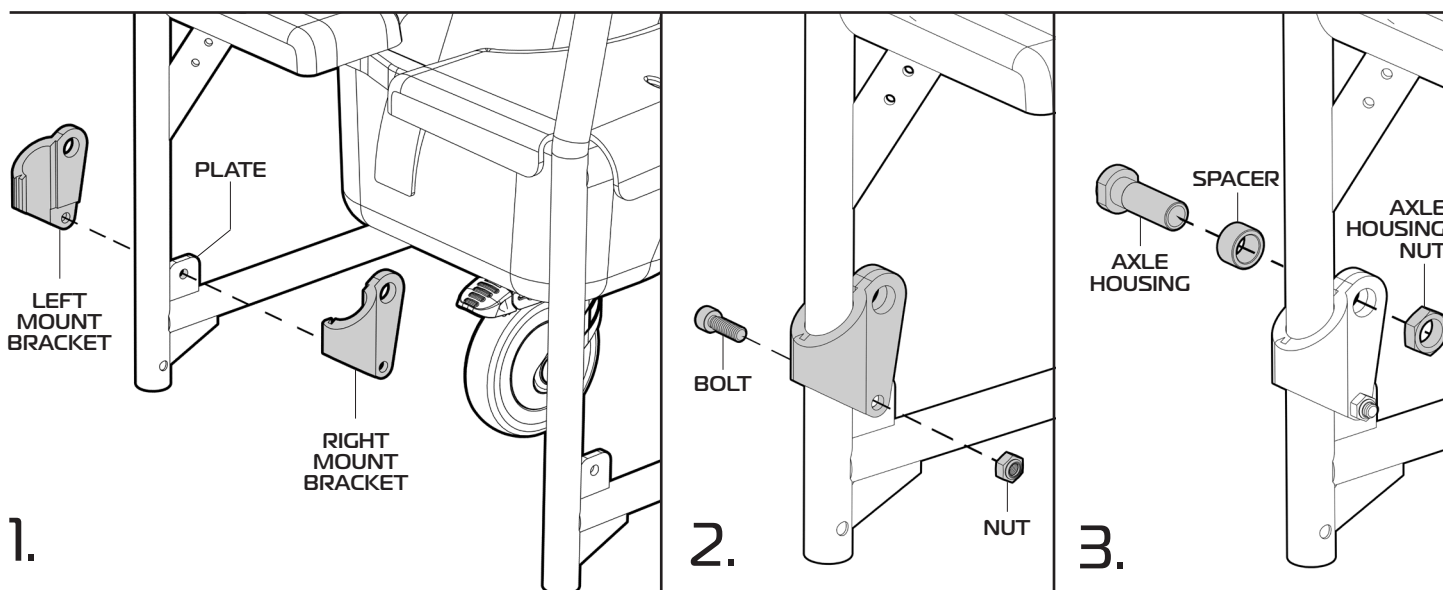

1. Clamp the left and right mount brackets together around tube and align holes in mount brackets with hole in plate on frame.
2. Slide bolt through holes and hand wind nut onto bolt. Use a 6mm Allen key and adjustable spanner to tighten fixing.
3. Slide spacer onto axle housing. Then slide axle housing with spacer through large hole on mount bracket and hand wind axle housing nut onto axle housing. Use adjustable spanners to tighten.

Repeat process on other leg of Mobile Shower Commode.

Please turn over for more instructions

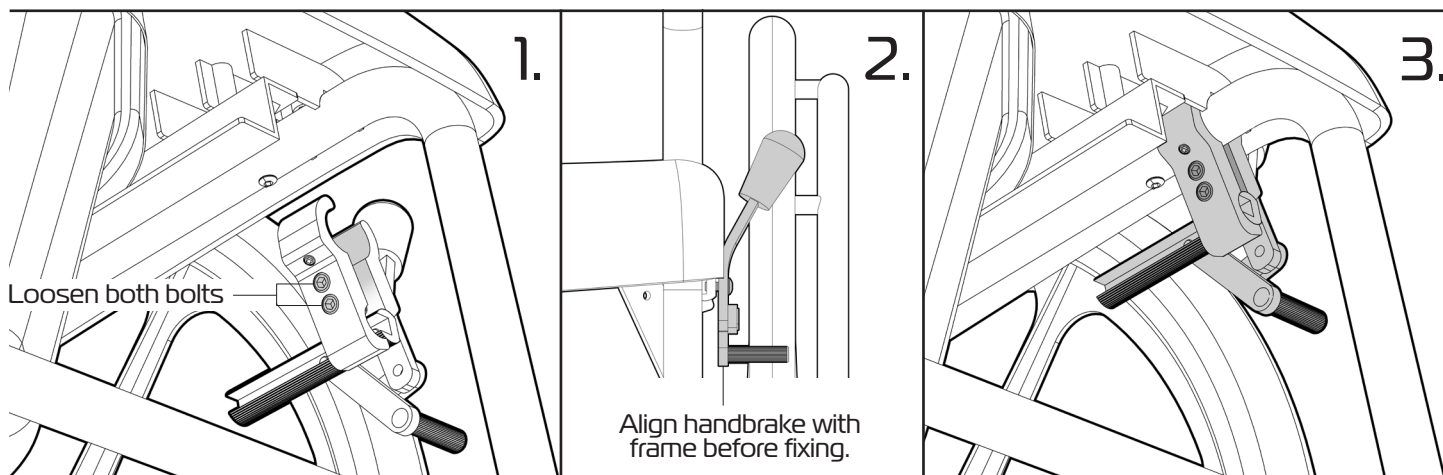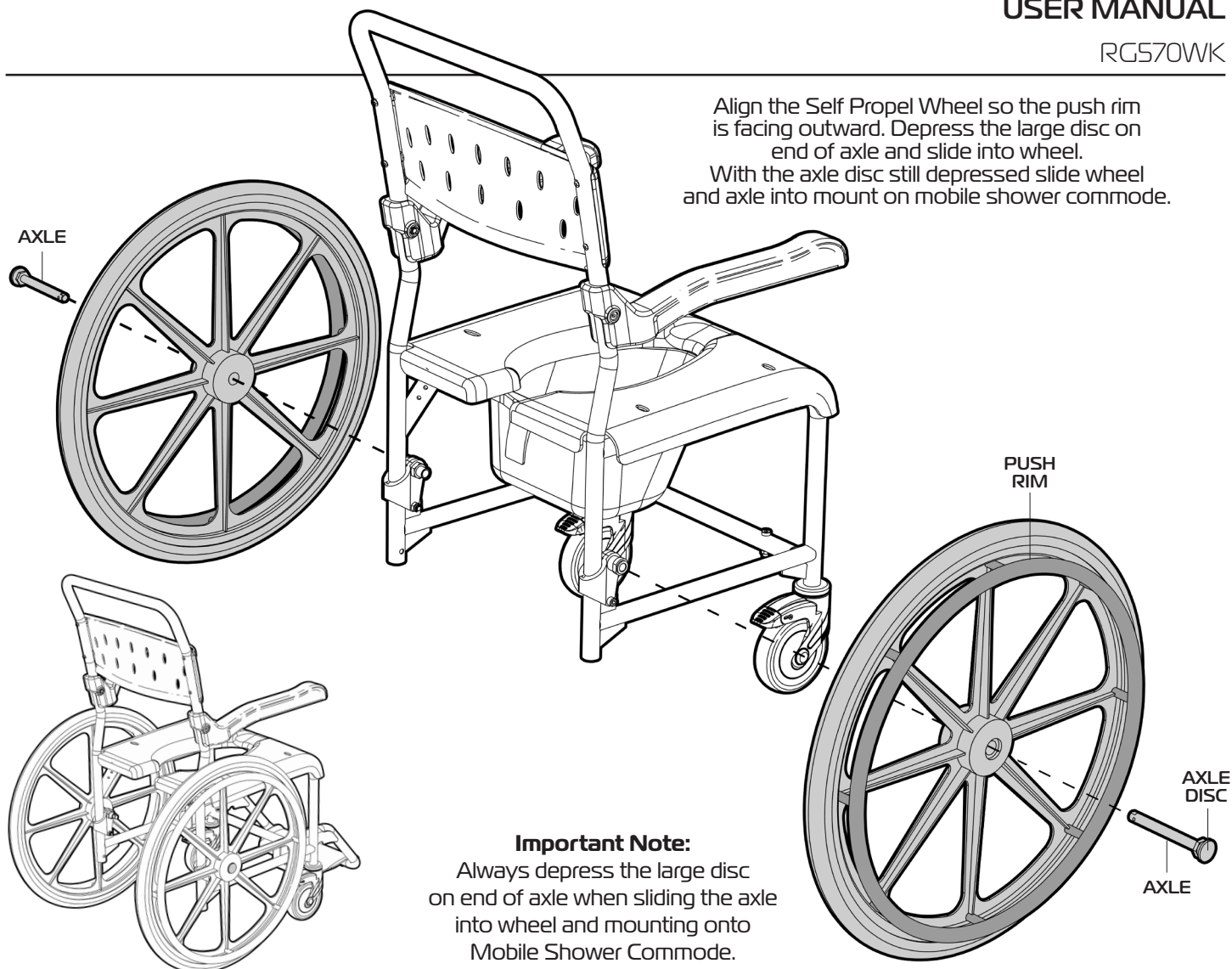

Determine the correct handbrake to mount on the side of the Mobile Shower Commode in front of the wheel.

1. Using a 5mm allen key slightly undo the two bolts on inside of hand brake mount until the clamp can fit over the tube under seat on Mobile Shower Commode as shown.
2. Align the handbrake with C ommode frame as shown
3. Re tighten both bolts to fix handbrake in place.

Repeat process to install other handbrake on Mobile Shower Commode.

Please turn over for more instructions

USAGE GUIDELINES

Please follow these guidelines to get the optimal benefit from the Self Propel Wheels when fitted to the Pluo Mobile Shower Commode.

Also refer to the Pluo Mobile Shower Commode user manual for information.

Failure to follow these guidelines may result in damage to the product and put the user at risk.

Periodically inspect to the Self Propel Wheels prior to use to ensure they are in good working order.



Always check that both handbrakes are working as required prior to using the commode. Always engage both handbrakes when commode is stationary and when the user is getting on and off the Mobile Shower Commode.

The Mobile Shower Commode is designed to be used indoors only.

Take care when using the Mobile Shower Commode on uneven or wet ground.

Ensure all four wheels remain in contact with the ground when the Mobile Shower Commode is in use. Do not lean too far to one side or lean backwards causing the wheels to leave the ground.

If you have any issues or questions about the PLUO Self Propel Wheels please contact your supplier or **REDGUM Brand** direct.

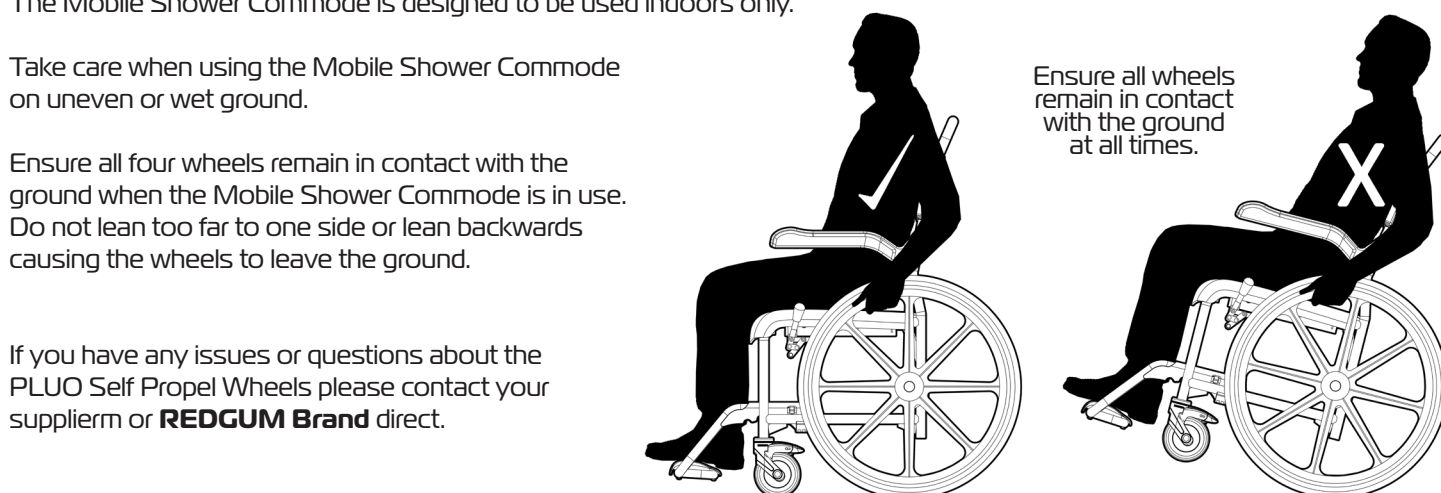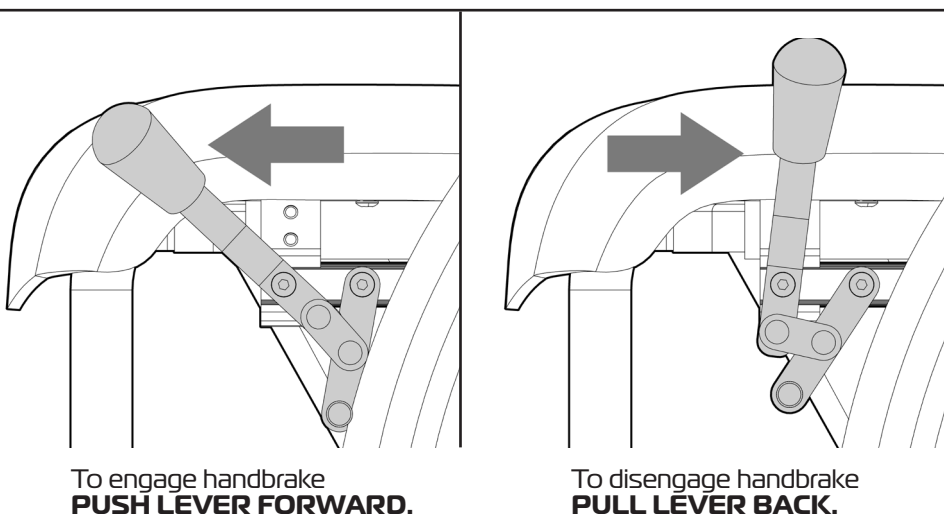

INSPECTIONS

We recommend that the Self Propel Wheels are visually checked periodically for signs of damage or wear prior to use. Remove the Mobile Shower Commode from use immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for support.

CLEANING

We recommend that you attend to any required cleaning as soon as possible. Clean the Self Propel Wheels using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners or abrasive cleaning pads could damage the product and should not be used.

WARRANTY

PLUO SELF PROPEL WHEELS (RG570WK) are covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure. If you have any questions please contact the supplier or REDGUM Brand direct.