




# **HANDI GRAB RAIL**

300mm: RG5930 / 450mm: RG5945

**USER MANUAL** 

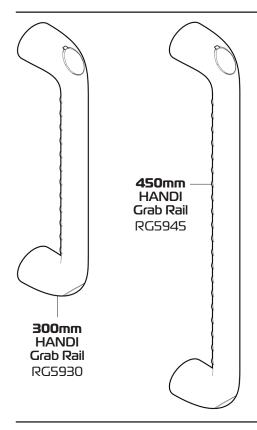

### **FEATURES**

- Assistive balancing aid.
- Modern discreet design.
- Ribbed grab rail for improved grip.
- Reinforced Plastic construction.
- Stainless Steel fixings supplied.

### **SPECIFICATIONS**

#### 300mm Handi Grab Bar

Overall: 340mm L x 50mm W x 75mm H

Rail Diameter: Ø30mm Wall to inside of rail: 45mm Product Weight: 200g Max. Supportive Load: 100kg

#### 450mm Handi Grab Bar

Overall: 490mm L x 50mm W x 75mm H

Rail Diameter: Ø30mm


Wall to inside of rail: 45mm

Product Weight: 300g

Max. Supportive Load: 100kg


## **INSPECT SUPPLIED PARTS**

If you missing any parts, see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.

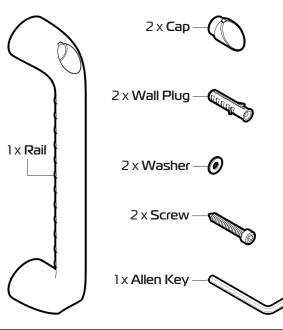

# TOOLS REQUIRED

(Not supplied)

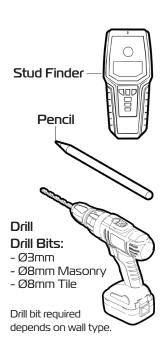

Thank you for purchasing this **REDGUM Brand** product. If you have any issues or questions about the HANDI Grab Rail please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180 E: orders@for-de.com.au A: 1 Business Way, Malaga, Western Australia, 6090


### **INTENDED USE**

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only to be used to assistive balancing aid (Do not use the Grab Rail to support your full body weight). Do not use for any other purpose. Handi Grab Rail must be fitted to a structural wall surface.

Please turn over for more instructions

#### INSTALLATION

Please follow these guidelines when installing the HANDI Grab Rail. **IMPORTANT NOTE:** The Grab Rail should only be fitted by an experienced tradesperson. The Grab Rail must be fitted to a structural wall surface (Must be fixed over a stud when being fitted to a hollow wall).

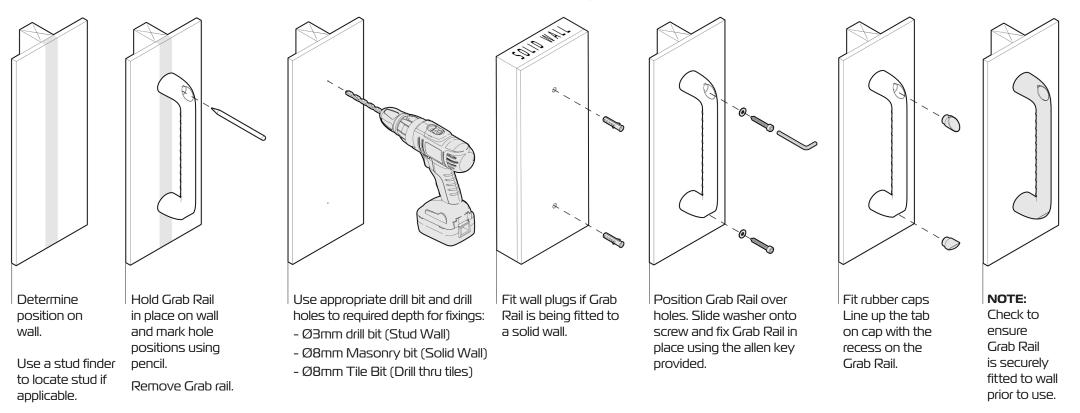

### **CLEANING**

Clean the Grab Rail using a non-abrasive cleaner or mild detergent with a soft cloth. We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining. Abrasive cleaners or abrasive cleaning pads could damage the Grab Rail and should not be used. If you have any questions about the cleaning of the product please contact your supplier or REDGUM brand direct.

## **WARRANTY**

300mm HANDI Grab Rail (RG5930) / 450mm HANDI Grab Rail (RG5945) are covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage.

You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure. Do not modify or fit any parts or accessories to the Grab Rail that are manufactured by another company.

To do so will void your warranty and may seriously compromise your safety.