



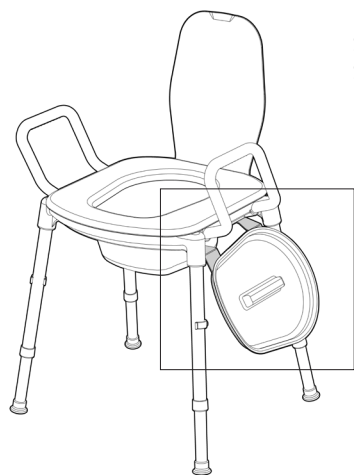

IMPORTANT NOTE

SPACE SAVER **BOWL LID HOLDER** can only be used on a
SPACE SAVER **TOILET SEAT RAISER** - RG515

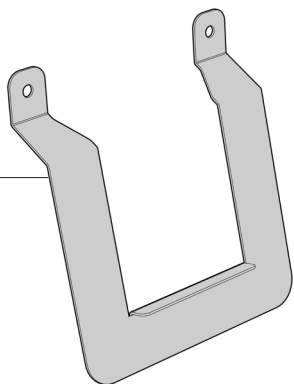

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the SPACE SAVER **BOWL LID HOLDER**
please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180** E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

FEATURES

- Bowl lid holder can only be used on a SPACE SAVER **TOILET SEAT RAISER** (RG515)
- Holds bowl lid within easy reach on side of Toilet Seat Raiser.
- Can be positioned on either side of the Toilet Seat Raiser.
- Easy to fit and remove when required.

SPECIFICATIONS


- Overall: **200mm (W) x 220mm (H) x 50mm (D)**
- Product Weight: **0.4kg**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Bowl Lid Holder to hold the lid on the side of the Space Saver Toilet Seat Raiser when not being used. Not to be used for any other purpose.

INSPECT SUPPLIED PARTS


1 x Bowl lid holder

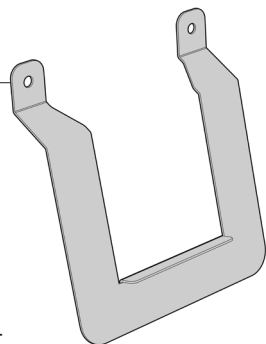


If you see any
damage or
suspect a fault
DO NOT USE
THE PRODUCT
and contact your
supplier for support.

**FITTING
INSTRUCTIONS**


1. Flip up the seat flap
and remove the bowl
from the Toilet
Seat Raiser.

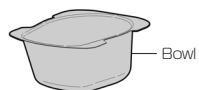


Seat
flap

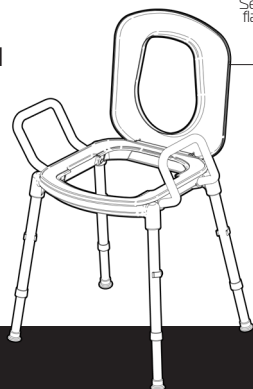

2. Flip the Toilet Seat Raiser
over to access the underside.

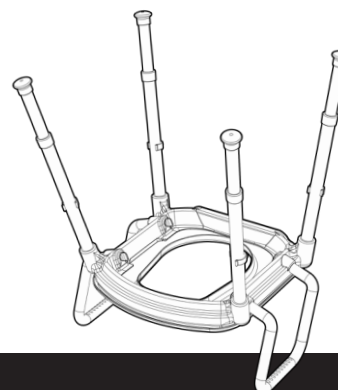

3. Determine which side the Bowl Lid Holder
will be positioned and remove both the red
thumbscrews on that side only.

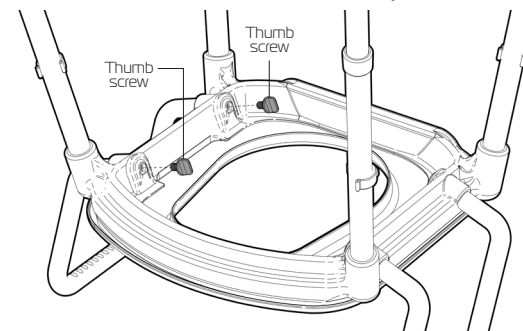


ASK FOR THE BRAND

redgumbrand.com.au


Please turn over for more instructions

FITTING INSTRUCTIONS


4. Position Bowl Lid Holder so it is angled towards outside of Toilet Seat Raiser.

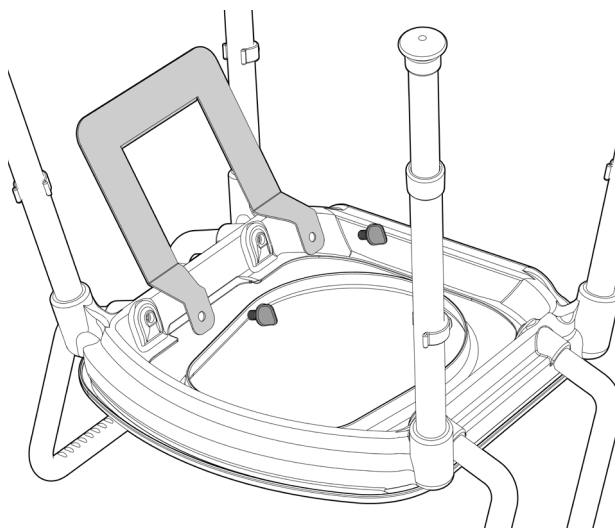

5. Line up holes in both the Bowl Lid Holder and Toilet Seat Raiser thumbscrew holes and wind in thumbscrews. (DO NOT OVERTIGHTEN)

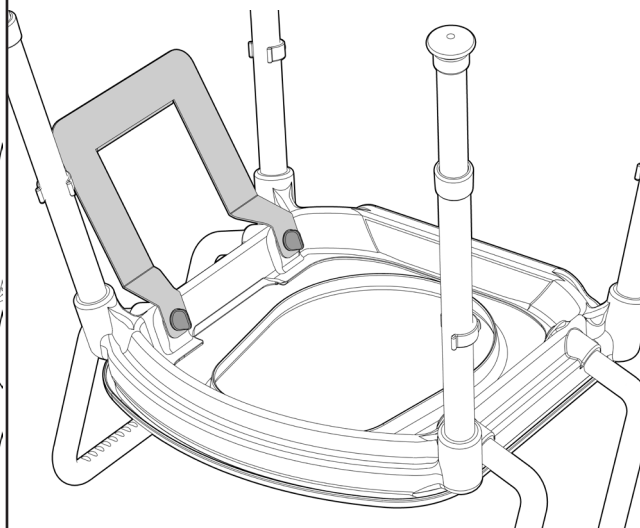

6. Flip Toilet Seat Raiser back onto its feet. Place bowl back into position and locate Bowl Lid as shown onto holder when not being used.

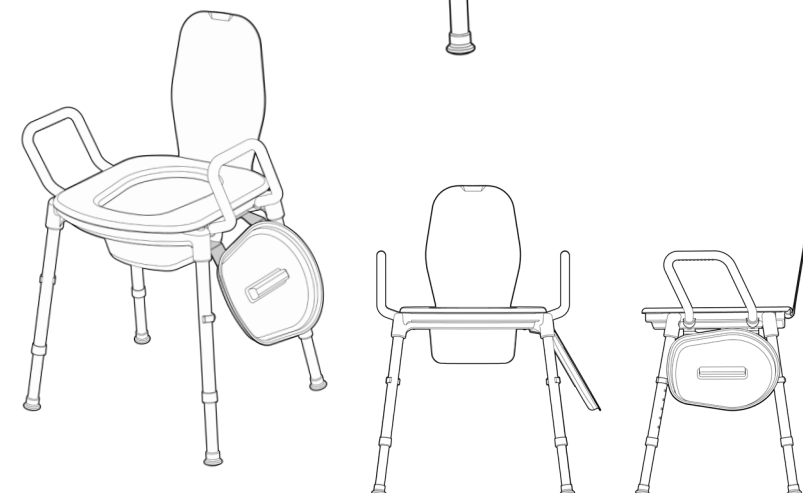

INSPECTIONS

We recommend that the Bowl Lid Holder is visually checked periodically for signs of damage or wear prior to use. Remove the product from use immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for support.

CLEANING

We recommend that you attend to any required cleaning as soon as possible. Clean the Bowl Lid Holder using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners or abrasive cleaning pads could damage the Bowl Lid Holder and should not be used.

WARRANTY

Space Saver Bowl Lid Holder (RG5BLH) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure. If you have any questions please contact the supplier or REDGUM Brand direct.