


HEIGHT ADJUSTABLE BED RAIL


RG613

Edition 1

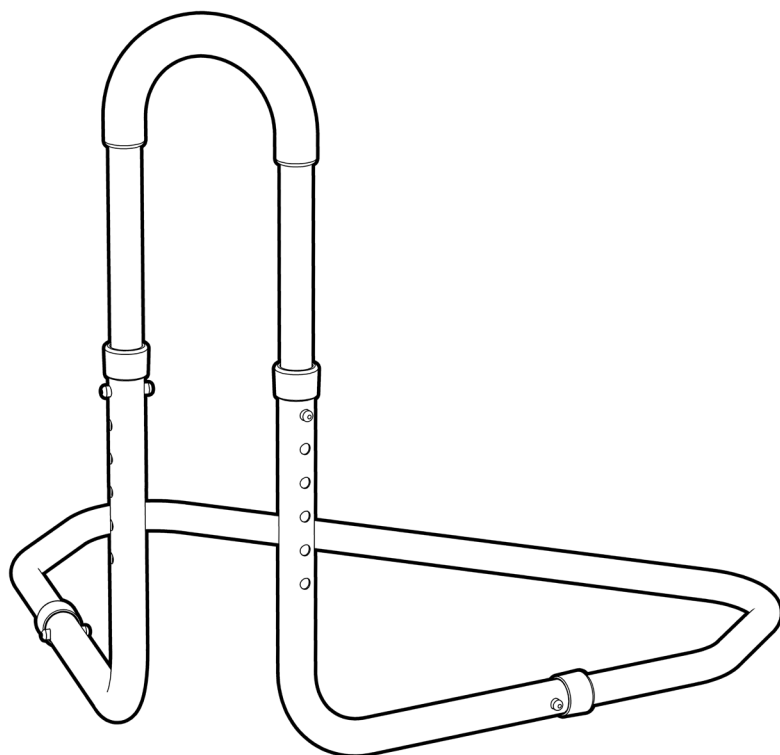

USER MANUAL

Please read and keep these instructions

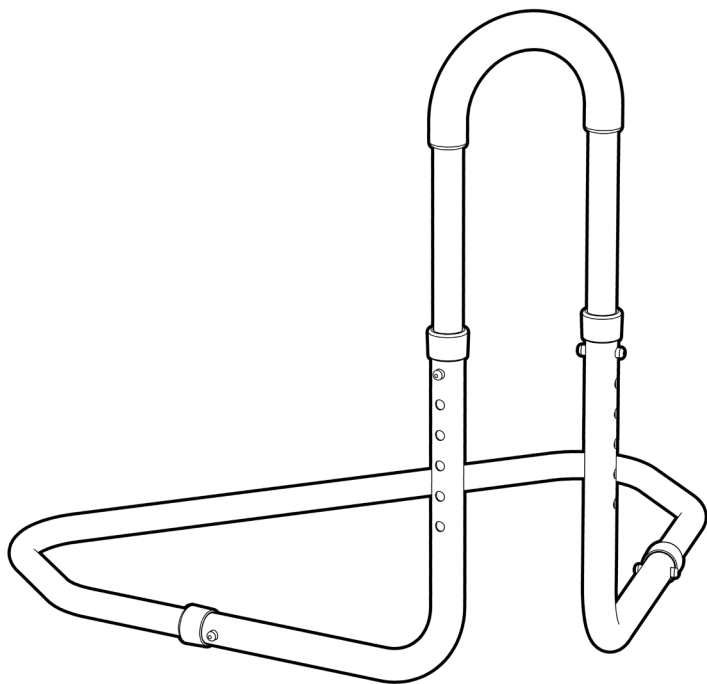

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Height Adjustable Bed Rail user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**


REDGUM Brand is part of **FOR-DE Group Pty Ltd**



ASK FOR THE BRAND

CONTENTS

- 1. INTRODUCTION** - INTENDED USE
- 2. PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
- 3. ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
- 4. ASSEMBLY** - BED RAIL
- 5. ASSEMBLY** - BED RAIL
- 6. USAGE GUIDELINES** - DO's + DON'Ts
- 7. PRODUCT CARE** - INSPECTIONS / CLEANING
- 8. WARRANTY**

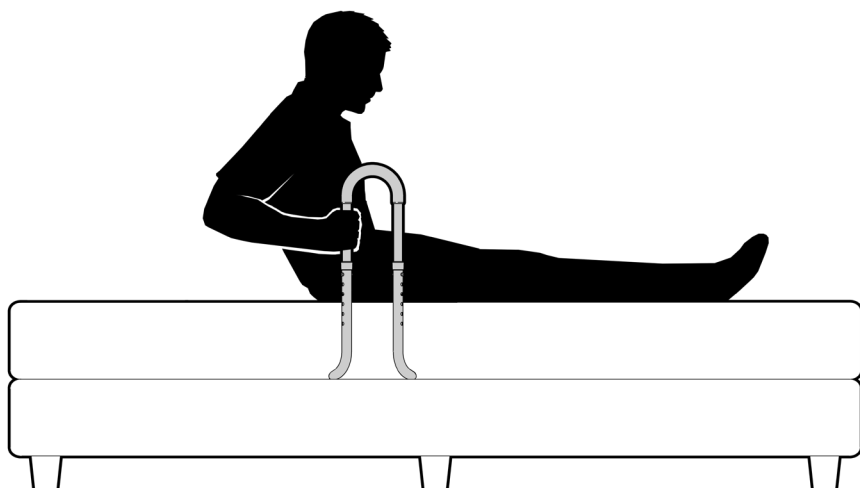

INTRODUCTION

Thank you for purchasing this quality REDGUM Brand product.

If you have any issues or questions about the Height Adjustable Bed Rail please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**


INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your Bed Rail for the purpose for which it is intended, which is as an assistive grab rail solution that can positioned to suit the users individual requirements.

IMPORTANT NOTE

This equipment *MUST* be assembled and put in place by a competent person.

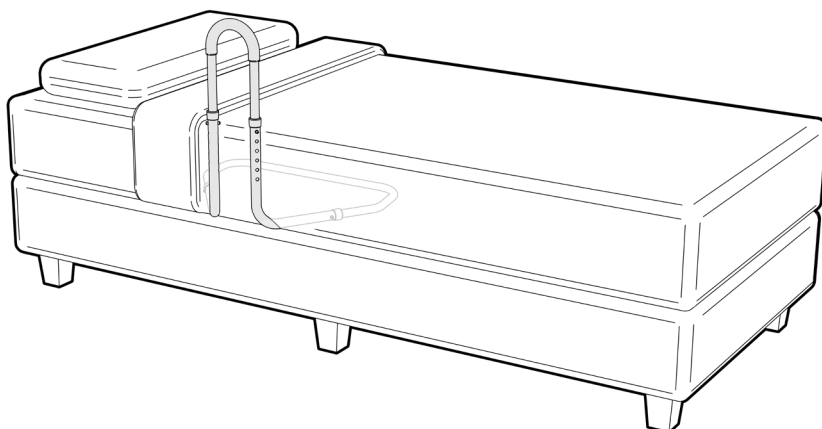

1.

ASK FOR THE BRAND

PRODUCT OVERVIEW

FEATURES

Assistive grab rail to help the user to sit up and get in and out of bed

Height adjustable

Padded hand grip

Easy to assemble - NO tools required

Easy to locate and remove for bed making

Strong lightweight construction

Can be pulled apart for storage and transport

Optional tie down straps available
(RG61BS)

SPECIFICATIONS

(All dimensions shown are nominal)

Overall Dimensions:

Width: 600mm

Depth: 485mm

Height Settings:

Hole 1: 545mm

Hole 2: 520mm

Hole 3: 495mm


Hole 4: 470mm

Hole 5: 445mm

Hole 6: 420mm

Product Weight:


Approx. 2kg

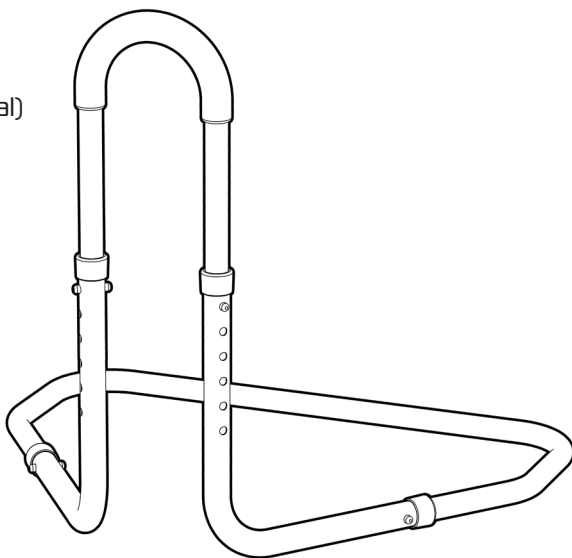

Please contact REDGUM Brand if you would like any further information on the Height Adjustable Bed Rail.

ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts prior to assembly:

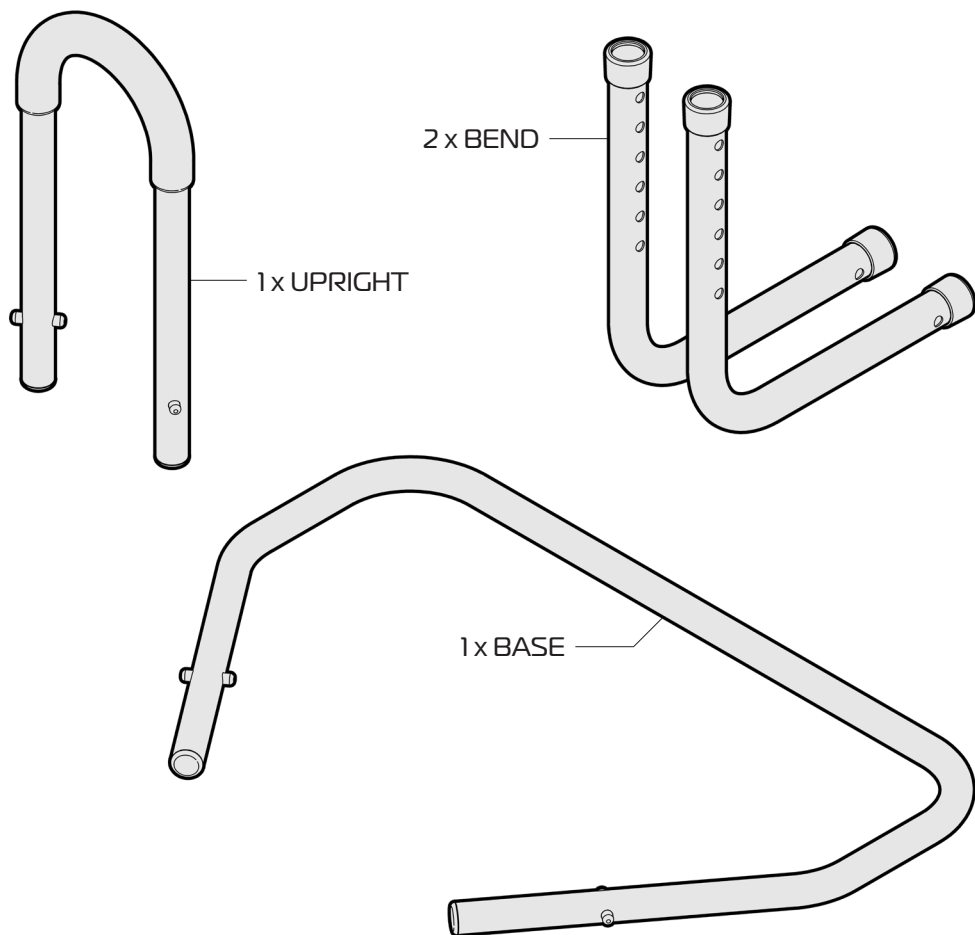

IMPORTANT NOTE

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

3.


ASK FOR THE BRAND

ASSEMBLY - BED RAIL


The bed rail requires assembly prior to use.

Please follow these steps to fit the bends to the base.

1. Align the bend with the base as shown.
Ensure the hole in bend lines up with
press buttons on the base.

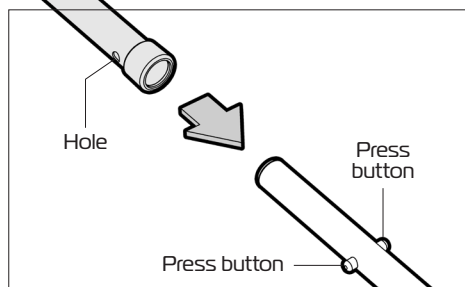

2. Slide the bend onto the base
until it makes contact with the
press buttons.

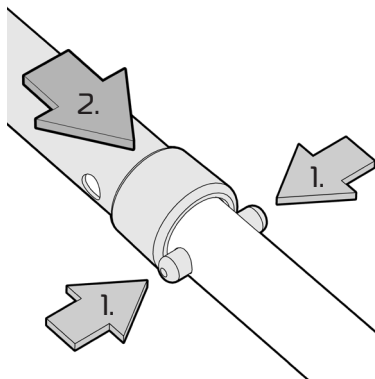

3. Depress the press button on either side and
at the same time continue to slide the bend
onto the base until both press buttons 'POP'
out of the holes in bend.

4. Repeat process
to fit other bend.





ASSEMBLY - BED RAIL

Please follow these steps to fit the upright to the base.

1. Align the upright with the bends as shown. Ensure the press buttons in upright line up with holes on the bends.

2. Slide the upright into the bends until it makes contact with the press buttons.

3. Depress the press button on either side and at the same time continue to slide the upright down onto the bends until all press buttons 'POP' out of the holes in bend.

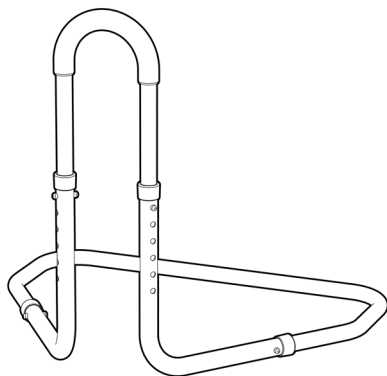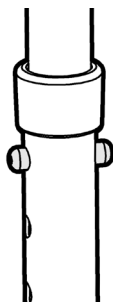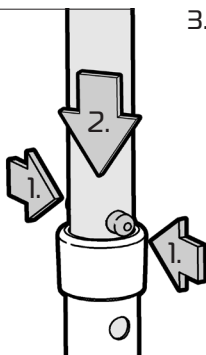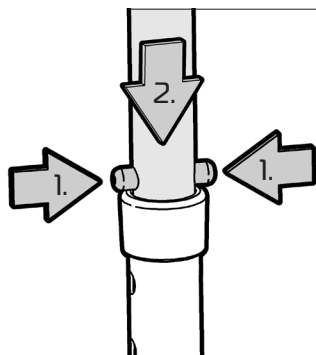

5.


ASK FOR THE BRAND

USAGE GUIDELINES - DO's + DON'TS

Please follow these guidelines to get the optimal benefit from the Bed Rail.

Failure to follow these guidelines may result in damage to the Bed Rail and put the user at risk.





Ensure the Bed Rail base frame is **FULLY INSERTED** in between the mattress and the bed base.

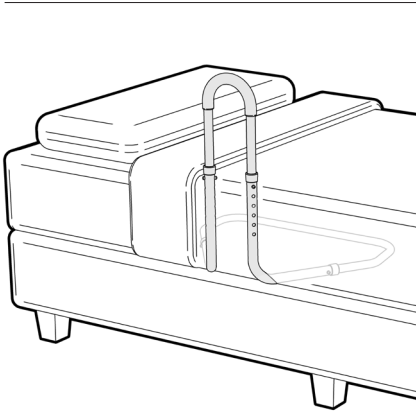

An optional strap is available to hold the Bed Rail in place if required. (RG61BS)

The Bed Rail is designed to be an **ASSISTIVE** grab rail and is not designed to take a user's total body weight.

Adjust the height of the Bed Rail to suit the user. Top of Bed Rail should be within easy reach when getting in and out of bed.

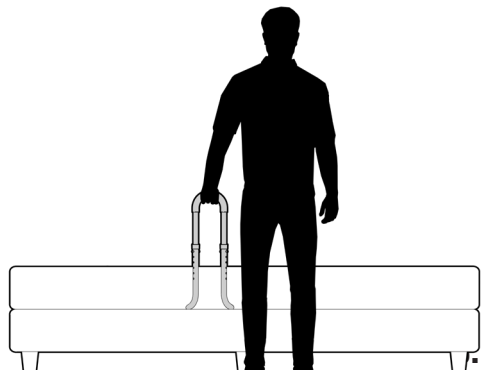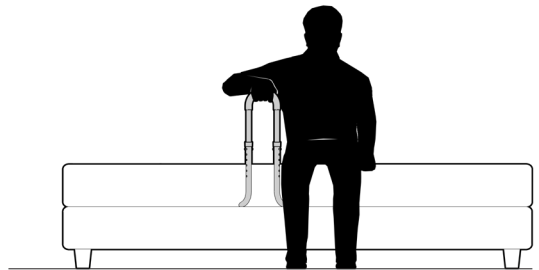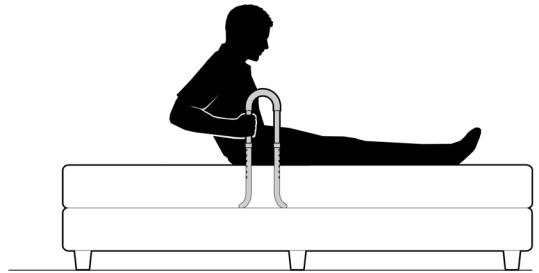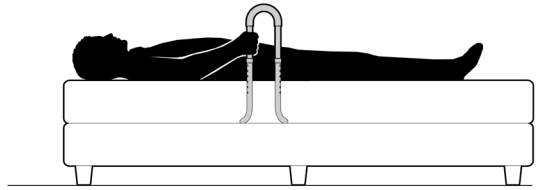

PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you check the Height Adjustable Bed Rail visually periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the Bed Rail immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT NOTE

Do not fit any parts or accessories to the Height Adjustable Bed Rail that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.


CLEANING

Clean the Bed Rail using disinfectant wipes. We do not recommend cleaning the product with soap and water.

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Do not use any abrasive cleaners or abrasive cleaning pads as they could potentially damage the product.

If you have any questions about the cleaning of the Bed Rail please contact your supplier or REDGUM brand direct.

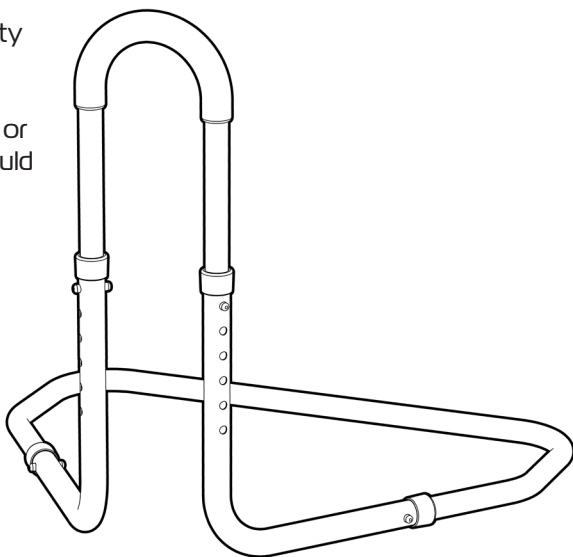

WARRANTY

REDGUM Brand RG613 - Height Adjustable Bed Rail
is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if the product needs to be returned to the place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser.
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the Height Adjustable Bed Rail in any way or fit any parts or accessories that are not specified as an option. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the Height Adjustable Bed Rail please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

DISTRIBUTOR:



**Explore our wide
range of products!**