




BED RAIL

RG611 - STANDARD

RG612 - SLOPE

RG614 - PEAK

AUGUST 2019

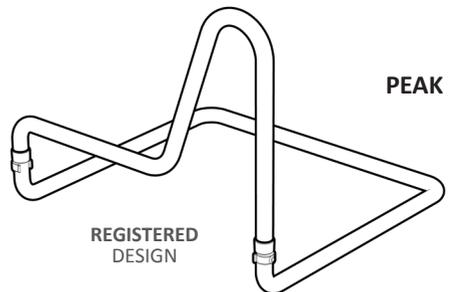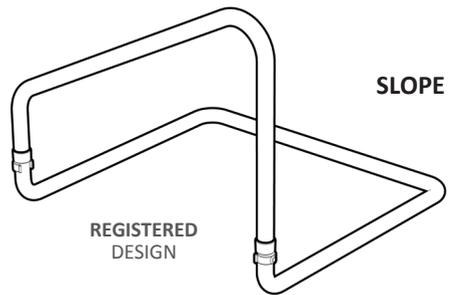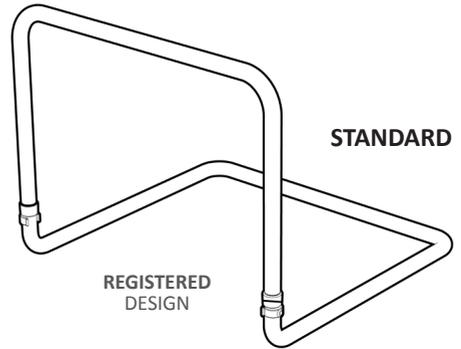

USER MANUAL

Please read and keep these instructions



REDGUM Brand owns the **copyright** to the Bed Rail user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand** 2019






REDGUM Brand is part of **FOR-DE Group Pty Ltd**

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - BED RAIL
5. **USAGE GUIDELINES** - DO's + DON'Ts
6. **PRODUCT CARE** - INSPECTIONS / CLEANING
7. **WARRANTY**
8. **REDGUM BRAND** PRODUCT RANGE

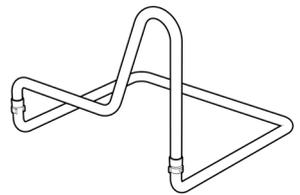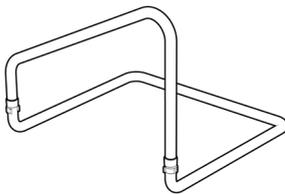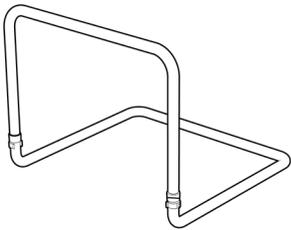

INTRODUCTION

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the **Bed Rail** please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

F: +61 8 9248 9452

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090




INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your **Bed Rail** for the purpose for which it is intended, which is as an assistive grab rail solution that can be positioned to suit the user's individual requirements.

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

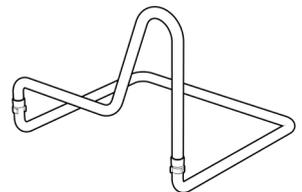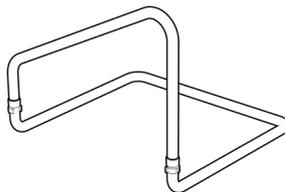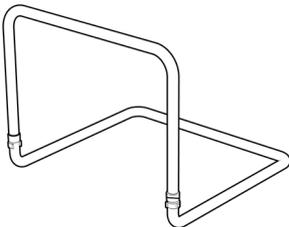

PRODUCT OVERVIEW

FEATURES

Assistive grab rail aids user to get in and out of bed comfortably, securely and maintain independence

Left or right bed location - Top rail on Slope / Peak bed rail can be flipped to suit

Easy to assemble - NO tools required

Easy to locate and remove for bed making

Strong lightweight construction

Can be pulled apart for storage and transport

Optional tie down straps available

SPECIFICATIONS

(All dimensions shown are nominal)

Overall Dimensions:

Length: 600mm




Depth: 550mm

Height: 495mm

Product Weight:

All three rail types: approx. 2kg


Please contact **REDGUM Brand** if you would like any further information on your **Bed Rail**.

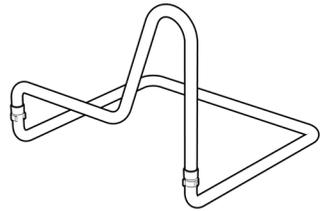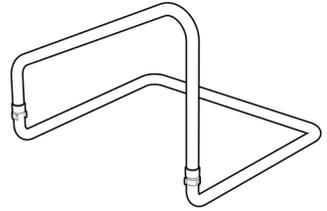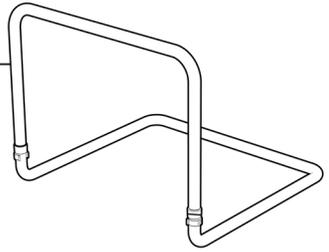


ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:
Only the chosen top rail is supplied (not all three types)


PARTS:

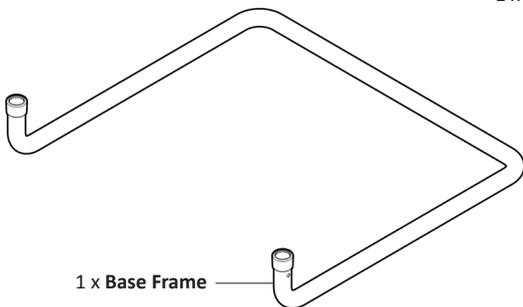

1 x STANDARD
Top Rail

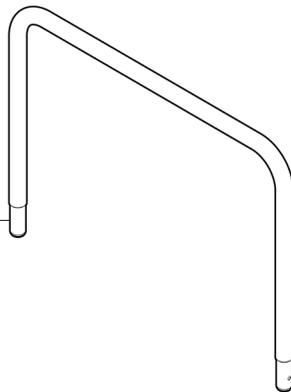


1 x SLOPE
Top Rail

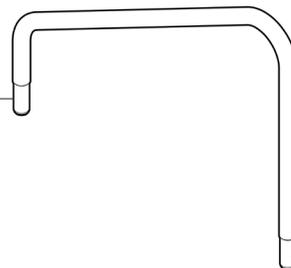

2 x Locking Clip



1 x PEAK
Top Rail

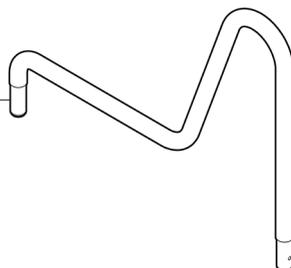

IMPORTANT NOTE


Check the product for any visible damage. If you see any damage or suspect a fault,
DO NOT USE YOUR PRODUCT and contact your supplier for support.

ASSEMBLY - BED RAIL

The bed rail requires assembly prior to use. Please follow these steps:

1. Insert the ends of top rail into the ends of the base frame,
2. Insert pin on locking clip into hole on base and push clip until it locates around tube,
3. Locate other clip on opposite side of bed rail.

The bed rail is now ready to be used.

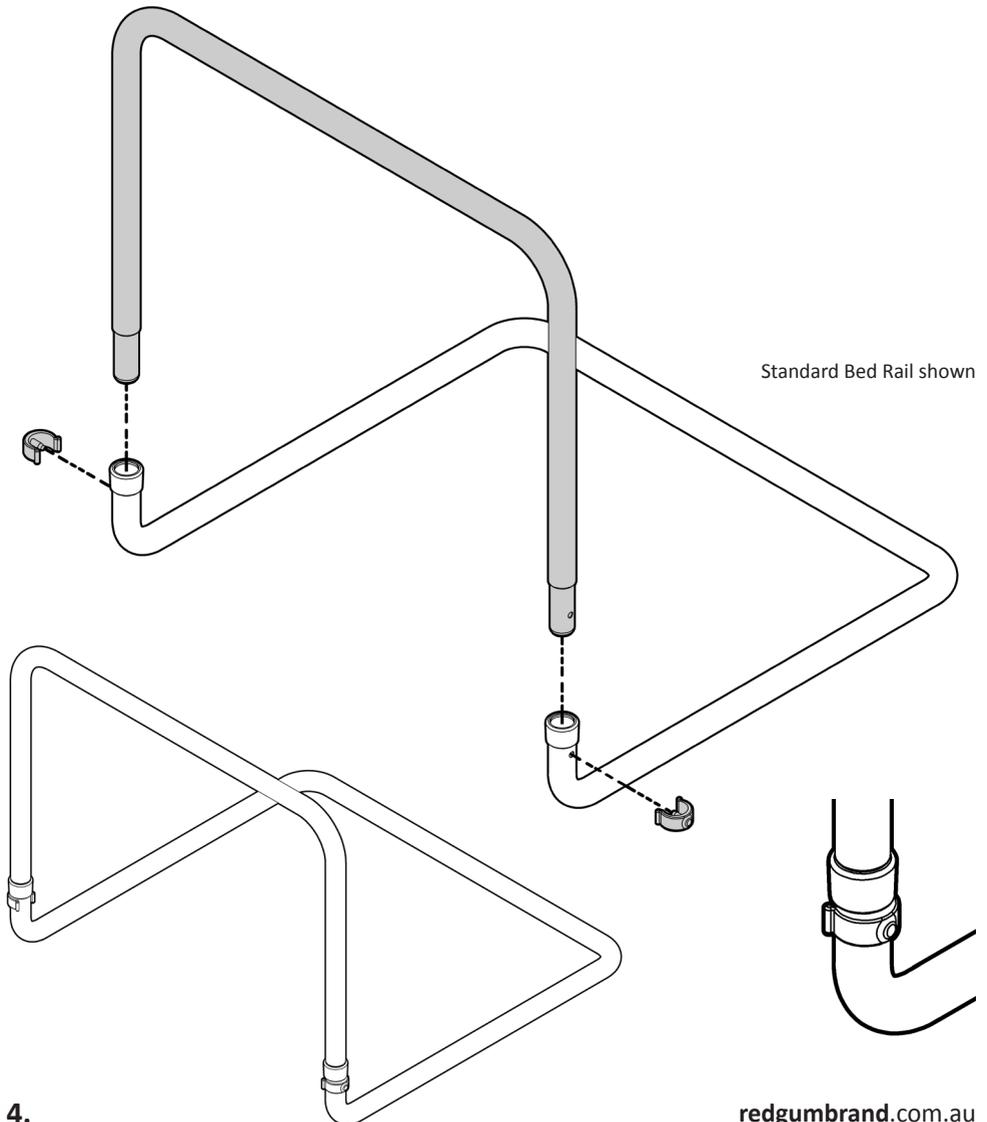



USAGE GUIDELINES - DO's + DON'TS

Please follow these guidelines to get the optimal benefit from your Bed Rail.

Failure to follow these guidelines may result in damage to the Bed Rail and put the user at risk.

Ensure the Bed Rail base frame is **FULLY INSERTED** in between the mattress and the bed base / frame. An optional strap is available to hold the Bed Rail in place if required.

The Bed Rail is designed to be an **ASSISTIVE** grab rail and is not designed to take a user's total body weight.

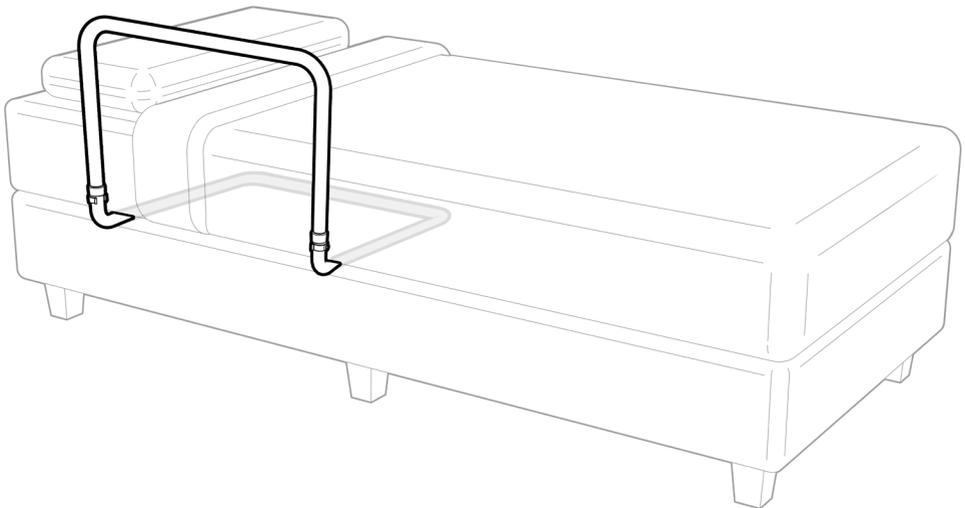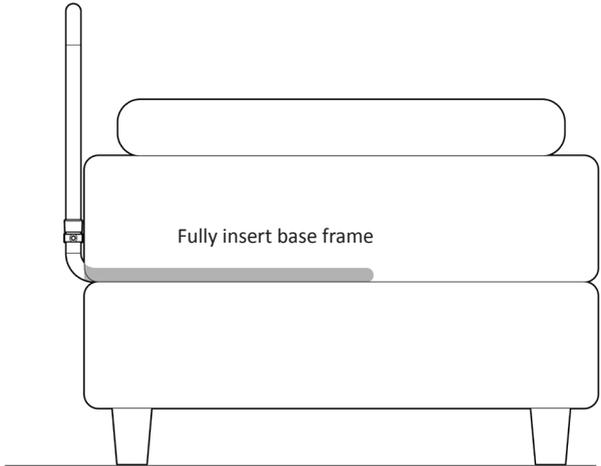

PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you check your Bed Rail visually periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the Bed Rail immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.




IMPORTANT NOTE

Do not fit any parts or accessories to your Bed Rail that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

CLEANING

Clean your Bed Rail using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Bed Rail and should not be used.

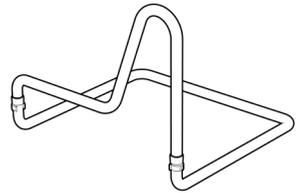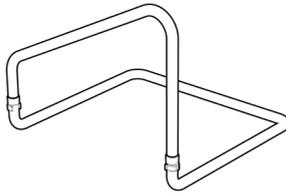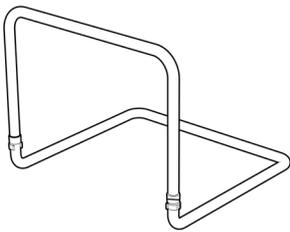

WARRANTY

Your REDGUM Brand:

RG611 - STANDARD Bed Rail

RG612 - SLOPE Bed Rail

RG614 - PEAK Bed Rail

are all covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

If you have any issues or questions about your Bed Rail please contact your supplier or REDGUM Brand direct on:




T: +61 8 9248 4180

F: +61 8 9248 9452

E: sales@for-de.com.au

A: 1 Business Way, Malaga
Western Australia 6090


Please keep this User Manual in a safe place for future reference.

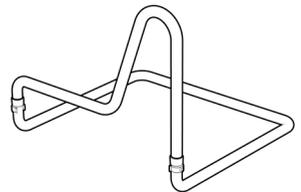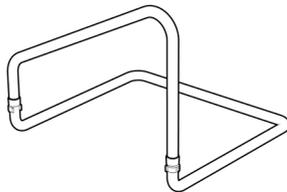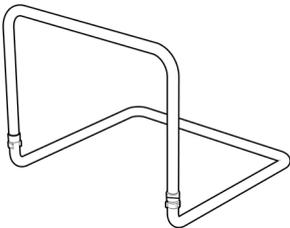

REDGUM Brand - Product Range

REDGUM Brand offer a comprehensive range of products including:

- Seat Walkers**
- Walking Frames**
- Wheelchairs**
- Walking Sticks + Quads**
- Crutches**
- Reachers**
- Shower / Bathroom products**
- Toilet products**
- Cushions: Air / Foam**
- Electric Lift Chairs**
- Electric Beds**
- Portable / Fixed Aluminium Ramps**
- Exercisers**
- Personal Alarms**

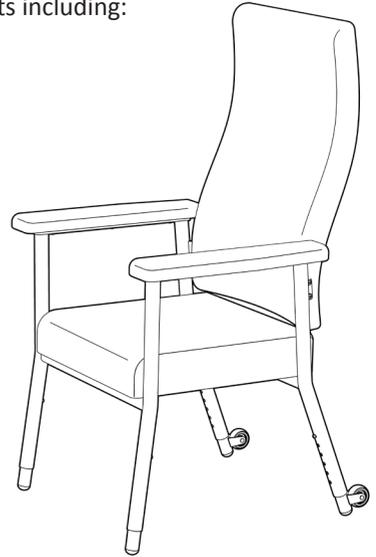

REDGUM Brand stock a large range of Spare Parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:


T: +61 8 9248 4180

F: +61 8 9248 9452

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

W: www.redgumbrand.com.au

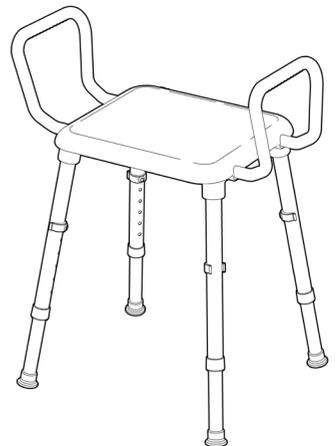

DISTRIBUTOR:

REDGUM BRAND
*Daily Living and
Lifestyle Products*



Proudly Australian Owned and Operated

redgumbrand.com.au