


DELUXE BEDSIDE COMMODE


RG650

Edition 1

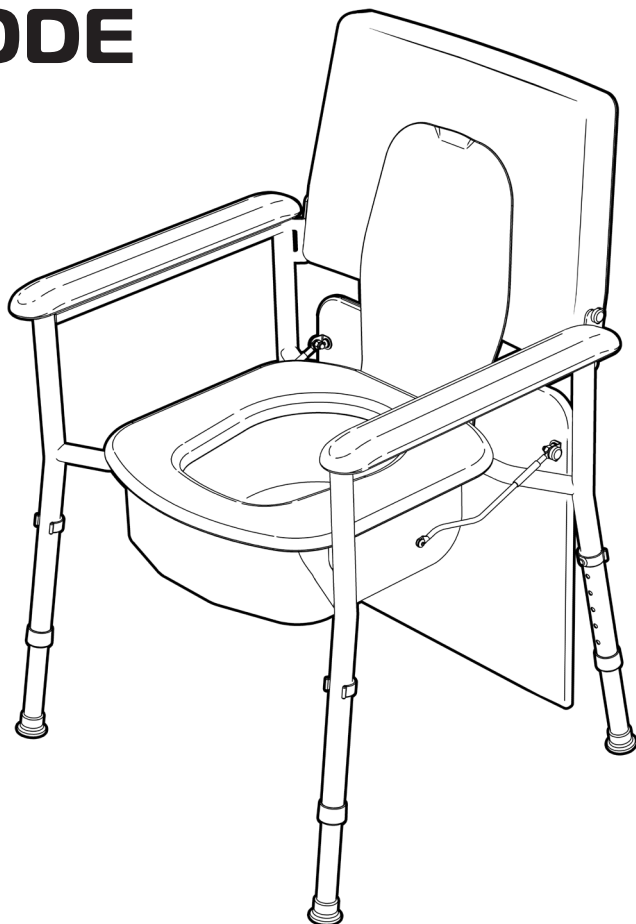

USER MANUAL

Please read and keep these instructions

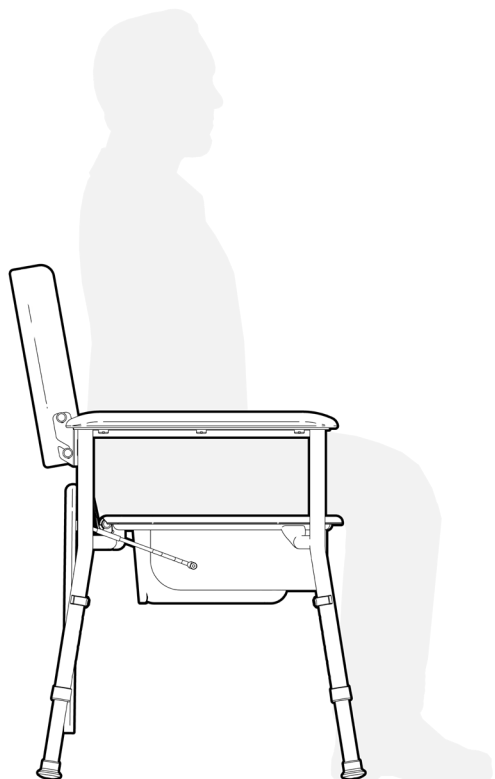

REDGUM brand

ASSISTED LIVING PRODUCTS

REDGUM Brand owns the copyright to the Deluxe Bedside Commode user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**




REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

CONTENTS

1. **INTRODUCTION**
2. **PRODUCT OVERVIEW:** FEATURES / SPECIFICATIONS
3. **ASSEMBLY:** UNPACK / INSPECT PARTS
4. **ASSEMBLY:** FIT LEGS
5. **ASSEMBLY:** FIT LEGS continued
6. **ASSEMBLY:** FIT BACKREST
7. **ASSEMBLY:** FIT BACKREST continued
8. **USAGE:** SEAT OVERLAY
9. **USAGE:** COMMUNE ACCESS
10. **USAGE GUIDELINES**
11. **INSPECTIONS / CLEANING**
12. **WARRANTY**

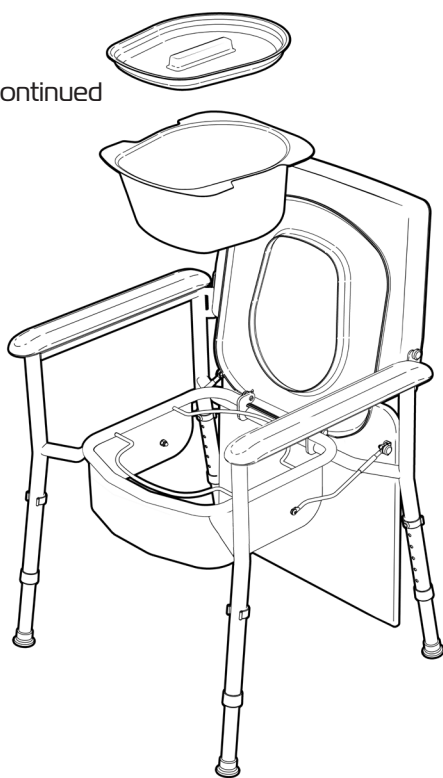

INTRODUCTION

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the Deluxe Bedside Commode please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180** E: **orders@for-de.com.au**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use.

The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.


Only use the Deluxe Bedside Commode for its intended purpose which is as a toileting solution away from the toilet.

IMPORTANT NOTE

This equipment **MUST** be set up by a competent person.

Maximum User Weight: 130kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

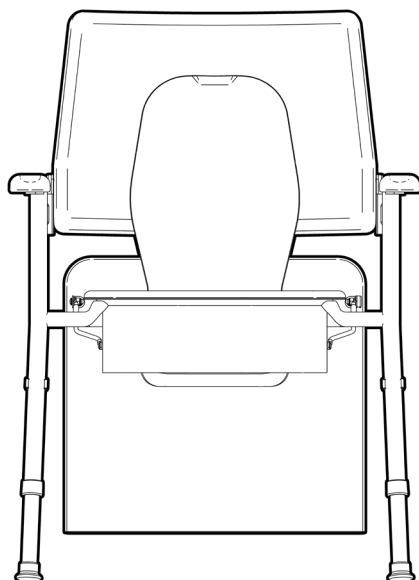

1.

ASK FOR THE BRAND

PRODUCT OVERVIEW

FEATURES

- Perfect for toileting away from the toilet
- Can be used as a bedside chair
- Commode feature can be concealed by padded seat overlay
- Upholstered backrest, armrests and seat overlay
- Plastic flip up toilet seat flap and lid
- Removable bowl and lid
- Height adjustable legs
- Non slip leg tips

SPECIFICATIONS (All dimensions shown are nominal)

Overall (at lowest height setting)

610mm W x 590mm D x 850mm H

Width between handles: **490mm**

Seat depth: **450mm**

Seat heights (Floor to top of seat flap)

6 settings

Hole 1: **430mm**

Hole 2: **455mm**

Hole 3: **480mm**


Hole 4: **505mm**

Hole 5: **530mm**

Hole 6: **555mm**

Product Weight: **8kg**


Maximum User Weight: **130kg**

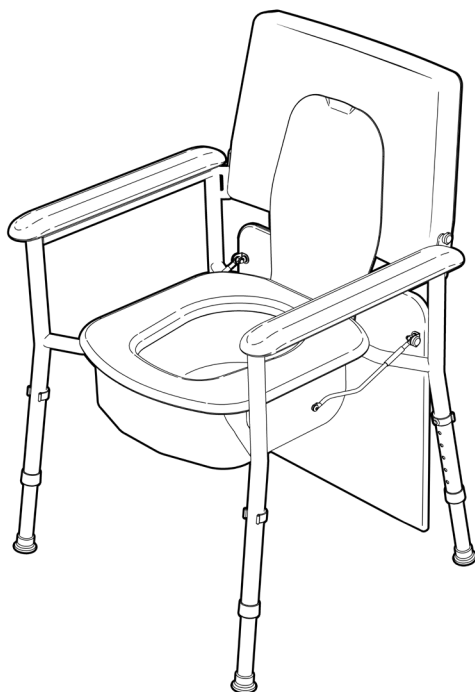

Please contact **REDGUM Brand** if you would like any further information on the Deluxe Bedside Commode.

ASSEMBLY UNPACK AND INSPECT PARTS

Check to ensure that the following parts have been supplied prior to assembly:

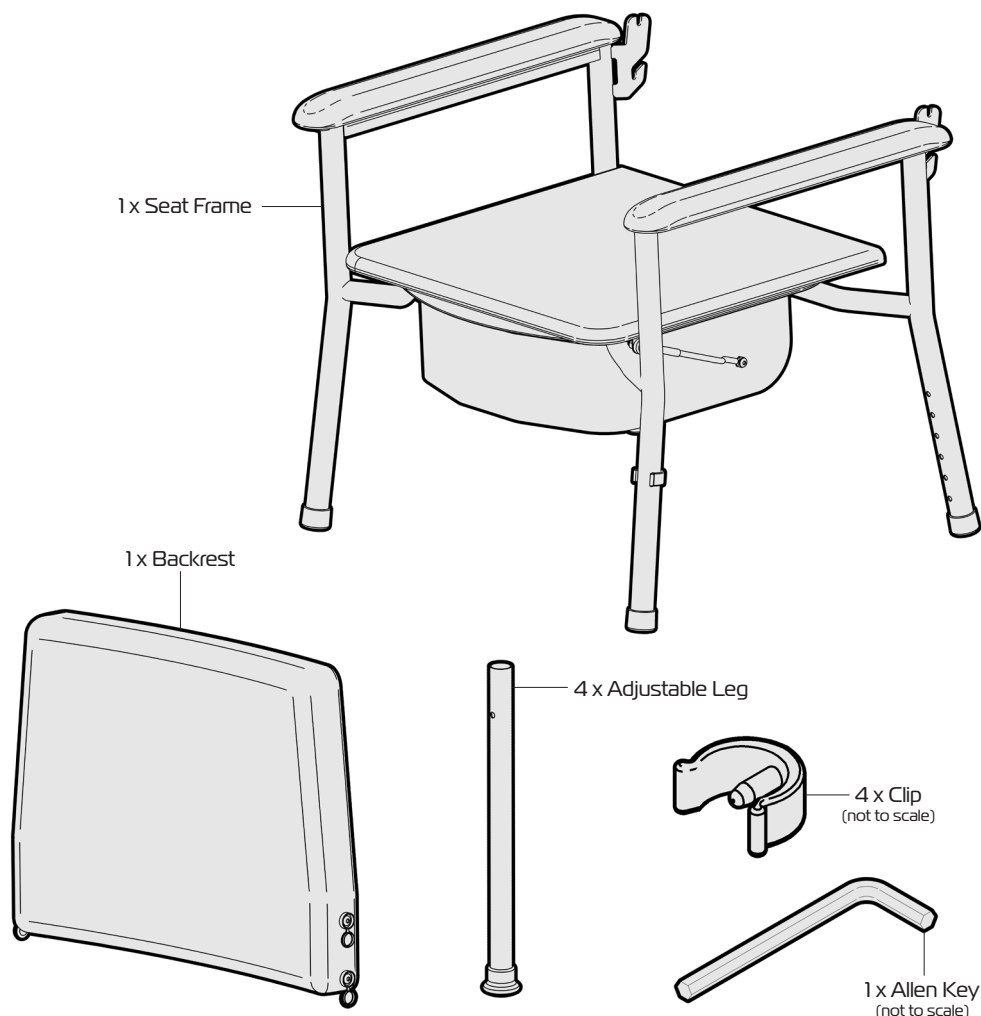

IMPORTANT NOTE

Check the parts for any visible damage. If you see any damage or suspect a fault, **DO NOT USE THE PRODUCT** and contact your supplier for support.

3.

ASK FOR THE BRAND

ASSEMBLY FIT LEGS

The Deluxe Bedside Commode has height adjustable legs that can be set to the correct height to suit the user.

All four legs should be set so the top of the seat flap is inline with the back of the users knee. Measure from the floor to the back of the users knee and adjust each leg to the closest hole setting shown below.

Seat Height Setting
(Floor to top of seat flap)

Hole 1: **430mm**


Hole 2: **455mm**

Hole 3: **480mm**

Hole 4: **505mm**

Hole 5: **530mm**


Hole 6: **555mm**

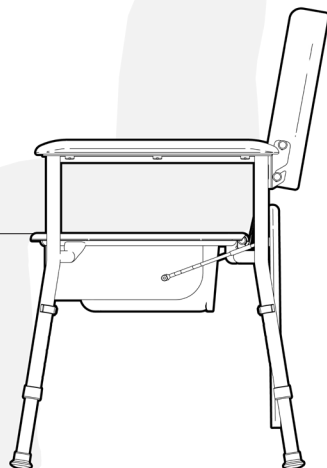

Please follow these instructions to fit the legs to the seat frame.

1. Carefully place the seat frame on its side as shown.


Refer to next page for further instruction....

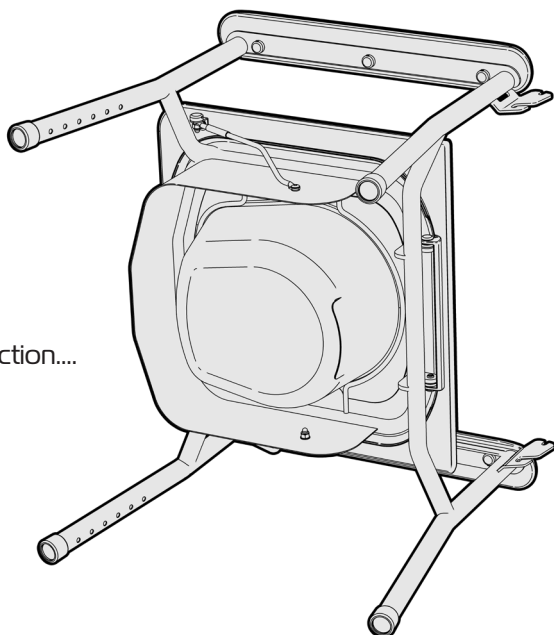


4.

ASSEMBLY FIT LEGS

2. Align the hole in the adjustable leg with the holes in the seat frame.

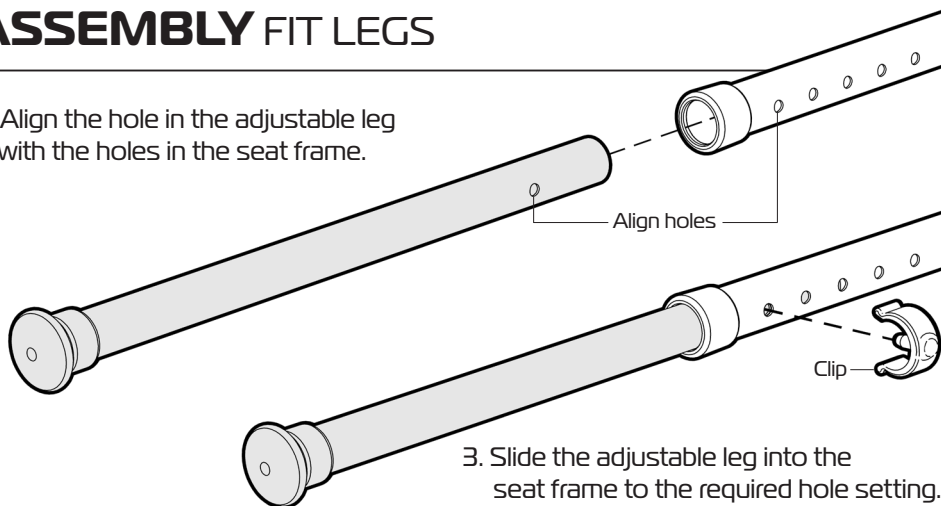

3. Slide the adjustable leg into the seat frame to the required hole setting. (Both holes must align)

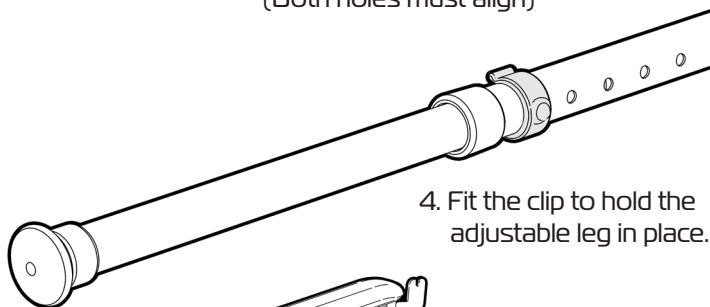

4. Fit the clip to hold the adjustable leg in place.


5. Repeat the process to fit all 4 legs.

Ensure all legs are set at the same height prior to use.

6. Carefully place the seat frame onto its feet.

5.



ASK FOR THE BRAND

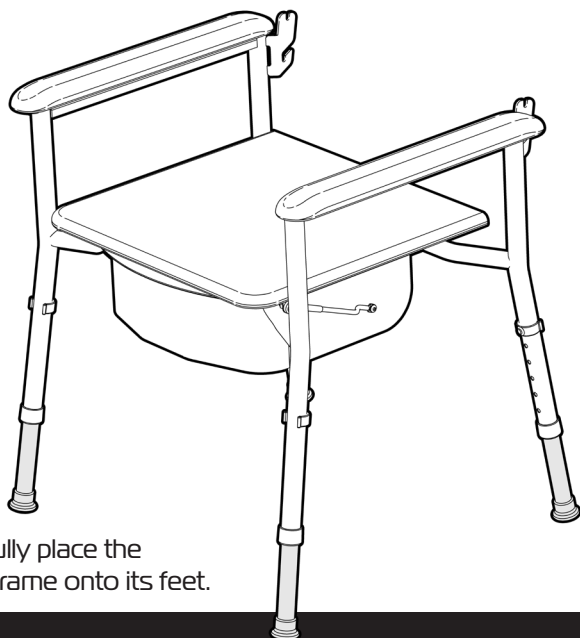

ASSEMBLY FIT BACKREST

Please follow these steps to fit the backrest onto the seat frame correctly:


1. Align the backrest with the seat frame as shown.

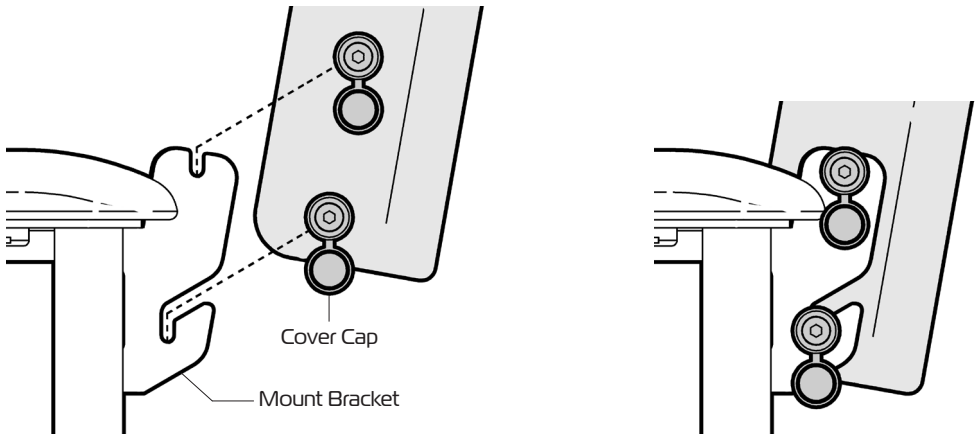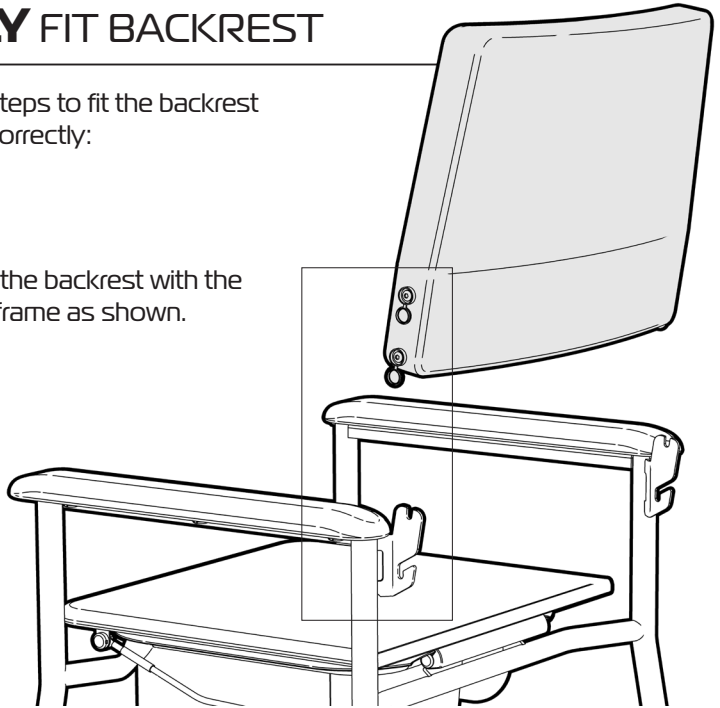


2. Align the fixings on the backrest with the mount brackets on the seat frame.
With the cover caps on the outside of the mount brackets - carefully locate the fixings onto the slots on the mount brackets on both sides.

Please refer to next page for more instruction on fitting backrest....


ASSEMBLY FIT BACKREST

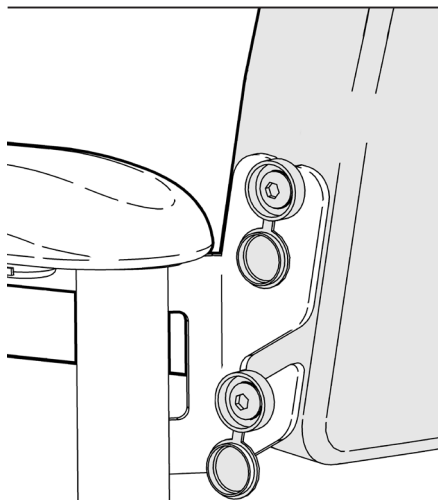


3. Ensure the backrest is fully seated on the mount brackets

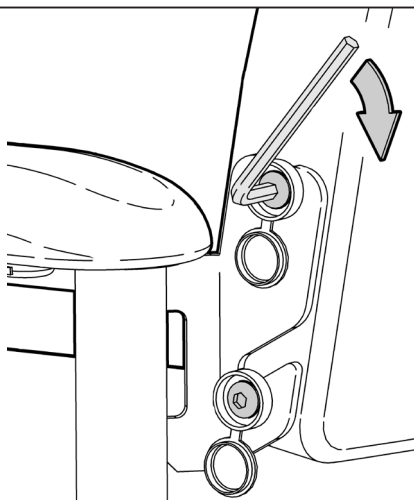

4. Using the allen key provided tighten all four fixings.

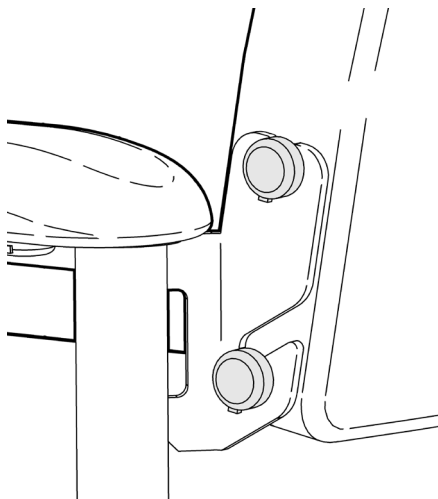

5. Once all four fixings are sufficiently tightened close the cover caps.

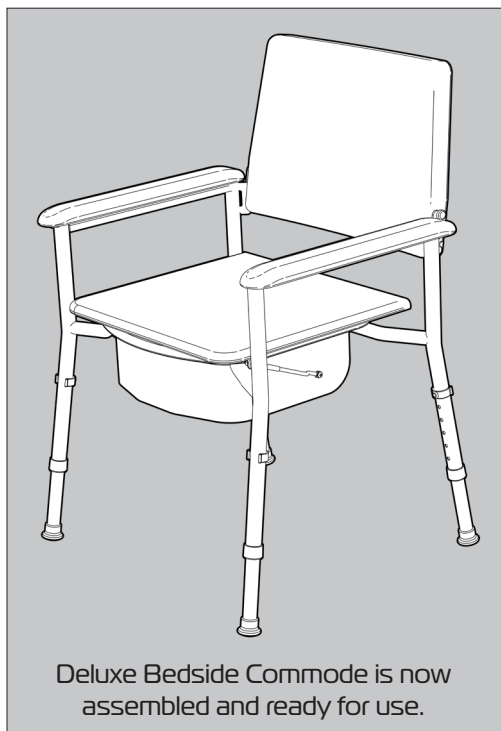


Deluxe Bedside Commode is now assembled and ready for use.

7.

ASK FOR THE BRAND


USAGE SEAT OVERLAY

This product can be used as a bedside chair when the seat overlay is on. Please follow these steps to stow the seat overlay and convert the product into a commode.

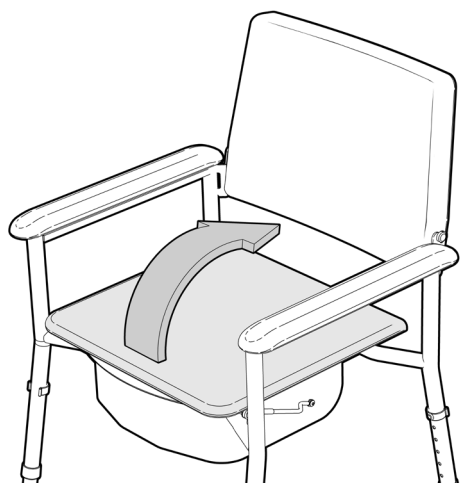


1. Lift the front of the seat overlay and pivot back into the gap between the seat and the backrest.

2. Carefully guide the overlay back until the pivot arms rest on the rear cross tube of the seat frame.

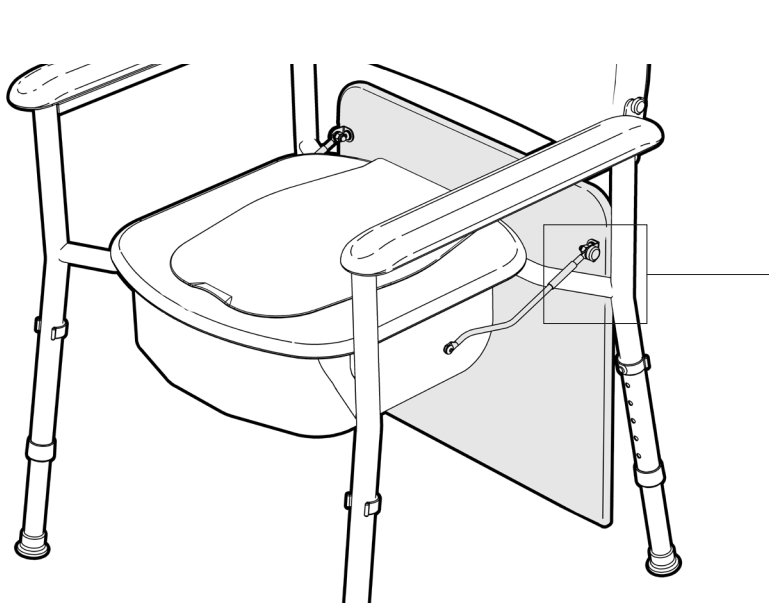


USAGE COMMODE ACCESS

Please follow these instructions to access the bowl and lid on the commode:

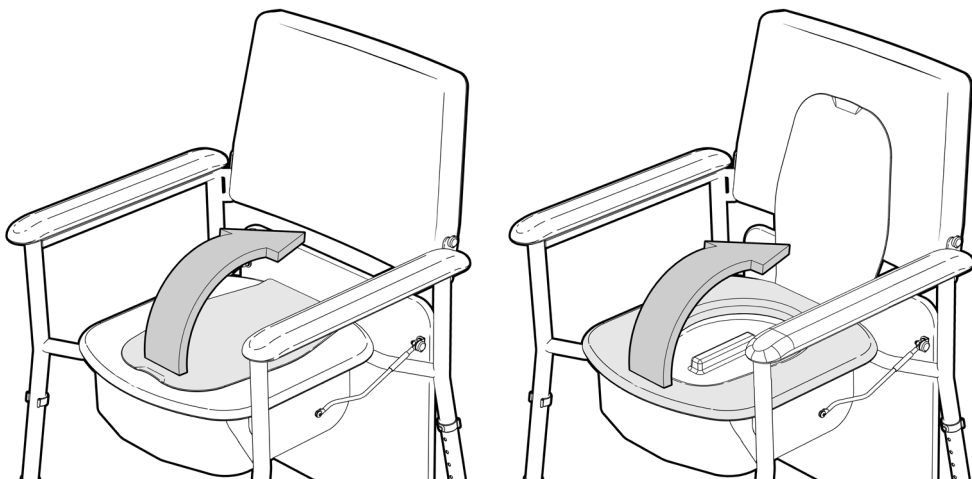

1. Gently rotate up the toilet seat lid and toilet seat flap until they stay up on their own.

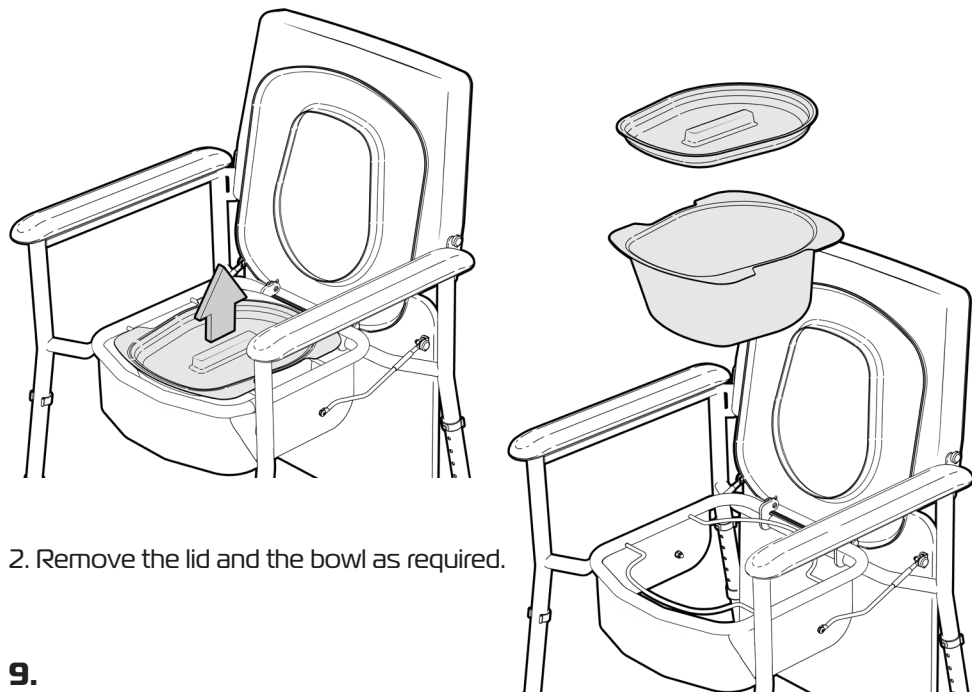

2. Remove the lid and the bowl as required.

9.

ASK FOR THE BRAND

USAGE GUIDELINES

Please follow these guidelines to get the optimal benefit from the Deluxe Bedside Commode.

Failure to follow these guidelines may result in damage to the product and put the user at risk.

Maximum user weight: **130kg**



Ensure that the Deluxe Bedside Commode has been set up at the correct height to suit the user. (Refer to page 4.)

Top of handles should be within reach when standing in front of the Deluxe Bedside Commode.

Always have both hands on the armrests and ensure your body weight is central over the commode when getting on and off.

Ensure ALL FOUR LEGS remain on the ground at all times when the commode is being used.

Take care when removing the bowl after toileting. Fit lid to cover contents and avoid spillage when moving bowl.

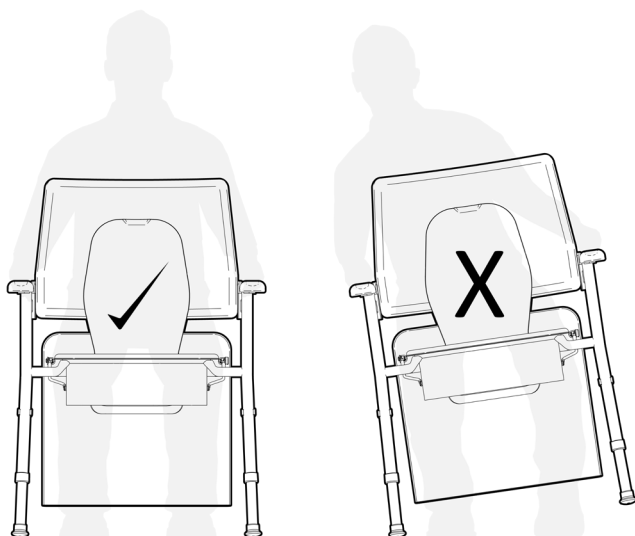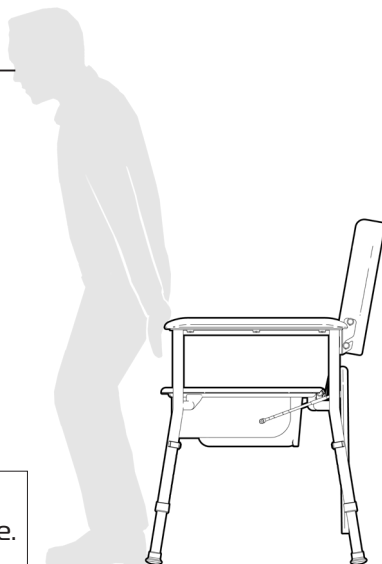

IMPORTANT NOTE

Do not lean too far to the side causing the legs to leave the ground.

INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the Deluxe Bedside Commode periodically for signs of damage or wear prior to use. Overall check the following:

- Frame has no distortion.
- Plastic parts are not damaged or cracked.
- Upholstery is in a usage state.



Withdraw the Deluxe Bedside Commode immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING


Clean the Deluxe Bedside Commode using disinfectant wipes. We do not recommend cleaning the product with soap and water.

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Clean the bowl and lid each time after use.

Do not use any abrasive cleaners or abrasive cleaning pads as they could potentially damage the Deluxe Bedside Commode.

If you have any questions about the cleaning of the Deluxe Bedside Commode please contact your supplier or REDGUM brand direct.

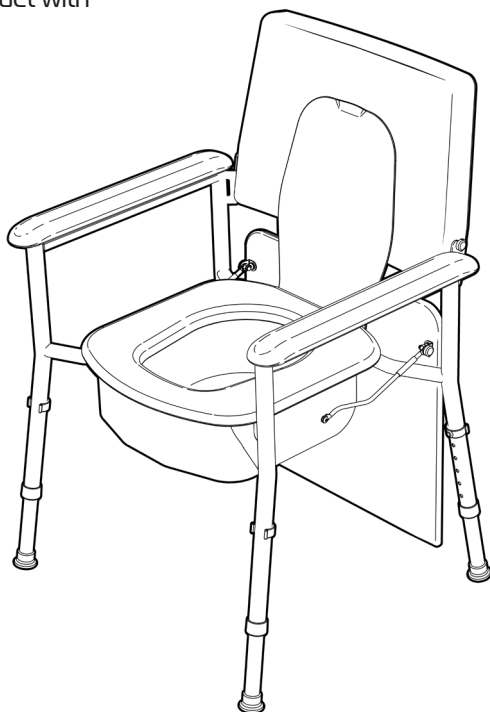

WARRANTY

REDGUM Brand Deluxe Bedside Commode (RG650) is covered by a 12 month warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if the product needs to be returned to the place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser.
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the Deluxe Bedside Commode in any way or fit any parts or accessories that are not specified as an option. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the Deluxe Bedside Commode please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

DISTRIBUTOR:



Explore the
REDGUM Brand
range of products

