

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Dressing Stick for its intended purpose which is being used as a device to extend the users reach.

INSPECT SUPPLIED PRODUCT If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.


Please turn over for more instructions

redgumbrand.com.au


REDGUM brand ASSISTED LIVING PRODUCTS

USAGE GUIDELINES


DRESSING STICK

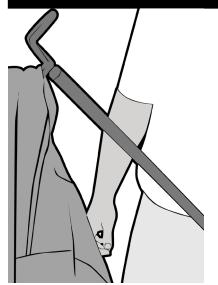


1. Place iacket onto the dressing stick.

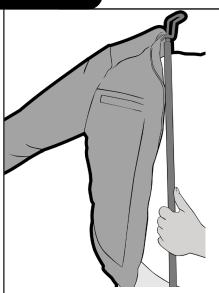


Holding the dressing stick, place one side of the jacket onto your shoulder and slide arm through.

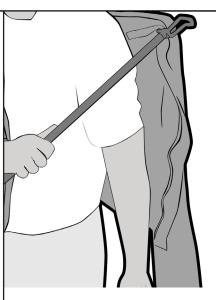

3. Repeat on the other side using the dressing stick to hold up the jacket

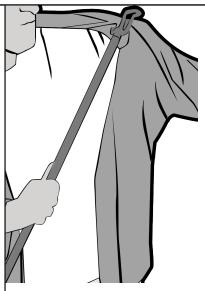

Slide the other arm through the jacket whilst using the dressing stick to hold it in place.

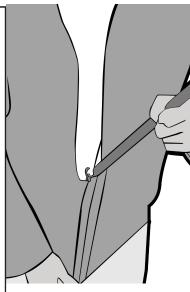

 Using the hook on the other side of the dressing stick, carefully place it on the zipper to zip up jacket.

Dressing Stick can also be used to pull on and push off pants, socks, shoes etc...

INSPECTIONS

We suggest that you check the Dressing Stick visually periodically for signs of damage or wear prior to use. Withdraw the Dressing Stick immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for support.

CLEANING

Periodically clean the Dressing Stick using anti bacterial wipes or a non-abrasive cleaner. Abrasive cleaners or abrasive cleaning pads could damage the Dressing Stick and should not be used.

WARRANTY

Redgum Brand Dressing Stick (RG71DS) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage.

You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

ASK FOR THE BRAND

redgumbrand.com.au