


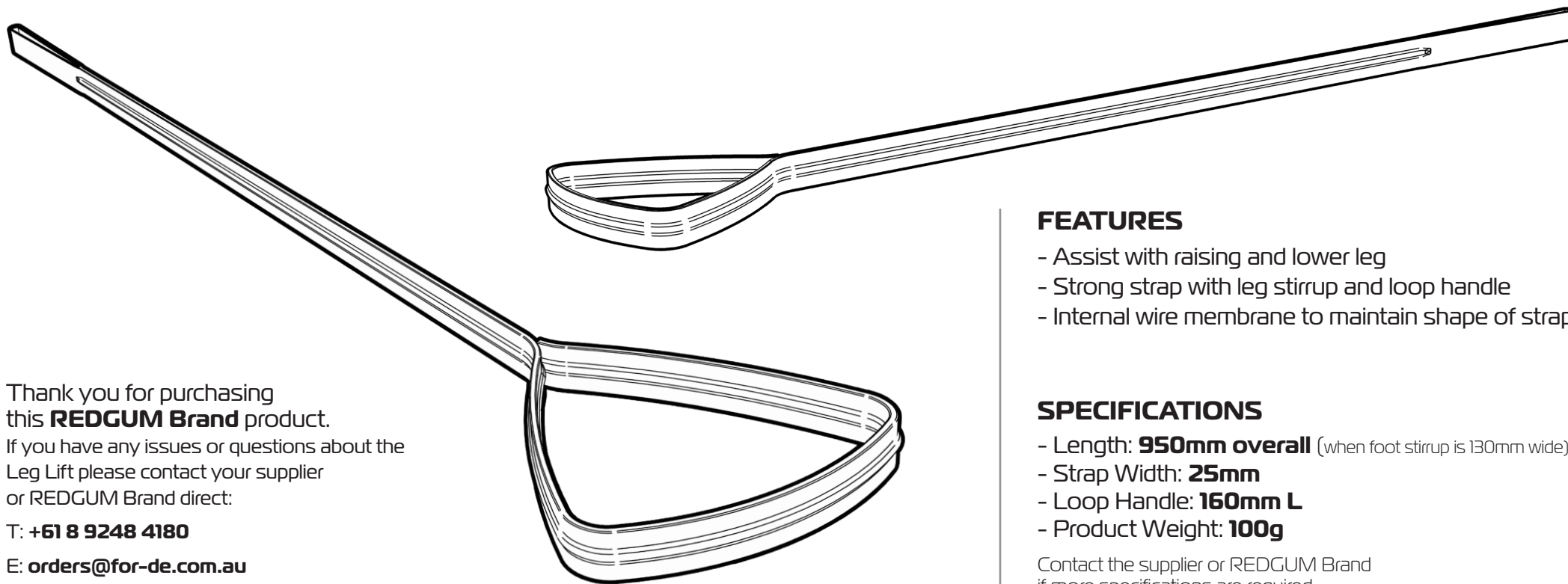

Thank you for purchasing
this **REDGUM Brand** product.
If you have any issues or questions about the
Leg Lift please contact your supplier
or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

FEATURES

- Assist with raising and lower leg
- Strong strap with leg stirrup and loop handle
- Internal wire membrane to maintain shape of strap

SPECIFICATIONS

- Length: **950mm overall** (when foot stirrup is 130mm wide)
- Strap Width: **25mm**
- Loop Handle: **160mm L**
- Product Weight: **100g**

Contact the supplier or REDGUM Brand
if more specifications are required.

INTENDED USE


This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Leg Lift for its intended purpose which is being used as a device to assist with the raising and lowering of ones leg.

INSPECT SUPPLIED PRODUCT If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.

Please turn over for more instructions

ASK FOR THE BRAND

redgumbrand.com.au

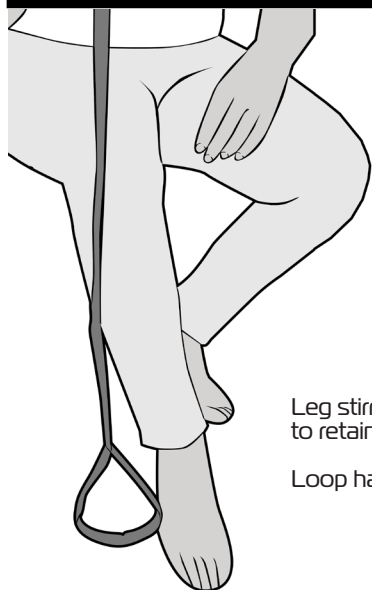

Leg stirrup has internal wire frame to retain the shape required.

Loop handle is flexible.

1. Determine which end of the leg lift is the leg stirrup.



2. Slide your foot into the leg stirrup.



3. Pull up on the strap to assist with leg lift.

INSPECTIONS

We suggest that you check the Leg Lift visually periodically for signs of damage or wear prior to use. Withdraw the Leg Lift immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for support.

CLEANING

Clean the Leg Lift with warm soapy water if required and allow to dry prior to use. Abrasive cleaners or abrasive cleaning pads could damage the Leg Lift and should not be used.

WARRANTY

Redgum Brand Leg Lift (RG71LL) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.