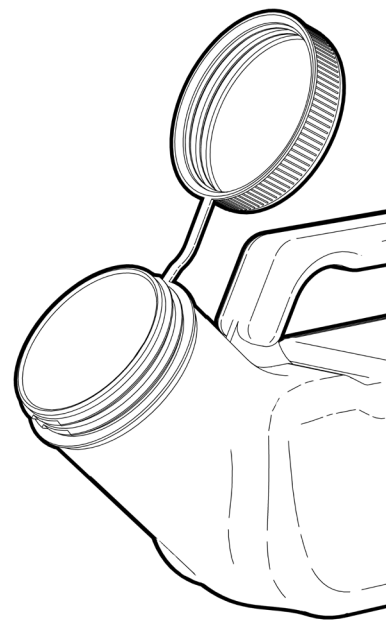
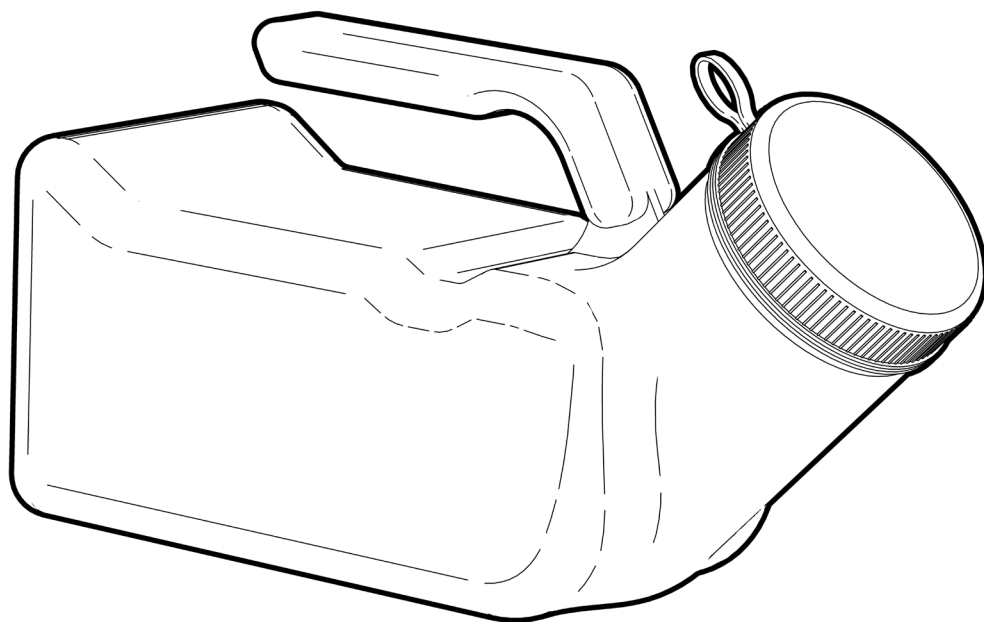


FEATURES

- Assist men with toileting away from toilet
- Screw on lid with retaining ring
- Volume marking on bottle (50ml increments)
- Long handle for effective handling
- Easy to clean

SPECIFICATIONS

- Size: **255mm L x 95mm W x 130mm H**
- Spout: **Ø52mm int.**
- Capacity: **1 litre**
- Product Weight: **120g**

Contact the supplier or REDGUM Brand if more specifications are required.

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the Male Urinal Bottle please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Male Urinal Bottle for its intended purpose which is being used as a device to assist with collecting a males urine.

INSPECT SUPPLIED PRODUCT If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.

Please turn over for more instructions

ASK FOR THE BRAND

redgumbrand.com.au


USAGE GUIDELINES

Unscrew the lid and gently place between the legs of the user.

Get the user to hold the bottle in place with their thighs when toileting.

Use as required and then carefully remove the bottle and screw on the lid to prevent spillage.

Stand the bottle up if required to determine volume.

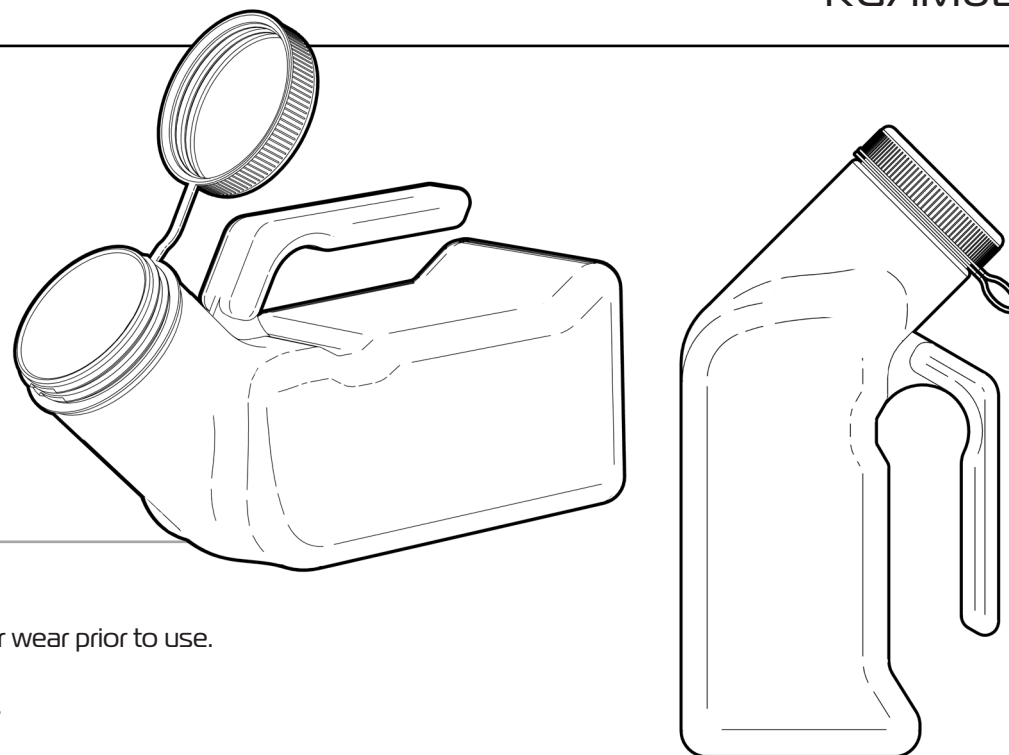

INSPECTIONS

We suggest that you check the Male Urinal Bottle visually periodically for signs of damage or wear prior to use.

Withdraw the Male Urinal Bottle immediately if you suspect damage or any other fault.

If in any doubt, please do not issue or use, but immediately contact your supplier for support.

CLEANING

Clean the Male Urinal Bottle with warm soapy water and allow to dry prior to use. Male Urinal Bottle can be autoclaved if required.

Abrasive cleaners or abrasive cleaning pads could damage the Male Urinal Bottle and should not be used.

WARRANTY

Redgum Brand Male Urinal Bottle (RG71MUB) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment.

Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law.

You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage.