


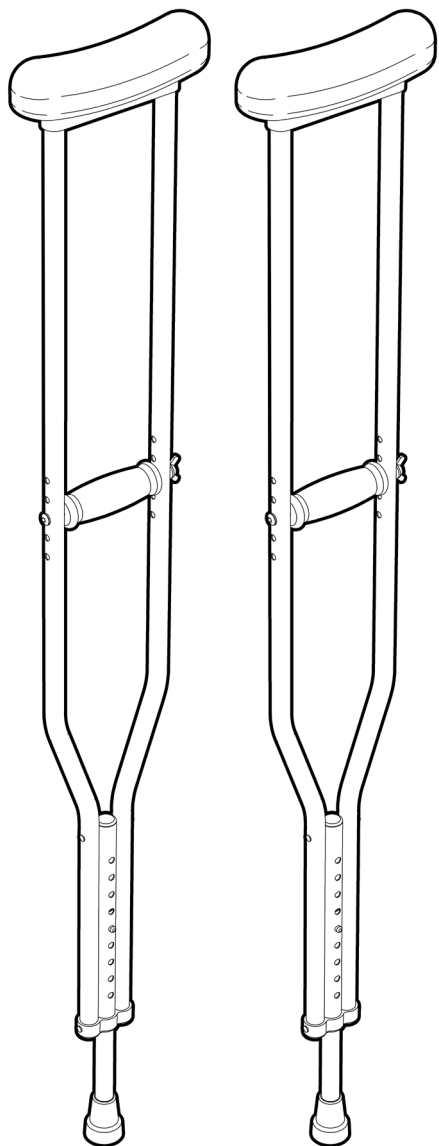

Thank you for purchasing this **REDGUM Brand** product. If you have any issues or questions about the Adjustable Underarm Crutches please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

E: orders@for-de.com.au

A: 1 Business Way, Malaga, Western Australia, 6090

FEATURES

- Excellent aid to take pressure off an injured leg.
- Supplied in pairs
- Height adjustable leg and handgrip.
- Comfortable ergonomic underarm padding.
- Comfortable ergonomic handle grip.
- Non-slip rubber leg tip.

SPECIFICATIONS

	RG7200S	RG7200M	RG7200T
Overall height	950-1150mm	1150-1350mm	1350-1550mm
Handgrip height (from top of crutch to top of handgrip)	195-350mm	395-550mm	450-600mm
Product weight	1.5kg per pair	1.5kg per pair	1.8kg per pair
Max. user weight	136kg		

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

INSPECT SUPPLIED PARTS

If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.


INSPECTIONS

We recommend that the Crutches are visually checked periodically for signs of damage or wear prior to use. Withdraw the Crutches immediately from use if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for support.

Please turn over for more instructions

To adjust the height of the Crutch:


Push in the press buttons on both sides of the Crutch, turn the leg slightly, lift or lower the leg to the required height and then realign the leg until both press buttons pop out of the nearest height adjustment hole.

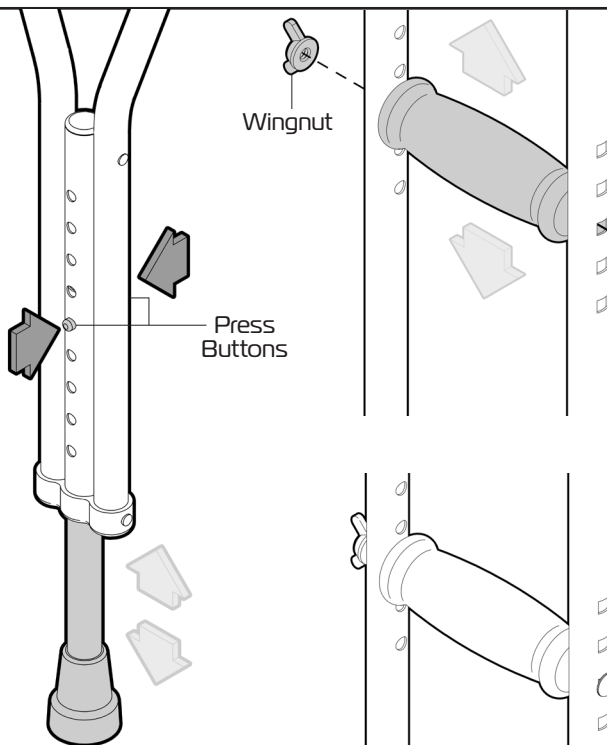

To adjust the height of the handgrip:

Undo the wingnut and remove the bolt.

Align the handgrip with the required height adjustment hole. Insert the bolt (making sure the square shape under the bolt head locates into square hole) and screw on the wingnut.

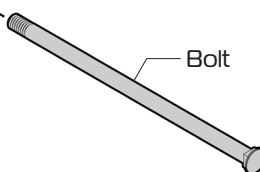


USAGE GUIDELINES

The Underarm Crutches should be set up to suit the users individual requirements.

Adjust the overall height of the Crutch to be approx. 50mm lower than users armpit. Adjust the handgrip to be level the wrist when their arm is down by their side.

Ensure that both Crutches are set up the same.

We suggest that you check the Crutches periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn. If you see any damage or suspect a fault, **DO NOT USE THE PRODUCT** and contact your supplier for support.

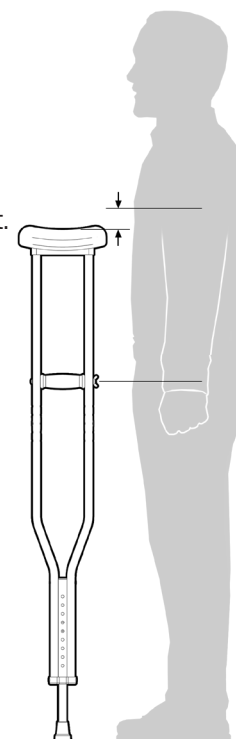

CLEANING

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining. Clean the Crutches using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners or abrasive cleaning pads could damage the Crutches and should not be used. If you have any questions about the cleaning of the product please contact your supplier or REDGUM brand direct.

WARRANTY

Adjustable Underarm Crutches (pair) (RG7200S, RG7200M, RG7200T) are covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage.

You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure. Do not modify or fit any parts or accessories to the Reacher that are manufactured by another company.

To do so will void your warranty and may seriously compromise your safety.